MOPEDIA®

ITALIANO

MA MP170-MP171 02 A_ITA_09 2020

SPONDA TELESCOPICA

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INDICE

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CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

MP170 Cromed Universal Telescopic bedside rails

MP171 White Universal Telescopic bedside rails

2. INTRODUCTION

Thank you for choosing a Telescopic bedside rail by MOPEDIA, manufactured by MORETTI S.p.A. Their design and quality are a guarantee of comfort, safety and reliability. Telescopic bedside rails by MOPEDIA by MORETTI S.p.A. are designed and built to meet all your demands for a practical, correct use. This user manual provides for some suggestions as to how correctly use the aid you have chosen and offers a lot of valuable advices for your safety. Please read carefully through the manual before using the Telescopic bedside rails. Should you have any queries, please contact your retailer for advice and assistance.

NOTE: Check that the product has not been damaged during shipment. In case of damage do not use it and contact your dealer for further instructions.

3. INTENDED USE

The bedside rail is a universal accessory designed to prevent an accidental fall of a person from a bed for hospital or home care.

WARNING!

- Do not use the product for a purpose not indicated in this manual.
- Moretti S.P.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer reserves the right to change the information contained in this document without previous notice.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of ACCESSORIES FOR BEDS complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

4.1 Standards and guidelines reference

Safety tests required by UNI EN 12182.

5. GENERAL WARNINGS

WARNING!

• Always pay close attention to the presence of moving parts that could cause limb entrapment and injury.

- For proper use of the device refer to this manual.
- For proper use of the device consult your doctor or therapist.
- Keep the packaged product away from any source of heat because the packaging is made of cardboard.
- The useful life of the accessory is determined by wear of parts cannot be repaired and / or replaced.
- Be careful when children are nearby.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Conditions of disposal

7. GENERAL DESCRIPTION

The Universal Telescopic bedside rails are designed to fit any bed thanks to the telescopic tube to be placed under the mattress.

7.1 Parts List MP170 and MP171

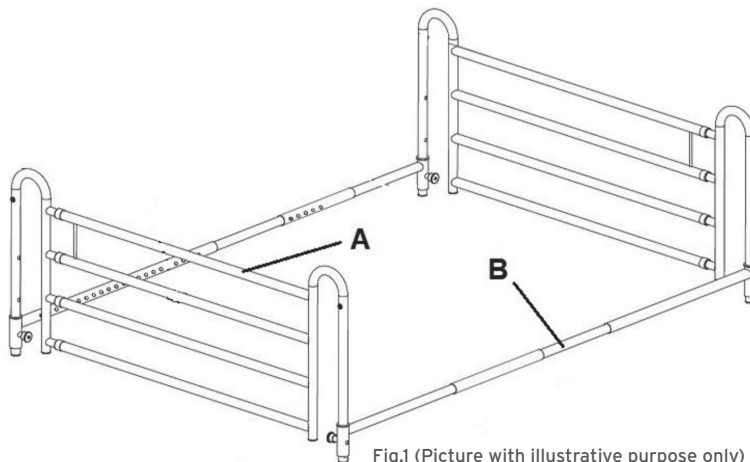
Fig.1 (Picture with illustrative purpose only)

A. Lateral bedside rails (2 Pcs)

B. Telescopic tubes (2 Pcs)

8. ASSEMBLING/DISASSEMBLING

8.1 Assembling

1. Place the telescopic tubes on the floor and insert the bedside rails into the vertical tubes. (Fig.2)
2. Place the assembled structure on the bed.
3. Adjust the width of the transverse elements to the width of the bed by using the push button.

Note: Push the telescopic tubes as close as possible to the bed frame, so as to eliminate the play that can be created between the bed frame and the bedside rail.

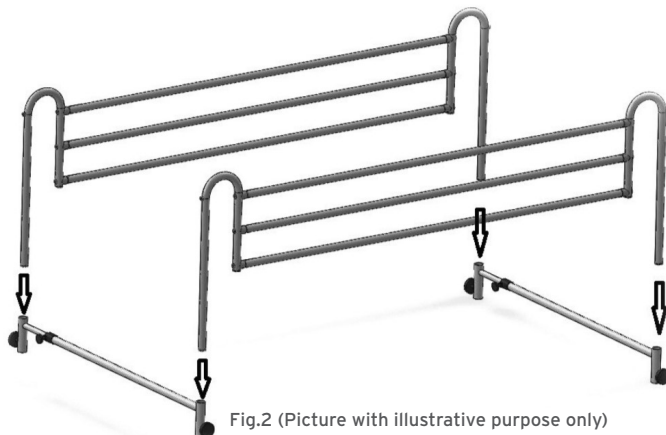
Fig.2 (Picture with illustrative purpose only)

4. Then place the mattress above the bed (the weight of the mattress and the patient help to keep securely the bed side rail).
5. If you need, you can fix the telescopic bedside rails on the bed frame using some belts (not included).

**WARNING!**

- Always pay attention to the presence of moving parts that could cause limb entrapment and injury.
- The bedside rails is not recommended use on beds with limbs lifting device.

8.2 Disassembling

To disassemble do the exact opposite of the above instructions.

9. BEFORE TO USE

- Always check the wear of the mechanical parts in the game so as to ensure the use of the device in total safety for persons and things.
- Check carefully the proper assembly of the device and pay attention to fix the bedside rails on the bed frame.

10. HOW TO USE

Positioning and regulation of the bedside rails are operations that the assistant has to do with maximum care to avoid dangerous installations. Using the knobs you can adjust the height of the bedside rails, until the complete lowering.

11. MAINTENANCE

The MOPEDIA devices by Moretti S.P.A. are checked carefully and supplied with CE mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. In case of repair, use only original spare parts and accessories. The bedside rails do not require special maintenance, although it is advisable to perform periodic checks on the structure.

Check the correct positioning of the tubes, check the connections, the hooks, and the complete tightening of screws and/or clamps.

12. CLEANING AND DISINFECTION**12.1 Cleaning**

To clean the bedside rails use a damp cloth with a solution of alcohol or neutral soap (in equal parts) mixed with water.

**WARNING!**

Do not use abrasive substances, acids, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of metal surfaces.

The manufacturing company cannot be held responsible for the damage that is caused by using materials that could damage the product surface or corrosive chemicals during cleaning.

12.2 Disinfection

If you need to disinfect the product use a common disinfectant neutral detergent.

13. CONDITIONS OF DISPOSAL 


In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the bedside rail in the appropriate disposal areas for recycling.

14. TECHNICAL FEATURES

14.1 Dimension and weight

MP170 Chromed Universal Telescopic bedside rails

MP171 White Universal Telescopic bedside rails

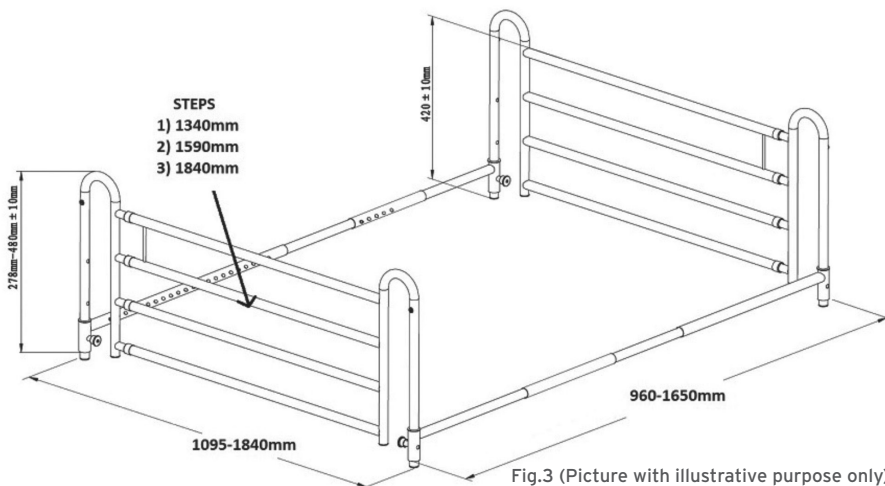Weight: 12 Kg approx.

14.2 Technical specifications

- Chrome plated (MP170) steel frame, Ø25mm
- Powder painted (MP171) steel frame, Ø25mm
- Polyethylene Details
- longitudinal bars in aluminum
- Knobs in PP for a rapid lowering of the bedside rail
- Telescopic tubes for wide beds from 90cm up to 163cm

The dimensions maybe change $\pm 0,5$ cm

15. WARRANTY

Every Moretti product is guaranteed for 2 (two) YEARS from the purchase date, except the restrictions specified below. The warranty shall not apply in case of damage caused by improper use, abuse, alterations of the product or any other use not in compliance with the user manual. The proper use of the product is indicated in the user manual. Moretti S.p.A. shall not be liable for any damages or personal injuries due to incorrect assembly/improper use of the device, not compliant with the user manual. Moretti does not guarantee its products against damages or faults in the following cases: natural disasters, non authorized maintenance or reparations, damages caused by electrical supply problems (if provided), use of parts or components not provided by Moretti, failure in accomplishing the guidelines and instructions for use, non authorized alterations, damages due to the transport (different from Moretti original delivery) or failure in maintaining the device as indicated in the manual. The components subject to wear are not covered by this warranty if the damage is due to the normal use of the product.

16. REPAIRING

Warranty repair

If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

Repair not covered by warranty

A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for goods out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the goods received.

Non defective device

The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the goods will be sent back to customer, the shipping charge will be to be paid by the customer.

17. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

18. EXEMPT CLAUSES

Moretti does not offer any other declaration, explicit or implicit warranty or conditions, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

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