MOPEDIA®

MA MP310_MP315 02 A_ITA_07 2020

PARASPONDE

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ITALIANO

INDICE

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 14.1 Exempt clauses pag. 6

CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

MP310 Premium bed rail protector cm 134 x 33 x 3h

MP315 Folding bed rail protector cm 127 x 66 x 2.5h

2. INTRODUCTION

Thank you for purchasing a Mopedia bed rail protector by Moretti S.p.A. These devices are designed and realized to meet all your needs for a practical, proper and safe use. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance.

3. INTENDED USE

The MOPEDIA by Moretti bed rail protectors are designed to protect the patient from possible critical accidental injury against the sides of the bed.

ATTENZIONE!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of ACCESSORIES FOR BEDS complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

- **READ** carefully the entire manual before using the device .
- For a correct use of the device always consult your physician or therapist
- Make sure that the product is delivered complete and keep it packed away from heat sources
- We recommend the use of a single user
- The device must be sanitized before any re-use
- **SERVICE LIFE** the device use limit is defined by the wear of the parts
- **DO NOT** allow children to play or operate the device
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Conditions of disposal

7. GENERAL DESCRIPTION

The MOPEDIA by MORETTI bed rail protectors are made of memory foam (polyurethane) and complete with waterproof cover (40% PU and 60% polyester) - (removable by zip for model MP310 non-removable for model MP315).

The cover is made with fabric with the following features:

- impermeable to liquids,
- vapor permeable,
- two-way stretch,
- antibacterial, antifungal treatment Sanitized
- latex-free (hypoallergenic)

8. HOW TO USE

Place the device with the long side facing the bed.

MP310

Adaptable to any type of bed rail thanks to the 4 adjustable straps

MP315

Folding model is easy to install thanks to the velcro closure

9. CLEANING

MP310

Indication for washing ONLY FOR COVER:

Wash by hand; Maximum temperature 30°C

Do not bleach

Do not iron

Do not tumble dry

Do not dry clean

The internal foam cannot be washed

MP315

Use only a damp cloth to remove dust, use a damp cloth with a mild soap to remove more difficult dirt. Dry thoroughly before use

10. WARNINGS AND ADVISES

We recommend the use of a single user.

- Store and use away from open flames and heat sources
- The devices must be sanitized prior to any use

11. TECHNICAL FEATURES

MP310 Dimensions: 134 x 33 x 3h cm

MP315 Dimensions: 127 x 66 x 2,5h cm

MP310 & MP315 composition: Internal material -> PU - Cover -> 40% PU 60% Polyester

12. MAINTENANCE

The MOPEDIA devices by MORETTI are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by authorized laboratory every 2 years.

13. CONDITION OF DISPOSAL

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the bed side rail in protector the appropriate disposal areas for recycling.

14. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual. Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

14.1 Exempt clauses

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

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