MOPEDIA®

MA RP400-XX_RP410-XX 02 A_ITA_09-2020

RIALZI UNIVERSALI PER WATER

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ITALIANO

INDICE

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CE I Class Medical Device
REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND
OF THE COUNCIL of 5 April 2017 concerning medical devices

1. CODES

RP400-6	Universal raised toilet seat (6 cm)
RP410-6	Universal raised toilet seat with lid (6 cm)
RP400-10	Universal raised toilet seat (10 cm)
RP410-10	Universal raised toilet seat with lid (10 Cm)
RP400-14	Universal raised toilet seat (14 cm)
RP410-14	Universal raised toilet seat with lid (14 cm)

2. INTRODUCTION

Thank you for purchasing universal raised toilet seats MOPEDIA by Moretti manufactured to support people with movement difficulty. Their design and their quality guarantee maximum comfort, liability and safety. MOPEDIA by Moretti universal raised toilet seats are designed and realised to satisfy all your needs. This instruction manual contains some useful tips for a correct use of the device you have bought as well as some important advises for your safety. In case of doubts, please contact your retailer who will be able to help you correctly.

3. INTENDED USE

The MOPEDIA universal raised toilet seat is a device to help people with movement difficulties using the wc.

WARNING!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

4. EU DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of RAISED TOILET SEAT - MOPEDIA, complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation

6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

READ carefully the entire manual before using the device.


- Make sure that the product is delivered complete and keep it packed away from heat sources
- We recommend a single patient use
- The device must be sanitized before any re-use
- SERVICE LIFE the device use limit is defined by the wear of the parts that can't be repaired/replaced. Service life limit won't be in any case greather than 10 years
- DO NOT allow children to play or operate the device
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

	Product code		. Batch Lot
	Unique Device Identification		Medical Device...
	Read the instruction manual		CE mark
	Conditions of disposal		Manufacturer


7. PARTS VIEW

A. Raised toilet seat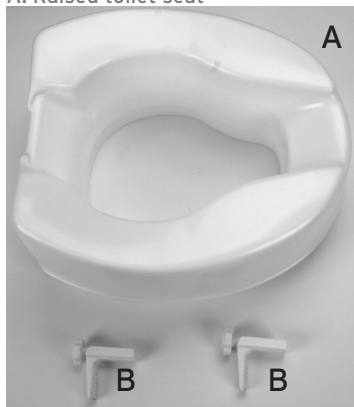

(Image for illustrative purpose only)

B. Brackets

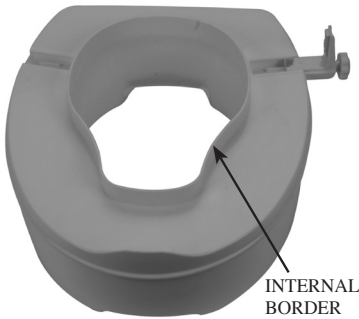(Image for illustrative purpose only)

8. INSTRUCTIONS FOR INSTALLATION

1. Insert the lateral brackets proceeding in the following way: turn the Mopedia universal raised toilet seat upside down and place it in a plane surface. Insert the brackets in the lateral appropriate places. Push them and turn the handle of the screw clockwise in order to block the brackets on the raised toilet seat. Place the brackets in the most external part to avoid problems with raised toilet seat position.

WARNING!

If the knob rotates loosely: push well the bracket against the thread molded in the raised toilet seat maintaining a constant pressure on the knob. So as to ensure the contact between the screw and the thread.

N.B. At this stage do not tighten the brackets otherwise it will be no possible to position the MOPEDIA raised toilet seat upward on the toilet bowl

2. Lift up lid and toilet seat and put the Mopedia universal raised toilet seat on the pottery. Brackets have to be turned to the rear part of the toilet. Be sure that the internal border is placed inside of toilet and the fore border is placed outside. Please note that the raised toilet seat has to fit together with pottery

N.B. The raised toilet seat should be in a perfect flat position on the toilet bowl

3. Turn the handles of the brackets clockwise until the MOPEDIA raised toilet seat is well fixed and can't be moved

WARNING!

The side-lock system has been especially designed to allow easy tightening therefore turning should not be continued once a resistance is felt.


9. CARE AND CLEANING

All parts of the raised toilet seat can be cleaned with soap and hot water or disinfected with specific products.

10. CAUTIONS

- The raised toilet seat should not be used to support more than 136 kg
- Make sure the toilet seat is securely in place before putting weight on it. The raised toilet seat should be checked on a monthly basis to ensure it fits securely
- Do not hold onto the toilet seat as support when getting on or off of the toilet

11. TECHNICAL FEATURES

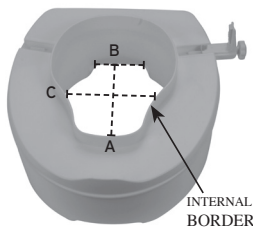External dimensions	 L: 42 cm	 W: 37,5 cm	
Internal border			
	A: 27 cm	B: 10 cm	C: 22 cm
Max. loading weight	136 kg		

12. SPARE PARTS AND ACCESSORIES

For spare parts and accessories please refer to Moretti master catalogue

-RP431 Screws kit

13. CONDITIONS OF DISPOSAL

When disposing of the device never use the normal return systems of municipal solid waste. It is recommended instead to disposal through the joint municipal ecological islands for the expected operations of recycling the materials used.

14. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual.

Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

15. REPAIR**15.1 Warranty repair**

In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location . The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty.

15.2 Repair not covered by warranty

A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

15.3 Non-defective device

The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer.

16. SPARE SPARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the receiving date.

17. EXEMPT CLAUSES

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

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