MOPEDIA®

MA RS940-RS941 02 A_ITA_07 2020

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ITALIANO

INDICE

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Via _____ Località _____

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com**MADE IN P.R.C.**

NOTE

MOPEDIA®

MA RS940-RS941 02 A_ENG_07 2020

SHOWER CHAIR

INSTRUCTION MANUAL

ENGLISH

INDEX

1. CODES.....	PAG.3
2. INTRODUCTION.....	PAG.3
3. INTENDED USE.....	PAG.3
4. DECLARATION OF CONFORMITY	PAG.3
5. GENERAL WARNINGS	PAG.4
6. SYMBOLS	PAG.4
7. GENERAL DESCRIPTION.....	PAG.4
8. BEFORE TO USE.....	PAG.5
9. PRECAUTIONS FOR USE.....	PAG.5
10. MONTAGGIO.....	PAG.5
10.1 Footrest adjustment	pag. 5
11. HOW TO USE.....	PAG.6
11.1 Brakes	pag. 6
11.2 Casters	pag. 6
11.3 Armrest.....	pag. 6
11.4 WC.....	pag. 7
12. MAINTENANCE	PAG.7
13. CLEANING AND SANITISATION	PAG.7
13.1 Cleaning.....	pag. 7
13.2 Disinfection.....	pag. 7
14. CONDITIONS OF DISPOSAL	PAG.7
15. TECHNICAL FEATURES.....	PAG.8
15.1 Dimension* and weight.....	pag. 8
15.2 Technical specifications RS940	pag. 8
15.3 Technical specifications	pag. 8
16. WARRANTY	PAG.8
17. REPAIRING.....	PAG.9
18. SPARE PARTS.....	PAG.9
19. EXEMPT CLAUSES	PAG.9

CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

RS940 - Shower and commode chair

RS941 - Shower and commode chair

2. INTRODUCTION

Thank you for choosing a MOPEDIA Shower chair by Moretti S.p.A. MOPEDIA Shower chair manufactured by Moretti S.p.A. are designed to satisfy all your requirements for a practical, correct and safe use. This manual provides some useful suggestions to correctly use the device you have selected and offers valuable advices for your safety. We recommend carefully reading the entire manual before using the hospital bed you have purchased. Should you have any queries, please contact your retailer for proper advice or assistance.

Note: Check all parts of the product have not been damaged during shipment. In case of damage do not use the product and contact your dealer for further instructions.

3. INTENDED USE

The shower chair MOPEDIA are created for people with movement difficulty and can be used indoors as substitution of toilet or as aid under the shower.

CAUTION!

- Do not use the following device for purposes other than those described in this manual.
- Moretti S.p.A. disclaims any liability for damage resulting from improper use of the device or use other than that described in this manual.
- We reserve the right to make changes to the device and this manual without notice in order to improve the characteristics.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of SHOWER CHAIRS complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

WARNINGS!

Always pay close attention to the presence of moving parts that could cause limb entrapment and injury.

- For proper use of the device refer to this manual.
- For proper use of the device consult your doctor or therapist.
- Keep the packaged product away from any source of heat because the packaging is made of cardboard.
- The useful life of the accessory is determined by wear of parts cannot be repaired and / or replaced.
- Be careful when children are nearby.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Conditions of disposal

7. GENERAL DESCRIPTION

RS940 Shower and commode chair RS941 Shower and commode chair

The Chair is delivered with disassembled backrest and footrest.

8. BEFORE TO USE


- Always check the status of the mechanical parts in order to guarantee they are used in total safety for persons and environment.
- Check the proper assembly of the device with attention to the correct insertion and locking screws and nuts of the backrest and footrest.

9. PRECAUTIONS FOR USE

- Don't attempt inclines greater than 10% without assistance.
- Completely engage the wheels locks when entering and leaving wheelchair or using it in elevator or wheelchair lift.
- Wheelchair should be on a stable level surface and wheels locks should be engaged before transferring user to or from the wheelchair.
- Before leaning or reaching forward, rotate the casters toward the front of the wheelchair and engage wheel locks to help prevent wheelchair from tipping over.
- Do not lean over the top of the wheelchair back since this can cause the wheelchair to tip over.
- Pay attention that the clothes does not get entangled in the wheels of the chair.
- Pay attention to avoid the crushing of fingers during the assembling or the regulation of footrest, backrest or wheels and during the use of the chair.
- Keep the chair far from fire.


10. MONTAGGIO

To assemble the seat insert the tubes in the opening on the back of the seat (Picture 1). The adjustable footrest have to be fixed on the external side of the seat with the pins (Picture 2).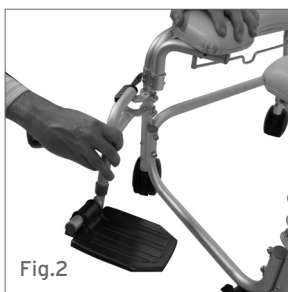

Pay attention to the right footrest to use and note that the footrest have to be fixed inwards. To fix the footrest be sure that the locking lever is well fixed inwards the chair. Insert the WC from the back part of the chair using the rails (Picture 3).

10.1 Footrest adjustment

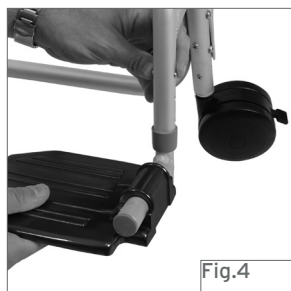The footrest of shower chair can be adjusted according to various requirements. You have to push the locking clip on the footrest (Picture 4) not less than 6.5cm distant from the floor. Check that both footrest are adjusted at the same height

** illustrative purpose images

11. HOW TO USE

Use only the back handle to drive the chair, do not use other supports.

Know your own capabilities and limitations in terms of strength and endurance before attempting to go up a ramp. You and your attendant should first consult a qualified professional for correct techniques on ramp negotiation. Assistance is recommended when going up or down steep inclines greater than 10%. If it becomes necessary to stop on an incline, avoid sudden, abrupt forward movement when you resume propelling the chair.

If you want to lift the chair up, hold it from back handles and frame only.

WARNING! The patient must not put pressure on footrests when go up and down from the chair.

11.1 Brakes

RS940

Fig.5

Engage wheel locks on both wheels before entering the chair, leaving the chair, leaning forward in the chair or while being transported in an elevator or wheelchair lift. On models with small wheels, push the lever on each wheel; (Picture 5) on the other models lift up the lever to brake.

The lever on the brake must tighten the wheel. In case of necessity check the pressure of the casters. If necessary adjust the loosen nut and then tighten up the nut.

RS941

Fig.6

Engage wheel locks on both wheels before entering the chair, leaving the chair, leaning forward in the chair or while being transported in an elevator or wheelchair lift. On models with small wheels, push the lever on each wheel; (Picture 5) on the other models lift up the lever to brake.

The lever on the brake must tighten the wheel. In case of necessity check the pressure of the casters. If necessary adjust the loosen nut and then tighten up the nut.

** illustrative purpose images

11.2 Casters

RS940

Casters should be inspected periodically and lubricated when necessary.

If the wheel wobbles, tighten the nut of the pivot slightly. To test for correct tightness, lean the chair to one side and spin the adjusted wheel.

RS941

Casters should be inspected periodically and lubricated when necessary.

The large rear wheels must also be checked periodically and must not swing in any way.

If the wheel wobbles, tighten the nut of the pivot slightly. To test for correct tightness, lean the chair to one side and spin the adjusted wheel.

11.3 Armrest

To facilitate the transport of the wheelchair you can remove or overturn the armrests. Never use them to move or to lift the chair.

11.4 WC

Use the toilet when the wheelchair is still. To use the toilet take out the removable part of the seat using the handle on the front part. Take out the toilet using the handle (Picture 6).

12. MAINTENANCE

The MOPEDIA by MORETTI chairs need minimal maintenance and general care, but we suggest you to control regularly:

- Braking system: monthly
 - Cleaning parts (seat, backrest, armpad, wheels and so on) if necessary
 - Cleaning the upholstery of the seat as required.
 - General check by qualified personnel every two years.
- It is extremely important that the wheelchair is checked at the first use. In particular, please check that all screws and nuts are tighten enough.

13. CLEANING AND SANITISATION

13.1 Cleaning

To clean the aid and the structure use a solution of alcohol and neutral soap (in equal parts) mixed with water.

WARNING! Do not use abrasive substances, acids, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of metal surfaces.

The manufacture company cannot be held responsible for the damage that is caused by using materials that could damage the product surface or corrosive chemicals during cleaning.

13.2 Disinfection

If you need to disinfect the product use a common disinfectant neutral detergent.

WARNING! Sanitising agents are corrosive. Avoid contact with the eyes and skin. Wear protective clothing, gloves, protective glasses and safety shoes. In case of accidental contact, rinse the affected part thoroughly using warm water and consult a doctor immediately.



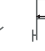
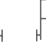
14. CONDITIONS OF DISPOSAL

When disposing of the device never use the normal return systems of municipal solid waste. It is recommended instead to disposal through the joint municipal ecological islands for the expected operations of recycling the materials used.

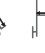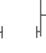15. TECHNICAL FEATURES

15.1 Dimension* and weight

Dimension* and weight														
MEA-SURES IN CM														
RS940	90	59	41	95	73	38/43	50	41	40	57	9,5	10	10	
Dimension* and weight														
MEA-SURES IN CM														
RS941	90	81	41	96	68	38/43	48	41	41	65	14,5	62	20	

*The dimension could be change $\pm 0.5\text{cm}$

15.2 Technical specifications RS940

- Anodized aluminium frame
- PVC padded seat and backrest
- Tip-up armrests
- 4 swivel castors; 2 front castors with brakes
- Swing-away detachable leg-rests, length adjustable
- Removable WC bucket
- Max User weight 110 Kg

15.3 Technical specifications

- Anodized aluminium frame
- PVC padded seat and backrest
- Tip-up armrests
- Swing-away detachable leg-rests, length adjustable
- Removable WC bucket
- Max User weight 110 Kg

16. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual. Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to wrong installation or improper use.

Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, not authorized alteration, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

17. REPAIRING

-Warranty repair-

If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be cover from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

-Repair not cover by warranty-

A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

-Non defective device-

The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be paid by the customer.

18. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

19. EXEMPT CLAUSES

Moretti does not offer any other declaration, explicit or implicit warranty or conditions, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

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