ITALIANO

MA CP900-xx_CP910-xx_02_A_ITA_09-2020

CARROZZINA RELIEF

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INDICE

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CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1 CODES

CP900-40	RELIEF wheelchair- 30 cm wheels - 40 cm seat
CP900-45	RELIEF wheelchair- 30 cm wheels - 45 cm seat
CP910-40	RELIEF wheelchair- 60 cm wheels - 40 cm seat
CP910-45	RELIEF wheelchair- 60 cm - 45 cm seat

2. INTRODUCTION

Thank you for purchasing an **ARDEA ONE RELIEF** wheelchair by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly.

NOTE: check all parts for shipping damages and test before using. In case of damage, do not use the device. Contact your dealer for further instructions.

3. INTENDED USE

This wheelchair is intended for people with mobility problems and can be used for self-propelling or with the aid of an operator. In all cases however it requires the intervention of an operator for the various adjustments you require.

CAUTION!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of MANUAL WHEELCHAIRS COMFY/RELIEF complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

- For correct use of the device, carefully refer to the following manual
- For correct use of the device always consult your doctor or therapist
- Keep the packaged product away from any source of heat as the packaging it is made of cardboard
- The useful life of the device is determined by the wear of non-repairable and / or replaceable parts
- Always pay close attention to the presence of moving parts which could cause limb entrapment and personal injury
- Always pay attention to the presence of children
- Do not lift the wheelchair by removable parts which could accidentally come off during lifting
- The user and / or the patient must report any serious accident occurring in relation to the device to the manufacturer and to the competent authority of the State in which the user and / or the patient is established.

6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Conditions of disposal


Serial Number

7. GENERAL DESCRIPTION

7.1 Parts view

Depending on the model chosen, the composing parts are as follow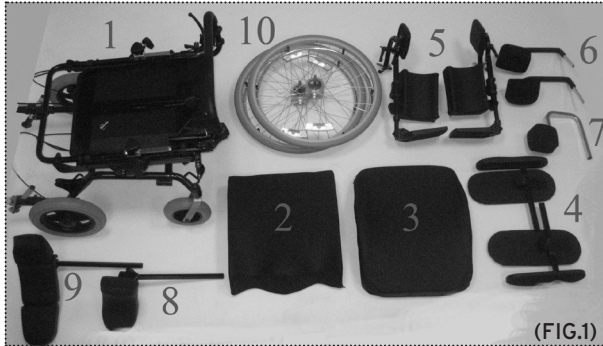


1. Frame
2. Padded seat
3. Padded backrest
4. Armrest and armrest sideboard
5. Elevating legrest
6. Shoulder stopper
7. Hip stopper
8. Headrest
9. Three-dimensional headrest (optional)
10. 60cm back wheels

(FIG.1)

8. ASSEMBLING

8.1 Open the wheelchair

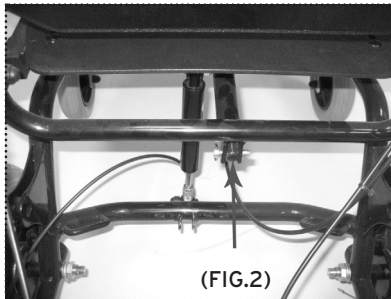(FIG.2)

Lift the backrest and connect the backrest to the piston by the hinge (FIG.2)


8.2 Backrest and seat assembly

(FIG.3)


Lean the padded seat and backrest on the frame, in correspondence to the Velcro strips. Fix them with a bit pressure (FIG.3)

8.3 Armrest assembly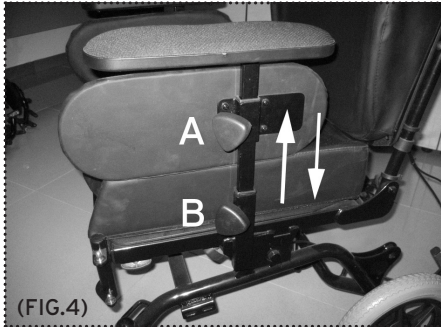

Insert the armrest as shown in (FIG.4) adjust the height as your needs and lock with "A" and "B" knobs.

8.4 Footrest assembly

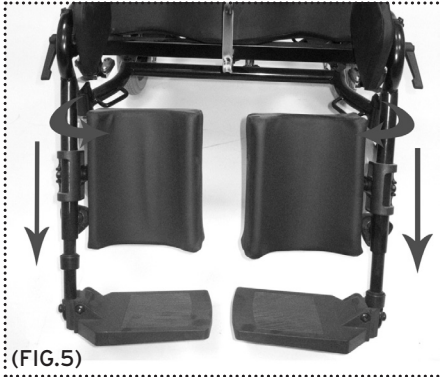Insert the legrest on the hinge on the frame, then rotate inwards until the level are locked (FIG.5)

8.5 Hip stopper assembly


Insert the hip stopper in the tube under the seat. Adjust the position and lock with knobs (FIG.6)

8.6 Shoulder stopper assembly

Insert the shoulder stopper in the supports behind the backrest. Adjust the position and lock with knobs (FIG.7)

8.7 Headrest assembly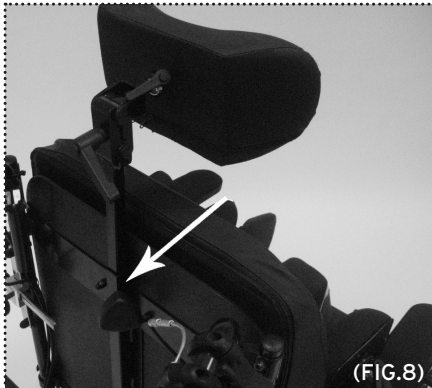

Insert the headrest in the support behind the backrest. Adjust the position and lock with knobs (FIG.8)

9. BEFORE USE

- Check the automotive parts usage, in order to guarantee a safe use of the device for people and things
- Assure all the parts are correctly assembled. Checking carefully the correct assembling of the rear wheels
- Assure the brakes on the back wheel correctly works

10. USING WARNINGS

- Do not use the wheelchair as seat for vehicle
- Assure the brake handles is duly adjusted and correctly works
- Avoid entrapment of finger into wheels or brakes during the wheelchair use
- The wheelchair temperature increase if exposed to the sun for long period

11. HOW TO USE

WARNING!

- Getting in and out of the wheelchair may be performed independently, provided you have an adequate mobility and upper body strength
- Do not stand on removable parts

DO NOT STAND ON THE FOOTREST WHEN GETTING IN OR OUT OF THE WHEELCHAIR, OVERTURNING RISK!!!

11.1 Get in the wheelchair

1. Position the wheelchair as close as possible to you
2. Engage wheel locks . We suggest to remove the armrest to ease the patient getting in the wheelchair
3. Lean on the frame with arms and slowly go down towards the seat

11.2 Get out of the wheelchair

1. Position the wheelchair as close as possible alongside the seat to which you are transferring
2. Engage wheel locks. We suggest to remove the armrest to ease the patient getting out of the wheelchair
3. Shift body weight into seat with transfer

11.3 Stairs and height difference (how to face them)

WARNING!

The following method must be used only from experienced patients. If you are not experienced, let you help from an assistant.

GET ON

Position the wheelchair at the curb to be crossed. Balance the wheelchair on the rear wheels, in order to lift the front wheels, enough to lean them on the obstacle. Bend your trunk forward and vigorously push the handrail. Continue pushing until the rear wheels are climbing over the obstacle.

GET OFF

To get off the stair follow the reverse procedure, positioning the rear side of the wheelchair at the curb, keep the handrails vigorously and get off the obstacle slowly; then balance the wheelchair on the rear wheels in order to complete the descent.

WARNING!

WE RECOMMEND TO USE THIS METHOD ONLY IN PRESENCE OF OBSTACLE LOWER THAN 10CM !!

11.4 Use the armrest

The armrest can be adjusted in height and width:

Adjust the height - unscrew the knob for armrest lock and adjust the height

Adjust the width - unscrew the knob under the seat and adjust the width

11.5 Use the legrest

The legrests can be adjusted in length, inclination, width and depth:

Adjust the length - Unscrew the knob on the legrests frame. Adjust the length of the legrests according to the patient's needs. Lock the knob again.

Adjust the inclination - Unscrew the side knob placed at the knees height and change the inclination. Lock the knob again

Adjust the width - You need a 6 mm allen wrench. Unscrew the screw under the seat and adjust the width according to the patient's needs. Lock the screw again

Adjust the depth - Unscrew both the side knobs placed under the seat, adjust the depth according to the patient's needs and lock the knobs again.

11.6 Use the reclining backrest

Adjust the backrest inclination by pressing the left lever on the handle bar and press the backrest down. Release the backrest to take it back to the usual inclination

11.7 Use the seat

Adjust the inclination of the seat by pressing the right lever on the handle bar and press the seat down. Release the seat to take it back to the usual inclination

WARNING!

These techniques are intended to be used only by experienced users. In contrary case it is required a technical assistance

11.8 Use the sliding seat

The top of the seat can be adjusted in depth to give the patient more comfort. Unscrew the 10mm nuts under the seat and adjust the position onwards or backwards.

11.9 Use the hip stopper

Use the hip stopper to avoid the continuous contact between lower limbs. It can be adjusted by using the knob under the seat.

11.10 Use the rear big wheels

The rear 60cm wheels are used when self-propelled. Grasp the handrail and wheel through the desired direction

11.11 Use the rear small wheels

The rear 30cm casters are used when wheel by an assistant through the handles

11.12 Use the quick release wheels

WARNING!

1. Before removing a wheel make sure that the wheelchair does not incur accidental falls
2. Do not remove a wheel with the patient seated on the wheelchair

The Relief wheelchair is equipped with quick release wheels. To remove the wheel press the button in the centre and extract the wheel outwards. Follow the procedure reversed to position the wheel.

11.13 Use the shoulder stopper

The shoulder stopper can be adjusted in height, length and in orientation:

Adjust the height/width - unloose the knob on the backrest to adjust height and length

Adjust the orientation - in the back of each stopper there are spherical pins which allow to adjust the shoulder stopper by unloosing the knob in the back.

11.14 Use the headrest

The headrest can be adjusted in height, depth and orientation:

Adjust the height - unloose the knob in the back of the headrest

Adjust the depth - unloose the knob in the back of the headrest. The support allow the headrest to be adjusted in depth and inclination.

Adjust the orientation - the back of the headrest is equipped with a spherical pin which allow the headrest to be oriented

11.15 Use the handle bar

The handle bar is intended to wheel the wheelchair or to adjust the backrest and the seat inclination. The handle bar can be adjusted in height by unloosing the knobs on the bar, then lift or lower it.

11.16 Use the anti-tipper castors

Adjust the height by pulling the metallic ring, release it when the correct adjustment is found

11.17 Use the brakes

The back wheels are provided with drum brakes. Press the correspondent lever on the handle bar. Release to wheel the wheelchair.

NOTE THE BRAKES CAN AUTOMATICALLY REMAIN LOCKED. TO USE THIS FEATURE PUSH THE LOCKING ELEMENT UNDER THE BRAKE LEVER. TO UNLOCK THE BRAKE PRESS THE LEVER UNTIL THE LOCKING ELEMENT IS COMPLETELY OUT FROM THE GEAR.

11.18 Carrying

The wheelchair has been designed to be easily carried. Follow the reversed procedure described in "Assembly" chapter. The closed wheelchair can be carry in the car boot.

12. MAINTENANCE

The ARDEA ONE devices by Moretti S.p.A are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to:

- When using the wheelchair for the first time and during the first month, check the fixing screws of the frame and if necessary tighten carefully
- Every 3 months make a check for the condition of all components and the correct ones locking of the screws and nuts of all mechanical parts, especially those concerning the wheels and the braking system

In case of repair, use only original spare parts and accessories.
Follow the suggestions in chapter "BEFORE USE" .

13. CLEANING AND DISINFECTION

13.1 Cleaning Cover

The cover is in artificial leather and they all can be removed. Clean with warm water and mild soap

13.2 Cleaning pad

Clean with warm water and mild soap

13.3 Cleaning frame and metallic parts

Clean with warm water and mild soap. Dry with a soft cloth in order to not scratch the painting.

13.4 Cleaning brakes and wheels

Clean the wheels with warm water and mild soap. Pay attention in drying the wheels, it's dangerous use the wheelchair with wet wheels.

13.5 Disinfection

If you need to disinfect the device use a common disinfectant cleaner

NOTE NEVER USE ACIDS, ALKALIS OR SOLVENTS SUCH AS ACETONE

14. CONDITIONS OF DISPOSAL

14.1 General conditions of disposal

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the wheelchair in the appropriate disposal areas for recycling.

15. SPARE PARTS/ACCESSORIES

For spare parts and accessories refer to main catalogue

16. TECHNICAL FEATURES

16.1 Dimension

Dimensions and weight

Measures in cm																
CP900-40	113	81	45	108	64/77	57	40/60	40	49	57	-	41	30	20	5	120
CP900-45								45		62		43				
CP910-40	113	81	45	108	64/77	57	40/60	40	49	60	-	45	60	20	5	120
CP910-45								45		65		46				

16.2 Technical specifics

• Frame	Powder coated steel
• Cover	Black nylon
• Backrest/seat/headrest pad	Moulded PU
• Armrest/ hip and shoulder stoppers /legrest pad	Foam
• Front wheels	20 cm PU
• Small rear wheels	30 cm PU
• Big rear wheels	60 cm PU
• Rear brakes	Drum brakes
• Seat angle	6° front 16° rear
• Backrest inclination	0-50°
• Max weight	120 kg
• Resistance to ignition	the materials used for cover and padding have been treated with methods specified in Directive EN1021-1/2 EN12183
• Reference regulation	-20°/+40°C
• Environmental temperature	Store in a cool place, away from heat sources, such as sunlight (suggested temperature between +10°C and +40°C)
• Storage	

17. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual.

Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to wrong installation or improper use.

Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, not authorized alteration, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

18. REPAIRING

18.1 Warranty repairing

If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

18.2 Repair not covered by warranty

A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

18.3 Not-defective device

The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be paid by the customer.

19. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

20. EXEMPT CLAUSES

Moretti does not offer any other declaration, explicit or implicit warranty or conditions, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

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