**SKEMA<sup>®</sup>FORM**

ITALIANO

MA MO41X.02 A \_ ITA \_ 09-2020

**PARAVENTI IN METALLO**

---

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## INDICE

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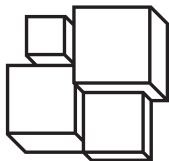
**CE** | Class Medical DeviceREGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL  
of 5 April 2017 concerning medical devices**1. CODES**

- MO411A** Aluminum ward screen with 3 panels
- MO412C** Chromed steel ward screen with 3 panels
- MO414C** Chromed steel ward screen with 4 panels
- MO415A** Aluminum ward screen with 4 panels

**2. INTRODUCTION**

Thank you for choosing a **metal ward screen** from the **SKEMAFORM by MORETTI S.p.A.** Their design and quality are a guarantee of comfort, safety and reliability. The metal ward screens from the **SKEMAFORM by MORETTI S.p.A.** are designed and built to meet all your demands for a practical and correct use. This user manual provides for some suggestions as how to use the item you have chosen correctly and offers a lots of valuable advices for your safety. Please read carefully this manual before using the metal ward screen. Should you have any queries, please contact your retailer for advice and assistance.

**3. INTENDED USE**

The metal ward screen by SKEMAFORM is a device formed by 3 or 4 panels linked together by zippers, intended and used as a mobile or partition wall.

**WARNING!**

- Do not use the product for a purpose not indicated in this manual.
- Moretti S.p.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

**4. DECLARATION OF CONFORMITY**

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of WARD SCREENS complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

**Note:** Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

## 5. GENERAL WARNINGS

For a proper use of the device refer to this manual.

- Keep the packaged product away from any source of heat because the packaging is made of cardboard
- The shelf-life of the accessory is determined by the wear of parts that cannot be repaired and/or replaced.
- Before using the device, always check that the mounting screws are properly tightened.
- Be careful when children are nearby.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

## 6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

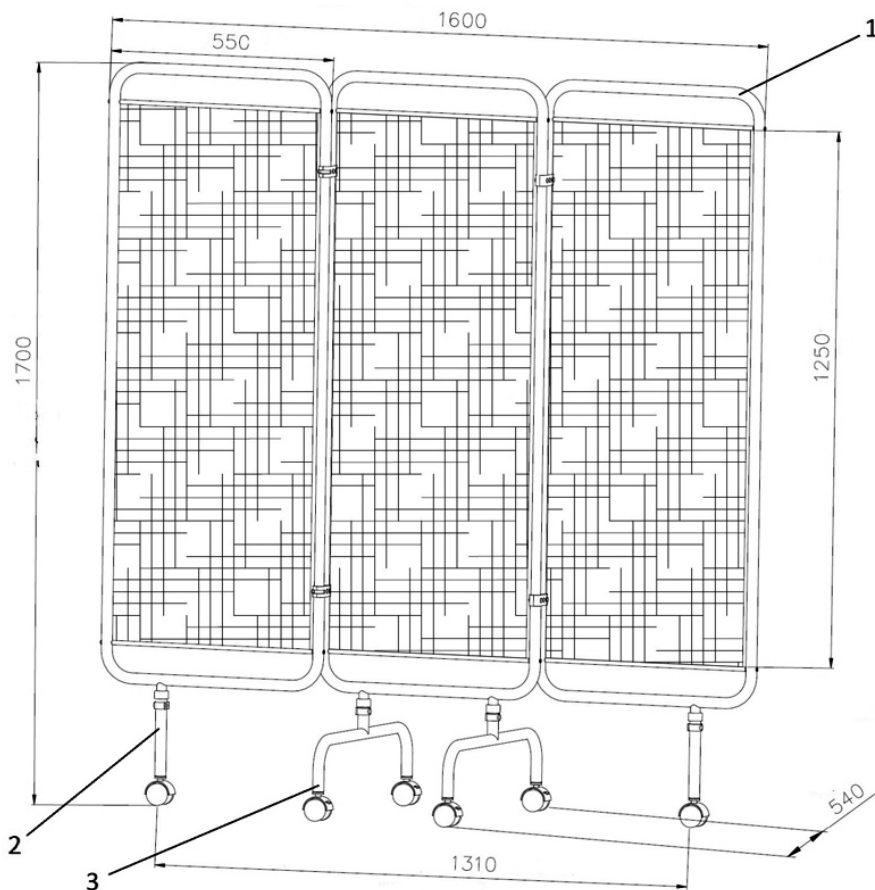
Read the instruction manual

Medical Device

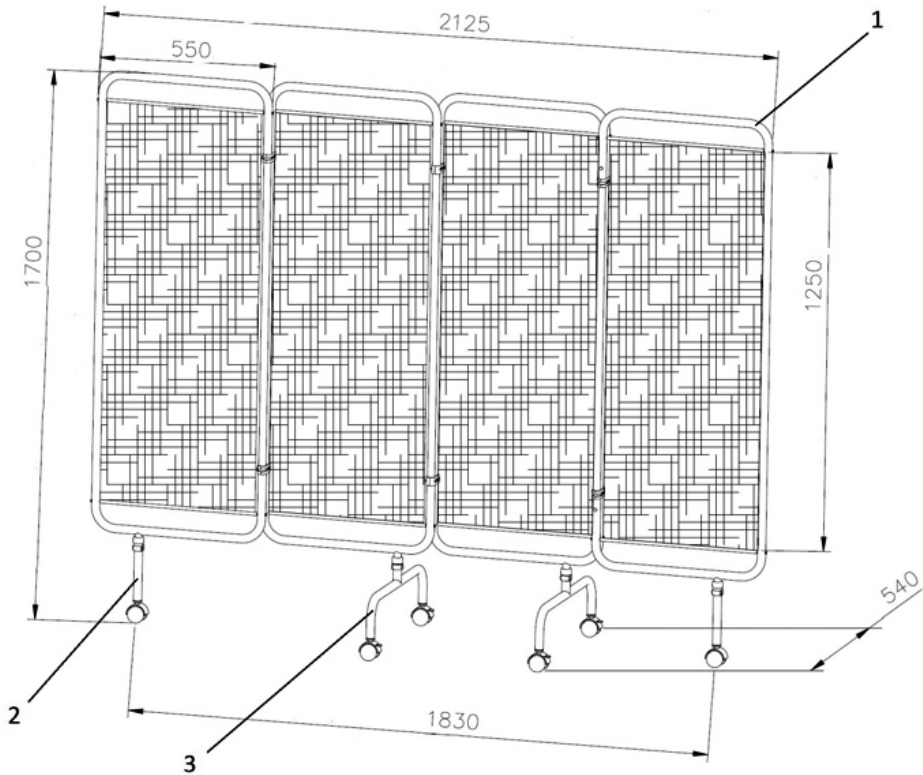
Conditions of disposal

**7. GENERAL DESCRIPTION**
**7.1 Parts view and description MO411A and MO412C**

**List of the parts**

1. Pre-assembled frame with 3 doors (1 pc)
2. Leg with castor (2 pcs)
3. Central leg with swivel castors and brake (2 pcs)

## 7.2 Parts view and description MO414C and MO415A

### List of the parts

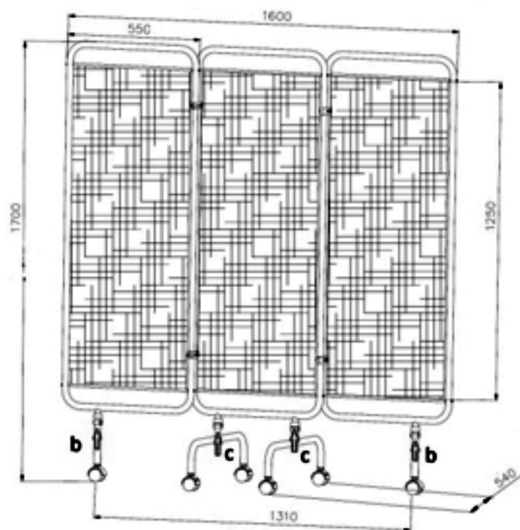
1. Pre-assembled frame with 4 doors (1 pc)
2. Leg with castor wheel (2 pcs)
3. Central leg with swivel castors and brake (2 pcs)

## 8. ASSEMBLY

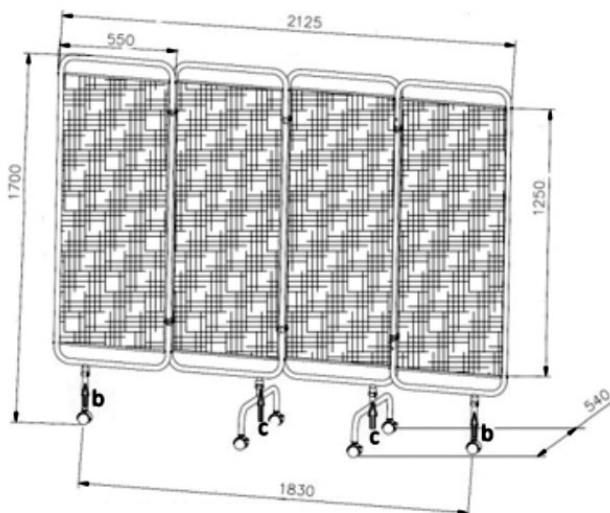
### WARNING!

Perform these operations paying close attention to prevent injury and/or property damage when moving the device

**8.1 Installation instructions for MO411A and MO412C**

- A. Unpack the product.
- B. Insert the legs (n.2) into the special housing on the frame (n.1) as in figure, and fix using the pins provided.
- C. Insert the legs (n.3) in the special housing on the frame (n.1) as shown, and fix using the pins provided.
- D. At this point the screen is mounted and ready for use.

**8.2 Installation instructions for MO414C and MO415A**

- A. Unpack the product.
- B. Insert the legs (n.2) into the special housing on the frame (n.1) as in figure, and fix using the pins provided.
- c. Insert the legs (n.3) in the special housing on the frame (n.1) as shown, and fix using the pins provided.
- D. At this point the ward screen is mounted and ready for use.

## 9. PRECAUTIONS FOR USE

- Before each use always check the state of wear of the mechanical parts to ensure a safe use of the device for persons and things.
- Before each use check the correct assembly of the device, with particular attention to the joints connecting screws, wheels, walls.
- Be careful to entrapment while moving the ward screen.
- When the ward screen is not moving, make sure that the parking brakes are blocked

## 10. MAINTENANCE

The MOPELIA devices by MORETTI are checked carefully and supplied with EC mark once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. In case of repair, use only original spare parts and accessories.

- More specifically, please check that all screws and bolts are well tightened

## 11. CLEANING AND DISINFECTION

### 11.1 Cleaning

To clean the aid and the structure use a solution of alcohol and neutral soap (in equal parts) mixed with water.

#### WARNING!

Do not use abrasive substances, acids, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of metal surfaces

The manufacture company cannot be held responsible for the damage that is caused by using materials that could damage the product surface or corrosive chemicals during cleaning.

### 11.2 Disinfection

If you need to disinfect the product use a common disinfectant neutral detergent.

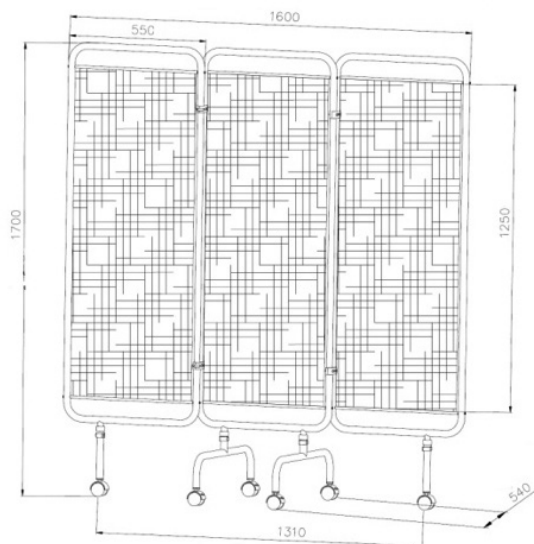
## 12. CONDITIONS OF DISPOSAL

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the kit in the appropriate disposal areas for recycling.

### 13. TECHNICAL FEATURES

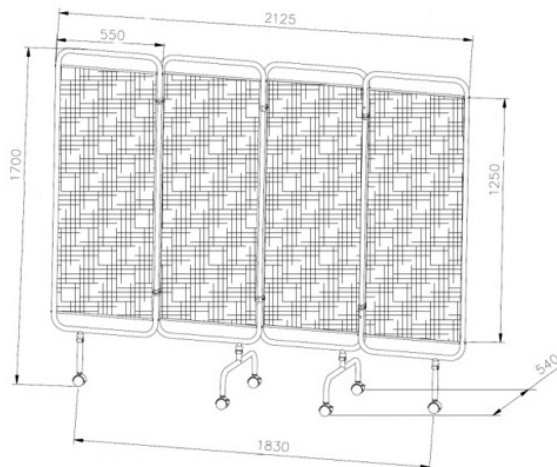
#### 13.1 Dimensions and weight of 3 panels ward screens M0411A and M0412C

- 3 swiveling panels screen in aluminium M0411A or chromed steel M0412C
- Washable and flame retardant PVC curtains
- 2 Pivoting castors  $\varnothing 50\text{mm}$
- 2 Pivoting castors with brakes  $\varnothing 50\text{mm}$

Height: 170 cm  
 Width: 160 cm  
 Depth: 54 cm  
 Weight M0411A: 7,7 kg  
 Weight M0412C: 9,2 kg

#### 13.2 Dimensions and weight of 4 panels ward screens M0414C and M0415A

- 3 swiveling panels screen in aluminium M0415A or chromed steel M0414C
- Washable and flame retardant PVC curtains
- 2 Pivoting castors  $\varnothing 50\text{mm}$
- 2 Pivoting castors with brakes  $\varnothing 50\text{mm}$

Height: 170 cm  
 Width: 212 cm  
 Depth: 54 cm  
 Weight M0414C: 14 kg  
 Weight M0415A: 10,5 kg

#### 14. WARRANTY

Every Moretti product is guaranteed for 2 (two) YEARS from the purchase date, except the restrictions specified below. The warranty shall not apply in case of damage caused by improper use, abuse, alterations of the product or any other use not in compliance with the user manual. The proper use of the product is indicated in the user manual. Moretti S.p.A. shall not be liable for any damages or personal injuries due to incorrect assembly/improper use of the device, not compliant with the user manual. Moretti does not guarantee its products against damages or faults in the following cases: natural disasters, non authorized maintenance or reparations, damages caused by electrical supply problems (if provided), use of parts or components not provided by Moretti, failure in accomplishing the guidelines and instructions for use, non authorized alterations, damages due to the transport (different from Moretti original delivery) or failure in maintaining the device as indicated in the manual. The components subject to wear are not covered by this warranty if the damage is due to the normal use of the product.

#### 15. REPAIRING

##### 15.1 Warranty repair

If a Moretti product shows material or manufacturing faults during the warranty period, Moretti S.p.A., together with the client, will consider whether the product fault is covered by warranty. Moretti S.p.A., at its unquestionable discretion, may replace or repair the item under warranty in a Moretti reseller point or at its own qualified offices. Labour costs for repairing the product may be at Moretti's expense if it is determined the reparation is under warranty. A reparation or replacement does not renew nor extend the warranty.

##### 15.2 Repair not covered by warranty

A product, not covered by warranty, may be returned for reparation only after receiving prior authorization from Moretti client service. Labour and delivery costs for an out-of-warranty reparation will be fully at the client's or reseller's expense. Reparations of out-of-warranty products are guaranteed for 6 (six) months, from the day the client receives the repaired product.

##### 15.3 Non defective device

The client is informed if, after examining and testing a returning product, Moretti decides the product is not defective. The product will be sent back to the client and all the return delivery costs will be at his expense.

#### 16. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the day you receive them.

#### 17. EXEMPT CLAUSES

Except what clearly specified in this warranty and in accordance with the law, Moretti S.p.A. does not offer its clients any other declaration, warranty or condition, expressed or implicit, including any declaration, warranty or condition of merchantability, of non violation or non interference, of suitability for a specific aim. Moretti S.p.A. does not guarantee that the use of Moretti products will be continuous and faultless. The duration of possible implicit warranties that may be applied in accordance with the law is limited to the warranty period. Some states or countries do not allow limitations to the duration of an implicit warranty, or the exoneration, or the limitations of accidental or indirect damages related to products for consumers. In such states or countries some exonerations or limitation of this warranty may not be applied to the user. This warranty is subject to modifications without any prior notice.

**WARRANTY CERTIFICATE**

ENGLISH

**Product** \_\_\_\_\_

**Purchased on (date)** \_\_\_\_\_

**Retailer** \_\_\_\_\_

**Address** \_\_\_\_\_ **Town/city** \_\_\_\_\_

**Sold to** \_\_\_\_\_

**Address** \_\_\_\_\_ **Town/city** \_\_\_\_\_

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

[www.morettispa.com](http://www.morettispa.com) email: [info@morettispa.com](mailto:info@morettispa.com)

**MADE IN P.R.C.**

**MORETTI S.P.A.**

Via Bruxelles, 3 - Meleto  
52022 Cavriglia (Arezzo)

Tel. +39 055 96 21 11  
Fax. +39 055 96 21 200

[www.morettispa.com](http://www.morettispa.com)  
[info@morettispa.com](mailto:info@morettispa.com)