SKEMA®FORM

MA M0743-M0745-M0746 01 A_ITA_10 2020

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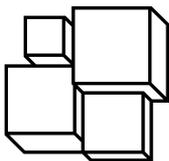
SKEMA[®]FORM

MA M0743-M0745-M0746 01 A_ENG_10 2020

WOODEN PORTABLE EXAMINATION COUCHES

INSTRUCTION MANUAL

ENGLISH

INDEX

1. CODES.....	PAG.3
2. INTRODUCTION.....	PAG.3
3. INTENDED USE.....	PAG.3
4. DECLARATION OF CONFORMITY	PAG.3
5. WARNINGS.....	PAG.4
6. SYMBOLS	PAG.4
7. GENERAL DESCRIPTION.....	PAG.4
7.1 Drawing and description of M0745 parts	pag. 4
7.2 Drawing and description of M0743-M0746 parts	pag. 5
8. ASSEMBLY	PAG.5
8.1 Assembly M0743-M0745-M0746	pag. 5
9. AVAILABLE ACCESSORIES	PAG.5
10. BEFORE EACH USE.....	PAG.5
11. PRELIMINARY CHECK.....	PAG.5
12. INSTRUCTIONS FOR USE FOR THE MODELS M0743-M0746.....	PAG.5
13. MAINTENANCE	PAG.6
14. CLEANING AND DISINFECTION	PAG.6
14.1 Cleaning.....	pag. 6
14.2 Disinfection	pag. 6
15. CONDITIONS OF DISPOSAL	PAG.6
16. TECHNICAL SPECIFICATIONS.....	PAG.6
16.1 Material specifications for all models	pag. 6
16.2 Dimensions and weight of M0743	pag. 6
16.3 Dimensions and weight of M0745	pag. 6
16.4 Dimensions and weight of M0746	pag. 6
17. WARRANTY	PAG.7
18. REPAIRS	PAG.7
18.1 Repairs under guarantee.....	pag. 7
18.2 Repair of a product no longer covered by guarantee	pag. 7
18.3 Non-defective products	pag. 7
19. SPARE PARTS	PAG.7
20. DISCLAIMER CLAUSES	PAG.7

CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

M0743	Polaris adjustable wooden portable couch with backrest
M0745	Polaris adjustable wooden portable couch - 70 cm
M0746	Polaris adjustable wooden portable couch with backrest - 70 cm

2. INTRODUCTION

Thank you for choosing an examination couch from the SKEMA range by Moretti. Moretti examination couches have been designed and manufactured to meet all your needs. They are practical and safe to use. This manual contains instructions to help you to get the most from the medical device you have chosen and also valuable safety advice. Please read carefully this manual before you use the examination couch. If you require any further information or advice, contact your reseller. They will be glad to help.

3. INTENDED USE

Moretti examination couches have been designed for short physiotherapy and examination sessions.

CAUTION!

- Using this device for purposes other than those defined in this manual is strictly forbidden
- Moretti S.p.A. declines all liability for damage or injury caused by improper use of the device or for uses other than those specified in this manual
- The manufacturer operates a policy of continuous improvement and reserves the right to make changes to the device and this manual without any prior notice

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of EXAMINATION COUCHES - SKEMA complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that

Moretti S.p.A. releases and makes available through its channels.

5. WARNINGS

Please read this manual. It contains important information and instructions on how to use and maintain this device correctly


- Keep the packaged product away from sources of heat. The packaging is made of cardboard and is flammable
- The useful life of the device depends on the wear of its non-replaceable and non-repairable parts and is in any case no longer than ten years
- Pay particular attention to moving parts. Limbs may become trapped in moving parts resulting in personal injury
- Take particular care when children are present
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

	Product code
	Unique Device Identification
	CE mark
	Manufacturer
	Batch Lot
	Read the instruction manual
	Medical Device
	Conditions of disposal

7. GENERAL DESCRIPTION

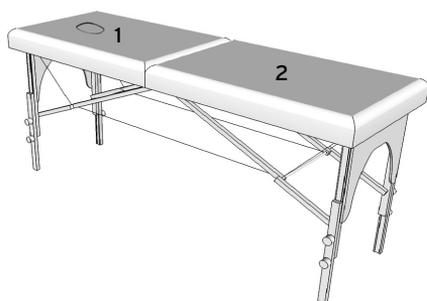
7.1 Drawing and description of M0745 parts

List of parts

1. Padded backrest
2. Padded couch

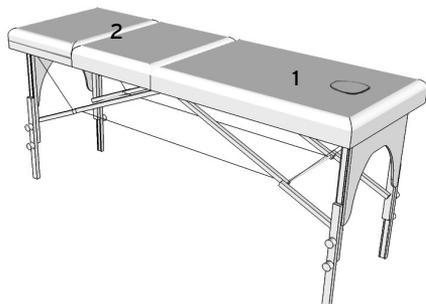7.2 Drawing and description of M0743-M0746 parts

List of parts

1. Padded backrest
2. Padded couch

8. ASSEMBLY

CAUTION!

Perform these operations with the help of another person. Take precautions to prevent injury or damage when moving the couch

8.1 Assembly M0743-M0745-M0746

All the foldable couches are delivered assembled. For the adjustable models: in order to adjust the height, unscrew the two control knobs and pull out the foot. Place the leg at the desired height and screw the knobs. Repeat the same procedure for the other three feet, making sure the four of them have the same height

9. AVAILABLE ACCESSORIES

M0755: Transport bag (except M0745-M0746)

M0750: Side arms (except M0746-M0746)

10. BEFORE EACH USE

- Check the mechanical parts for wear. Check that the device can be used in perfect safety
- Check that the device has been correctly assembled. Pay particular attention to the pins and stops, locking the backrest and the mechanical joints in general

11. PRELIMINARY CHECK

- Before you make any adjustments to the couch, ensure that it is in a stable position on its feet.
- Avoid contact with massage creams and oils. Always place a sheet on the couch before every use

12. INSTRUCTIONS FOR USE FOR THE MODELS M0743-M0746

The couch backrest can be adjusted to angles from 0° to 45°. Raise the backrest and fix it at the required angle, using the metallic adjuster stay

CAUTION!

Before you position a patient on the couch, check that the backrest and the knobs on the feet are safely locked in position

13. MAINTENANCE

The SKEMA series devices by Moretti are thoroughly checked before they leave our production facility and have the CE mark. For the safety of the patient and physician we recommend that you check the suitability of your product for its intended use at least once a year. Regularly inspect the following:

- Load bearing structure of the couch.

For repairs, use only original spare parts and accessories

14. CLEANING AND DISINFECTION

14.1 Cleaning

Use a damp cloth and neutral soap only. Dry thoroughly before use. Use water with a temperature not exceeding 30 °C.

14.2 Disinfection

When necessary, disinfect the product with a standard household disinfectant.

N.B. Do not use acid or alkaline detergents. Do not use solvents (e.g. acetone, diluent).

15. CONDITIONS OF DISPOSAL

When disposing of the device at the end of its life cycle, do not use the usual systems for the disposal of solid urban refuse. Dispose of the device at a municipal waste separation and recycling centre

16. TECHNICAL SPECIFICATIONS

16.1 Material specifications for all models

Frame material:	Wood
Padding:	Polyurethane density 25 Height 4 cm (M0745-M0746) Height 3 cm (all the other models)
Upholstery:	Washable, class 1IM fire retardant leatherette
Painting:	Water-based paints

16.2 Dimensions and weight of M0743

Dimensions, opened:	L: 1820 (2100 with headrest) W: 600 (830 with armrests) H: 600+800 mm
Dimensions, closed:	L: 910 W: 200 H: 600 mm
Weight:	14 Kg
Max. load bearing capacity:	120 Kg

16.3 Dimensions and weight of M0745

Dimensions, opened:	L: 1820 (2100 with headrest) W: 700 H: 600+800 mm
Dimensions, closed:	L: 910 W: 200 H: 700 mm
Weight:	13 Kg
Max. load bearing capacity:	120 Kg

16.4 Dimensions and weight of M0746

Dimensions, opened:	L: 1820 (2100 with headrest) W: 700 H: 600+800 mm
Dimensions, closed:	L: 910 W: 210 H: 700 mm
Weight:	16 Kg
Max. load bearing capacity:	120 Kg

17. WARRANTY

All Moretti products are guaranteed against material and manufacturing defects for a period of two (2) years from the date of sale of the product, subject to any exclusions or limitations specified below. This guarantee becomes invalid in the event of improper use, misuse or modification of the product or failure to follow the use instructions in this manual. The intended use of this product is specified in the instruction manual. Moretti declines all liability for damage or personal injury caused by failure to follow the instructions in the installation, assembly and use manuals. Moretti does not guarantee Moretti products against damage or defects in the following circumstances: natural disasters; unauthorised maintenance or repairs; damage deriving from electrical power supply problems (where present); use of parts or components not supplied by Moretti; failure to follow guidelines and operating instructions; unauthorised modifications; shipping damage (other than original Moretti shipping); failure to follow the maintenance instructions in this manual. This guarantee does not cover wear components if the damage can be attributed to normal use of the product.

18. REPAIRS

18.1 Repairs under guarantee

In the event that a Moretti product appears to have material or manufacturing defects during the guarantee period, Moretti will assess together with the customer if the product defect is covered under guarantee. Moretti can decide, at its sole discretion, to substitute or repair the item under guarantee at the premises of a specified Moretti reseller or at its own premises. Labour costs for the product repair can be charged to Moretti if the repair comes within the terms of the guarantee conditions. Repairs or substitutions do not renew or extend the guarantee.

18.2 Repair of a product no longer covered by guarantee

You may return a product no longer under guarantee for repair to Moretti provided that you obtain an authorisation to do so from Moretti Customer Assistance Service beforehand. Labour and shipping costs for repairs not covered by guarantee will be invoiced in full to the customer or reseller concerned. Repairs to products not covered by guarantee are guaranteed for six (6) months from the day on which the repaired product was received.

18.3 Non-defective products

The customer will be duly informed if Moretti, after examining and testing a returned product, finds that the product is not defective. The product will be returned to the customer and the customer will be billed for the costs of the return.

19. SPARE PARTS

Moretti original spare parts are guaranteed for six (6) from the day on which the spare part was received

20. DISCLAIMER CLAUSES

Unless otherwise expressly specified in this guarantee and within the limits of the law, Moretti offers no further statements, warranties or guarantees, expressed or implied, including any statements regarding warranties and conditions of merchantability, fitness for particular use, non violations and non interference. Moretti does not guarantee that the use of a Moretti product will be uninterrupted or error-free.

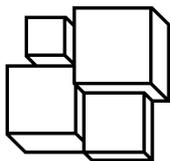
The duration of any implied guarantees which may be implied by law is limited to the guarantee period within the limits of the law. Some states or countries do not allow limitation on how long an implied warranty lasts and some do not allow exclusions or limitations of incidental or consequential damages in relation to consumer products. In some states and countries, some exclusions and limitations to this guarantee do not apply to the user. This guarantee is subject to variations without prior notice.

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