MA ST745_ST746 03 A_ITA_05-2021

GUANCIALI IN POLIURETANO ESPANSO

MANUALE D'ISTRUZIONE

INDICE

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Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com**MADE IN ITALY**

LEVITAS®

MA ST745_ST746 03 A_ENG_05-2021

POLYURETHANE FOAM PILLOWS
INSTRUCTION MANUAL

ENGLISH

INDEX

1. CODES.....	PAG.3
2. INTRODUCTION.....	PAG.3
3. INTENDED USE.....	PAG.3
4. DECLARATION OF CONFORMITY	PAG.3
5. GENERAL WARNINGS	PAG.4
6. SYMBOLS	PAG.4
6.1 Washing symbols	pag. 4
7. GENERAL DESCRIPTION	PAG.4
8. CLEANING.....	PAG.5
9. CONDITIONS OF DISPOSAL	PAG.5
10. TECHNICAL FEATURES	PAG.5
11. WARRANTY	PAG.5
11.1 Repairs	pag. 5
11.2 Spare parts	pag. 6
11.3 Exempt clauses	pag. 6

CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

ST745 Cushion with fireproof Trevira cover

ST746 Cushion with fireproof Politex cover

2. INTRODUCTION

Thank you for choosing a LEVITAS polyurethane pillow by Moretti.

Moretti polyurethane pillow has been designed and manufactured to satisfy all your needs for practical, correct and safe use. This manual contains useful suggestions for using your device properly and safely. Please read this entire manual carefully before you use the mattress.

Should you have any queries, please contact your dealer for suitable advice and assistance.

3. INTENDED USE

CAUTION!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of ANTIDECUBITUS MATTRESSES AND CUSHIONS complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

- For proper use of the device please refer to the following manual carefully
- Always consult your doctor or therapist for proper use of the device
- We recommend the use to one user only
- The device must be sanitized before re-use
- The service life of the device is determined by the wear of parts that cannot be repaired and/or replaced
- Store and use the device away from naked flames and heat sources
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to

6. SYMBOLS

	Product code
	Unique Device Identification
	CE mark
	Manufacturer
	Batch Lot
	Read the instruction manual
	Medical Device
	Conditions of disposal

6.1 Washing symbols

	Maximum washing temperature 60°C; normal process
	Do not bleach
	Do not dry clean
	Do not tumble dry
	Do not iron

7. GENERAL DESCRIPTION

Pillows are made in expanded polyurethane with open cells. Pillows are with holes to prevent asphyxiation, are anti-fungal, breathable, non-deformable, antibacterial.

8. CLEANING

Inside part :

We recommend washing with water and detergents. The wrinkling can be performed both with mangane both in low speed centrifuge. Drying can take place in a room, not sunny, with air currents or with air circulation stove (maximum temperature 70 ° C.).

Autoclave treatment with saturated water vapour at max. temperature of 106 pieces C. for maximum ten minutes.

External part/cover :

ST745 Wash in water at temperatures below 95 ° C. Does not bear the treatment with chlorine, do not tumble dry, do not iron, do not dry clean.

ST746 Washable in water at a maximum temperature of 60 ° C. Do not iron.

The device must be sanitized before it is used by another patient.

Follow the symbols on the label. The meaning of each symbol is given in paragraph 6.1

9. CONDITIONS OF DISPOSAL

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the bath chair in the appropriate disposal areas for recycling.

10. TECHNICAL FEATURES

Dimension and weight:

ST745 70 x 40 x 9 h cm - 500 g

ST746 70 x 40 x 9 h cm - 500 g

Density: 21 Kg/m³

11. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed.

The correct intended use of the device is specified in this manual. Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use.

Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair , damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

11.1 Repairs

Warranty repairs In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location . The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty

Repair not covered by warranty A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

Non-defective device The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer .

11.2 Spare parts

The original Moretti spare parts are guaranteed for 6 (six) months from the receiving date.

11.3 Exempt clauses

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

WARRANTY CERTIFICATE

ENGLISH

Product _____

Purchased on (date) _____

Retailer _____

Address _____ **Town/city** _____

Sold to _____

Address _____ **Town/city** _____

MORETTI S.P.A.

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN ITALY

MORETTI S.P.A.

Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)

Tel. +39 055 96 21 11
Fax. +39 055 96 21 200

www.morettispa.com
info@morettispa.com