MORETTI S.p.A.
Via Bruxelles, 3 - Meleto
52022 Cavriglia (AR) Italy

Tel. + 39 055 96 21 11
Fax. + 39 055 96 21 200
www.morettispa.com
info@morettispa.com

Part. IVA 00306090515
Meccanografico AR 005370
Reg. Imp. Arezzo 00306090515

R.E.A. Arezzo 68869
Cap. Soc. € 1.500.000,00
C/C Postale 13682521

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Larghezza: 65 cm

Lunghezza: 80 cm

Scooter Accessories Assembly and Use Manual

Codes:

CNA106 Rear storage basket
CNA107 Rear storage basket, large
CNA110 Tank holder
CNA116 Stick holder
CNA117 Double cane holder
CNA118 Crutch holder
CNA122 Walker holder
CNA145 Multi-attachment 4 holes
CNA150 Sun cover for scooter

WARNING!

- It is forbidden to use the following device for purposes other than those defined in the following manual.
- Moretti S.p.A. declines all liability for damage resulting from improper use of the device or from use other than that indicated in this manual.
- The manufacturer reserves the right to make changes to the device and the following manual without prior notice in order to improve its features, please consult our website for the latest available version of the manual

WARNING!

The maximum weight allowed to be applied on any support is 10kg

Assembly CNA106 CNA107

- 1 Remove the cap from the bracket
- 2 Securely fasten the bracket to the structure
- 3 Insert the locking bar into the basket
- 4 Insert pins and tighten knob
- 5 Insert the tube into the holder, screw the knob into the hole and tighten firmly.

Images for illustration purposes only

Assembly CNA110 CNA116 CNA117 CNA118

- 1 Remove the cap from the bracket
- 2 Screw the support firmly onto the frame
- 3 Insert the tube into the bracket, screw the knob into the hole and tighten firmly

Montaggio CNA122

- 1 Remove the support cap
- 2 Securely fasten the bracket to the structure
- 3 Insert tubes A into support B and tighten the knobs
- 4 Insert the assembled frame into support C and tighten the knobs
- 5 Insert support E on top
- 6 Insert fitting D on top and tighten knob
- 7 Insert hooks F on both sides of the fitting and tighten the knobs
- 8 Insert tubing into the holder and screw the knob into the hole and tighten firmly
- 9 Always use the Velcro strap to secure the walker to the frame during transport

Images for illustration purposes only

Assembly CNA145

- 1 Remove the support cap
- 2 Screw the support firmly onto the frame
- 3 Insert tubing into the support, screw the knob into the hole and tighten firmly
- 4 Insert the accessories into the holes and tighten the knobs firmly

Images for illustration purposes only

ATTENTION!

Accessories such as baskets and sun cover must always be placed in the central position

Assembly CNA150

Images for illustration purposes only

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