MOPEDIA®

MA RS972xx 03 A_ITA_07 2020

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ITALIANO

INDICE

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**MORETTI S.P.A.**

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com**MADE IN P.R.C.**

MOPEDIA®

MA RS972xx 03_A_ENG_07 2020

BATH HANDLES

INSTRUCTION MANUAL

ENGLISH

INDICE

1. CODES..... PAG.3

2. INTRODUCTION..... PAG.3

3. INTENDED USE..... PAG.3

4. DECLARATION OF CONFORMITY PAG.3

5. GENERAL WARNINGS PAG.4

 5.1 General warnings pag. 4

 5.2 Warnings for use..... pag. 4

6. SYMBOLS PAG.4

7. BEFORE USE pag.4

8. ASSEMBLING..... PAG.4

9. HOW TO USE PAG.5

10. TECHNICAL FEATURES..... PAG.5

11. CLEANING AND DISINFECTION PAG.5

 11.1 Cleaning..... pag. 5

 11.2 Disinfection pag. 5

12. CONDITIONS OF DISPOSAL PAG.5

14. WARRANTY PAG.6

15. REPAIR..... PAG.6

16. SPARE PARTS PAG.6

17. EXEMPT CLAUSES..... PAG.6

ENGLISH

CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL of 5 April 2017 concerning medical devices

1. CODES

RS972-30 Ø 36 mm - bath handle 30 cm

RS972-45 Ø 36 mm - bath handle 45 cm

RS972-60 Ø 36 mm - bath handle 60 cm

2. INTRODUCTION

Thank you for purchasing a MOPEDIA bath handle by Moretti S.p.A. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly.

NOTE Check if any part has shipping damages and test before using. In case of damage, do not use the device. Contact your dealer for further instructions.

WARNING!

- Do not use the product for a purpose not indicated in this manual.
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer reserves the right to change the information contained in this document without previous notice.

3. INTENDED USE

The MOPEDIA safety bath handle by Moretti is a bathroom aid for people with motor difficulty.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of SAFETY BATH HANDLES complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

5.1 General warnings

- DO NOT use this product or any available optional equipment without first completely reading and understanding this instruction manual.
- Keep the packed product away from heat sources
- The bath handle is not designed to hold or support your full body weight.
- This handle bar is designed to assist your natural motion, as when stepping into or out of a shower.
- Do not attach to moving surfaces.
- DO NOT allow children to play on or operate the bath chair.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

5.2 Warnings for use

For a correct use please make reference to the following manual. Pay attention to the presence of children.

6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Conditions of disposal

7. BEFORE USE

- Check if the frame is damaged in order to guarantee a safe use for people and things.
- Check the correct assembly of the device.

8. ASSEMBLING

- Choose a solid and safe wall to install the handle.
- Make sure that it is possible to drill the wall safely for people and things.
- Make sure to position the handle bar at the most suitable height for users.
- Lift the flange cover to show the hole screw.
- Place the handle up to the wall in the desired position and mark the screw hole position.

- Drill the wall and push correct sized plugs for the screw you are using into the wall.
- Place the handle up to the wall, and drive the screws
- Tighten well the screws.
- Finally, clip the screw cover over the flanges. This will give to your handle an elegant appearance and hide the screw head.

Note: Stainless steel screws are recommended to reduce the risk of rust.

9. HOW TO USE

The safety bath handle if horizontally placed becomes an efficient foothold for movement such as sit or stand up from wc.

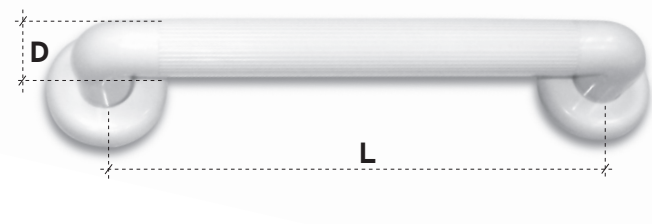
WARNING!

Don't bear down on the handle with whole body weight, just use the handle as support for movement.

10. TECHNICAL FEATURES

MATERIAL: the grab is composed by PVC and the end parts in ABS

Dimension

D = diameter
L = length

RS972-
30 diameter 36 mm -
length 30 cm

RS972-45 diameter 36 mm - length 45 cm
RS972-60 diameter 36 mm - length 60 cm

11. CLEANING AND DISINFECTION

11.1 Cleaning

Use a damp cloth and mild soap. Then dry thoroughly before use.

11.2 Disinfection

If you need to disinfect the device use a common disinfectant cleaner.

Note: Never use acids, alkalis or solvents such as acetone.

12. CONDITIONS OF DISPOSAL

In case of disposal don't use the inserting container for municipal waste. It is recommended to dispose of the device through the municipal sorted waste collection centre for the planned recycling operations of the materials used.

13. SPARE PARTS AND ACCESSORIES

For spare parts only refer to MORETTI master catalogue.

14. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual. Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

15. REPAIR

Warranty repair

In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location. The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty.

Repair not covered by warranty

A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

Non defective devices

The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer.

16. SPARE PARTS

For spare parts only refer to MORETTI master catalogue.

17. EXEMPT CLAUSES

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

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