KYARA®

MA ST191 01 A_ITA_11-2020

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ITALIANO

INDICE

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
**MORETTI S.P.A.**

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: **info@morettispa.com****MADE IN P.R.C.**

KYARA®

MA ST191 01 A_ENG_11-2020

ICE BAG

INSTRUCTION MANUAL

ENGLISH

INDEX

1. CODE	PAG.3
2. INTRODUCTION.....	PAG.3
3. INTENDED USE.....	PAG.3
4. DECLARATION OF CONFORMITY	PAG.3
5. GENERAL WARNINGS	PAG.4
6. SYMBOLS	PAG.4
7. DESCRIPTION AND COMPONENTS.....	PAG.4
8. BEFORE USE	PAG.5
9. INSTRUCTIONS FOR USE	PAG.5
10. HOW TO USE.....	PAG.5
11. MAINTENANCE AND STORAGE	PAG.5
11.1 Maintenance	pag. 5
11.2 Storage	pag. 5
12. CLEANING AND DISINFECTION.....	PAG.5
12.1 Cleaning	pag. 5
12.2 Disinfection	pag. 5
13. CONDITIONS OF DISPOSAL	PAG.6
14. TECHNICAL FEATURES.....	PAG.6
15. WARRANTY	PAG.6
15.1 Exempt clauses.....	pag. 6

CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL

of 5 April 2017 concerning medical devices

1. CODE

ST191 ICE BAG 28 CM

2. INTRODUCTION

Thank you for purchasing the ice bag Kyara by Moretti S.p.A. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance.

In case of doubts, we recommend to contact the dealer who can help and advice you properly.

NOTE Check all parts for shipping damages and test before using. In case of damage, do not use the device. Contact your dealer for further instructions.

3. INTENDED USE

The ice bag Kyara is intended to cool parts of the human body.

CAUTION!

- Do not use the product for a purpose not indicated in this manual.
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of PRODUCTS FOR PERSONAL CARE complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation

6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

- For a correct use of the device, refer to this manual
- Make sure that the product is delivered complete and keep it packed away from sources of heat
- The useful life of the device is determined by wear of parts can not be repaired and / or replaced
- Always pay attention to the presence of children.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

- | | |
|--|------------------------------|
|  | Product code |
|  | Unique Device Identification |
|  | CE mark |
|  | Manufacturer |
|  | Batch Lot |
|  | Read the instruction manual |
|  | Medical Device |
|  | Conditions of disposal |

7. DESCRIPTION AND COMPONENTS

The ice bag is realized in durable fabric equipped with screw cap, rubber seal in the cap. Inside in PVC which guarantees the preservation of ice for long time. Size 28 cm diameter

8. BEFORE USE

Always check the state of wear of the parts in such a way as to ensure it is used in complete safety for persons and property

9. INSTRUCTIONS FOR USE

- Never use the bag in a manner different from that stated
- The Moretti Spa will not be liable for any purpose other than that stated or intended for a use different from that reported
- Use only on intact skin and not on injuries or lacerations, in any case always use a protective cloth between the skin and the bag.
- Use only with ice or ice water
- In the case of use by children always keep under control the bag to prevent accidental opening

10. HOW TO USE

Before use check that the surface of the bag is not present injuries, lacerations or wear likely to impair the adhesive strength of the liquid. If you can make a washing the of the bag. Fill the bag with ice. Close the bag with the cap included taking care to drain the excess air from the inside. Check the correct closing controlling spills. Place the bag on the chosen body part using a soft cloth between the skin and the bag.

11. MAINTENANCE AND STORAGE

11.1 Maintenance

The Kyara ice bags by MORETTI are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. In case of repair, use only original spare parts and accessories.

11.2 Storage

For longer storage of the product we recommend placing the bag completely dry away from light and heat sources within sprinkling a bit talcum powder.

12. CLEANING AND DISINFECTION

12.1 Cleaning

For cleaning both inside and outside of the bag is recommended to use warm water and a neutral detergent. Allow to dry naturally. Do not expose to direct sunlight or sources of artificial heat

12.2 Disinfection

If you need to use to disinfect the products a common disinfectant cleaner

NOTE Never use acids, alkalis or solvents such as acetone or thinner

13. CONDITIONS OF DISPOSAL

When disposing of the device never use the normal systems of provision of municipal solid waste. It is recommended to dispose of the device at the local recycling center.

14. TECHNICAL FEATURES

Material:	Inside PVC
	Outside Nylon
Size:	28 cm diameter

15. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and for the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual.

Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to mistaken installation or improper use.

Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, alteration not authorized, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

15.1 Exempt clauses

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
Moretti does not offer any other declarations, warranty or conditions, explicit or implicit, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

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MORETTI S.p.A.