ITALIANO

MA CPA600\_CPA605 02 A ITA\_11-2020

## **PORTABOMBOLA E ASTA FLEBO**

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## INDICE

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**www.morettispa.com email: info@morettispa.com**

**MADE IN P.R.C.**

## This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

**NOTE**

ENGLISH

MA CPA600\_CPA605 02 A\_ENG\_11-2020

**OXYGEN BOTTLE HOLDER AND I.V.POLE**

---

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## INDEX

<b>1. CODES.....</b>	<b>PAG.3</b>
<b>2. INTRODUCTION.....</b>	<b>PAG.3</b>
<b>3. INTENDED USE.....</b>	<b>PAG.3</b>
<b>4. DECLARATION OF CONFORMITY .....</b>	<b>PAG.3</b>
4.1 Standards and guidelines reference.....	pag. 4
<b>5. GENERAL WARNINGS .....</b>	<b>PAG.4</b>
<b>6. SYMBOLS .....</b>	<b>PAG.4</b>
<b>7. GENERAL DESCRIPTION.....</b>	<b>PAG.5</b>
7.1 View and List of Components CPA600 - I.V. pole.....	pag. 5
7.2 View and List of Components CPA605 - Metal Oxygen bottle holder .....	pag. 5
<b>8. ASSEMBLING/DISASSEMBLING .....</b>	<b>PAG.6</b>
8.1 Assembling CPA600 - I.V. pole.....	pag. 6
8.2 Assembling CPA605 - Metal Oxygen bottle holder.....	pag. 6
8.3 Disassembling.....	pag. 6
<b>9. BEFORE USE.....</b>	<b>PAG.6</b>
<b>10. PRECAUTIONS FOR USE .....</b>	<b>PAG.6</b>
<b>11. HOW TO USE.....</b>	<b>PAG.6</b>
<b>12. MAINTENANCE.....</b>	<b>PAG.7</b>
<b>13. CLEANING AND DISINFECTION .....</b>	<b>PAG.7</b>
13.1 Cleaning.....	pag. 7
13.2 Disinfection.....	pag. 7
<b>14. CONDITIONS OF DISPOSAL .....</b>	<b>PAG.7</b>
<b>15. SPARE PARTS .....</b>	<b>PAG.7</b>
<b>16. TECHNICAL FEATURES.....</b>	<b>PAG.7</b>
16.1 Dimension .....	pag. 7
16.2 Technical specifications.....	pag. 7
<b>17. WARRANTY .....</b>	<b>PAG.8</b>
<b>18. REPAIRING .....</b>	<b>PAG.8</b>
<b>19. SPARE PARTS .....</b>	<b>PAG.8</b>
<b>20. EXEMPT CLAUSES .....</b>	<b>PAG.8</b>

**CE** I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL  
of 5 April 2017 concerning medical devices

**1. CODES****CPA600** I.V. pole**CPA605** Metal Oxygen bottle holder**2. INTRODUCTION**

Thank you for choosing an accessories manufacturer by MORETTI S.p.A. Their design and quality are a guarantee of comfort, safety and reliability. The accessory by MORETTI S.p.A. is designed and built to meet all your demands for a practical, correct use. This user manual provides some suggestions to correctly use the accessory you have chosen and offers a lot of valuable advice for your safety. Please read carefully through the manual before using the accessory. Should you have any queries, please contact your retailer for advice and assistance.

**3. INTENDED USE**

The **Oxygen bottle holder** and **I.V. pole** is primarily intended for people who have mobility and respiratory problems. However they are adaptable both in hospitals and home.

**NOTE:** Check that the product has not been damaged during shipment. In case of damage do not use it and contact your dealer for further instructions.

**WARNING!**

- Do not use the product for a purpose not indicated in this manual.
- Moretti S.p.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

**4. DECLARATION OF CONFORMITY**

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of ACCESSORIES FOR WHEELCHAIRS complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

**Note:** Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

#### 4.1 Standards and guidelines reference

Safety tests required by UNI EN 12182

### 5. GENERAL WARNINGS

#### WARNING!

Always pay close attention to the presence of moving parts that could cause limb entrapment and injury.

- For proper use of the device refer to this manual.
- For proper use of the device consult your doctor or therapist.
- Keep the packaged product away from any source of heat because the packaging is made of cardboard.
- The shelf-life of the accessory is determined by the wear of parts that cannot be repaired and / or replaced.
- Be careful when children are nearby.
- Do not exceed the maximum payload.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

### 6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

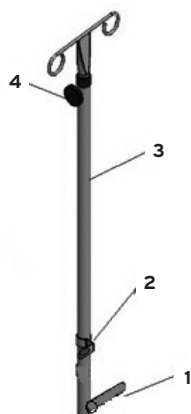
Conditions of disposal

## 7. GENERAL DESCRIPTION


### 7.1 View and List of Components CPA600 - I.V. pole

1. Fixing tube
2. Fixing supports
3. Telescopic tube
4. Knob

### 7.2 View and List of Components CPA605 - Metal Oxygen bottle holder

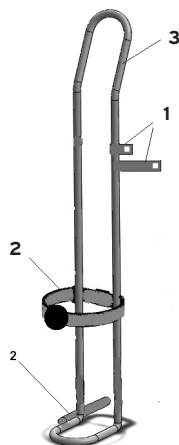1. Slidings Plates Fixing (4pcs)
2. Knob(2pcs)
3. Frame

## 8. ASSEMBLING/DISASSEMBLING

### 8.1 Assembling CPA600 - I.V. pole

1. Unpack the product and check that the content matches with the description of the parts in the manual.
2. Insert the fixing tube "1" on the bottom frame on the wheelchair.
3. Fix the IV pole by the support "2" on the backrest using the existing screws of the wheelchair.
4. Now the accessory is ready for use.

Picture with illustrative purpose only

### 8.2 Assembling CPA605 - Metal Oxygen bottle holder

#### **WARNING!**

Make the adjustment operations without of the oxygen bottle.

1. Unpack the product and check that the content matches with the description of the parts in the manual.
2. Insert the fixing tube "2" on the bottom frame on the wheelchair.
3. Fix the Oxygen bottle holder by the support "1" on wheelchair frame using the existing screws of the wheelchair and the Oxygen bottle holder.
4. TIGHTEN THE SCREWS of the supports.
5. Make sure that everything is fixed in your aid.
6. Now the accessory is ready for use.

Picture with illustrative purpose only

### 8.3 Disassembling

To disassemble do the exact opposite of the above instructions.

## 9. BEFORE USE

- Always check the status of the mechanical parts to guarantee they are used in total safety.
- Check the device is properly assembled. Pay particular attention to the locking of the belt

## 10. PRECAUTIONS FOR USE

- Do not exceed the maximum load capacity.
- Make the adjustment operations without Oxygen bottle.
- Check all components are well fixed.

## 11. HOW TO USE

The assistant must place and adjust the accessory with the maximum care to avoid sudden movements or dangerous oscillations by the hooks. An assistant has to check any movement of the patient.

## 12. MAINTENANCE

Check the structure periodically: the correct positioning of the tubes, the connections, the hooks, and the complete tightening of screws and/or terminals.

## 13. CLEANING AND DISINFECTION

### 13.1 Cleaning

Use a damp cloth or a solution of lukewarm water and neutral soap.

#### WARNING!

Do not use abrasive substances, acids, alkali, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of the metal surfaces.

The manufacture company is not responsible for any damage caused by using materials that can damage the product surface or corrosive chemicals during cleaning.

### 13.2 Disinfection

If you need to disinfect the product, use a common delicate disinfectant neutral detergent.

## 14. CONDITIONS OF DISPOSAL

When disposing of the device, never use the normal return systems of municipal solid waste. It is recommended instead the disposal through the joint municipal ecological islands for the expected operations of recycling the materials used.

## 15. SPARE PARTS

For spare parts and accessories, please always refer to the MORETTI master catalogue. ONLY USE ORIGINAL MORETTI ACCESSORIES. MORETTI S.P.A. COMPANY SHALL NOT BE LIABLE FOR ANY INCIDENT CAUSED BY IMPROPER USE.

## 16. TECHNICAL FEATURES

### 16.1 Dimension

#### CPA600 - I.V. POLE

Height "A": 100cm

Width "B": 22cm

Pay load: 5 kg

#### CPA605 - Metal Oxygen bottle holder

Height "A": 78cm

Width "B": 15cm

Oxygen bottle MAX diameter: 15cm

Pay load: 20 kg circa

### 16.2 Technical specifications

- Chromed plated steel

The dimensions maybe change  $\pm 0,5\text{cm}$

## 17. WARRANTY

Every Moretti product is guaranteed for 2 (two) YEARS from the purchase date, except the restrictions specified below. The warranty shall not apply in case of damage caused by improper use, abuse, alterations of the product or any other use not in compliance with the user manual. The proper use of the product is indicated in the user manual. Moretti S.p.A. shall not be liable for any damages or personal injuries due to incorrect assembly/improper use of the device, not compliant with the user manual. Moretti does not guarantee its products against damages or faults in the following cases: natural disasters, non authorized maintenance or reparations, damages caused by electrical supply problems (if provided), use of parts or components not provided by Moretti, failure in accomplishing the guidelines and instructions for use, non authorized alterations, damages due to the transport (different from Moretti original delivery) or failure in maintaining the device as indicated in the manual. The components subject to wear are not covered by this warranty if the damage is due to the normal use of the product.

## 18. REPAIRING

### Warranty repair

If a Moretti product shows material or manufacturing faults during the warranty period, Moretti S.p.A., together with the customer, will consider whether the product fault is covered by warranty. Moretti S.p.A., at its unquestionable discretion, may replace or repair the item under warranty in a Moretti reseller point or at its own qualified offices. Labour costs for repairing the product may be at Moretti's expense if it is determined the reparation is under warranty. A reparation or replacement does not renew nor extend the warranty.

### Repair not covered by warranty

A product, not covered by warranty, may be returned for reparation only after receiving prior authorization from Moretti customer service. Labour and delivery costs for an out-of-warranty reparation will be fully at the customer's or reseller's expense. Reparations of out-of-warranty products are guaranteed for 6 (six) months, from the day the customer receives the repaired product.

### Non defective device

The customer is informed if, after examining and testing a returning product, Moretti decides the product is not defective. The product will be sent back to the customer and all the return delivery costs will be at his expense.

## 19. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the day you receive them.

## 20. EXEMPT CLAUSES

Except what clearly specified in this warranty and in accordance with the law, Moretti S.p.A. does not offer its customers any other declaration, warranty or condition, expressed or implicit, including any declaration, warranty or condition of merchantability, of non violation or non interference, of suitability for a specific aim. Moretti S.p.A. does not guarantee that the use of Moretti products will be continuous and faultless. The duration of possible implicit warranties that may be applied in accordance with the law is limited to the warranty period. Some states or countries do not allow limitations to the duration of an implicit warranty, or the exoneration, or the limitations of accidental or indirect damages related to products for consumers. In such states or countries some exoneration or limitation of this warranty may not be applied to the user. This warranty is subject to modifications without any prior notice.

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