MA RC400-XX_RC410-XX_03 C* IT_08 2025

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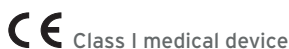
EN

MA RC400-XX_RC410-XX 03 C* EN_08 2025

**Fixed frame wheelchair
INSTRUCTION MANUAL**

INDEX

1. CODES		PAG.3
2. INTRODUCTION		PAG.3
3. INTENDED USE		PAG.3
4. DECLARATION OF CONFORMITY		PAG.3
5. GENERAL WARNINGS		PAG.4
6. SYMBOLS		PAG.4
7. GENERAL DESCRIPTION		PAG.5
8. ASSEMBLY/ DISASSEMBLY		PAG.5
8.1 Assembly RC400-xx		pag. 5
8.2 Assembly RC410-xx		pag. 5
8.3 Footrest adjustment		pag. 5
8.4 Disassembly		pag. 6
9. BEFORE EVERY USE		PAG.6
10. USAGE WARNING		PAG.6
10.1 Brakes		pag. 6
10.2 Wheels.....		pag. 6
11. HOW TO USE		PAG.6
11.1 Getting into the fixed frame wheelchair.....		pag. 6
11.2 Getting out of the fixed frame wheelchair.....		pag. 6
12. MAINTENANCE		PAG.7
13. CLEANING AND DISINFECTION		PAG.7
13.1 Cleaning.....		pag. 7
13.2 Disinfection.....		pag. 7
14. CONDITIONS OF DISPOSAL		PAG.8
15. TECHNICAL SPECIFICATIONS		PAG.8
15.1 Dimensions and weight		pag. 8
15.2 Technical specifications.....		pag. 8
16. SPARE PARTS AND ACCESSORIES		PAG.8
17. WARRANTY		PAG.9
18. REPAIRS		PAG.9
18.1 Repairs under warranty.....		pag. 9
18.2 Repairing a product not covered by the warranty.....		pag. 9
18.3 Non-defective products		pag. 9
19. REPLACEMENT PARTS		PAG.9
20. NON-LIABILITY CLAUSE		PAG.9

Class I medical device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 on medical devices

EN

1. CODES

- RC400-XX Propelled fixed frame wheelchair
RC410-XX Self-propelled fixed frame wheelchair

*XX = seat sizes 40-43-45-48 cm

2. INTRODUCTION

Thank you for choosing a fixed frame wheelchair in the MOPEDIA range by Moretti. The MOPEDIA fixed frame wheelchairs by Moretti have been designed and constructed to satisfy all your needs for practical, correct and safe use. This manual contains useful suggestions for using your device properly and safely.

Please read this manual in full before you use the fixed frame wheelchair you have purchased. Should you have any queries, please contact your dealer for suitable advice and assistance.

NB Check to ensure that no parts of the product have been damaged during shipment. Do not use the product if it is damaged and contact the retailer for further instructions.

3. INTENDED USE

The MOPEDIA fixed frame wheelchairs are intended for the transport and movement of people with mobility impairments in indoor environments: at hospitals and/or healthcare structures and entities or at home.

WARNING!

- Do not use the product for any purpose other than that specified in this manual
- Moretti S.p.A. declines any and all liability for damages resulting from improper use of the device or any use other than that specified in this manual
- The manufacturer reserves the right to modify the device and its relevant manual without prior notice, in order to improve the product's features

4. DECLARATION OF CONFORMITY

MORETTI SpA declares, under its own exclusive responsibility, that the products made and sold by MORETTI SpA in the **fixed frame wheelchairs - MOPEDIA** product family conform with the applicable provisions of Regulation (EU) 2017/745 of 5 April 2017 on MEDICAL DEVICES.

To this end, MORETTI SpA guarantees and declares as follows, under its own exclusive responsibility:

1. The devices in question satisfy the general safety and performance requirements set out in Annex I to Regulation (EU) 2017/745, as prescribed by Annex IV to said regulation.
2. The devices in question ARE NOT MEASUREMENT TOOLS.
3. The devices in question ARE NOT INTENDED FOR CLINICAL INVESTIGATION.
4. The devices in question are sold in NON STERILE packing.
5. The devices in question should be considered as belonging to class I, in accordance with the rules set out in Annex VIII to said regulation.
6. MORETTI SpA maintains and provides to the competent authorities, for at least ten years from the date of manufacture of the last production lot, the technical documentation proving conformity with Regulation (EU) 2017/745.

Note: The complete product codes, the manufacturer's single registration number (SRN), the basic UDI-DI code and any references to standards used are indicated in the EU Declaration of Conformity that MORETTI SPA issues and provides through its own channels.

EN

5. GENERAL WARNINGS

- DO NOT install or operate the product until you have read and understood this instruction manual. If you do not understand the warnings, cautions, and instructions, consult a health care professional, dealer, or service technician before attempting to install this product and avoid damage or personal injury;
- For a correct usage of the product always consult your doctor or therapist
- Keep the packaged product away from any heat source as the packaging is made of cardboard;
- The life of the device is determined by the wear and tear of non-repairable and/or replaceable parts;
- Always be aware of the presence of children;
- Do not lift the wheelchair by removable parts that may accidentally become detached during lifting.
- The user and/or patient should report any serious accident that occurs with the device to the manufacturer and to the competent authority in the Member State where the user and/or patient is located. the user and/or patient is established.

WARNING!

Always be alert for moving parts that could cause limb entrapment and personal injury.

6. SYMBOLS

	Product code
	Unique Device Identification
	CE mark
	Manufacturer
	Batch Lot
	Read the instruction manual
	Medical Device
	Conditions of disposal

7. GENERAL DESCRIPTION

RC400-xx /RC410-xx

1. Backrest
2. Seat
3. Pivoting front castors
4. Rear wheels with parking brake
(60 cm for RC410-xx)
5. Armrests
6. Footrest

Images for illustrative purpose

EN

8. ASSEMBLY/ DISASSEMBLY

8.1 Assembly RC400-xx

The MOPEDIA rigid frame wheelchair is supplied with the backrest, seat, armrests and footrests disassembled.

- To assemble the backrest, simply insert the two ends of the tubes into the openings on the rear frame of the chair. Then fix the backrest using the supplied knobs.
- To assemble the seat, match the c-clips (fixed on the underside) to the frame, and press down on the perimeter of the seat to lock it in place.
- To fit the armrests, insert the iron ends into the relevant pairs of holes on the side of the seat.
- The footplates must be inserted from the outside of the wheelchair, matching the two pins on the frame with the two holes on the footplate. The footplates must always face the inside of the wheelchair. To fasten the footplate, make sure that the locking lever has clicked into place and that it engages around the corresponding iron tab on the frame.

8.2 Assembly RC410-xx

The MOPEDIA rigid frame wheelchair is supplied with the backrest, seat, rear wheels, armrests and footrests disassembled.

- To assemble the backrest, simply insert the two ends of the tubes into the openings at the back of the chair. Then fix the backrest using the supplied knobs.
- To assemble the seat, match the c-clips (fixed on the underside) to the frame, and press down on the perimeter of the seat to lock it in place.
- To mount the 60 cm rear wheels use the only 2 bolts + nut supplied with the wheelchair (TIGHTEN THE NUTS TO SUFFICIENCY IN ORDER TO AVOID THE WHEELS LOSING DURING NORMAL USE).
- To fit the armrests, insert the iron ends into the relevant pairs of holes on the side of the seat.
- The footplates must be inserted from the outside of the wheelchair, matching the two pins on the frame with the two holes on the footplate. The footplates must always face the inside of the wheelchair. To fasten the footplate, make sure that the locking lever has clicked into place and that it engages around the corresponding iron tab on the frame.

8.3 Footrest adjustment

The footrests of rigid frame wheelchairs can be adjusted according to the height that offers the best comfort by using the nuts+screws located on the oblique tube of the footrest itself. Check always that the two footrests are at the same height.

8.4 Disassembly

Follow the assembly procedure backwards.

EN

9. BEFORE EVERY USE

- Always check for wear and tear of moving mechanical parts, in order to ensure that the wheelchair can be used in total safety for both people and property.
- Check that the device has been assembled correctly and particularly that the wheels and all components and accessories* (optional) have been inserted properly.
- Check correct functioning of the brake on the wheels.

10. USAGE WARNING

- The fixed frame wheelchair is for indoor use only; it cannot be used as a seat in motor vehicles;
- Always use the rear handle only to push the rigid frame wheelchair, Never push using any other support;
- Before moving on an inclined surface make sure of your abilities and limitations. Consult movement techniques with qualified personnel;
- However, assistance is recommended for surfaces with a slope greater than 10%. If it is necessary to stop on a sloped surface, avoid sudden movements;
- If two people are lifting the rigid-frame wheelchair the wheelchair in places that are dangerous for the patient, such as footrests and armrests, and use only the frame
- It is advisable to store the product away from heat sources, as some parts such as the back and seat upholstery and armrests, if exposed to sunlight, could reach temperatures that could cause skin damage.
- Make sure the tray has not been moved or removed;
- Do not lean on the footrests to sit or get out of the wheelchair;
- Do not lean out of the wheelchair to pick up objects on furniture or other surfaces;
- Avoid sitting on the edge to prevent tipping it over.

10.1 Brakes

Lock the wheels of the chair before going up, down, stretching forward. The brakes are applied by pressing the levers on the sides of the rear wheels.

10.2 Wheels

Wheels should be periodically checked and lubricated as necessary.

11. HOW TO USE

Note: Consult the general warnings and user instructions in this manual

WARNING!

- Getting into and out of a fixed frame wheelchair on your own can be potentially risky: do so only if you are physically able and with care.
- Do not lean on removable parts, but rather on the seat frame. **DO NOT SHIFT YOUR WEIGHT ONTO THE FOOTPLATE WHEN GETTING INTO OR OUT OF THE FIXED FRAME WHEELCHAIR, AS IT COULD TIP OVER!!!**

11.1 Getting into the fixed frame wheelchair

1. Get as close as possible to the chair, in a manner that it is positioned alongside you
2. Lock the brakes on the front wheels. Getting into the fixed frame wheelchair is easier if the armrest is removed on the side where this operation is being performed.
3. Rest your weight on the frame structure using your arms and slowly lower yourself into the chair

11.2 Getting out of the fixed frame wheelchair

1. Get as close as possible to the side of another seat
2. Lock the brakes on the front wheels. Getting out of the fixed frame wheelchair is easier

- if the armrest is removed on the side where this operation is being performed.
- Rest your arms on the frame of the seat into which you are moving and raise yourself slowly towards it

11.3 Stairs and height differences (advice on how best to deal with them)

For fixed frame wheelchair RC240-xx, take stairs (from around 1 cm up to 3 cm) in reverse, with rear wheels of \varnothing 600 mm

EN

12. MAINTENANCE

The products in the MOPEDIA range by Moretti have been carefully checked and CE marked when placed on sale.

For patient and doctor safety, have the product checked at least every two years by the manufacturer or by an authorised laboratory to verify its suitability for use. Use only original replacement parts and accessories for repairs.

- It is extremely important to check the device before using it for the first time. In particular, check that all the screws and bolts are properly tightened.
- The mechanical parts of the device can move correctly (backrest, leg rest, armrests, wheels, etc.).
- Check that the wheel brakes are working properly.
- Check the mechanical parts (hinges and pins) once a month and lubricate them with a drop of oil to ensure they slide easily and to avoid wear.

The fixed frame wheelchairs in the MOPEDIA range by MORETTI only need a few simple and periodic maintenance operations. The periodic checks to carry out are listed below:

Element	Frequency
Braking capacity	Monthly
Cleaning the seat upholstery	As needed
General check by qualified personnel	Every two years

WARNING!

Before using the device, perform all of the checks and controls suggested in the section "BEFORE USE" of this instruction manual.

13. CLEANING AND DISINFECTION

13.1 Cleaning

Only use a damp cloth to clean the fixed frame wheelchair and dry it with a clean and dry cloth, or use warm water and mild soap.

WARNING!

- Do not use abrasive substances, acids, alcohol, chlorine-based detergents, disinfectants and acetone, as these substances will scratch the plastic parts and cause the metal surfaces to rust.
- The manufacture will not be responsible for damage caused by use of materials that could damage the surface of the product or by use of corrosive chemical products during cleaning.

13.2 Disinfection

Use a mild disinfectant cleaner if you need to disinfect the device.

WARNING!

Sanitising products are corrosive. Avoid contact with the eyes and skin. Wear protective clothing, gloves, protective goggles and safety shoes. In the case of accidental contact, wash the part concerned with plenty of warm water and seek medical advice immediately.

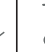
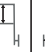
14. CONDITIONS OF DISPOSAL

Never dispose of the product as normal domestic waste. Dispose of the product at a sorted waste collection centre for recycling.

EN

15. TECHNICAL SPECIFICATIONS

15.1 Dimensions and weight

Dimensions and weight													
Measurements in cm*													
RC400-XX	95	77	48	94	74	54	32÷41	40	43,5	60	15,5 Kg	30	20
								43		63			
								45		65			
								48		68			
RC410-XX	105	81	48	92	74	54	32÷41	40	43,5	62,5	15,5 Kg	60	20
								43		65,5			
								45		68,5			
								48		71,5			

Measures above written could vary by +/- 1 cm

* = For RC410-xx models 60 cm self-propelled rear wheels

15.2 Technical specifications

FRAME	PAINTED STEEL
COATING	PCV
SEAT	PADDED
 MAX LOAD	135 KG
OPERATING ENVIRONMENT TEMPERATURE	- 20 C° / + 40 C°
RECOMMENDED STORAGE	STORE IN A COOL, DRY PLACE AND ALWAYS AWAY FROM DIRECT SOURCES OF HEAT SUCH AS SUNLIGHT
WHEELS	REAR Ø 30/60CM / FRONT Ø 12 CM

16. SPARE PARTS AND ACCESSORIES

About spare parts and accessories please refer to general catalogue

17. WARRANTY

Moretti products are guaranteed for 2 (two) years from the date of sale against material and manufacturing defects, subject the following limitations. The warranty is voided by improper use, abuse, modifications to the product and failure to follow the instructions. The intended use of the product is given in the user manual.

Moretti is not liable for damage, injury or any other consequences resulting from installation or use which are not scrupulously conforming with the instructions given in the installation, assembly and user manual. Moretti does not guarantee its products against damage or defects in the following circumstances: natural disasters, unauthorised repair or maintenance, improper electric power supply (as applicable), use of parts or components not supplied by Moretti, failure to follow the guidelines and instructions for use, tampering, shipping damage (other than the original shipping by Moretti), or failure to perform maintenance as indicated in the manual. Components subject to wear and tear are not covered by this warranty if the damage is caused by normal use of the product.

18. REPAIRS

18.1 Repairs under warranty

If a Moretti product has material or manufacturing defects during the warranty period, Moretti will agree with the client whether the defect is covered by the warranty. Moretti, at its sole discretion, may replace or repair the article at a specified Moretti reseller or its own premises. The costs of labour incurred in repairing the product will be borne by Moretti if it determines that the repair is covered by the warranty. Repair and replacement do not renew the warranty period.

18.2 Repairing a product not covered by the warranty

A product not covered by warranty may be returned for repair only if authorised in advance by Moretti customer service. The costs of labour and shipping incurred by repairs not covered by the warranty are borne by the client or reseller in their entirety. Repairs on products not covered by the warranty are themselves guaranteed for 6 (six) months from the day of reception of the repaired product.

18.3 Non-defective products

The client will be notified if Moretti concludes that the product is not defective after having received and examined it. The product will be returned to the client at his expense.

19. REPLACEMENT PARTS

Moretti original replacement parts are guaranteed for 6 (six) months from the day of delivery.

20. NON-LIABILITY CLAUSE

Unless otherwise expressly specified in this warranty and within the limits of the law, Moretti makes no declaration, guarantee or condition, express or implicit, including any future declaration, guarantee or condition of sale, suitability for a given purpose, non violation and non interference. Moretti does not guarantee that the use of its product will be uninterrupted and problem-free. The duration of any implicit guarantee under the law is limited to the warranty period, within the limits of the law. Certain states and countries do not permit limitations on the duration of an implicit guarantee or the exclusion of limitation of accidental or indirect damages in relation to consumer products. In said states and countries, certain exclusions and limitations of this warranty may not apply to the user. This warranty is subject to modification without notice.

EN

WARRANTY CERTIFICATE

Product _____

Purchased on (date) _____

Retailer _____

Address _____ **Town/city** _____

Sold to _____

Address _____ **Town/city** _____

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN P.R.C.

MORETTI S.p.A.
Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)

Tel. +39 055 96 21 11
Fax. +39 055 96 21 200

www.morettispa.com
info@morettispa.com