MOPEDIA®

MA RP505_RP523_RP547_RH547_02 A*_ITA_06 2023

ROLLATORS IN ALLUMINIO

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ITALIANO

INDICE

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Via _____ Località _____

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com**MADE IN P.R.C.**

MOPEDIA®

MA RP505_RP523_RP547_RH547_01 A*_ENG_06 2023

ALUMINUM ROLLATORS

INSTRUCTION MANUAL

ENGLISH

INDEX

| | |
|--------------------------------------------|--------------|
| 1. CODES | PAG.3 |
| 2. INTRODUCTION | PAG.3 |
| 3. INTENDED USE | PAG.3 |
| 4. DECLARATION OF CONFORMITY | PAG.3 |
| 5. WARNINGS | PAG.4 |
| 5.1 General warnings | pag. 4 |
| 5.2 Warnings for use..... | pag. 4 |
| 5.3 Stability warnings | pag. 4 |
| 5.4 Assembling & folding warnings | pag. 4 |
| 6. SYMBOLS | PAG.5 |
| 7. BEFORE USE | PAG.5 |
| 8. ASSEMBLING INSTRUCTION | PAG.5 |
| 8.1 Before assembling the product..... | pag. 5 |
| 8.2 Assembly and disassembly RP505..... | pag. 5 |
| 8.2.1 Handle adjustment | pag. 5 |
| 8.3 Assembling RP523 | pag. 6 |
| 8.3.1 Seat adjustment..... | pag. 6 |
| 8.3.2 Adjusting handles | pag. 6 |
| 8.3.3 Folding the rollator | pag. 6 |
| 8.3.4 Seat | pag. 6 |
| 8.4 Assembling RP547 and RH547 | pag. 7 |
| 8.4.1 Seat adjustment..... | pag. 7 |
| 8.4.2 Adjusting handles | pag. 7 |
| 8.4.3 Seat | pag. 7 |
| 9. HOW TO USE | PAG.7 |
| 10. BRAKES | PAG.7 |
| 10.1 To brake | pag. 7 |
| 10.2 Lock the brake (handbrake) | pag. 8 |
| 11. CARE AND MAINTANANCE | PAG.8 |
| 12. CLEANING | PAG.8 |
| 13. WHEEL CARE | PAG.8 |
| 14. TECHNICAL FEATURES | PAG.8 |
| 15. CON DITIONS OF DISPOSAL | PAG.9 |
| 16. SPARE PARTS / ACCESSORIES | PAG.9 |
| 17. WARRANTY | PAG.9 |
| 17.1 Repairing..... | pag. 9 |
| 17.2 Spare parts | pag. 9 |
| 17.3 Exempt clauses | pag. 9 |

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

| | |
|--------------|-----------------------------------------------------|
| RP505 | Indoor aluminium rollator SOLE |
| RP523 | Ultralight aluminium rollator GALATEA |
| RP547 | Aluminium rollator with height-adjustable seat FEBE |
| RH547 | Aluminium rollator Febe HD |

2. INTRODUCTION

Thank you for choosing a MOPEDIA range rollators by Moretti S.p.A. MOPEDIA range rollators manufactured by Moretti S.p.A. are designed to satisfy all your requirements for practical, correct and safe use. This manual provides some useful suggestions for correctly using the device you have selected and offers valuable advice for your safety. We recommend reading the entire manual carefully before using the rollator you have purchased. Should you have any queries, please contact your retailer for proper advice or assistance.

3. INTENDED USE

This rollator fit for people with movement problems. Can be used as assistance to facilitate ambulation or even servo-assisted ambulatory. Anyway, it can't be used by people with evident malfunction of the hand or the arm, even by people with balance problems. The rollator can be used inward and outward. We recommend using on flat surface where the wheels can reach the land. It can't be used on rock ground or on slanted area as the rollator can be unstable. The device is not intended to carry people or things.

WARNING!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of ROLLATORS - MOPEDIA complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017. For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. WARNINGS

5.1 General warnings

- Do NOT install or use the device before reading and understanding this manual; If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional dealer or technical personnel before attempting to install this equipment to avoid injure or damage;
- The rollator should only be used by patients with a suitable motor capacity which ensures their use without risks;
- Advise with physician or therapist to determinate the correct adjustment and the correct use;
- Physician or therapist should contribute to height adjustment for the maximum support, for a correct brake activation and for a general assembly;
- The rollators are not intended to be used as self-propelled devices.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

5.2 Warnings for use

1. Insure the brake cable to be correctly set;
2. Verify the correct operation of the brake after every height adjustment to avoid a free flexion of the cable;
3. Breaking this simple rules can cause personal damages;
4. The rollator can be used only on a hard, flat surface. Do not use the rollator to climb the stairs
5. Leaning always on both dumbbells to maintain balance;
6. Pay particular attention to wet, icy or snowy;
7. Do not overload the rollator;
8. Do not use the rollator seat (where present) to carry persons or property, the seat is for the user's session with the parking brake engaged;
9. All wheels must be in contact with the floor at any time during use to ensure the proper balancing of the rollator and maximum safety;
10. The brake must be engaged before using the seat.

Note When the rollator is used in steady position, the handbrake must be locked

5.3 Stability warnings

- The rollator can provide ambulatory assistance to an individual weighing up to 136 kg, INCLUDING the loading bag;
- The rollator bag has a weight limitation of 5 kg.

5.4 Assembling & folding warnings

- Verify that both handle support tubes are correctly inserted in the frame and secured with locking knobs locked into the holder with its main wing nuts and knobs without movement;
- Both handle tubes are securely locked in the main support part with wing nuts and knobs without any movements;
- Do not hanging anything on the rollator frame to avoid distortions of the tubes when the rollator is open and damage the stability. Injury may also be caused by distortions.

Note Take care to entrapment of fingers and / or clothing during the opening and closing.

6. SYMBOLS

-  Product code
-  Unique Device Identification
-  CE mark
-  Manufacturer
-  Batch Lot
-  Read the instruction manual
-  Medical Device
-  Conditions of disposal
-  Maximun loading weight

7. BEFORE USE

Before using the device, always check:

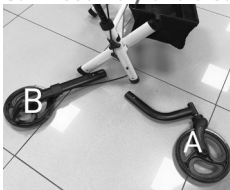
- That all parts of the rollator are locked in a secure manner.
- Check that all screws are tight.
- Make sure that the seat is properly locked (if that 'not press further with the power seat bottom frame).
- Check that the locking of the seat is correctly inserted.
- Check the correct operation of the brake and the regular brake.

8. ASSEMBLING INSTRUCTION

8.1 Before assembling the product

Unpack the product and check that no damage has occurred during shipping. In the event when the rollator is damaged, please contact your nearest dealer.

8.2 Assembly and disassembly RP505

- Open the rollator and basket until the plastic tab snaps out
- Insert the front castors A and rear castors B into their places
- Place the transparent plastic tray on top of the basket

To remove and close follow the steps in reverse order

WARNING! Make sure that the knobs is located into the holes. Before use, always make sure that the rollator does not tend to close up under the influence a small lateral force.

8.2.1 Handle adjustment

To adjust the handle, it is necessary to unscrew the knobs and remove the plastic blocks on both sides. Then raise or lower the handle to the desired height by exerting equal force on both sides reposition the plastic block and screw the knob on, making sure that the hole is well drilled.

**WARNING!**

This rollator does not allow you to sit on the front surface consisting of the basket and tray, so it is strictly forbidden to sit on it

8.3 Assembling RP523

- Insert legs and handles into the appropriate housings, correctly matching the letters shown on the product parts
- Insert the backrest into the backrest slots by pressing on the side pins

8.3.1 Seat adjustment

- To adjust the seat it is necessary to adjust the front wheels A by removing the pin from the holes and reposition them once the desired height has been reached;
- For rear wheels B press the side pins and position the wheel at the desired height by snapping them

**WARNING!**

Adjust the wheels making sure that the seat is always parallel to the ground

8.3.2 Adjusting handles

Unscrew the grips and remove them from the hole, choose the desired height and reinsert them in the hole by screwing them in.

**WARNING!** Make sure that the knobs is located into the holes.

Before use, always make sure that the rollator does not tend to close up under the influence a small lateral force.

8.3.3 Folding the rollator

Lift the metal handle under the seat and pull it upwards

8.3.4 Seat

Before sitting , place the rollator on a flat and solid surface and then lock the brakes. Make sure the parking brake is secured properly and that the rollator is in a stable position.

WARNING! Sit down distributing the weight over the whole seat. Do not sit by pushing only on one side.

8.4 Assembling RP547 and RH547

- Insert legs and handles into the appropriate housings, correctly matching the letters shown on the product parts
- Insert the backrest into the backrest slots by pressing on the side pins

WARNING! Before use always make sure that the rollator does not tend to close under the effect of a small lateral force. Take care not to weave the brake cables when connecting the handles and the wheels

8.4.1 Seat adjustment

To adjust the seat it will be necessary to press the locking pins on the front and rear wheels and slide them to the desired height

8.4.2 Adjusting handles

Press the side buttons and position the handle at the desired height by snapping them into place

8.4.3 Seat

Before sitting, place the rollator on a flat and solid surface and then lock the brakes. Make sure the parking brake is secured properly and that the rollator is in a stable position.

WARNING! Sit down distributing the weight over the whole seat. Do not sit by pushing only on one side.

9. HOW TO USE

- Start pushing when you are in an upright position with both hands on the handle;
- Make sure your hands are dry before using the rollator to avoid the risk of slipping;
- Walking maintaining an upright position with his eyes facing forward and not down;
- Do not push the rollator too far away from the body, this may cause falls and loads inadequate;
- Do not use the rollator to overcome steps higher than 20mm;
- When using outdoors, prefer the sidewalks properly prepared in accordance regulations for the removal of architectural barriers;
- To overcome small obstacles to leverage the rear footrests to raise the front of the rollator

10. BRAKES

The good functioning of the brakes depends on the wear and dirt accumulated on the wheels check the condition of the brakes and wheels before use.

10.1 To brake

To brake the rollator pull the brake lever with your fingers upwards until the rollator slows down or stops. To continue to move the rollator, release the brake handles.

10.2 Lock the brake (handbrake)

Press down on the brake lever with the palm of your hand, when you hear a “click” wheels are locked. Lift the handle of the brake releasing it from the stop position.

WARNING! When the user is stationary, the brakes must be locked

11. CARE AND MAINTANANCE

The devices **MOPEDIA** by Moretti SpA on the market are carefully checked and carry the CE mark. For the safety of the patient and the doctor is recommended to be checked by the manufacturer or by an approved laboratory, at least every 2 years, the suitability for use of your rollator. If repairs are to be used only original spare parts and accessories.

The useful life of the product depends on the use, maintenance, storage and cleaning.

12. CLEANING

To clean parts, use only a damp cloth with a neutral detergent, not use cleaners containing solvents and metal brushes or abrasive or sharp objects, may damage the product.

13. WHEEL CARE

1. Clean the wheels with warm water and mild detergent. Wipe dry with a clean cloth.
2. Periodically lubricate the wheels.

14. TECHNICAL FEATURES

Series SOLE

| | |
|--------------|----------|
| Product code | RP505 |
| Tray | 25x39 cm |
| Frame | Aluminum |

Series GALATEA

| | |
|--------------|----------|
| Product code | RP523 |
| Seat | 37x24 cm |
| Frame | Aluminum |

Series

FEBE

FEBE HD

| | | |
|--------------|----------|----------|
| Product code | RP547 | RH547 |
| Seat | 35x35 cm | 60x40 cm |
| Frame | Aluminum | |

Dimensioni e pesi

| |  |  |  |  |  |  |  |  |  |  |
|-------|-------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------|
| RP505 | 75 | 86 97 | 69 tray | 57 | 40 | 95x56x34cm | 6,8 | | | |
| RP523 | 70 80 | 85 102 | 54 61 | 64 | 47 | 95x58x33cm | 7,4 | 20 | 20 | 136 kg |
| RP547 | 70 80 | 80 99 | 50 57 | 63 | 47 | 90x62x39cm | 8 | | | |
| RH547 | 80 | 80 100 | 50 57 | 70 | 55 | 110x70x39cm | 10 | 25 | 25 | 180 kg |

The dimensions are in cm and could be changed $\pm 0,5$ cm

* Maximum size

15. CON DITIONS OF DISPOSAL

When disposing of the device never use the normal return systems of municipal solid waste. It is recommended instead to disposal through the joint municipal ecological islands for the expected operations of recycling the materials used.

16. SPARE PARTS / ACCESSORIES

For spare parts and accessories please refer to the Moretti S.p.A. master catalogue

17. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be apply in the event damages caused by improper use, abuse or alteration, and for the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual.

Moretti is not responsible of consequent damages, personal injuries or whatever caused by or in relation with mistaken installation or with device improper use.

Moretti does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, alteration not authorized, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

17.1 Repairing

Warranty repair: If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will estimate with customer if the fault can be cover from warranty. Moretti, at its unquestionable discretion, could repair or replace the item, with a Moretti dealer or with own base. Labor cost can be in charge to Moretti if the repair is cover from warranty. A repair or a replacement doesn't extend the warranty.

Repair not cover from warranty: A product without warranty can be given back after Moretti authorization. The labor and shipping costs for a bath lift out of warranty are to be payed by the customer or by the dealer. The repairs are guaranteed for 6 months as from the good received.

Non-defective device: The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be payed by the customer.

17.2 Spare parts

The original Moretti spare parts are guaranteed for 6 months as from the receiving date.

17.3 Exempt clauses

Moretti doesn't offer any other declarations, warranty or conditions, explicit or implicit, including possible declarations, warranties or condition of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti doesn't guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be fixed by according to the law is limited by the warranty period, in the limits of law. Some states or countries don't allow to limit the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user.

The present warranty may be modified without prior notification.

WARRANTY CERTIFICATE

ENGLISH

Product _____

Purchased on (date) _____

Retailer _____

Address _____ Town/city _____

Sold to _____

Address _____ Town/city _____

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN P.R.C.

** Please consult our website for the latest available version of the user manual*

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