ITALIANO

MA YDC-6A1/EM216 03 A_ITA_11-2021

MATERASSO A DEPRESSIONE

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INDICE

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Venduto a _____

Via _____ Località _____

JIANGSU RIXIN MEDICAL EQUIPMENT CO.,LTD.

No.427 Yangjin Road, Jinfeng, Zhangjiagang, Jiangsu Province, CHINA

SUNGO EUROPE B.V.

Fascinatio Boulevard 522, Unit 1.7, 2909VA Capelle aan den IJssel, The Netherlands.

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MORETTI S.P.A.

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

EASYRED®

MA YDC-6A1/EM216 03 A_ENG_11-2021

VACUUM MATTRESS STRETCHER

INSTRUCTION MANUAL

ENGLISH

INDEX

1. CODE	PAG.3
2. INTRODUCTION.....	PAG.3
3. INTENDED USE.....	PAG.3
4. DECLARATION OF CONFORMITY	PAG.3
5. GENERAL WARNINGS	PAG.4
6. SYMBOLS	PAG.4
7. BEFORE USE	PAG.4
8. WARNINGS FOR USE.....	PAG.4
9. HOW TO USE	PAG.5
10. MAINTENANCE	PAG.5
11. CLEANING AND DISINFECTION	PAG.5
12. GENERAL CONDITIONS OF DISPOSAL	PAG.5
13. TECHNICAL FEATURES	PAG.5
14. WARRANTY	PAG.5
15. REPAIRS	PAG.6
15.1 Warranty repair	pag. 6
15.2 Repair not covered by warranty	pag. 6
15.3 Non defective device	pag. 6
16. SPARE PARTS	PAG.6
17. EXEMPT CLAUSES.....	PAG.6

 I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODE

YDC-6A1/EM216 VACUUM MATTRESS STRETCHER

2. INTRODUCTION

Thank you for purchasing a EASYRED by Moretti S.p.A. device. This product was designed to immobilize the patient during transport. The product is completely radiolucent. Made from TPU (thermoplastic polyurethane) very durable, with 4 belts and eight handles (4 on each side). Complete with pump and carry bag. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance.

3. INTENDED USE

Device for the immobilization and transportation of patients in emergency.

This product is intended for exclusive use by professional and qualified staff, duly trained for emergency care. It is always recommended to follow the procedures designed for emergency response. Improper use of the device can cause damage to the patient who is cared for in an emergency.

WARNINGS!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of PRODUCTS FOR EMERGENCY INTERVENTIONS complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017. For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

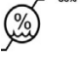
1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

- READ carefully the entire manual before using the device .
- The product should be used only by qualified personnel
- Make sure that the product is delivered complete and keep it packed away from heat sources
- SERVICE LIFE- the device use limit is defined by the wear of the parts
- Store in a dry place
- DO NOT allow children to play or operate the device
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

	Product code
	CE mark
	Manufacturer
	Production dates
	Batch Lot
	Attention
	Keep dry
	Medical Device
	Name and address of the European Representative
	Expiration date
	Read the instruction manual
	Not sterile
	Humidity limit
	Conditions of disposal

7. BEFORE USE

- Always check the state of wear of the parts in such a way as to ensure it is used in complete safety for persons and property.

8. WARNINGS FOR USE

- In the case of air leakage during the depression verify that the valve is tightly connected
- Make sure the mattress is to avoid contact with sharp objects pierce it.
- Use belts when necessary
- Do not use with patient who exceed the weight indicated in the product label and in the technical features chapter of the present manual

- Periodically check its sturdiness, the stability and the rubber tips are in good conditions. Replace them once worn out .
- In case of doubts about use, adjustment or maintenance, contact your dealer
- Keep it away from children to avoid any possible risks.

9. HOW TO USE

- Turn the valve to the "open" position.
- Lay the mattress so that the parts are distributed equitably.
- Insert the pump into the valve and suck air until the parties reach a certain rigidity. Then turn the valve on the move "Closed".
- Correctly position the patient on the mattress, wrap and hook the safety belts.
- Re-position the valve in the "open" position and pull out the air fi no in that the mattress becomes stiff enough to allow the complete immobilization of the patient. Then turn the valve to "closed" and remove the

10. MAINTENANCE

The EASYRED devices by MORETTI are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years.

11. CLEANING AND DISINFECTION

Use water and a mild disinfectant, dry away from direct sunlight.

Note Never use acids, alkalis or solvents such as acetone

12. GENERAL CONDITIONS OF DISPOSAL

General conditions of disposal. In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the bath chair in the appropriate disposal areas for recycling.

13. TECHNICAL FEATURES

Code	Open dimensions	Fold dimensions	Weight	Max user's weight
YDC-6A1/EM216	212x75x7 cm	70x60x37 cm	5 kg	160kg

14. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product , and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual. Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair , damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

15. REPAIRS

15.1 Warranty repair

In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location . The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty

15.2 Repair not covered by warranty

A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

15.3 Non defective device

The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer .

16. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the receiving date.

17. EXEMPT CLAUSES

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference.

Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

ENGLISH

WARRANTY CERTIFICATE

Product _____

Purchased on (date) _____

Retailer _____

Address _____ **Town/city** _____

Sold to _____

Address _____ **Town/city** _____

JIANGSU RIXIN MEDICAL EQUIPMENT CO.,LTD.

No.427 Yangjin Road, Jinfeng, Zhangjiagang, Jiangsu Province, CHINA

SUNGO EUROPE B.V.

Fascinatio Boulevard 522, Unit 1.7, 2909VA Capelle aan den IJssel, The Netherlands.

Imported and distributed by:

MORETTI S.P.A.

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MORETTI S.P.A.
Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)

Tel. +39 055 96 21 11
Fax. +39 055 96 21 200

www.morettispa.com
info@morettispa.com