ITALIANO

MA CP850-37 02 B_ITA_08 2021

CARROZZINA SERIE TRAVEL

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INDICE

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CE | Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices**1. CODES****CP850-37** Wheelchair TRAVEL seat 37 cm**2. INTRODUCTION**

Thank you for purchasing a **ARDEA ONE** wheelchair by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly.

Note: check all parts for shipping damage and test before using. In case of damage, do not use the device. Contact your dealer for further instruction.

3. INTENDED USE

The wheelchairs are intended for people with movement difficulties. The wheelchairs can be for self-propulsion (with 60cm casters) or can be wheeled. In all cases however it requires the intervention of an operator for the various adjustments you require.

WARNING!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without notice

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of MANUAL WHEELCHAIRS ARDEA ONE complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

- For correct use of the device, carefully refer to the following manual
- For correct use of the device always consult your doctor or therapist
- Keep the packaged product away from any source of heat as the packaging it is made of cardboard
- The useful life of the device is determined by the wear of non-repairable and/or replaceable parts
- Always pay close attention to the presence of moving parts which could cause limb entrapment and personal injury
- Always pay attention to the presence of children
- Do not lift the wheelchair by removable parts which could accidentally come off during lifting
- The user and / or the patient must report any serious accident occurring in relation to the device to the manufacturer and to the competent authority of the State in which the user and / or the patient is established.

6. SYMBOLS

	Product code
	Unique Device Identification
	CE mark
	Manufacturer
	Batch Lot
	Read the instruction manual
	Medical Device
	Conditions of disposal
	Serial Number

7. OPENING / CLOSING

The wheelchair can be opened easily, without tools, just following a few manual operations.

Note: Make reference to safety cautions in this manual

7.1 Opening of CP850-37

Lift up the backrest until levers block (fig. 1); pull the footrest outward (fig. 2); push the seat downward until it is completely opened (fig. 3); push-down the armrests until 90° degrees.

(FIG.1)

(FIG.2)

(FIG.3)

7.2 Closing of CP850-37

Lift up the armrests (fig. 4); Press backrest levers and push it downward (fig. 5); pull up seat and footrest until the wheelchair is totally folded (fig. 6); pull up the footrest (fig. 7).

(FIG.4)

(FIG.5)

(FIG.6)

(FIG.7)

8. BEFORE USE

- Check the mechanical parts usage, in order to guarantee a safe use of the device for people and things
- Assure all the parts are correctly assembled. Checking carefully the correct assembling of the rear wheels
- Assure the brake handles are well regulated
- Check the seat is completely open. If not push the seat down from the lateral frame.

9. WARNING FOR A CORRECT USE

- Do not use the wheelchair as seat for vehicle
- Assure the brake handles is duly adjusted and correctly works
- Avoid entrapment of finger into wheels or brakes during the wheelchair use
- The wheelchair temperature increase if exposed to the sun for long period

10. HOW TO USE

Note: make reference to "GENERAL WARNINGS" chapter

WARNING!

- Getting in and out of the wheelchair may be performed independently, provided you have a adequate mobility and upper body strength
- Do not stand on removable parts

DO NOT STAND ON THE FOOTRESTS WHEN GETTING IN OR OUT OF THE WHEELCHAIR, OVERTURNING RISK!!!

10.1 Get in the wheelchair

1. Position the wheelchair as close as possible to you
2. Engage wheel locks . We suggest to remove the armrest to ease the patient getting in the wheelchair
3. Lean on the frame with arms and slowly go down towards the seat

10.2 Get out in the wheelchair

1. Position the wheelchair as close as possible along side the seat to which you are transferring
2. Engage wheel locks. We suggest to remove the armrest to ease the patient getting out of the wheelchair
3. Shift body weight into seat with transfer

10.3 Stairs and height difference

WARNING!

The following method must be used only from experienced patients. If you are not experienced, let you help from an assistant!

Before moving on an inclined surface make sure of your ability and limits. Refer to the movement techniques with qualified personnel. An assistance for surfaces with a slope greater than 10% is recommended. If you need to stop on an inclined surface, avoid sudden movements. To slow in a slope do not use the wheel brakes. Not suddenly change the downward direction.

- Get in

Position the wheelchair at the curb to be crossed. Balance the wheelchair on the rear wheels, in order to lift the front wheels, enough to lean them on the obstacle. Bend your trunk forward and vigorously push the wheelchair. Continue pushing until the rear wheels are climbing over the obstacle.

- Get off

To get off the stair follow the reverse procedure, positioning the rear side of the wheelchair at the curb, keep the handle vigorously and get off the obstacle slowly; then balance the wheelchair on the rear wheels in order to complete the descent

WARNING!

It is recommended not to use this technique in the presence of a vertical drop more than 10 cm!

10.4 Brakes

Stop wheels before getting in/out, stretching out or getting on lift. Pull completely lever up to stop chair. If correctly done, brakes press 5/6 mm against the rubber. If needed, use the equipment key to unscrew nut while clamping bit forwards and backwards. Fix correctly nut. to adjust brakes.

10.5 Wheels

The wheels must be periodically checked and lubricated if necessary. Remove the fork taking out the screw and lubricate the peg. Remove the wheel from the fork as well and lubricate the pegs. Assembly again the wheel and the peg and tighten the screws well.

10.6 Carrying

The wheelchair has been designed to be easily carried.

Once the wheelchair are closed, you can position it in the car boot, for example. If this can't be possible, position the wheelchair in front of the car rear seat so that it is stable and cannot slip or slide. Lock the wheelchair with the seatbelt.

11. MAINTENANCE

The ARDEA ONE devices by Moretti are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. In case of repair use

only original spare parts. Before using follow the "BEFORE USE" chapter. Moretti ARDEA ONE wheelchairs require few elementary and periodic maintenance, below we list the checks to be carried out periodically.

- Braking capacity - Monthly
- Seat cleaning - As needed
- General check by qualified staff- Biennial
- Pressure tyre - Monthly (if provided)

12. CLEANING AND DISINFECTION

12.1 Cover and padding

The cover and padding are in Nylon. Clean with warm water and mild soap.

12.2 Frame and metallic parts

Clean with warm water and mild soap. Dry with a soft cloth in order to not scratch the painting.

12.3 Brakes and wheels

Clean the wheels with warm water and mild soap. Pay attention in drying the wheels, it's dangerous use the wheelchair with wet wheels.

12.4 Disinfection

if you need to disinfect the device use a common disinfectant cleaner

Note: Do not use solvent and abrasive product

13. CONDITION OF DISPOSAL 

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the wheelchair in the appropriate disposal areas for recycle

Disposal instructions, as required by directive 2012/19/UE

At the end of its service life, the product may not be disposed of together with normal municipal waste. It must be delivered to the municipal sorted disposal centre or to the reseller, who provides this service. Sorted waste disposal reduces the environmental and health consequences of improper disposal and enables its materials to be recycled, this resulting in energy and resource savings. To highlight the requirement for sorted disposal of electromedical equipment, the product bears the barred bin symbol.

14. SPARE PARTS AND ACCESSORIES

For spare parts only advert to MORETTI master catalogue.

15. TECHNICAL FEATURES

15.1 Dimensions

DIMENSIONS AND WEIGHT

Measures in CM																
CP850-37	86 VARIABLE	58 VARIABLE	38	89 VARIABLE	65 VARIABLE	43 VARIABLE	33	37	41	44	29	7	20	13	-	75

15.2 Technical specificatios CP850-37

- Aluminium frame
- Max weight **75 kg**
- PU rear casters, 13 cm diameter
- Rear casters: diameter 20cm, solid tires
- Nylon coating
- Resistance to ignition: the materials used for cover and padding have been treated with methods specified in Directive EN1021-1/2
- Reference regulation: EN12183
- Environmental temperature: -20°C/+ 40°C
- Storage: store in a cool place, away from heat sources, such as sunlight (suggested temperature between +10°C and +40°C)

16. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and for the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual. Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to mistaken installation or improper use. Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, alteration not authorized, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

16.1 Repairing

Warranty repair If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be cover from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

Repair not cover by warranty A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

Non defective device The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be paid by the customer.

16.2 Spare parts

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

16.3 Exempt clauses

Moretti does not offer any other declarations, warranty or conditions, explicit or implicit, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

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