EASYRED®

MA EM810 EM820 MA EM830 EM840 EM850 EM860 EM870 EM891 EM892 03 A_ITA_06-2021

BORSE E ZAINI PER EMERGENZA

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ITALIANO

INDICE

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1. CODES

- EM810** Multipurpose emergency bag
- EM820** Multipurpose emergency bag with modules
- EM830** 3-pocket multipurpose emergency bag
- EM840** 2-section multipurpose emergency bag
- EM850** 2-section emergency rucksack
- EM860** Emergency rucksack with 2 lateral pockets
- EM870** 4-pocket emergency rucksack
- EM891** Transparent multipurpose module with Velcro closure - 5 pcs/set
- EM892** Transparent multipurpose module without Velcro closure - 5 pcs/set

2. INTRODUCTION

Thank you for purchasing EASYRED by MORETTI emergency bag. Design and quality of the article guarantee maximum security and reliability. EASYRED by MORETTI emergency bags are designed and realized to meet all your requirements.

This user's manual contains quick advice for a correct use and maintenance of the bag. We suggest you to read this user's manual carefully before handling the bag, in case of any doubts, please contact the retailer who will help you.

3. INTENDED USE

EASYRED by MORETTI emergency bags are used to transport medical equipment, medications, medicines and all that is necessary for an emergency operation.

WARNINGS!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

4. GENERAL WARNINGS

- READ carefully the entire manual before using the device .
- The product should be used only by qualified personnel
- Make sure that the product is delivered complete and keep it packed away from heat sources
- SERVICE LIFE- the device use limit is defined by the wear of the parts
- Store in a dry place
- DO NOT allow children to play or operate the device

5. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Conditions of disposal

6. COMPOSITION

All EASYRED by MORETTI emergency bags are made with the following materials :

Polyester 600D, Nylon 210D e 420D, Transparent vinyl

7. MAXIMUM WEIGHT

EASYRED by MORETTI emergency bags are used to transport medical equipments and all that is necessary for an emergency operation: the maximum weight should not be more than 30Kg.

8. WARNINGS FOR USE

For a correct use of the bag, pay your attention to the following:

- All zip and fasteners have to operate properly, assuring the closing of the bag during the transport.
- Medicines and other liquids contained in the bag have to be closed in order to avoid any accidental emission.
- The bag has to be always clean and no residue has to be present inside the bag. In this case we recommend to follow the cleaning instructions.

9. CLEANING AND DISINFECTION

Hand wash:

Wash in warm water containing mild detergent. Let it dry free on air.

Washing machine:

Remove all interior-plastic parts and wash at 40 deg. C without centrifuge, let dry free on air. For bags/rucksacks with non-removable interior-plastic parts are not to be cleaned in the washing machine. Ironing is not permitted.

10. GENERAL CONDITIONS OF DISPOSAL

General conditions of disposal. In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the bath chair in the appropriate disposal areas for recycling.

11. TECHNICAL FEATURES


11.1 Emergency bags

EM810

Double lateral pocket to contain first aid material and back pocket with net partition. Main zip-opening; net compartments with velcro fastening and elastic bands.

EM810 - Dimensions

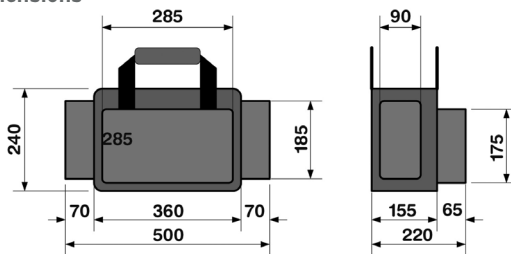
EM820

Upper zip-opening. Velcro modular sections. Complete with two net pockets.


EM820 - Dimensions

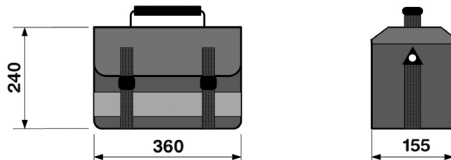
EM830

Velcro modular sections. Two lateral pockets, front pocket with three sections.

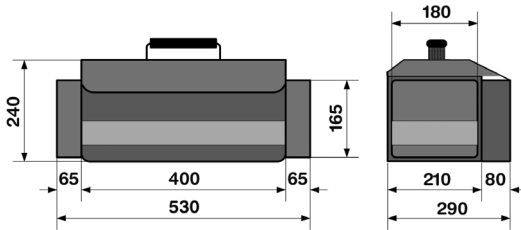**EM830 - Dimensions****EM840**

2-section emergency bag. Double opening zip, internal compartment suitable for clear vinyl modules, provided with elastic pockets for small objects.

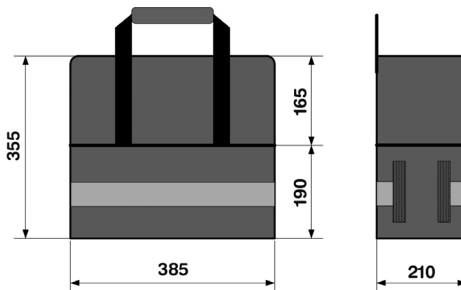**EM840 - Dimensions**

11.2 Emergency rucksacks


EM850

2-Section rucksack. Rigid bottom compartment secured with zip, upper part with interchangeable sections. Big internal extensible compartment with elastic bands and pocket with velcro.

ENGLISH

EM850 - Dimensions

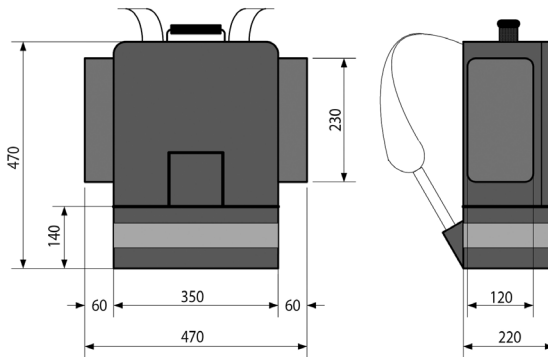
EM860

Single zip-opening rucksack, with velcro modular sections. Two lateral pockets for first aid material; net pocket with velcro fastening for emergency material.

EM860 - Dimensions

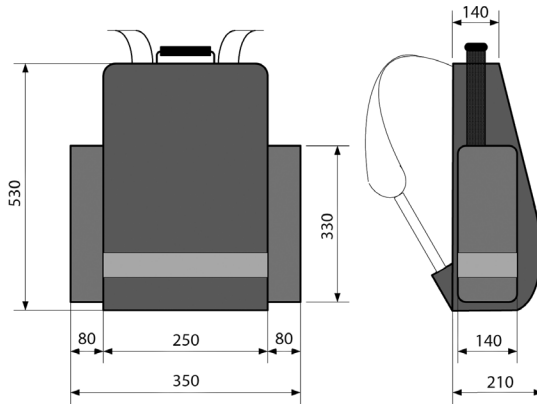
EM870

Single zip-opening, velcro modular sections. Two lateral pockets for first aid material, net pocket with velcro fastening for emergency material.

EM870 - Dimensions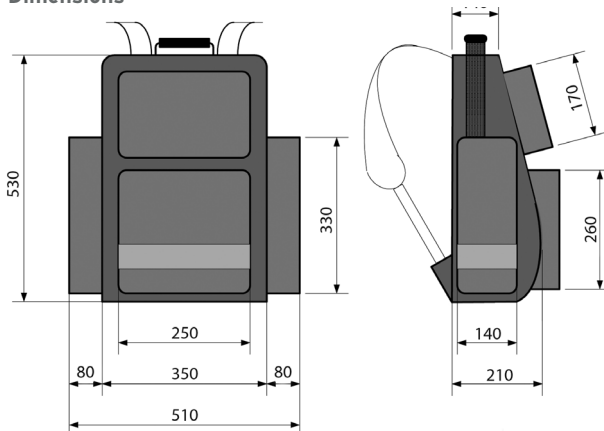

11.3 Modules for emergency bags and backpacks

EM891 con velcro

EM892 senza velcro

Set of 5 coloured, clear vinyl modules for rucksacks and emergency bags. Zip-opening. Available in different colours.

EM891-EM892 - Dimesnions

 L: 14 CM
  W: 15,5 CM
  H: 7,5 CM

12. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual. Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

13. REPAIRS

13.1 Warranty repair

In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location. The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty

13.2 Repair not covered by warranty

A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

13.3 Non defective device

The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer.

14. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the receiving date.

15. EXEMPT CLAUSES

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference.

Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

WARRANTY CERTIFICATE

ENGLISH

Product _____

Purchased on (date) _____

Retailer _____

Address _____ **Town/city** _____

Sold to _____

Address _____ **Town/city** _____

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN INDONESIA

MORETTI S.P.A.

Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)

Tel. +39 055 96 21 11
Fax. +39 055 96 21 200

www.morettispa.com
info@morettispa.com