SKEMA[®]FORM

ITALIANO

MA MI310-MI311-MI320-MI321 01 E* ITA_06-2024

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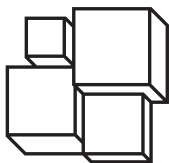
REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

- MI310X** LYTUS 3 sections electric examination couch 62 cm
MI311X LYTUS 3 sections electric examination couch 62 cm, with castors
MI320X LYTUS 3 sections electric gynecological couch 62 cm
MI321X LYTUS 3 sections electric gynecological couch 62 cm, with castors
X: Different pad colors

2. INTRODUCTION

Thank you for purchasing a SKEMA examination couch by Moretti S.p.A.

The se products are designed and manufactured to meet all your needs for a practical, correct and safe use. This manual contains small suggestions for proper use of the device you have chosen and valuable advice for your security. You may read the complete manual before using the product, in case of doubts please contact your dealer, who will help and advise you properly.

3. INTENDED USE

Moretti examination beds are intended for patient diagnosis, treatment and monitoring.

WARNING!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of THREE-SECTIONS LYTUS EXAMINATION COUCHES complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

4.1 Applicable regulations and directives

In order to satisfy safety standards for users, Moretti S.p.A. complies with the following standards: UNI CEI EN 60601-2-52:2016 Electro-medical equipment. Part 2-52. Special requirements for the basic safety and essential performance of medical beds

UNI EN 12182:2012 Products intended for the assistance of people with disabilities

5. GENERAL WARNINGS

For a correct use of the product, please refer to the present manual

- Keep the packed examination bed away from heat sources
- **SERVICE LIFE**- the examination bed service life is determined by wear of parts not repairable and/or replaceable and in any case not exceed 10 years
- **ALWAYS** pay close attention to the presence of moving parts that could cause entrapment limbs and injuries
- **DO NOT** allow children to play on or operate the bed
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Serial number

Read the instruction manual

Medical Device

Conditions of disposal

Attention, should read the instructions

Production dates

Waste Electrical & Electronic Equipment (WEEE)

IPX6

Degree of protection against dusts and liquids

B Type

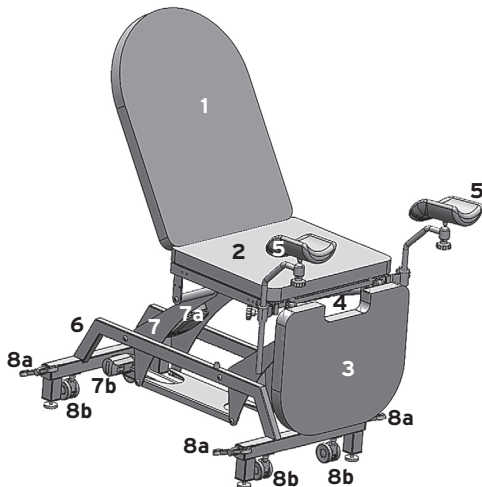
Double insulation class

Max loading weight


Caution, danger of crushing - entrapment


7. GENERAL DESCRIPTION
7.1 Parts view

Parts list

- 1 Back-rest section, equipped with electric actuator for movement
- 2 Seat section
- 3 Leg-rest section equipped with gas spring for movement, with release lever placed under the seat
- 4 Stainless steel tray (MI320X-MI321X models only)
- 5 Adjustable anatomical PU thigh supports (MI320X-MI321X models only)
- 6 Frame and base
- 7 Lifting mechanism
 - 7a Electric actuator control unit
 - 7b Elevation actuator
- 8a Castors pedals (MI311X-MI321X models only)
- 8b Castors (MI311X-MI321X models only)

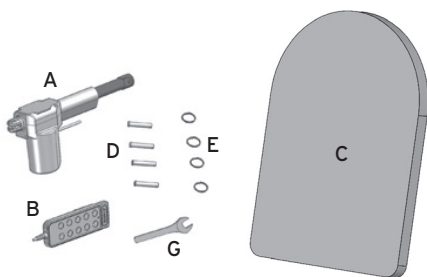Image for illustrative purposes

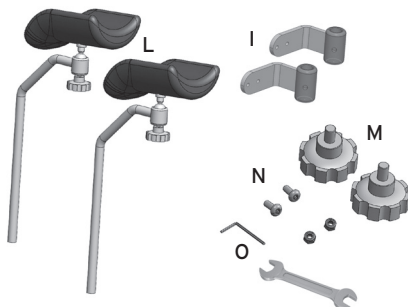Components to be assembled by the customer (for all models)

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Components to be assembled by the customer (for models MI320, MI321 gynecological couches)

- I. n.2 supports for thigh supports/accessories
- L. N.2 complete thigh supports
- M. n.2 handwheels
- N. n.4 M6 screws with nuts for leg support
- O. Tools (wrenches) for assembling the thigh supports

Note. The above components could be already installed on the bed

8. ASSEMBLING

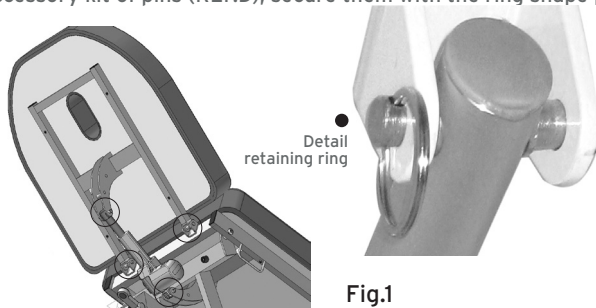
WARNING!

Perform these operations with an operator aid, be careful to avoid personal injuries or property damages during bed movements


8.1 Bed assembling

- Remove the product from the packaging and make sure that the content matches the model you requested and that the parties have not been damaged during transport. In the contrary case contact your dealer immediately
- Proceed with the assembling of the headrest and the actuator to the bed frame using the accessory kit of pins (REF.D), secure them with the ring shape pins (REF.E) as shown in Fig.1

8.2 Pedal assembling (MI311X, MI321X)

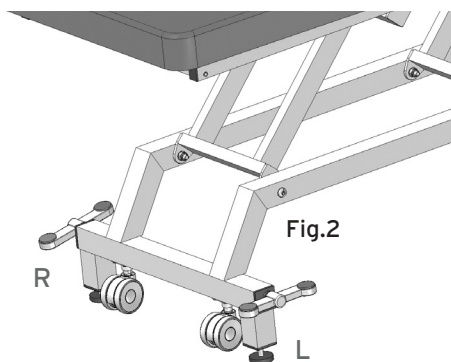Assemble the pedal control wheel (REF.F) arranging them as in Fig.2


- Presser foot adjustment

Adjust the presser foots to allow a correct operation of castors pedal control mechanism; to do this, adjust the presser foot height in such a way that the distance shown in the Fig.3 does not exceed 10 mm

Your bed is correctly assembled and ready for use.

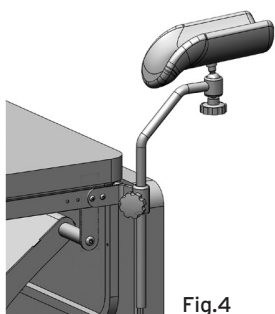8.3 Assembling thigh supports (MI320X, MI321X)

- Fix the thigh support by installing it through the appropriate holes in the front side part of the seat, using m6 through screws with nuts to be tightened in the internal part of the lateral tubular
- Insert the complete thigh support on the metal bush just installed
- Adjust the position of the thigh supports by tightening the handwheel supplied in the desired position

Fig.4

8.4 Assembling and connection of accessory footswitch kit MR333

MR333 accessories components view and list

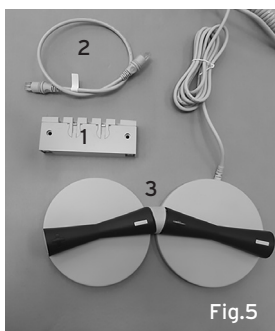
Fig.5

Assembling procedure:

- Remove the plastic cover that locks the connectors on the control unit
- Disconnect the electrical connector of the remote control on the control unit (if already installed) and replace it with the cable provided with the Hub
- Insert the cover for connectors lock on the control unit
- Connect to the Hub: the just installed cable coming from the control unit, the cable from the footswitch, the cable from the remote control
- Lock the connectors on the Hub with the plastic fastener

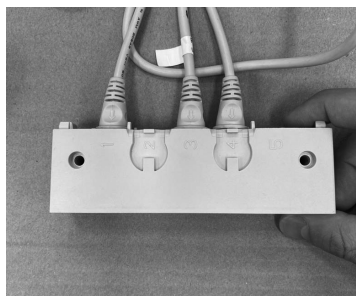
Fig.6

8.5 Assembling accessory paper roll holder head side MIA398

Fig.7

Assembling procedure:

Remove the hole caps at the upper ends of the headrest frame tubes. Insert the two tubes with brackets (of the accessory) inside the tubes of the frame whose cap has been removed, fix the brackets inserted on the headrest frame using the two screws supplied; Insert the roll holder tube between the two brackets and fix it with the brackets at the ends using the handwheels provided, as shown in Fig.7.

9. BEFORE USE

- Always check the state of wear of the mechanical parts in order to ensure a use of the device in total safety for people and things
- Check for proper assembly of the device with particular attention to the pins and locking clips of the headboard, actuators and mechanical joints in general

10. WARNINGS FOR A CORRECT USE

- Before making any adjustment on the bed make sure that this is in a stable position on the legs;
- Before cleaning or maintaining operations or simply move the bed disconnect the plug of the mains voltage;
- Do not use power strips or extension cords
- Do not connect to the device any electrical device not covered by the normal operation

WARNING!

Do not sit with the weight weighing all on the head, since the maximum declared load for the bed is to be understood "evenly distributed on its surface"; otherwise the frame could be seriously and irreparably damaged. Moretti Spa declines any liability for damages arising from improper use and/or other than that reported in this user manual.

WARNING!

Before starting the movement and during the movement of the table top in elevation and the backrest and legrest sections in inclination, make sure that there are no limbs or other parts of the patient's body that could get trapped in the gaps between the sections or in the structure moving, resulting in damage from trapping or crushing. To avoid accidental falls, before moving the table in height or in its sections, make sure that the patient has assumed a stable and balanced position and posture.

11. HOW TO USE

Before each use, connect the power supply cable to the 220 vac mains socket. Use the supplied remote control to adjust the height of the bed and the inclination of the headboard. As for the leg section, operate the lever to manually adjust the position of the section. To reposition the leg section, pull the gas spring release lever located under the seat.

12. USE THE REMOTE CONTROL

Rif.8	Backrest button "UP"
Rif.9	Backrest button "DOWN"
Rif.10	Bed lift button "UP"
Rif.11	Bed lift button "DOWN"

13. CASTORS USE (MI311X, MI321X)

WARNING!

- Before moving the bed, always unplug the supply cable
- Do not move the bed with patients on board

These models of beds have 4 swivel wheels to facilitate movement. The system of wheels has been studied to ensure a fast and easy intervention through a pedal control. When it is desired to move the bed is sufficient to press with a foot the two pedals on the same side of the bed (right or left) in such a way that the bed is lifted by the supporting feet. Move the bed to the desired location, then press the footrests to lower the wheels and replace the bed on its feet so that it is stable.

14. MAINTENANCE

The SKEMA devices by MORETTI are checked carefully and supplied with EC mark, once launched on the market. For the safety of the patient and the doctor we recommend to check the suitability of your product at least once a year.

For periodic inspection we refer to the check of the following parts:

- The supporting frame of the bed and the lifting mechanism with their fittings, safety devices
- Check installation and operation of the control mechanism wheels
- Check the integrity of cables and plugs
- Check all welding points

If a repair is needed, please use only approved parts and accessories

15. CLEANING AND DISINFECTION
15.1 Cleaning

Use a damp cloth and mild soap. Then dry thoroughly before use

Use water at a temperature not exceeding 30° C

Do not use washing machines jet of water and steam

15.2 Disinfection

If you need to disinfect the device use a common disinfectant cleaner

NOTE Never use acids, alkalis or solvents such as acetone

16. CONDITIONS OF DISPOSAL

16.1 General conditions of disposal

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the beds in the appropriate disposal areas for recycling

16.2 Correct treatment of electrical parts (Directive 2012/19/UE)

At the end of its life, the product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment or at retailers that provide this service. By ensuring these batteries are disposed of correctly, you will help prevent potentially negative consequences for the environment and human health which could otherwise be caused by inappropriate waste handling of the battery. To remark the need to dispose of electrical equipment separately, the products is marked with crossed mobile waste bin

17. DECLARATION OF ELECTROMAGNETIC COMPATIBILITY

17.1 Guidance and Manufacturer's Declaration - Electromagnetic Emissions

This device is intended for use in the electromagnetic environment specified below. The user of this device should make sure it is used in such an environment.

EMISSIONS TEST	COMPLIANCE	ELECTROMAGNETIC ENVIRONMENT - GUIDANCE
RF emissions CISPR 11	Group 1	The device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network
RF emissions CISPR 11	Class B	
Harmonic emissions IEC61000-3-2	Class A	
Voltage fluctuations / Flicker emissions IEC61000-3-3	Compliant	

⚠ Warning:

- The device should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the device should be observed to verify normal operation in the configuration in which it will be used.
- Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the pump, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

17.2 Guidance and Manufacturer's Declaration - Electromagnetic Immunity

This device is intended for use in the electromagnetic environment specified below. The user of this device should make sure it is used in such an environment.

Basic EMC standard	Immunity Test Levels		Compliance Levels	Electromagnetic Environment-Guidance
	Professional healthcare facility environment	HOME HEALTHCARE ENVIRONMENT		
Electrostatic Discharge (ESD) IEC61000-4-2	±8kV contact ±15kV air	HOME HEALTHCARE ENVIRONMENT	±8kV contact ±15kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.

Basic EMC standard	Immunity Test Levels		Compliance Levels	Electromagnetic Environment-Guidance
	Professional healthcare facility environment	HOME HEALTHCARE ENVIRONMENT		
Electrical fast transient/ burst IEC61000-4-4	±2kV for power supply line ±1kV for input/output line		±2kV for power supply line ±1kV for input/output line	Mains power quality should be that of a typical commercial or hospital environment
Surge IEC61000-4-5	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	± 1 kV line(s) to line(s)	± 1 kV line(s) to line(s)	Mains power quality should be that of atypical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC61000-4-11	Voltage Dips: i) 100% reduction for 0,5 period, ii) 100% reduction for 1 period, iii) 30% reduction for 25/30 period, Voltage Interruptions: 100% reduction for 250/300 period		230V	Mains power quality should be that of a typical commercial or hospital environment. If the user of this device requires continued operation during power mains interruptions, it is recommended that the device be powered from an uninterruptible power supply or a battery.
Power frequency (50/60Hz) magnetic field IEC61000-4-8	30 A/m	30 A/m	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
Conducted RF IEC 61000-4-6	3 Vrms 0,15 MHz - 80 MHz 6 Vrms in ISM bands between 0,15 MHz and 80 MHz 80 % AM at 1 kHz	3 Vrms 0,15 MHz - 80 MHz 6 Vrms in ISM and amateur radio bands between 0,15 MHz and 80 MHz 80 % AM at 1 kHz	6Vrms	Portable and mobile RF communications equipment should be used no closer to any part of this device, including cables, than there commended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance =√P 150kHz to 80MHz =0.6√P 80MHz to 800MHz =1.2√P 800 MHz to 2.7 GHz Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). b Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, ^a should be less than the compliance level in each frequency ranged. Interference may occur in the vicinity of equipment marked with the following symbol:
Radiated RF EM Fields IEC61000-4-3	3 V/m 80 MHz to 2.7 GHz 80 % AM at 1 kHz 385-6000 MHz, 9-28V/m, 80% AM(1kHz) pulse mode and other modulation	10 V/m 80 MHz to 2,7 GHz 80 % AM at 1 kHz 385-6000 MHz, 9-28V/m, 80% AM(1kHz) pulse mode and other modulation	10V/m	

NOTE 1: UT is the a.c. mains voltage prior to the application of the test level

NOTE2: At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people

- a) Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the device is used exceeds the applicable RF compliance level above, the device should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the device.
- b) Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 10 V/m.

17.3 Recommended separation distances between portable and mobile RF communications equipment and this device

This device is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of this device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and this device as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz = \sqrt{P}	80 MHz to 800 MHz = $0.6\sqrt{P}$	800 MHz to 2,7 GHz $d=1.2\sqrt{P}$
0.01	0.1	0.06	0.12
0.1	0.31	0.19	0.38
1	1	0.6	1.2
10	3.1	1.9	3.8
100	10	6	12

For transmitters specified for a maximum output not listed above, the recommended separation distance "d" in meters (m) can be calculated using the equation applicable to the frequency of the transmitter, where "P" is the maximum rated power d' transmitter output in Watts (W) according to the transmitter manufacturer.

NOTE 1 At 80MHz and 800MHz if the high frequency range is applied

NOTE 2: Guideline question for not applying in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people

18. SPARE PARTS AND ACCESSORIES

For spare parts and accessories refer to Moretti main catalogue

MR333 Kit footswitch 2 channels kit

MIA398 Paper roll holder (head side)

MIA306 IV pole (2 hooks)

MIA312 Pair of folding side rails

MIA315X Pair of padded, adjustable side armrests ("X" is for the color of upholstery)

MIA316 Pair of blood sampling armrests in black PU

MIA320 Pair of anatomical thigh supports for gynecology in black PU

19. TECHNICAL FEATURES

19.1 Dimension and weight

Table dimensions (extended bed):	mm 1930x620
External dimensions MI310X:	mm 1930x620
External dimensions MI311X:	mm 1930x720
External dimensions MI32XX:	mm 1930x850 (with thigh support)
Min height:	mm 520
Max height:	mm 920
Headrest section max angle:	70°
Legrest section max angle:	90°

 Pack size	 L: 1785 mm	 P: 730 mm	 H: 535 mm
 Pack weight	MI310: 65 Kg MI311: 68 Kg MI320: 70 Kg MI321: 73 Kg		
 Bed weight	MI310: 57 Kg MI311: 60 Kg MI320: 62 Kg MI321: 65 Kg		
 Max weight	200 Kg		

19.2 Technical specifications

Required force to operate the controls on the remote control	5 N
Required force to push the elevation pedal	30 N
Input	220-240V AC 50Hz, 2.5A
Output	24 VDC
Protection Class	IPX6
Noise level	< 45 dB
Insulation Class	Class 2
Used materials for bed mainframe	Varnished steel frame, foam bed cover in PVC

20. TROUBLE SHOOTING

SYMPTOM	CAUSES	SOLUTION
No control running	<ol style="list-style-type: none"> 1. The plug is not connected 2. The remote control is not connected to the control unit 3. The remote control or the control unit are defective 	<ol style="list-style-type: none"> 1. Connect the plug 2. Check and connect the remote control 3. Contact the service center for a more precise diagnosis
The head-section elevation doesn't work	<ol style="list-style-type: none"> 1. The head-section actuator is not connected to the control unit 2. The actuator or the remote control are defective 	<ol style="list-style-type: none"> 1. Check and connect the elevator to the control unit 2. Contact the service center for a more precise diagnosis
The bed elevation doesn't work	<ol style="list-style-type: none"> 1. The bed actuator is not connected to the control unit 2. The actuator or the remote control are defective 	<ol style="list-style-type: none"> 1. Check and connect the actuator to the control unit 2. Contact the service center for a more precise diagnosis
The footswitch doesn't work	<ol style="list-style-type: none"> 1. The footswitch is not connected to the Y cable 2. The Y cable is not connected to the control unit 3. The control unit or the actuator are defective 	<ol style="list-style-type: none"> 1. Check and connect the footswitch to the Y cable 2. Connect the Y cable to the control unit 3. Contact the service center for a more precise diagnosis

21. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual.

Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to wrong installation or improper use.

Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, not authorized alteration, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

22. REPAIRING

22.1 Warranty repair

If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

22.2 Repair not covered by warranty

A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received

22.3 Non-defective devices

The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be paid by the customer

23. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 months from the receiving date

24. EXEMPT CLAUSES

Moretti does not offer any other declarations, explicit or implicit warranty or conditions, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage.

The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

WARRANTY CERTIFICATE

ENGLISH

Product _____

Purchased on (date) _____

Retailer _____

Address _____ **Town/city** _____

Sold to _____

Address _____ **Town/city** _____

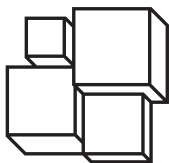
Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN ITALY

** Please consult our website for the latest available version of the user manual*

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