ITALIANO

MA RP78x 01 ITA _12-2021

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MANUALE D'ISTRUZIONE

INDICE

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ENGLISH

MA RP78x 01_B_ENG_12-2021

MULTIFUNCTION CHAIRS FOR BATHROOM

INSTRUCTION MANUAL

INDEX

1. CODES..... PAG.3

2. INTRODUCTION..... PAG.3

3. INTENDED USE..... PAG.3

4. DECLARATION OF CONFORMITY PAG.3

5. GENERAL WARNINGS PAG.4

6. SYMBOLS PAG.4

7. GENERAL DESCRIPTION PAG.4

 7.1 Multifunction chairs for bathroom pag. 4

8. BEFORE USE PAG.4

9. BEFORE ASSEMBLY PAG.4

10. ASSEMBLY INSTRUCTION PAG.5

11. SEAT PAG.5

 11.1 To lift the armrest (only for rp763) pag. 5

 11.2 To lower arms (only for rp763) pag. 5

12. USING WARNINGS..... PAG.5

13. HOW TO USE..... PAG.6

14. MAINTENANCE PAG.6

15. CLEANING AND DISINFECTION PAG.6

 15.1 Cleaning..... pag. 6

 15.2 Disinfection pag. 6

16. CONDITIONS OF DISPOSAL PAG.6

17. TECHNICAL FEATURES PAG.7

18. SPARE PARTS AND ACCESSORIES..... PAG.7

19. WARRANTY PAG.8

20. REPAIRS PAG.8

 20.1 Warranty repair pag. 8

 20.2 Repair not covered by warranty..... pag. 8

 20.3 Non-defective device pag. 8

21. SPARE PARTS PAG.8

22. EXEMPT CLAUSES PAG.8

 I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

RP780	4 in 1 functions commode chair
RP781	4 in 1 functions foldable commode chair
RP782	4 in 1 functions commode chair with castors
RP783	4 in 1 functions commode chair with foldable armrests

2. INTRODUCTION

Thank you for purchasing a **MOPEDIA** multifunction chairs for bathroom Moretti S.p.A. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance.

3. INTENDED USE

The multifunction chairs for bathroom **MOPEDIA** by Moretti are intended to be used as an aid in shower and bathroom for people with walking difficulties. They include four functions in one: a stabilizer for water rise, support for water, commode chair and shower seat.

CAUTION!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of SEAT AND SUPPORT FOR WC complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

- READ carefully the entire manual before using the device
- For a correct use of the device always consult your physician or therapist
- Make sure that the product is delivered complete and keep it packed away from heat sources
- SERVICE LIFE- the device use limit is defined by the wear of the parts
- DO NOT allow children to play or operate the device
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Conditions of disposal

7. GENERAL DESCRIPTION

7.1 Multifunction chairs for bathroom

- RP780** Fixed model
RP781 Foldable model
RP782 Demountable model with castors
RP783 Steel frame with foldable armrest

The multifunction chairs for bathroom **MOPEDIA** are made of anodized aluminum (RP763 is steel), with a comfortable chair in PVC and non-slip rubber tips (except model RP762 equipped with castors). Adjustable height every 2,5 cm. They include four functions in one: a stabilizer for water rise, support for water, commode chair and shower seat. Available fixed model, foldable and demountable with castors.

8. BEFORE USE

- Always check if there is any part worn out in order to ensure the device is used in complete safety for persons and property
- Check the correct assembly of the device

9. BEFORE ASSEMBLY

Unpack the product and check that no damage has occurred during transport. In the event that the product is damaged, please contact your nearest dealer.

10. ASSEMBLY INSTRUCTION

- RP780** Insert the toilet support to the product which is supplied already assembled, making sure to fit correctly the support with the horizontal rear bar.
- RP781** Open the sides of the frame, lift the part with the toilet support already assembled and fix the two ends to lateral parts. Next, install the rear bar.
- RP782** Insert the two horizontal bars in the supports of the side parts making them block thanks to the safety clips. Insert the toilet support paying attention to fit it properly with the horizontal rear bar.

- RP783** Insert the toilet support paying attention to fit it properly with the horizontal rear bar. Insert the backrest bar upwards into the holes of the rear rails making it block thanks to the safety clips.

11. SEAT

(fig.1)

(fig.2)

Press the safety clip to adjust the height of the chairs by removing the inner part of the leg and bringing the holes to the desired height. Check that all four legs are adjusted to the same height.

11.1 To lift the armrest (only for RP783)

Swing the armrest upward inserting them in the safety clips. (fig.1)

11.2 To lower arms (only for RP783)

Press the safety clips inward and swing the armrest downward. (fig.2)

NOTE Before you sit down:

- Place the product on a flat and solid surface and if present, lock the brakes of the castors
- Make sure that the lock latch is secure and that the product is in a stable position

12. USING WARNINGS

- Use caution on wet and slippery surfaces
- DO NOT use with patient who exceeds the weight indicated in the product label and in the technical features chapter of the present manual

(Picture with illustrative purpose only)

- The chair can be used only on a hard, flat surface. Do not use the chair on stairs, inclines or declines
- Do not use the chair in a way other than that described in this manual
- We disclaim any responsibility for improper use of the chair
- Avoid to leave the device in contact with water for long periods of time
- Periodically check the stability and hardness of the product as well as the status of the tips and castors. Replace worn once
- In case of doubts about use, adjustment or maintenance, contact your dealer
- Keep it away from children to avoid any possible risks

13. HOW TO USE

- Setting out to sit with your weight on the seat assembly. Do not sit by pushing on one side only
- Leaning always on both arms to get up and keep your balance
- In case of use of the chair by people with movement difficulties, help them always in the sitting or in getting up from the chair

CAUTION!

Do not move the aid with the patient seated

14. MAINTENANCE

The MOPEDIA devices by MORETTI are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years.

For periodic inspection means an examination of the following parts:

- The frame of the chair and connecting parts
- Check mounting and operation of the wheels with its brake
- Check all the welding points
- Check the wear of the tips where present

In case of repair, use only original spare parts and accessories.

15. CLEANING AND DISINFECTION

15.1 Cleaning

To clean the aid and the structure use neutral soap (in equal parts) mixed with water.

WARNING!

Do not use abrasive substances, acids, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of metal surfaces

The manufacture company cannot be held responsible for the damage that is caused by using materials that could damage the product surface or corrosive chemicals during cleaning.

15.2 Disinfection

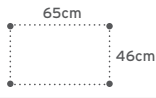
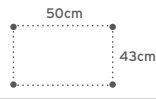
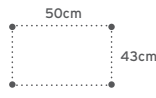
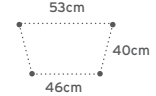
If you need to disinfect the product use a common disinfectant neutral detergent.

16. CONDITIONS OF DISPOSAL

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the bath chair in the appropriate disposal areas for recycling.

17. TECHNICAL FEATURES

Codes	Max user's weight	Adjustable height armrest	Adjustable height seat	Base	Material used
RP780	130 kg	From 66 to 76 cm	From 45 to 55 cm		Aluminum frame
RP781		From 64 to 74 cm	From 46 to 56 cm		Aluminum frame
RP782		From 64 to 74 cm	From 47 to 57 cm		Aluminum frame
RP783		From 60 to 78 cm	From 40 to 58 cm		Steel frame

The measures have a tolerance ± 1 cm

18. SPARE PARTS AND ACCESSORIES

RV7058-4

Spare seat and cover for commode chair **RP781-RP781-RP782**
Hole dimensions: 20,5 x 24 cm

RV7058-5

Spare seat and cover for commode chair **RP783**

RV7069-1

Bucket, hook and cover for commode chair **RP78x**

RV7024

\varnothing 29mm
Tips for commode chair **RP78x**
4 pcs

RV7051

Swivel casters for **RP782**
pairs

(Picture with illustrative purpose only)

19. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below.

The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual. Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use.

Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

20. REPAIRS

20.1 Warranty repair

In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location. The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty.

20.2 Repair not covered by warranty

A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

20.3 Non-defective device

The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer.

21. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the receiving date.

22. EXEMPT CLAUSES

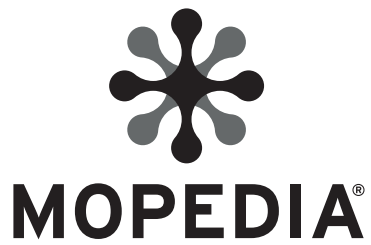
Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

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