ITALIANO

MA ST390 01 A* _ITA_ 03-2023

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INDICE

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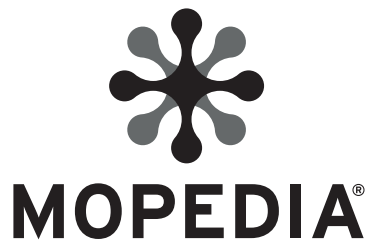
 **MORETTI S.P.A.**

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: **info@morettispa.com****MADE IN P.R.C.**

ENGLISH

MA ST390 01 A*_ENG_03 2023

POSITIONING INFLATABLE CUSHION

INSTRUCTION MANUAL

INDEX

1. CODES..... PAG.3

2. INTRODUCTION..... PAG.3

3. INTENDED USE..... PAG.3

4. DECLARATION OF CONFORMITY PAG.3

5. GENERAL WARNINGS PAG.3

6. SYMBOLS PAG.4

7. BEFORE USE PAG.4

8. WARNINGS FOR USE..... PAG.5

9. INDICATIONS FOR USE PAG.5

10. MAINTENANCE PAG.5

11. CLEANING AND DISINFECTION PAG.5

 11.1 Cleaning..... pag. 5

12. GENERAL CONDITIONS OF DISPOSAL PAG.6

13. TECHNICAL FEATURES PAG.6

14. WARRANTY PAG.6

 14.1 Exempt clauses pag. 6

ENGLISH

CE I Class Medical Device
REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND
OF THE COUNCIL of 5 April 2017 concerning medical devices

1. CODES

ST340 Positioning inflatable cushion

2. INTRODUCTION

Thank you for purchasing an OPERA cushion of the line **MOPEDIA** by Moretti. The cushions are designed and realized to meet all your needs for a practical, proper and safe use. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance.

3. INTENDED USE

The patient cushions of the line MOPEDIA by Moretti are devices to be used to give relief to the patient during sitting, rest, in case of post-operative courses.

CAUTION!

- Do not use the product for a purpose not indicated in this manual.
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer reserves the right to change the information contained in this document without previous notice.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of CUSHIONS FOR PATIENTS complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

- Please refer to this manual carefully for proper use of the device.
- Check that the product has been delivered to you intact and keep it packed away from heat sources.
- The service life of the device is determined by the wear of parts that cannot be repaired and/or replaced.
- Always pay attention to the presence of children.

- Always consult your physician or therapist for proper use of the device.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

	Product code
	Unique Device Identification
	CE mark
	Manufacturer
	Batch Lot
	Read the instruction manual
	Medical Device
	Conditions of disposal
	Do not bleach
	Wash by hand; Maximum temperature: 30°C
	Do not dry clean
	Do not tumble dry
	Do not iron

7. BEFORE USE

- Check the state of wear of the parts in order to guarantee the use of the device in total safety for people and things

8. WARNINGS FOR USE

WARNING!

- Do not use the device without the sheath
 - Do not use the device in direct contact with the skin
 - Ensure that the patient has no catheters, IV needles, electrodes or other devices anchored on the body before using the cushion
 - Do not turn the patient on cramped surfaces, or on the edge of the bed
 - The cushion does not guarantee stable static positioning, it is recommended to use other supports to support the patient in a lateral decubitus position
- Avoid leaving the device in contact with water for long periods of time.
 - In case of doubts about use, adjustment or maintenance, contact your dealer.
 - To avoid any risks, keep the device out of the reach of children

9. INDICATIONS FOR USE

- Place the cushion gently under the patient's legs as shown in (fig.1)
- Lift the cushion so that the legs fit into the leg rests (fig 2)
- Use both hands (one on the back and one on the cushion), to gently turn the patient onto their side (fig 3)

(Fig.1)

(Fig.2)

(Fig.3)

10. MAINTENANCE

The MOPEDIA devices by MORETTI are checked carefully and supplied with EC mark, once launched on the market.

11. CLEANING AND DISINFECTION

11.1 Cleaning

Washing indication for cover, where present:

Do not bleach

Wash by hand; Maximum temperature: 30°C

Do not dry clean

Do not tumble dry

Do not iron

**WARNING!**

The product cannot be washed. Only the cover can be washed as indicated above. Absolutely avoid using acidic, alkaline or solvent products such as acetone or diluent.

12. GENERAL CONDITIONS OF DISPOSAL 

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the bath chair in the appropriate disposal areas for recycling.

13. TECHNICAL FEATURES

CODES	MATERIAL	COVER	SIZE
ST390	PVC	100% Polyester	49x40x15 cm*

*measurements may vary depending on pressure applied

Note: All products are resistant to ignition according to the EN1021-1 standard, but it is recommended to pay close attention to open flames as the products could still catch fire

14. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual. Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

14.1 Exempt clauses

Except as expressly specified in this warranty and in the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, in the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

WARRANTY CERTIFICATE

ENGLISH

Product _____

Purchased on (date) _____

Retailer _____

Address _____ Town/city _____

Sold to _____

Address _____ Town/city _____

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN P.R.C.

** Please consult our website for the latest available version of the user manual*

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