SKEMA[®]FORM

MA MI481_MI482_03 A _ITA_11-2020

PREDELLINI

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ITALIANO

INDICE

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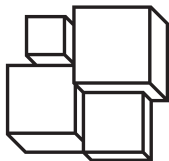
CE | Class Medical DeviceREGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices**1. CODES**

MI481	1 step stool 150 Kg
MI482	2 steps stool 150 Kg

2. INTRODUCTION

Thank you for choosing a step stool from the SKEMAFORM by MORETTI S.p.A. Their design and quality are a guarantee of comfort, safety and reliability. The step stools from the SKEMAFORM by MORETTI S.p.A. are designed and built to meet all your demands for a practical, correct use. This user manual provides for some suggestions as to how correctly use the item you have chosen and offers a lots of valuable advice for your safety. Please read carefully through the manual before using the step stools. Should you have any queries, please contact your retailer for advice and assistance.

3. INTENDED USE

The step stool by SKEMAFORM is a device designed to help people to easily get on and off onto medical examination bed.

**WARNING!**

- Do not use the product for a purpose not indicated in this manual
- Moretti S.P.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice

If you should have any doubts please contact your dealer who will help and advice you correctly.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of STEP STOOL complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

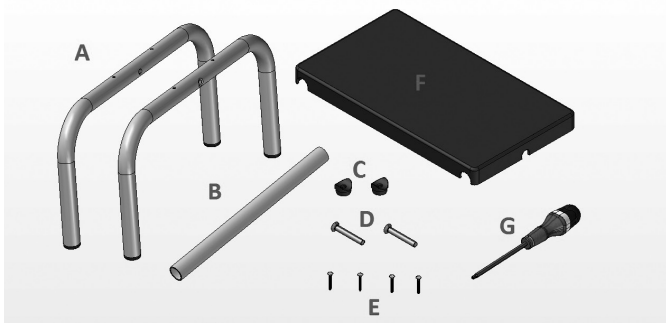
- For proper use of the device refer to this manual.
- Keep the packaged product away from any source of heat because the packaging is made of cardboard.
- The useful life of the accessory is determined by wear of parts cannot be repaired and/or replaced.
- Be careful when children are nearby.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

	Product code
	Unique Device Identification
	CE mark
	Manufacturer
	Batch Lot
	Read the instruction manual
	Medical Device
	Conditions of disposal

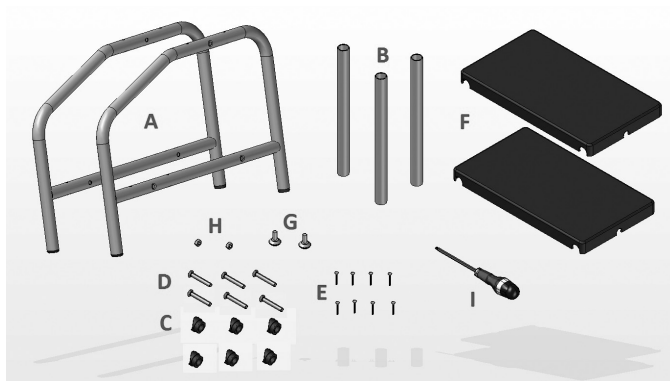
7. GENERAL DESCRIPTION

7.1 Parts view and description MI481 (1 step stool)

Parts list

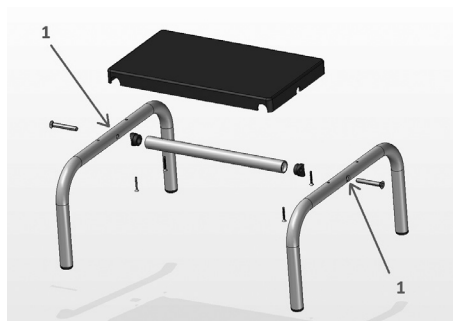
- A. Legs (2 pcs)
- B. Connecting crossbar (1 pc)
- C. Saddle washer (2 pcs)
- D. M8 Countersunk head screws (2 pcs)
- E. Self-thread screws (4 pcs)
- F. Platform (1 pc)
- G. Tool (1 pc)

7.2 Parts view and description MI482 (2 steps stool)

Parts list

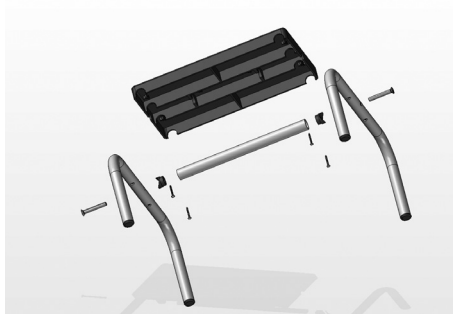
- A. Legs (2 pcs)
- B. Connecting crossbars (3 pcs)
- C. Saddle washer (6 pcs)
- D. M8 countersunk head screws (6 pcs)
- E. Self-thread screws (8 pcs)
- F. Platform (2 pcs)
- G. Adjustment feet (2 pcs)
- H. Nuts for Adjustment feet (2 pcs)
- I. Tool (1 pc)

8. ASSEMBLY
8.1 Installation instructions MI481 (1 step stool)

1. Place the two outer legs (ref.A) with the flaring of the central hole facing outwards (ref.1 pictures 1-2)
2. First, assemble the legs and the crossbar (ref.B) by the flared fittings and M8 screws like the picture (ref.1 pictures 1-2)

Pictures 1

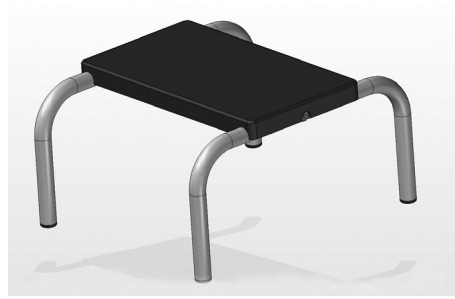
Pictures 2

3. Do not yet fully tighten these screws, but leave slightly loose components together so they can move
4. Place the platform on the crossbar of the previously assembled frame, making sure that all the tubes go to lodge perfectly into the platform slots (pictures 3)

Pictures 3

Pictures 4

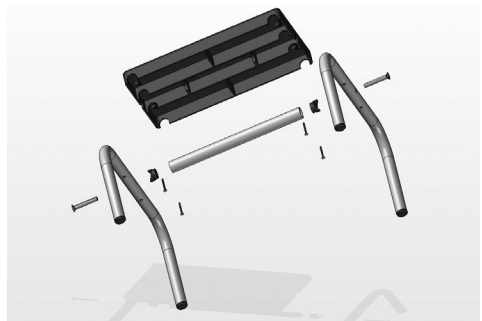
5. At this point, screw the self-thread screws (ref.E) by the side legs holes on the platform
6. Tighten them securely keeping the leg upright during this operation
7. At this point tighten well the frame screws previously assembled
8. Now your step stool MI481 is fully assembled and ready for use (pictures 4)

8.2 Installation instructions MI482 (2 steps stool)

1. Place the two outer legs (ref.A) with the flaring of the central hole facing outwards (ref.1 pictures 1-2)
2. First, Assemble the legs and the crossbars (ref.B) by the flared fittings and M8 screws like the picture(ref.1 pictures 5-6)
3. Do not yet fully tighten these screws, but leave slightly loose components together so they can move

Pictures 5

Pictures 6

4. Place the platform on the crossbar of the previously assembled frame, making sure that all the tubes go to lodge perfectly into the platform slots (pictures 6-7)

Pictures 7

Pictures 8

5. At this point, screw the self-thread screws (ref.E) by the side legs holes on the platform
6. Tighten them securely keeping the leg upright during this operation
7. At this point tighten well the frame screws previously assembled
8. Now your step stool MI482 is fully assembled and ready for use (pictures 8)

9. HOW TO USE

Choose the type of the step stool according to the height of the person who has to get on the bed. Place the step of SKEMAFORM line by Moretti on a firm and leveled surface to the side of a bed. Adjust the height of the back legs using the adjusters. Get on to the step with the aid of the hand placed on the bed. To get off, use the same procedure.

CAUTIONS!

- Do not exceed the maximum load of 150 kg
- Before use, always make sure that the step stool SKEMAFORM is securely assembled and hasn't any damage that could compromise the stability of the product.

10. MAINTENANCE

The MOPEDIA devices by MORETTI are checked carefully and supplied with EC mark, once launched on the market.

For patient and physician's safety we recommend to let check the device by the manufacturer or by an authorized laboratory every 2 years. In case of repair, use only original spare parts and accessories. More specifically, please check that all screws and bolts are well tightened.

11. CLEANING AND DISINFECTION

11.1 Cleaning

To clean the step stool use a solution of alcohol and neutral soap (in equal parts) mixed with water.

11.2 Disinfection

If you need to disinfect the product use a common disinfectant neutral detergent.

N.B. Never use acids, alkalines or solvents such as acetone or diluent. Keep away from light, moisture, heat.

12. CONDITIONS OF DISPOSAL

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the kit in the appropriate disposal areas for recycling.

13. TECHNICAL FEATURES

13.1 Dimensions and weight

MI481 - MI482

Step stool size: mm 380x200

Step height: mm 200

MI481 size	 L: 380 mm	 W: 450 mm	 H: 240 mm
MI482 size	 L: 380 mm	 W: 500 mm	 H: 410 mm
 MI481 Weight	4 kg		
 MI482 Weight	6,5 kg		
 Maximun Load	150 kg		

14. WARRANTY

Every Moretti product is guaranteed for 2 (two) YEARS from the purchase date, except the restrictions specified below. The warranty shall not apply in case of damage caused by improper use, abuse, alterations of the product or any other use not in compliance with the user manual. The proper use of the product is indicated in the user manual. Moretti S.p.A. shall not be liable for any damages or personal injuries due to incorrect assembly/improper use of the device, not compliant with the user manual. Moretti does not guarantee its products against damages or faults in the following cases: natural disasters, non authorized maintenance or reparations, damages caused by electrical supply problems (if provided), use of parts or components not provided by Moretti, failure in accomplishing the guidelines and instructions for use, non authorized alterations, damages due to the transport (different from Moretti original delivery) or failure in maintaining the device as indicated in the manual. The components subject to wear are not covered by this warranty if the damage is due to the normal use of the product.

15. REPAIRING

15.1 Warranty repair

If a Moretti product shows material or manufacturing faults during the warranty period, Moretti S.p.A., together with the client, will consider whether the product fault is covered by warranty. Moretti S.p.A., at its unquestionable discretion, may replace or repair the item under warranty in a Moretti reseller point or at its own qualified offices. Labour costs for repairing the product may be at Moretti's expense if it is determined the reparation is under warranty. A reparation or replacement does not renew nor extend the warranty.

15.2 Repair not covered by warranty

A product, not covered by warranty, may be returned for reparation only after receiving prior authorization from Moretti client service. Labour and delivery costs for an out-of-warranty reparation will be fully at the client's or reseller's expense. Reparations of out-of-warranty products are guaranteed for 6 (six) months, from the day the client receives the repaired product.

15.3 Non defective device

The client is informed if, after examining and testing a returning product, Moretti decides the product is not defective. The product will be sent back to the client and all the return delivery costs will be at his expense.

16. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the day you receive them.

17. EXEMPT CLAUSES

Except what clearly specified in this warranty and in accordance with the law, Moretti S.p.A. does not offer its clients any other declaration, warranty or condition, expressed or implicit, including any declaration, warranty or condition of merchantability, of non violation or non interference, of suitability for a specific aim. Moretti S.p.A. does not guarantee that the use of Moretti products will be continuous and faultless. The duration of possible implicit warranties that may be applied in accordance with the law is limited to the warranty period.

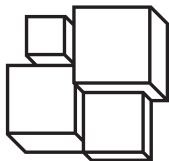
Some states or countries do not allow limitations to the duration of an implicit warranty, or the exoneration, or the limitations of accidental or indirect damages related to products for consumers. In such states or countries some exonerations or limitation of this warranty may not be applied to the user. This warranty is subject to modifications without any prior notice.

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