**MOPEDIA®**

MA RIA201\_RIA211\_RIA220 02 A\* ITA 11\_2023

**IMBRACATURE PER MUEVO HOME**

---

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ITALIANO

## INDICE

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**CE** I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL  
of 5 April 2017 concerning medical devices

**1. CODES**

<b>RIA201</b>	Standard sling for Muevo Home RI900 and RI905 patient lift
<b>RIA211</b>	Sling for toilet for Muevo Home RI900 and RI905 patient lift
<b>RIA220</b>	Sling for support and walking for Muevo Home RI900 and RI905 patient lift

**2. INTRODUCTION**

Thank you for purchasing a MOPEDIA sling by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly.

**3. INTENDED USE**

The MOPEDIA by MORETTI slings are designed to transport patient with mobility disease.

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice

**4. DECLARATION OF CONFORMITY**

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of MUEVO HOME PATIENT LIFT complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017. For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

**Note:** Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

## 5. GENERAL WARNING

- DO NOT use this product or any available optional equipment without first completely reading and understanding this instruction manual.
- Advise with physician or therapist to determinate the correct adjustment and the correct use of the sling
- Disinfect the item after every usage
- SERVICE LIFE- the sling use limit is defined by the wear of the parts
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

- Carefully read the patient lift RI900 and RI905 manual and follow the instruction suggested
- In case of use of harnesses with a different maximum load from the one provided for the patient lift, always use the lowest maximum load.

## 6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Conditions of disposal

Max person weight

Maximum washing temperature 30°C; normal process

Do not dry clean

Do not bleach

Do not tumble dry

Do not iron

## 7. GENERAL DESCRIPTION

Slings for patient lift Muevo Home. Made of 100% polyester fabric, tearproof, highly breathable, stitched with polyester thread and edged with sturdy tape in 100% polyester. The edging tape is continuous and sewn around the entire perimeter of the harness in order to ensure greater toughness.

The slings/seats are equipped with 4 attachment points with high-strength buckles to allow adjustment of the length of the support straps, in order to obtain different heights and different inclinations of the seat, to adapt to the anatomy of the person and the height of the patient's collection and unloading.

The eyelets present in the female part of the buckles must be anchored to the 4 hooks present on the horizontal lifting arms, the male buckles with the relative loop present in the harness laces to adjust in length (stretch or shorten) the seat support straps and consequently be able to fix the height of the harness and the inclination so as to adapt the positioning of the harness to the body and user needs.

Seats/slings available for R1900 and R1905:

- Standard version (RIA201)
- Toilet version (RIA211)
- Support/walking version (RIA220)

All slings are "one size".

**Nota:** The RIA201 and RIA211 slings can be used with the patient sitting on the sling; the RIA220 sling for support and walking is dedicated to use with an upright posture, with cooperative patients who are able to stand and walk using support, leaning on themselves to the arms of the muevo home and supported at the bottom of the pelvis by the harness.

## 8. BEFORE USE

Always check the good state of integrity of the harness by checking in particular the seams and the condition of the straps, buckles (male and female) and plastic loops.

## 9. WARNING FOR CORRECT USE

- Before use, make sure that the device is not damaged: the fabric, the border, the strap, the seams, the buckles and plastic loops with suspension loops must be intact, in a good state.
- Periodically check for any damage or usury, every 6 month at least, in particular if you often clean the sling.
- Make sure that the female part of the buckles with the relative tape eyelets, the buckles and the plastic loops are properly hooked to the hooks of the horizontal arms.
- Make sure that the buckles and loops on the 4 harness straps are in the correct position before use, and that when the buckle is hooked up between the male (seat side) and female (support arms side) there is a 'click' sound to confirm correct positioning (subsequently); also test the correct installation by pulling down the harness laces.
- Store and use away from flame or source of heat.
- Do not exceed the maximum load

- Always consult the user manual of the patient lift RI900 and RI905 on which you install the harness and strictly follow the instructions given.
- In case of use of sling with a different maximum load from the one provided for the stand up lifter, always use the lowest maximum load

## 10. INSTRUCTIONS FOR INSTALLATION AND USE

The RIA201, RIA211 and RIA220 slings in the MOPEDIA line by Moretti are suitable for use with Muevo Home RI900 and RI905 transfer and lifting device.

The RIA201, RIA211 and RIA220 harnesses are equipped with 4 attachment points (2 for each side) with adjustable heights using buckles and loops which, by sliding them on the straps, allow you to reach different adjustment positions of the person's position and height from the ground with the harness/seat. If the hooks of the horizontal arms of the RI900 or RI905 device are not equipped with the female part of the buckle or if it is necessary to replace them, insert the female buckles with the slotted strap on the hooks through the special slot as shown in the figure below (Fig.i).

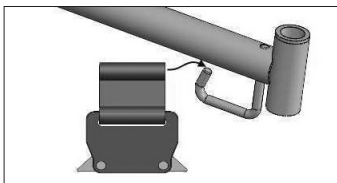
Fig. i  
Images for illustrative purpose

Attach the attachment points of the harnesses (buckles with "male") to the corresponding "female" buckles on the horizontal arm according to the following couplings: a with A, b with B, c with C, d with D (Figs. ii, iii, iv, v, vi, vii)

Fig. ii

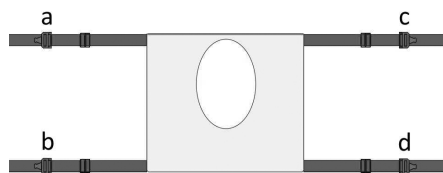
Fig. iii

Fig. iv

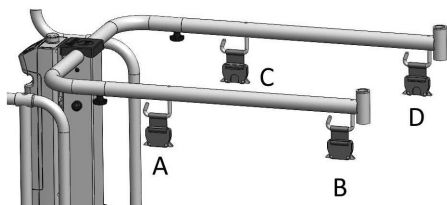
Fig. v

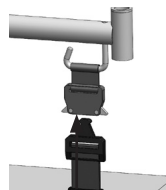
Fig. vi

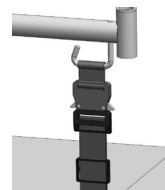
Fig. vii

Image for illustrative purposes

 **WARNING!** Confirmation of the correct installation of the harness buckles is the 'click' when the male buckle is inserted into the female. Also check manually, by pulling the straps downwards, that all the buckles are correctly fitted.

 **WARNING!** Do not move the patient without the backrest installed on the RI900 or RI905 device! Always take care, before moving the person, to check the correct installation of the seat by checking the correct fastening of the buckles and the backrest by checking the complete sliding of the lower tube on the end bushing of the arm.

**11. MAINTENANCE**

The devices MOPEDIA by MORETTI are checked carefully and supplied with EC mark, once launched on the market.

For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 6 months.

**12. CLEANING AND DISINFECTION**

Maximum washing temperature 30°C; normal process

Do not bleach

Do not tumble dry

Do not iron


Do not dry clean

### 13. CONDITIONS OF DISPOSAL


In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the sling in the appropriate disposal areas for recycling.

### 14. TECHNICAL FEATURES


MATERIAL 100% Polyester  
 DIMENSIONS MEASURES IN CM  
 MAX LOADING WEIGHT 180 Kg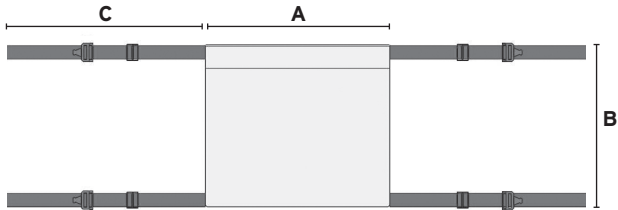

Cod.	A	B	C
RIA201	56	48	45

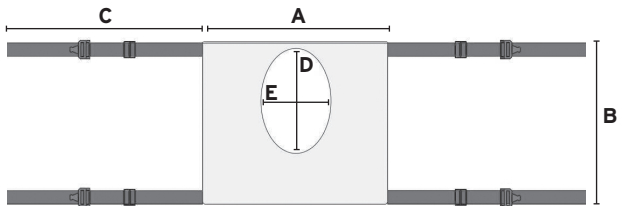Cod.	A	B	C	D	E
RIA211	56	48	45	32	21

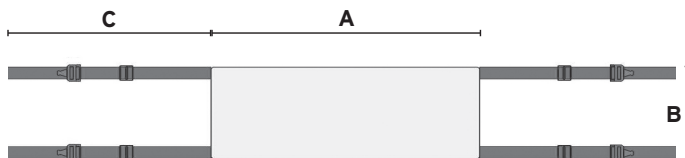Cod.	A	B	C
RIA220	65	22	45

## 15. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and the warranty will not be valid if the instructions for use are not strictly adhered to.

The correct intended use is specified in this manual.

Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to wrong installation or improper use.

Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, not authorized alteration, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

## 16. EXEMPT CLAUSES

Moretti does not offer any other declaration, explicit or implicit warranty or conditions, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

ENGLISH

**WARRANTY CERTIFICATE**

**Product** \_\_\_\_\_

**Purchased on (date)** \_\_\_\_\_

**Retailer** \_\_\_\_\_

**Address** \_\_\_\_\_ **Town/city** \_\_\_\_\_

**Sold to** \_\_\_\_\_

**Address** \_\_\_\_\_ **Town/city** \_\_\_\_\_

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

[www.morettispa.com](http://www.morettispa.com) email: [info@morettispa.com](mailto:info@morettispa.com)

**MADE IN ITALY**

*\* Please consult our website for the latest available version of the user manual*

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