**MOPEDIA®**

MA RP170\_RP171\_RP172\_RP175 02 A\_ITA\_08 2020

**REGGIBRACCIO**

---

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ITALIANO

## INDICE

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CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL  
of 5 April 2017 concerning medical devices**1. CODES**

RP170 Arm sling in cotton - Small

RP171 Arm sling in cotton - Medium

RP172 Arm sling in cotton - Large

RP175 Arm sling In foam - Covered with hypoallergenic tissue

**2. INTRODUCTION**

Thank you for purchasing a Mopedia ARM SLINGS by Moretti S.p.A. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance.

**3. INTENDED USE**

The arm slings MOPEDIA by Moretti are suitable to be used with plaster or after sprain.

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

**4. DECLARATION OF CONFORMITY**

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of ARM SLING complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

**Note:** Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

## 5. GENERAL WARNINGS

- READ carefully the entire manual before using the device .
- Make sure that the product is delivered complete and keep it packed away from heat sources
- SERVICE LIFE- the device use limit is defined by the wear of the parts
- DO NOT allow children to play or operate the device
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

- For the correct choice of the type of collar and for its proper use, always consult your doctor or therapist.

## 6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Conditions of disposal

Wash by hand; Maximum temperature 40°C

Rinse very gently, do not wring

Do not bleach

Do not iron

Flat drying

## 7. GENERAL DESCRIPTION

### 7.1 Arm sling in cotton

RP170 Small - RP171 Medium - RP172 Large

### 7.2 Arm sling In foam - Covered with hypoallergenic tissue

RP175 Unique size

## 8. BEFORE USE

- Check if the device is damaged in order to guarantee a safe use for people and things

## 9. USING WARNINGS

- For a correct use of the device, refer to this manual
- In the event of uncertainty regarding the adjustment or maintenance of the product, contact your dealer
- To avoid any risks to keep the device away from children.
- Do not use the device in the presence of deterioration or damage that may impair the effectiveness

## 6

### 10. HOW TO USE

Put your arm inside the cotton sling, adjust the regulation to immobilize the arm.

### 11. MAINTENANCE

The Mopedia devices by MORETTI are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years.

### 12. CLEANING

Wash by hand; Maximum temperature 40°C

Rinse very gently, do not wring

Do not bleach

Do not iron

Flat drying

**Note:** Never use acids, alkalis or solvents such as acetone

### 13. CONDITION OF DISPOSAL

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the bath chair in the appropriate disposal areas for recycling.

### 14. TECHNICAL FEATURES

#### 14.1 Dimensions RP170 - RP171 - RP172

Codici prodotto	RP170	RP171	RP172
 W:	34 cm	35,5 cm	37 cm
 H:	19,5 cm	20 cm	22 cm

### 15. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of ( two ) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual. Moretti is not responsible for

damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use.

Moretti Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

### 15.1 Exempt clauses

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

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