MOPEDIA®

ITALIANO

MA RP928-RP929 01A_ITA_10 2021

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INDICE

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Via _____ Località _____

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com**MADE IN P.R.C.**

MOPEDIA®

MA RP928-RP929 01A_ENG_10 2021

PEDAL EXERCISER

INSTRUCTION MANUAL

ENGLISH

INDICE

1. CODES	PAG.3
2. INTRODUCTION	PAG.3
3. INTENDED USE	PAG.3
4. DECLARATION OF CONFORMITY	PAG.3
5. WARNINGS	PAG.4
6. SYMBOLS	PAG.4
7. GENERAL DESCRIPTION	PAG.5
7.1 Description of parts RP928 - CROSSY UP	pag. 5
7.2 Description of parts RP929 - CROSSY DOUBLE	pag. 5
8. ASSEMBLY	PAG.5
8.1 Assembly RP928	pag. 5
8.2 Assembly RP929	pag. 6
9. BEFORE USE	PAG.7
10. WARNINGS FOR USE	PAG.7
11. INSTRUCTION FOR USE	PAG.7
11.1 Use with legs	pag. 8
11.2 Use with arms (only for RP929).....	pag. 8
12. MAINTENANCE	PAG.8
13. CLEANING AND DISINFECTION	PAG.9
13.1 Cleaning.....	pag. 9
14. CONDITIONS OF DISPOSAL	PAG.9
15. SPARE PARTS AND ACCESSORIES	PAG.9
16. TECHNICAL FEATURES	PAG.9
17. WARRANTY	PAG.10
18. REPAIRING	PAG.10
18.1 Warranty repair	pag. 10
18.2 Repair not covered by warranty.....	pag. 10
18.3 Non defective devices	pag. 10
19. SPARE PARTS	PAG.10
20. EXEMPT CLAUSES	PAG.10

CE | Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

- RP928** Painted steel high rehabilitation pedal exerciser with pedals and handles CROSSY UP
RP929 Painted steel high rehabilitation pedal exerciser with double pedals CROSSY DOUBLE

2. INTRODUCTION

Thank you for choosing a **MOPEDIA** pedal exerciser by Moretti S.p.A. The Moretti pedal exercisers are designed and built to meet all your demands for a practical, correct and safe use. This user manual provides some suggestions as to how correctly use the device you have chosen and offers a lots of valuable advice for your safety. Please read carefully through the manual before using the pedal exerciser. Should you have any queries, please contact your retailer for advice and assistance.

Note: Check that all parts of the product have not been damaged during the shipment. In case of damage do not use the product and contact the retailer for further instructions.

WARNING!

- Do not use the product for a purpose not indicated in this manual.
- Moretti S.P.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer reserves the right to change the information contained in this document without previous notice.

3. INTENDED USE

The pedal exerciser is a medical device indicated for people with mobility difficulties and it is indicated to reestablish coordination and the muscle mass of arms and legs.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of PEDAL EXERCISERS complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. WARNINGS

WARNING!

Always pay close attention to the presence of moving parts that could cause limb entrapment and personal injuries.

- For a proper use of the device always consult your doctor or therapist.
- Keep the packed product away from any heat source as the packaging is made of carton.
- The useful life of the accessory is determined by wear of parts that cannot be repaired and/or replaced.
- Be careful when children are nearby.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device


Conditions of disposal

7. GENERAL DESCRIPTION

Every CROSSY pedal exerciser of the line Mopedia by Moretti is equipped with four non-slip tips, pedals with adjustable foot straps, an handle for the intensity regulation.


7.1 Description of parts RP928 - CROSSY UP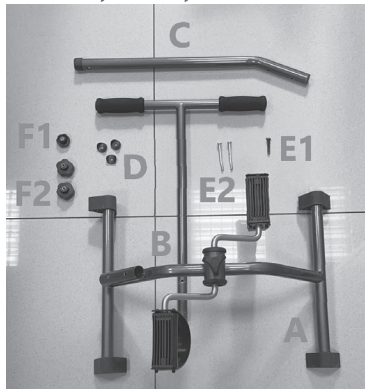

List of parts

- A - nr.1 lower base with pedals;
- B - nr.1 upper structure with handles;
- C - nr.1 inclined central structure;
- D - nr.3 semi-cylindrical spacers;
- E1 - nr.1 small pins;
- E2 - nr.2 big pins;
- F1 - nr.1 small handwheel;
- F2 - 2 large handwheels;

7.2 Description of parts RP929 - CROSSY DOUBLE

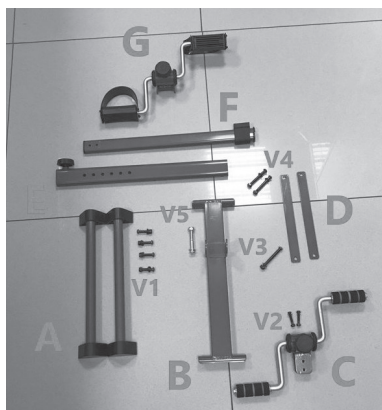List of parts

- A - nr.2 base elements with non-slip tips;
- B - nr.1 central base element;
- C - nr.1 element with handles;
- D - nr.2 rectangular tie rods;
- E - nr.1 lower column element with holes;
- F - nr.1 upper column element;
- G - nr.1 element with pedals;
- V1 - nr.4 short pins + concave washer + nut;
- V2 - nr.2 pins + washer + nut;
- V3 - nr.1 long pin + washer + nut;
- V4 - 2 pins + washer + nut;
- V5 - 1 silver plated pin + washer + nut;

Images for illustrative purpose

8. ASSEMBLY

8.1 Assembly RP928

The pedal set is supplied with the upper part disassembled, to assemble it proceed with the steps listed below:

- Insert the tubular (C), from the inclined part with the 2 holes, inside the seat on the base (A) (fig.1). Then use nr.2 pins (E2) + nr.2 semi-cylindrical spacers (D) + nr.2 large tightening handwheels (F2) to fix the parts together (fig.2).
- Then insert the tubular (B) and secure it to the structure using the remaining pin (E1) + spacer (D) + handwheel (F1). Your product is now ready to be used (fig.3).

fig.1

fig.2

fig.3

Images for illustrative purpose

8.2 Assembly RP929

The pedal set is supplied completely disassembled. Follow the steps below to assemble it:

- Assemble the tubes with rubber feet (A) and the central block of the base (B) using the 4 bolts + washer + nut (V1) as in fig.4;
- Assemble the upper handles (C) to the upper pedal element (F) using the two vertical holes and 2 bolts + washer + nut (V2) as in fig.5; For the moment, set aside the elements just assembled.
- Now place the lower pedal (G) in correspondence with the side holes on the lower element (E). Use 1 of the 2 bolts + washer + nut (V4) to assemble them as in fig.6. For the hole left free, use 1 bolt + washer + nut (V3), recognisable as slightly longer, remembering to fit the tie rods (D) as in fig.7;

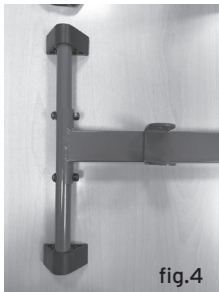
fig.4

fig.5

fig.6

fig.7

Images for illustrative purpose

- Now join the lower element (E) with the "U" shaped joint on the upper face of the base (B), using no.1 bolt + washer + nut (V5) as in fig.8. Take care to insert the pin on the side where the joint has the square hole.
- Now fix the tie rods (D) to 1 of the 3 holes on the side of the base (B) according to the desired inclination, using the second pin + washer + nut (V4) left available, as in fig.9;
- Unscrew the tightening handwheel from the lower element (E), raise the plastic cap of the upper element (F) and insert the upper element into the lower one (E) after having pressed the silver button as in fig.10; Once the upper element (F) is positioned at the desired height,

and after having made sure that the silver button has clicked in, lower again the plastic cap of the upper element (F) and screw the tightening handwheel as in fig.11;

fig.8

fig.9

fig.10

fig.11

Images for illustrative purpose

9. BEFORE USE

- Always verify the state of wear of the mechanical parts, in order to guarantee a safe use of the device, for people and things.
- Verify the correct assembly of the device. Pay particular attention to the blocking of the pedals and to insertion of the tips on the support legs.
- Verify the correct functioning of the brake for the regulation of the intensity effort.

WARNING!

Always consult your doctor or your therapist before doing every rehabilitative therapy.

10. WARNINGS FOR USE

- Each of the nr. 4 (four) support tips must lean against the surface in every moment during the use, in order to guarantee the correct balance of the pedal exerciser and the maximum safety.
- Do not straighten and reuse a deformed pedal exerciser. Risk of accident!
- Unauthorized changes to the pedal exercisers or use of non-original spare parts may change the product frame and create conditions of danger, which may cause serious damages and invalidate the warranty.
- The product is subject to particular efforts that depend on the quality of use and from the weight of the person. For this reason we suggest to check the integrity of the product every 3/4 months, especially if it is subjected to impacts or unnatural stress for its use.

11. INSTRUCTION FOR USE

WARNING!

The pedal exerciser require regular maintenance and must be daily checked to work in a proper way. We suggest to immediately substitute the worn or broken parts that may cause damages to the patient. It is recommended that an expert shows how to use the pedal exerciser and that the patient repeat the procedure under his/her supervision. For a correct rehabilitation it is necessary to maintain a correct posture. Before beginning the activity, consult your doctor/therapist.

- Ensure that the screws are all completely clamped before use;
- Place the product in a stable way and in a flat surface;
- The positioning of the pedal exerciser in relation to the height of the seat must permit the

- complete distension of the leg without having to incline the pelvis to the right and to the left;
- Start the activity only when the chest is almost straight;
- To gradually effectuate the rehabilitation, the pedal exerciser is equipped with brake for the regulation of the effort intensity;
- Do not take the feet off the pedals while the pedal exerciser is in fuction;
- At the end of the use, turn clockwise the handle for the brakes regulation, to increase the power and avoid that the pedals freely rotate causing possible injuries.

**WARNING!**

The non-compliance of the previous recommendations may cause injuries or serious health problems.

11.1 Use with legs

1. Place the pedal exerciser in a flat and non slip surface;
2. Sit down in front of the device at a distance that allows to easily insert the feet on the pedals. **Always place the right foot on the pedal labelled with the letter "R" and the left foot on the pedal labelled with the letter "L".**
3. Adjust the foot straps as necessary;
4. Adjust the brake hardness following the doctor or therapist's recommendations.

Note: To increase the effort intensity turn clockwise the brake handle. To decrease the intensity turn counterclockwise.

5. Activate the device doing a rotatory movement.

11.2 Use with arms (only for RP929)

1. Place the pedal exerciser in a flat and non slip surface;
2. Sit down in front of the device at a distance that allows to easily insert the feet on the pedals. **Always place the right foot on the pedal labelled with the letter "R" and the left foot on the pedal labelled with the letter "L".**
3. Regulate the hardness of the brake following your doctor or therapist's recommendations.

Note: To increase the effort intensity turn clockwise the brake handle. To decrease the intensity turn counterclockwise.

4. Firmly grab the pedals with the hands and activate the device doing a rotatory movement.

12. MAINTENANCE

The MOPEDIA devices by MORETTI are checked carefully and supplied with EC mark, once launched on the market. For patient and doctor's safety we recommend to let check the device by the manufacturer or by laboratory authorized once a year.

For periodic inspection is intended a check of the following parts:

- The supporting framework of the pedal exerciser and its connecting parts;
- Check the assembly of the pedals, of the tips and the functioning of the brake for the regulation of the effort intensity;
- Check that all the bolts are tightened;
- Verify all the welding points;
- Lubrication: the pedal exerciser is thought for a minimal maintenance; however a check every six months and the lubrication should ensure an excellent duration. The pedal exerciser must freely rotate.

In case of repair, use only original spare parts and accessories.

13. CLEANING AND DISINFECTION

13.1 Cleaning

- Use exclusively a wet cloth and neutral soap. Successively dry well before the use.
Do not use direct water jets.
- Use water at a temperature not superior than 30°C.
- Do not use pressure washer or steam.

13.2 Disinfection

- If necessary effectuate a disinfection of the product with a common disinfectant cleaner.
- A total disinfection of the product has to be effectuated regularly or when the product it is used by another user. The colour loss of some materials may be caused by particular kind of perspiration, by UV radiations, wear, heat, humidity and it is considered a natural process not covered by warranty.

Note: Absolutely avoid the use of acidic, alkaline or solvent products, such as acetone or diluent.

14. CONDITIONS OF DISPOSAL

WARNINGS FOR THE PROPER DISPOSAL OF THE PRODUCT, AS REQUIRED BY THE EUROPEAN DIRECTIVE 2012/19/UE:

At the end of its service life, the product must not be disposed of together with normal municipal waste. It must be delivered to the municipal sorted waste collection centre or to the resellers, in case they provide for this service. Sorted waste disposal reduces the environmental and health consequences of an improper disposal. Furthermore, the materials which make up the product may be recycled, thus resulting in energy and resource savings. In order to highlight the compulsory sorted disposal of the electro-medical equipment, the product itself bears the barred bin symbol.

15. SPARE PARTS AND ACCESSORIES

For spare parts and accessories, please always refer to the MORETTI master catalogue.

16. TECHNICAL FEATURES

Materials: powder coated tubular steel frame.

16.1 Dimensions and weight RP928 - CROSSY UP

External dimensions: 41.5 x 54.5 x 95 cm (L x W x H)

Product weight: 3.3 kg

16.2 Dimensions and weight of RP929 - CROSSY DOUBLE

External dimensions: 39 x 56 x (80.5-92.5) cm [W x D x (Hmin - Hmax)]

Product weight: 5.5 kg

17. WARRANTY

Every Moretti product is guaranteed for 2 (two) YEARS from the purchase date, except the restrictions specified below. The warranty shall not apply in case of damage caused by improper use, abuse, alterations of the product or any other use not in compliance with the user manual. The proper use of the product is indicated in the user manual.

Moretti S.p.A. shall not be liable for any damages or personal injuries due to incorrect assembly/improper use of the device, not compliant with the user manual. Moretti does not guarantee its products against damages or faults in the following cases: natural disasters, non authorized maintenance or reparations, damages caused by electrical supply problems (if provided), use of parts or components not provided by Moretti, failure in accomplishing the guidelines and instructions for use, non authorized alterations, damages due to the transport (different from Moretti original delivery) or failure in maintaining the device as indicated in the manual. The components subject to wear are not covered by this warranty if the damage is due to the normal use of the product.

18. REPAIRING

18.1 Warranty repair

If a Moretti product shows material or manufacturing faults during the warranty period, Moretti S.p.A., together with the customer, will consider whether the product fault is covered by warranty. Moretti S.p.A., at its unquestionable discretion, may replace or repair the item under warranty in a Moretti reseller point or at its own qualified offices. Labour costs for repairing the product may be at Moretti's expense if it is determined the reparation is under warranty. A reparation or replacement does not renew nor extend the warranty.

18.2 Repair not covered by warranty

A product, not covered by warranty, may be returned for reparation only after receiving prior authorization from Moretti customer service. Labour and delivery costs for an out-of-warranty reparation will be fully at the customer's or reseller's expense. Reparations of out-of-warranty products are guaranteed for 6 (six) months, from the day the customer receives the repaired product.

18.3 Non defective devices

The customer is informed if, after examining and testing a returning product, Moretti decides the product is not defective. The product will be sent back to the customer and all the return delivery costs will be at his expense.

19. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the day you receive them.

20. EXEMPT CLAUSES

Except what clearly specified in this warranty and in accordance with the law, Moretti S.p.A. does not offer its customers any other declaration, warranty or condition, expressed or implicit, including any declaration, warranty or condition of merchantability, of non violation or non interference, of suitability for a specific aim. Moretti S.p.A. does not guarantee that the use of Moretti products will be continuous and faultless. The duration of possible implicit warranties that may be applied in accordance with the law is limited to the warranty period.

Some states or countries do not allow limitations to the duration of an implicit warranty, or the exoneration, or the limitations of accidental or indirect damages related to products for consumers. In such states or countries some exoneration or limitation of this warranty may not be applied to the user. This warranty is subject to modifications without any prior notice.

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