SKEMA[®]FORM

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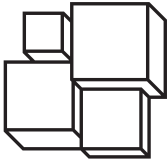
SKEMA[®]FORM

MA C11xx 03 A_ENG_05 2021

MULTI-PURPOSE MODULAR TROLLEYS "EOLO"

INSTRUCTION MANUAL

ENGLISH

INDICE

1. CODES.....	PAG.3
2. INTRODUCTION.....	PAG.3
3. INTENDED USE.....	PAG.3
4. DECLARATION OF CONFORMITY	PAG.3
5. GENERAL WARNINGS	PAG.4
6. SYMBOLS	PAG.4
7. GENERAL DESCRIPTION.....	PAG.5
7.1 Parts view and description CI100B-CI105B-CI110B-CI115B-CI122B-CI123B....	pag. 5
7.2 Parts view and description CI120B-CI125B-CI130B	pag. 6
8. ASSEMBLY	PAG.6
8.1 Assembling instructions - CI100B-CI105B-CI110B-CI115B-CI122B-CI123B....	pag. 6
8.2 Assembling instructions - CI120B-CI125B-CI130B	pag. 8
9. WARNINGS TO USE.....	PAG.10
10. CLEANING AND DISINFECTION	PAG.10
10.1 Cleaning.....	pag. 10
10.2 Disinfection	pag. 10
11. CONDITIONS OF DISPOSAL	PAG.10
12. ACCESSORIES.....	PAG.10
13. TECHNICAL FEATURES	PAG.11
13.1 Dimensioni e peso carrelli CI100B-CI105B-CI110B-CI115B-CI122B-CI123B ...	pag. 11
13.2 Dimensions e weight Trolleys CI120B-CI125B-CI130B	pag. 12
13.3 Specifications	pag. 12
14. WARRANTY	PAG.13
15. REPAIRING	PAG.13
15.1 Warranty repair	pag. 13
15.2 Repair not covered by warranty.....	pag. 13
15.3 Non defective device	pag. 13
16. SPARE PARTS	PAG.13
17. EXEMPT CLAUSES.....	PAG.13

CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

CI100B Trolley with 1 column H705mm, 1 handle and 1 shelf 300x400mm - Max Pay load 20kg
CI105B Trolley with 1 column H705mm, 1 handle and 1 shelf 350x400mm - Max Pay load 20kg
CI110B Trolley with 1 column H705mm, 1 handle and 1 shelf 300x400mm - Max Pay load 20kg
CI115B Trolley with 1 column H705mm, 1 handle and 1 shelf 350x400mm - Max Pay load 20kg
CI120B Trolley with 2 column H705mm, 2 handle and 2 shelf 350x400mm - Max Pay load 30kg
CI122B Trolley with 1 column H705mm, 1 handle and 1 shelf 350x400mm - Max Pay load 20kg
CI123B Trolley with 1 column H905mm, 1 handle and 1 shelf 350x400mm - Max Pay load 20kg
CI125B Trolley with 2 column H905mm, 2 handle and 2 shelf 350x400mm - Max Pay load 30kg
CI130B Trolley with 2 column H905mm, 2 handle and 2 shelf 350x400mm - Max Pay load 30kg

2. INTRODUCTION

Thank you for choosing a trolleys multi-purpose modular "EOLO" SKEMAFORM by MORETTI S.p.A. Their design and quality are a guarantee of comfort, safety and reliability. Trolleys multi-purpose modular from the SKEMAFORM by MORETTI S.p.A. are designed and built to meet all your demands for a practical, correct use. This user manual provides for some suggestions as to how correctly use the item you have chosen and offers a lots of valuable advice for your safety. Please read carefully through the manual before using the I.V. Poles. Should you have any queries, please contact your retailer for advice and assistance.

3. INTENDED USE

The modular multi-functional trolley "EOLO" the SKEMAFORM line is a device intended to support and hand the medical equipment and electromedical.

WARNING!

- Do not use the product for a purpose not indicated in this manual.
- Moretti S.p.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer reserve the right to change the information contained in this document without previous notice.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of TROLLEYS complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation

6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS


- For proper use of the device refer to this manual
- Keep the packaged product away from any source of heat because the packaging is made of paper
- The shelf-life of the accessory is determined by wear of parts cannot be repaired and/or replaced
- Be careful when children are nearby
- Do not exceed the max payload on the shelves shown in this manual to prevent deformation and breakage
- Before using the device, always check that the assembly screws are well tight
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

	Product code
	Unique Device Identification
	CE mark
	Manufacturer
	Batch Lot
	Read the instruction manual
	Medical Device
	Conditions of disposal

7. GENERAL DESCRIPTION

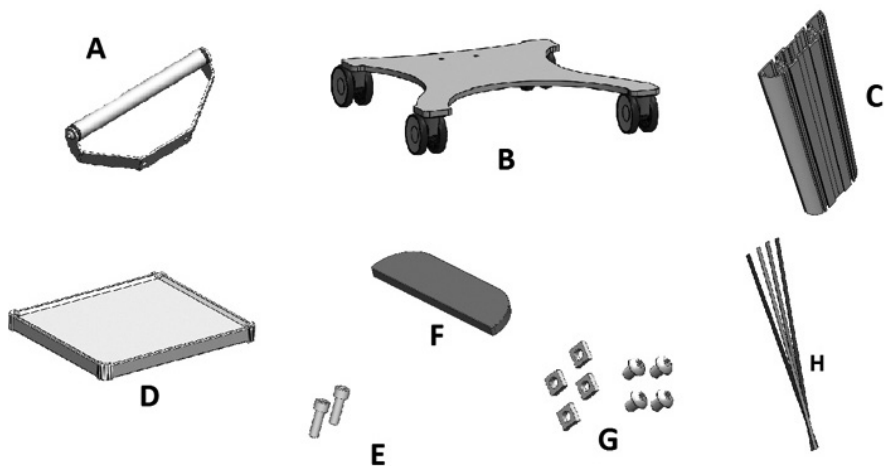7.1 Parts view and description C1100B-C1105B-C1110B-C1115B-C1122B-C1123B
(Multi-purpose Trolley with 1 column, 1 handle and 1 shelf)


ENGLISH

Parts list

- A. Push handle (1 pc)
- B. Base with casters (1 pc)
- C. Aluminum column (1 pc)
- D. Shelf with corner bumpers (1 pc)
- E. Screws for column (2 pcs)
- F. Plastic cap for column (1 pc)
- G. Screws and nuts to assemble accessories (4 pcs)
- H. Colored strips to customize the trolley (8 pcs)

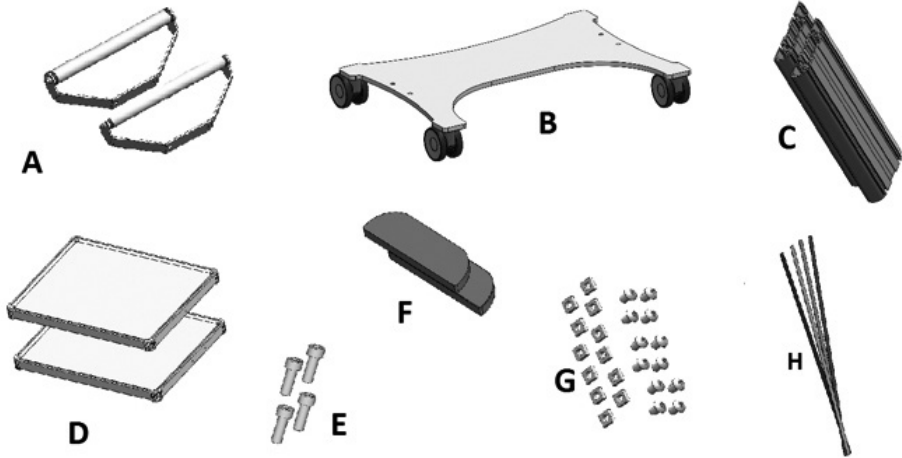7.2 Parts view and description CI120B-CI125B-CI130B (Multi-purpose Trolley with 2 columns, 2 handles and 2 shelves)

Parts list

- A. Push handles (2 pcs)
- B. Base with casters (1 pc)
- C. Aluminum columns (2 pcs)
- D. Shelves with corner bumpers (2 pcs)
- E. Screws for columns (4 pcs)
- F. Plastic caps for column (2 pcs)
- G. Screws and nuts to assemble accessories (12 pcs)
- H. Colored strips to customize the trolley (16 pcs)

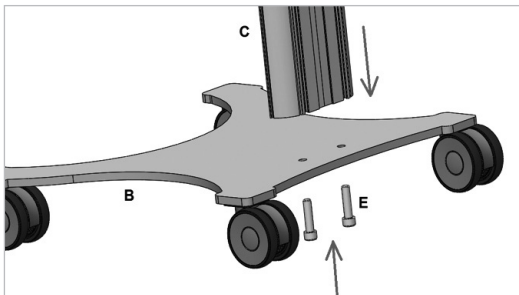
8. ASSEMBLY


WARNING!

During the assembling ask help to another operator, pay attention to avoid any injury to persons and/or damage to objects


8.1 Assembling instructions - CI100B-CI105B-CI110B-CI115B-CI122B-CI123B (Multi-purpose Trolley with 1 column, 1 handle and 1 shelf)

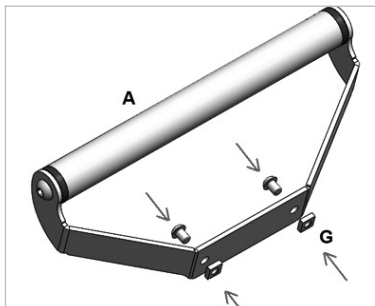
1. Place the base of the trolley (ref.B) on a horizontal plane and position the column (ref.C) as in the figure;
2. Use the screws (ref.E-2 pcs) to fix the column at the base;

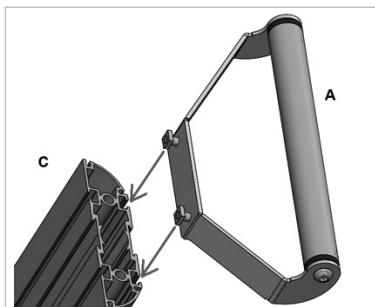
3. Prepare the handle (ref.A) with screws and nuts to assemble at the column (rif.G-2 pcs);

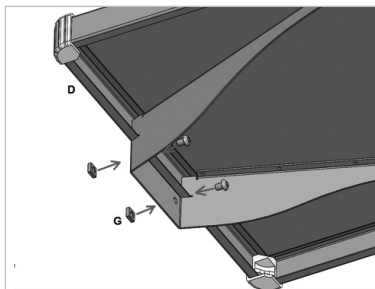
4. Insert the handle in the grooves of the column (ref.C);
5. Place the handle and tighten the screws (ref. G) at the desired height

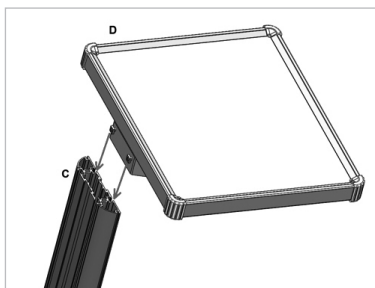
6. Prepare the shelf (ref.D) with screws and nuts to assemble at the column

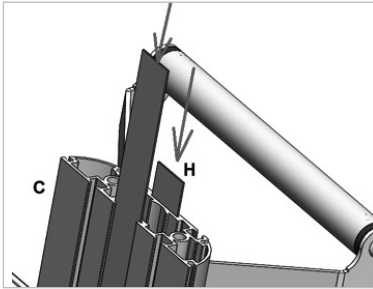7. Insert the shelf in the grooves of the column;
8. Place the shelf and tighten the screws (ref. G) at the desired height;

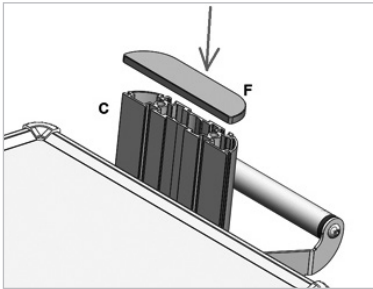
9. Now you can customize your trolley, so insert the colored strips (ref. H) in the central column grooves (ref. C)


10. With a light pressure, insert the cap (ref.F-2pc) in the ends of the column (ref.C-2pc)
11. At this point your multipurpose trolley "EOLO" is mounted and ready to use

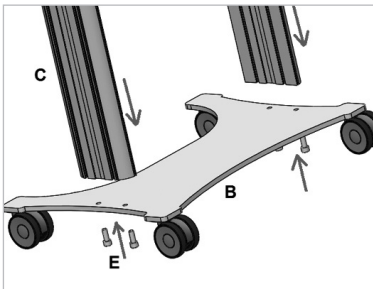
8.2 Assembling instructions - CI120B-CI125B-CI130B (Multi-purpose Trolley with 2 columns, 2 handles and 2 shelves)

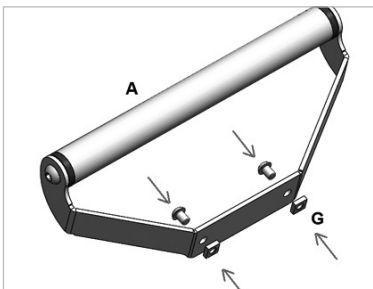
1. Place the base of the trolley (ref.B) on a horizontal plane and position the column (ref.C) as in the figure;
2. Use the screws (ref.E-2 pcs) to fix the columns at the base;

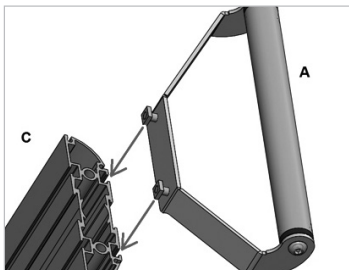
3. Prepare the handles (ref.A) with screws and nuts to assemble at the columns (rif.G-2 pcs);

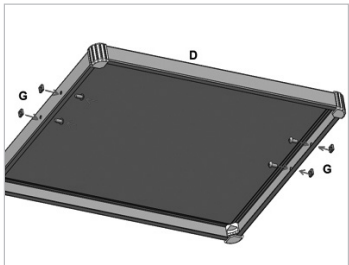
4. Insert the handles in the grooves of the column (ref.C);
5. Place the handles and tighten the screws (ref. G) at the desired height;

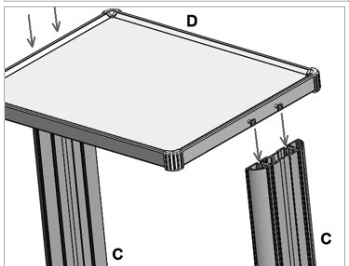
6. Prepare the shelves (ref.D) with screws and nuts to assemble at the column;

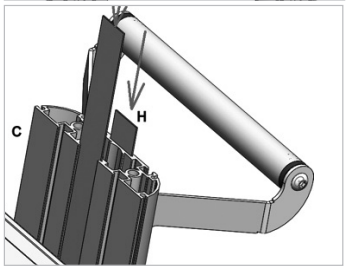7. Insert the shelves in the grooves of the column;
8. Place the shelves and tighten the screws (ref. G) at the desired height;

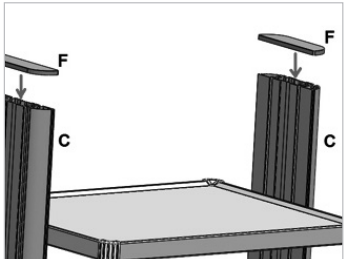9. Now you can customize your trolley, so insert the colored strips (ref. H) in the central columns grooves (ref.C);

10. With a light pressure, insert the caps (ref.F-2pc) in the ends of the columns (ref.C-2pc)
11. At this point your multipurpose trolley "EOLO" is mounted and ready to use

9. WARNINGS TO USE

- Before each use always check the state of wear of the mechanical parts so as to guarantee a use of the device in total safety for people and things
- Before each use check the correct assembly of the device, with a focus on connecting screws / and column / and at the base and those between shelves / accessories and column / and
- To move the trolley, use only the appropriate handle supplied
- Never exceed the Max payload for shelf indicated in this manual
- When The truck is not on motion, make sure that the parking brakes are operated properly

WARNING!

Do not overload the shelf, check the maximum weight on the technical features

10. CLEANING AND DISINFECTION

10.1 Cleaning

To clean the aid and the structure use a solution of alcohol and neutral soap (in equal parts) mixed with water.

WARNINGS!

Do not use abrasive substances, acids, alkaline, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of metal surfaces

The manufacture company cannot be held responsible for the damage that is caused by using materials that could damage the product surface or corrosive chemicals during cleaning.

10.2 Disinfection

If you need to disinfect the product use a common disinfectant neutral detergent.

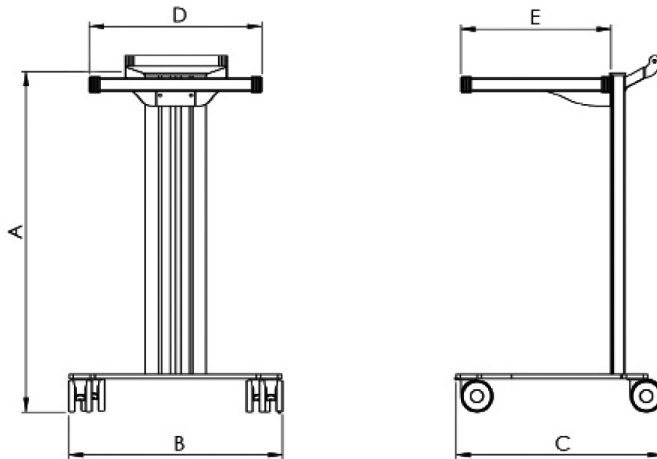
11. CONDITIONS OF DISPOSAL

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the product in the appropriate disposal areas for recycling.

12. ACCESSORIES

- CIA100** Shelf Moretti trolleys 400x300mm with cantilever joint for C1100B-C1105B-C1110B-C1115B-C1122B-C1123B
- CIA102** Shelf Moretti trolleys 400x350mm with cantilever joint for C1100B-C1105B-C1110B-C1115B-C1122B-C1123B
- CIA104** Shelf Moretti trolleys 400x350mm with side joint for C1120B
- CIA106** Shelf Moretti trolleys 500x400mm with side joint for C1125B-C1130B
- CIA110** Drawer Moretti trolleys 400x350mm for C1100B-C1105B-C1110B-C1115B-C1122B-C1123B
- CIA120** Big Drawer Moretti trolleys 500x400mm for C1120B-C1125B-C1130B
- CIA150** Wire basket Moretti trolleys, for the whole range C11XXX
- CIA152** Aluminum IV pole Moretti trolleys, for the whole range C11XXX
- CIA154** Stainless steel IV pole Moretti trolleys, for the entire range C11XXX
- CIA156** Cylinder holder Moretti trolleys for the whole range C11XXX
- CIA158** Monitor support for Moretti trolleys for the whole range C11XXX


13. TECHNICAL FEATURES
13.1 Dimensioni e peso carrelli CI100B-CI105B-CI110B-CI115B-CI122B-CI123B

Images only illustration purposes

Measures in mm

Codes	A	B	C	D	E	Weight (kg)	Max Pay Load (kg)
CI100B	800	500	500	400	300	14	20 kg
CI105B					350	14,5	
CI110B	820	480	500	400	300	10	
CI115B					350	10,5	
CI122B	800	630	500	400	350	20	
CI123B	1000					21	

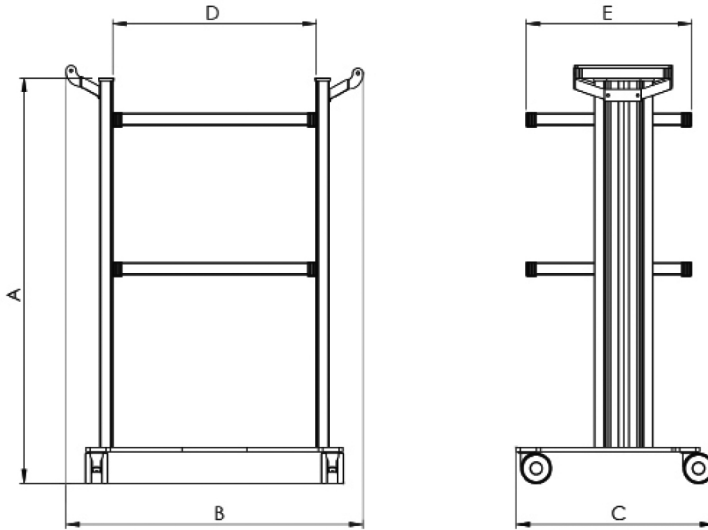13.2 Dimensions e weight Trolleys CI120B-CI125B-CI130B

Images only illustration purposes

Measures in mm							
Codes	A	B	C	D	E	Weight (kg)	Max Pay Load (kg)
CI120B	800	630	500	400	350	23,5	30 kg
CI125B	1000	730	500	500	400	25	
CI130B	1020	730	550	500	400	18	

13.3 Specifications

Column	Painted aluminum
Base and shelf	Powder coated steel
Handle	Powder coated steel and anodized aluminum tube
Castor	ø 75mm with brakes (2pcs) and without brakes (2pcs)
Corner bumpers	PLA
Colored stripes	Blue, yellow, red, green to customize the cart

Handle and shelf which can be positioned at any height

14. WARRANTY

Every Moretti product is guaranteed for 2 (two) YEARS from the purchase date, except the restrictions specified below. The warranty shall not apply in case of damage caused by improper use, abuse, alterations of the product or any other use not in compliance with the user manual. The proper use of the product is indicated in the user manual. Moretti S.p.A. shall not be liable for any damages or personal injuries due to incorrect assembly/improper use of the device, not compliant with the user manual. Moretti does not guarantee its products against damages or faults in the following cases: natural disasters, non authorized maintenance or reparations, damages caused by electrical supply problems (if provided), use of parts or components not provided by Moretti, failure in accomplishing the guidelines and instructions for use, non authorized alterations, damages due to the transport (different from Moretti original delivery) or failure in maintaining the device as indicated in the manual. The components subject to wear are not covered by this warranty if the damage is due to the normal use of the product

15. REPAIRING

15.1 Warranty repair

If a Moretti product shows material or manufacturing faults during the warranty period, Moretti S.p.A., together with the client, will consider whether the product fault is covered by warranty. Moretti S.p.A., at its unquestionable discretion, may replace or repair the item under warranty in a Moretti reseller point or at its own qualified offices. Labour costs for repairing the product may be at Moretti's expense if it is determined the reparation is under warranty. A reparation or replacement does not renew nor extend the warranty.

15.2 Repair not covered by warranty

A product, not covered by warranty, may be returned for reparation only after receiving prior authorization from Moretti client service. Labour and delivery costs for an out-of-warranty reparation will be fully at the client's or reseller's expense. Reparations of out-of-warranty products are guaranteed for 6 (six) months, from the day the client receives the repaired product.

15.3 Non defective device

The client is informed if, after examining and testing a returning product, Moretti decides the product is not defective. The product will be sent back to the client and all the return delivery costs will be at his expense.

16. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the day you receive them.

17. EXEMPT CLAUSES

Except what clearly specified in this warranty and in accordance with the law, Moretti S.p.A. does not offer its clients any other declaration, warranty or condition, expressed or implicit, including any declaration, warranty or condition of merchantability, of non violation or non interference, of suitability for a specific aim. Moretti S.p.A. does not guarantee that the use of Moretti products will be continuous and faultless. The duration of possible implicit warranties that may be applied in accordance with the law is limited to the warranty period.

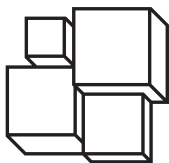
Some states or countries do not allow limitations to the duration of an implicit warranty, or the exoneration, or the limitations of accidental or indirect damages related to products for consumers. In such states or countries some exonerations or limitation of this warranty may not be applied to the user. This warranty is subject to modifications without any prior notice.

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