KYARA®

ITALIANO

MA LTK780_LTK785 01 A_IT_06-2021

SCALDALETTO

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INDICE

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 **MORETTI S.P.A.**

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com**MADE IN P.R.C.**

MA LTK780_LTK710_LTK785 01A_EN_06-2021

HEATING MATTRESS PAD

INSTRUCTION MANUAL

ENGLISH

INDEX

1. CODES PAG.3

2. INTRODUCTION..... PAG.3

3. SAFETY WARNINGS PAG.3

4. SYMBOLS PAG.5

5. GENERAL DESCRIPTION PAG.6

6. INSTRUCTION OF USE PAG.7

 6.1 How to place the heating mattress pad..... pag. 7

 6.2 Automatic switch-off function pag. 7

 6.3 Switch-off..... pag. 7

7. CLEANING AND MAINTENANCE PAG.8

 7.1. Storage pag. 8

8. GENERAL CONDITIONS OF DISPOSAL PAG.9

9. TECHNICAL FEATURES PAG.9

10. TROUBLESHOOTING..... PAG.10

11. WARRANTY PAG.10

 11.1 Exempt clauses pag. 10

ENGLISH

1. CODES

LTK780 HEATING MATTRESS PAD - SINGLE 150x80 cm

LTK785 HEATING MATTRESS PAD - DOUBLE 160x140 cm

2. INTRODUCTION

Thank you for choosing the heating mattress pad of the Kyara line by Moretti. Please read the operating instructions and safety warnings in particular before using the appliance and follow these at all times. Keep this manual and illustrative guide with the appliance for future consultation. Should you pass the device on to another user, make sure to also include this documentation. This manual includes suggestions for a correct use of the appliance and precious advices for your safety. If you have any doubts, please contact the retailer, who will be able to help you and advise you correctly.

NOTE Check that all parts of the product have not been damaged during the shipment. In case of damage do not use the product and contact the retailer for further instructions.

3. SAFETY WARNINGS

- After having removed the appliance from its packaging, verify its integrity and check for any damage caused during transport. If in doubt, do not use the appliance and contact the local retailer. Do not let children play with the packaging!
Keep the plastic wrapping out of the reach of children; risk of suffocation!
- Before connecting the appliance, check that the mains voltage data reported on the identification plate corresponds to that of the available electric mains. The identification plate is attached to the appliance.
- This appliance is intended exclusively as a heating mattress pad for home use. Any other use is improper and potentially dangerous.
- When using for the first time, you may notice a nasty odour due to the synthetic materials.
- Checking the heating mattress pad outside with your hands must not be taken into account as this may give a misleading result of the heat emitted.
- The appliance is not to be used by persons (including children) with reduced physical, sensorial or mental abilities, or with no experience or knowledge, unless supervised by a person responsible for their safety, supervision or after having received instructions regarding its use.
- Children must be supervised to ensure they do not play with the appliance.
- Do not use the appliance for ill people, for infants or for persons who are insensitive to heat. This appliance must not be used by children if the control devices have not been set beforehand by a parent or guardian or if the child has not been properly instructed on how to use the device safely.

-  **WARNING - NOT** suitable for children under the age of 3. Children under the age of 3 must not use the appliance as they are unable to react in the event of overheating.
- **DO NOT** activate the temperature control with wet hands.
- **DO NOT** pull the power cord or the appliance itself to remove the plug from the socket.
- **DO NOT** immerse the power cord and the remote control.
- To disconnect the device do not pull the cable, just unplug the plug from the socket.
- Always disconnect the appliance from the mains before cleaning or performing maintenance and when the appliance is not in use.
- To disconnect the device do not pull the cable, just unplug the plug from the socket.
- In case of a fault or malfunctioning of the appliance, switch it off and do not tamper with it. If the appliance needs to be repaired, contact an authorised technical service centre.
- If the power cord is damaged, it must be replaced by an authorised technical service centre, so as to prevent all risks.
- The appliance must be used solely with the types of control indicated on the label.
- **DO NOT** place the remote control of the heating mattress pad under the pillow, the blankets or the cover when the device is turned on.
-  **DO NOT** use the folded appliance.
 - Before storage, let the appliance cool before folding it.
 - **DO NOT** leave clothes inside the appliance if folded.
 - Inspect the appliance frequently to check for signs of wear and tear. If such signs are found or if the appliance has been used incorrectly, contact an authorised technical assistance centre before continuing to use it.
 - People with pacemakers **SHOULD NOT** use the device. The device can be used to preheat the bed, but it has to be turned off and disconnected before going to bed.
 - This appliance is not intended for medical use inside of hospitals.
 - **DO NOT** use the appliance if it is wet or in locations with an high level of humidity (e.g., bathroom).
 - **DO NOT** use the appliance to keep animals warm.
 - **DO NOT** leave the appliance running while you sleep. Always switch off the device before falling asleep
 - If the device is used for an extended period, the selector should be placed on the level 1-2 (minimum heat).
 - **DO NOT** use the appliance on swollen or painful parts of the body.
 - Immediately switch the device off if you feel pain during the use.
 - **DO NOT** use the device if highly flammable gases are present in the air.
 - **NEVER** use the device near to a gas source.
 - **DO NOT** place on the device flammable materials or materials that easily deteriorate due to the heat (e.g., carton, plastic, paper, candles).
 - **NEVER** expose the appliance to other heat sources (such as burners or ovens), sub-zero temperatures, direct sunlight, humidity.
-  **DO NOT INSERT PINS OR NEEDLES.**
 - Application of the appliance for a long time at high temperature can burn the skin.
 - **DO NOT** use the device with wet or damp hands and/or feet.
 - **DO NOT** use the device in the bathroom, laundries, always place it so as to avoid falling into the water.
 - Avoid using extensions; if the extension cable is not suitable, it may overheat.

- **DO NOT** use on adjustable beds; otherwise check before use that the cover and the cable cannot be trapped or blocked.
- **DO NOT** tuck.

WARNING!
In case of improper use, exist the risk of possible injuries.

4. SYMBOLS

Product code

Month and year of production

CE mark

Instruction manual

Disposal

Insulation class II

Do not use bleach

Hand wash at a temperature not higher than 30°

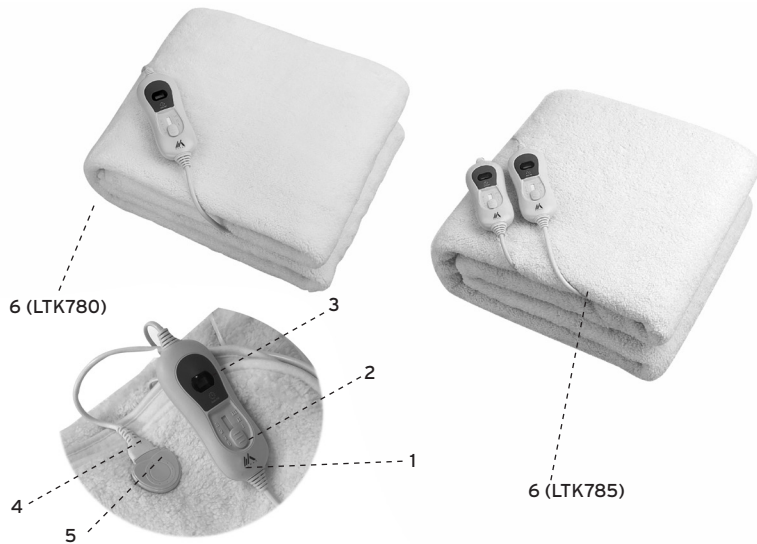
Do not dry-clean

Do not dry the heating mattress pad in the dryer

Do not iron

5. GENERAL DESCRIPTION

(Image for illustrative purpose only)

MAIN COMPONENTS

1. Remote control
2. Temperature selector
3. Display
4. Connector
5. Connector base
6. Heating mattress pad

6. INSTRUCTION OF USE

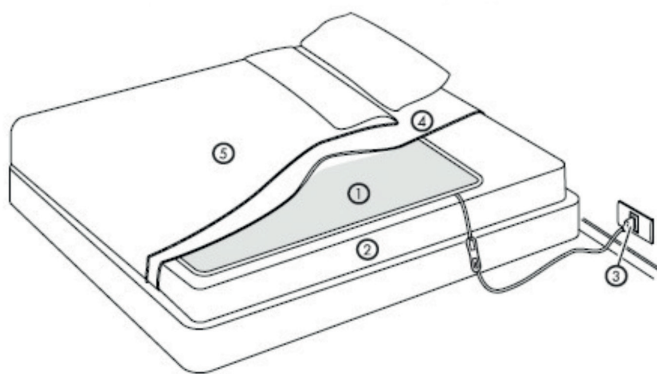
The heating mattress pad must be placed between the mattress and the first sheet, not over the bed or people.

ATTENTION!

The heating mattress pad can only be used with the control supplied. The relative model is specified on the label sewn on the panel.

6.1 How to place the heating mattress pad

(Image for illustrative purpose only)

1. Place the mattress pad directly on the mattress, with the plug facing the mattress. Make sure it is stretched out and secure it with the straps provided. **DO NOT** place the heating mattress pad under the pillow.
2. Place the first sheet over the mattress pad and prepare the bed as usual.
3. Insert the socket of the control into the plug to connect the adjustment control of the temperature to the mattress pad. For the double model it is necessary to repeat the operation also for the second grip. Unplug the heating mattress pad pressing the levers on the sides of the plug. Insert the plug of the control cable into the socket. Select the desired temperature by turning the knob adjustment. Select the high pre-heating temperature to obtain a rapid heating. Select temperatures 1 to 4 for prolonged heating.
 - Level 0 = off
 - Level 1 = Minimum temperature
 - Level 2 = Average temperature
 - Level 3 = Maximum temperature

6.2 Automatic switch-off function

- The heating mattress pad is provided with an automatic switch-off system. After 180 minutes of functioning it automatically turns off.
- To turn the device on once again, place the temperature selector in position "0", wait a few seconds, then select the desired temperature once again.

6.3 Switch-off

- Switch the heating mattress pad off by placing the temperature selector in position "0".
- Remove the plug from the socket.

7. CLEANING AND MAINTENANCE

Do not use bleach

Hand wash at a temperature not higher than 30°

Do not dry-clean

Do not dry the heating mattress pad in the dryer

Do not iron

- Over time, the fabric of the bedwarmer may undergo a natural yellowing. This does not compromise the safety and/or performance of the device itself.
- Before cleaning make sure that the heating mattress pad is not connected to the electric main and that it's cold.
- Before cleaning make sure the connection cable is detached.
- Washing in the washing machine, dry cleaning or tumble drying can be damaging the internal heating wire due to the high speed force. The HEATING MATTRESS PAD can be washed by hand at a temperature not exceeding 30 °C. To dry the heating mattress pad, place it on a surface that absorbs moisture.
- Do NOT compress or squeeze.
- Use a neutral cleanser.
- DO NOT use bleach.
- DO NOT dry in the dryer, leave the heating mattress pad dry to the air. DO NOT place it on radiators or other heating systems.
- DO NOT iron.
- DO NOT use pins on electrical parts of the device to prevent damage.
- Make sure that the heating mattress pad is completely dried, before using it again.
- While cleaning do not use chemical or abrasive products.
- Never immerse the power cord or remote control in water or other liquids, this would cause irreparable damage.
- DO NOT use the heating mattress pad until it is completely dry.
- LAY OUT THE BED WARMER
 - It is recommended to lay the bed warmer trying to reach the original size. The best result is obtained when the product is still wet, by pulling the side strips one by one. It may be necessary to repeat this operation several times until the product will not be dry.
 - Check that the cables are not twisted or bent. Such control is better if done against the light.
 - In case of cable problems, after laying out the heating mattress pad, or if the mattress pad does not takes its original form, contact your nearest dealer for a security check.

7.1. Storage

- Allow the product to cool before storing.
- Do not place objects on top of the product during storage.
- Always store the heating mattress pad in a dry place away from other heat sources.

8. GENERAL CONDITIONS OF DISPOSAL

The product is packaged in recyclable materials. Dispose of it in compliance with the environmental protection standards.

The product must be disposed of appropriately, in compliance with European standard 2002/96/EC. The recyclable materials contained in the appliance must be recovered so as to prevent environmental pollution. For further information, contact the local disposal body or the appliance dealer.

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(Image for illustrative purpose only)

Cod. LTK780

- Measure 150x80 cm
- Power 60W
- Voltage 220-240V ~ 50/60Hz
- Automatic power off approximately 180 min
- Weight 880 g about
- Material 100% polyester
- Upper part: sherpa pile

Cod. LTK785

- Measure 160x140cm
- Potenza 2x60W (Max 120W)
- Voltage 220-240V ~ 50/60Hz
- Automatic power off approximately 180 min
- Weight 1,7 kg about
- Material 100% polyester
- Upper part: sherpa pile

10. TROUBLESHOOTING

ATTENTION!

If the electrical device does not work properly and it is defective, do not attempt to repair it by yourself.

PROBLEM	POSSIBLE CAUSE	SOLUTION
The device does not work.	The device does not receive the power source.	Verify if: a. the power cable is connected to the connecting socket; b. the plug is insert in the socket; c. the socket is defective; d. the fuse of your mains connection has been activated.
	Any temperature level has been selected. The temperature selector is in position 0.	Move the temperature selector to position 1, 2 or 3.
	The protection against overheating has been activated.	Have the device checked by the manufacturer.

11. WARRANTY

Every Moretti product is guaranteed for 2 (two) YEARS from the purchase date, except the restrictions specified below. The warranty shall not apply in case of damage caused by improper use, abuse, alterations of the product or any other use not in compliance with the user manual.

The proper use of the product is indicated in the user manual. Moretti S.p.A. shall not be liable for any damages or personal injuries due to incorrect assembly/improper use of the device, not compliant with the user manual.

Moretti does not guarantee its products against damages or faults in the following cases: natural disasters, non authorized maintenance or reparations, damages caused by electrical supply problems (if provided), use of parts or components not provided by Moretti, failure in accomplishing the guidelines and instructions for use, non authorized alterations, damages due to the transport (different from Moretti original delivery) or failure in maintaining the device as indicated in the manual.

The components subject to wear are not covered by this warranty if the damage is due to the normal use of the product.

11.1 Exempt clauses

Except what clearly specified in this warranty and in accordance with the law, Moretti S.p.A. does not offer its customers any other declaration, warranty or condition, expressed or implicit, including any declaration, warranty or condition of merchantability, of non violation

or non interference, of suitability for a specific aim. Moretti S.p.A. does not guarantee that the use of Moretti products will be continuous and faultless. The duration of possible implicit warranties that may be applied in accordance with the law is limited to the warranty period. Some states or countries do not allow limitations to the duration of an implicit warranty, or the exoneration, or the limitations of accidental or indirect damages related to products for consumers. In such states or countries some exoneration or limitation of this warranty may not be applied to the user. This warranty is subject to modifications without any prior notice.

**WARRANTY CERTIFICATE****Product** _____**Purchased on (date)** _____**Retailer** _____**Address** _____ **Town/city** _____**Sold to** _____**Address** _____ **Town/city** _____ **MORETTI S.P.A.**

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: **info@morettispa.com****MADE IN P.R.C.**

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