SKEMA[®]FORM

MA MI474_MI476_MI477_MI478N_MI479_MI480N_02 A ITA_1f-2020

SGABELLI

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ITALIANO

INDICE

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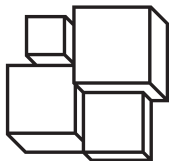
CE | Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

MI474A	Stool with chromed piston - Chromed casters - Steel seat
MI474B	Stool with chromed piston - Chromed casters - White padded seat
MI474BLU	Stool with chromed piston - Chromed casters - Blue padded seat
MI474N	Stool with chromed piston - Chromed casters - Black padded seat
MI474V	Stool with chromed piston - Chromed casters - Green padded seat
MI476B	Stool with casters and screw-lifting system - White padded seat
MI476BLU	Stool with casters and screw-lifting system - Blue padded seat
MI476N	Stool with casters and screw-lifting system - Black padded seat
MI476V	Stool with casters and screw-lifting system - Green padded seat
MI477B	Stool with casters and gas-operating system - White padded seat
MI477BLU	Stool with casters and gas-operating system - Blue padded seat
MI477N	Stool with casters and gas-operating system - Black padded seat
MI477V	Stool with casters and gas-operating system - Green padded seat
MI478N	Stool with casters and gas-operating system - Black seat in expanded PU with backrest
MI479A	Stool with 20-cm chromed piston - Chromed casters - Steel seat
MI479B	Stool with 20-cm chromed piston - Chromed casters - White padded seat
MI479BLU	Stool with 20-cm chromed piston - Chromed casters - Blue padded seat
MI479N	Stool with 20-cm chromed piston - Chromed casters - Black padded seat
MI479V	Stool with 20-cm chromed piston - Chromed casters - Green padded seat
MI480N	Stool with casters and gas-operating system - Saddle seat in black PU

2. INTRODUCTION

Thank you for choosing a stool from the SKEMA product range by MORETTI S.p.A.. Moretti stools have been designed and realized in order to guarantee comfort, safety and reliability. This user manual provides for some suggestions as to how correctly use the stool you have chosen and offers a lots of valuable advice for your safety. Please read carefully through the manual before using the stool. Should you have any queries, please contact your retailer for advice and assistance.

3. INTENDED USE

Supporting stools for medical activities.

WARNING!

- It is strictly forbidden to use this device for different purposes from those specified here in this user manual.
- Moretti S.p.A. declines any responsibility for damages caused by an improper use of the device or by a different purpose from that one specified here in this manual.
- The manufacturer reserves the right to modify the product and this user manual without prior notice, in order to improve the characteristics of the device.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of STOOLS complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

For all matters concerning a proper use of the product, please refer to this user manual:

- Keep the packing away from any heat source because it is a cardboard box
- The service life of the device is determined by the wear of its non-repairable and/or spare parts
- Please, pay attention to moving parts that may trap limbs or cause personal injuries
- Always pay attention to children
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

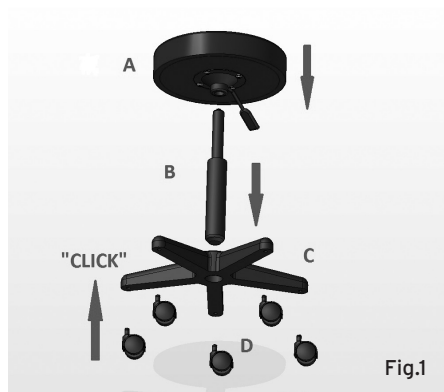
Medical Device

Conditions of disposal

7. GENERAL DESCRIPTION

7.1 Drawing and description of the parts

Parts list

- A- Seat
- B- Lifting piston
- C - 5-spoke base
- D - Swivel casters

8. ASSEMBLY

8.1 Stool assembly

1. Open the packing box and check that the content corresponds to the requested model and the parts have not been damaged by the transportation. Otherwise, please contact immediately your reseller.

N.B. - For assembling these stools no other tool is requested. (Fig.1)

1. Fit the swivel casters into the 5-spoke base, until you hear a “click”;
2. Fit the piston into the central housing hole of the base;
3. Fit the seat into the piston which has been just assembled.

9. BEFORE EACH USE

- Always check the mechanical parts for wear so that you can guarantee a totally safe use of the device for people and things
- Check the correct assembly of the device

10. INSTRUCTIONS FOR USE

10.1 For models MI474A - MI474B - MI474BLU - MI474N - MI474V - MI477B - MI477N - MI477V MI478N - MI479A -MI479B - MI479BLU - MI479N - MI479V - MI480N:

The stools are equipped with a gas-operating system piston and an activation handle, placed under the seat. In order to raise or lower the seat, activate manually the handle. Release it to block the piston again.

10.2 For models MI476B - Mi476BLU - MI476N - MI476V:

These stools are equipped with a screw-lifting system piston, without any activation handle. Screw the seat counter-clockwise to raise the stool; screw the seat clockwise to lower it.

11. MAINTENANCE

The devices from the SKEMA product range by MORETTI S.p.A. are carefully checked and provided with the EC label before being launched on the market. As far as patient's and doctor's safety is concerned, you are recommended to have the product suitability checked by the manufacturer or by an authorized lab, at least once a year.

12. CLEANING & DISINFECTION

12.1 Cleaning

Use only a damp cloth and neutral soap. Then, dry well before use. Do not use water warmer than 30° C. Do not use water and/or vapour jet cleaning machine.

12.2 Disinfection

If necessary, disinfect the device using a common cleaning disinfectant product.

N.B. In no way use acid, alkaline or solvent products as like acetone or thinner

13. CONDITIONS OF DISPOSAL

As far as the product disposal is concerned, never use the common urban solid waste disposal systems. The product must be disposed of by the municipal sorted waste collection centres, designated for recycling the used materials.

14. TECHNICAL SPECIFICATIONS

Dimension MI474A	 L: 500 mm	 W: 500 mm	 H: 430÷520 mm
Seat type	Seat Ø 29 cm - Inox steel		
Piston type	Chromed piston, gas-operating system 140 mm		
Base type with casters	Aluminium base Ø 50 cm with 5 chromed casters		
 Stool weight	5 kg		
 Max. load	150 kg		
Dimension MI474B MI474BLU - MI474N - MI474V	 L: 500 mm	 W: 500 mm	 H: 430÷520 mm
Seat type	Seat Ø 32 cm padded in PU density 40 height 5 cm upholstery in washable and self-extinguishing synthetic leather, 11M		
Piston type	Chromed piston, gas-operating system 140 mm		
Base type with casters	Aluminium base Ø 50 cm with 5 chromed casters		
 Stool weight	5 kg		
 Max. load	150 kg		

Dimension MI476B MI476BLU - MI476N - MI476V	 L: 500 mm	 W: 500 mm	 H: 420÷560 mm
Seat type	Seat Ø 32 cm padded in PU density 40 height 5 cm upholstery in washable and self-extinguishing synthetic leather, 11M		
Piston type	Screw-lifting system 140 mm		
Base type with casters	Black plastic base Ø 50 cm with 5 plastic casters		
 Stool weight	5 kg		
 Max. load	150 kg		
Dimension MI477B MI477BLU - MI477N - MI477V	 L: 500 mm	 W: 500 mm	 H: 420÷560 mm
Seat type	Seat Ø 32 cm padded in PU density 40 height 5 cm upholstery in washable and self-extinguishing synthetic leather, 11M		
Piston type	Black painted piston, gas-operating system 140 mm		
Base type	Black plastic base Ø 50 cm with 5 plastic casters		
 Stool weight	5 kg		
 Max. load	150 kg		
Dimension MI478N	 L: 500 mm	 W: 500 mm	 H: 400÷530 mm
Seat type	Seat L:33xW:35 cm in expanded PU with backrest H:15 cm		
Elevation type	Screw-lifting system 140 mm		
Base type	Black plastic base Ø 50 cm with 5 plastic casters		
 Stool weight	5 kg		
 Max. load	150 kg		
Dimension MI479A	 L: 500 mm	 W: 500 mm	 H: 500÷700 mm
Seat type	Seat Ø 35 cm - Inox steel		
Piston type	Chromed piston, gas-operating system 200 mm		
Base type	Aluminium base Ø 50 cm with 5 chromed casters		
 Stool weight	5 kg		
 Max. load	150 kg		

Dimension MI476B MI476BLU - MI476N - MI476V	 L: 500 mm	 W: 500 mm	 H: 530+730 mm
Seat type	Seat Ø 32 cm padded in PU density 40 height 5 cm upholstery in washable and self-extinguishing synthetic leather, IIM		
Piston type	Chromed piston, gas-operating system 200 mm		
Base type	Aluminium base Ø 50 cm with 5 chromed casters		
 Stool weight	5 kg		
 Max. load	150 kg		
Dimension MI480N	 L: 500 mm	 W: 500 mm	 H: 430+580 mm
Seat type	Saddle seat L:38 x W:34 cm in expanded PU		
Piston type	Black painted piston, gas-operating system 140 mm		
Base type with casters	Black plastic base Ø 50 cm with 5 plastic casters		
 Stool weight	5 kg		
 Max. load	150 kg		

15. WARRANTY

Every Moretti product is guaranteed for 2 (two) years from the purchase date, except the restrictions specified below. The warranty shall not apply in case of damage caused by improper use, abuse, alterations of the product or any other use not in compliance with the user manual. The proper use of the product is indicated in the user manual.

Moretti S.p.A. shall not be liable for any damages or personal injuries due to incorrect assembly/ improper use of the device, not compliant with the user manual.

Moretti does not guarantee its products against damages or faults in the following cases: natural disasters, non authorized maintenance or reparations, damages caused by electrical supply problems (if provided), use of parts or components not provided by Moretti, failure in accomplishing the guidelines and instructions for use, non authorized alterations, damages due to the transport (different from Moretti original delivery) or failure in maintaining the device as indicated in the manual. The components subject to wear are not covered by this warranty if the damage is due to the normal use of the product.

16. REPARATIONS

16.1 Repairation under warranty

If a Moretti product shows material or manufacturing faults during the warranty period, Moretti S.p.A., together with the client, will consider whether the product fault is covered by warranty. Moretti S.p.A., at its unquestionable discretion, may replace or repair the item under warranty in a Moretti reseller point or at its own qualified offices. Labour costs for repairing the product may be at Moretti's expense if it is determined the repairation is under warranty. A repairation or replacement does not renew nor extend the warranty.

**16.2 Reparation of an out-of-warranty product**

A product, not covered by warranty, may be returned for reparation only after receiving prior authorization from Moretti Client Service. Labour and delivery costs for an out-of-warranty reparation will be fully at the client's or reseller's expense. Reparations of out-of-warranty products are guaranteed for 6 (six) months, from the day the client receives the repaired product.

16.3 Non-defective products

The client is informed if, after examining and testing a returning product, Moretti decides the product is not defective. The product will be sent back to the client and all the return delivery costs will be at his expense.

17. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the day you receive them.

18. EXONERATION CLAUSES

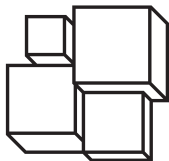
Except what clearly specified in this warranty and in accordance with the law, Moretti S.p.A. does not offer its clients any other declaration, warranty or condition, expressed or implicit, including any declaration, warranty or condition of merchantability, of non violation or non interference, of suitability for a specific aim. Moretti S.p.A. does not guarantee that the use of Moretti products will be continuous and faultless. The duration of possible implicit warranties that may be applied in accordance with the law is limited to the warranty period. Some states or countries do not allow limitations to the duration of an implicit warranty, or the exoneration, or the limitations of accidental or indirect damages related to products for consumers. In such states or countries some exonerations or limitations of this warranty may not be applied to the user. This warranty is subject to modifications without any prior notice.

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