MOPEDIA®

MA RS9xx 02 A_ITA_09-2020

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ITALIANO

INDICE

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REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

RS988X - Overbed table with folding legs

RS990X - Overbed table with folding legs, reclining in 3 positions

RS992X - Overbed table with wheels, painted steel frame, manual regulation

RS997X - Overbed table with wheels, painted steel frame, automatic regulation

2. INTRODUCTION

Thank you for choosing an overbed table by MOPEDIA range, manufacturer by MORETTI S.p.A. Their design and quality are a guarantee of comfort, safety and reliability. Overbed tables by MOPEDIA by MORETTI S.p.A. are designed and built to meet all your demands for a practical, correct use. This user manual provides some suggestions to correctly use the tables you have chosen and offers a lot of valuable advice for your safety. Please read carefully through the manual before using the overbed table. Should you have any queries, please contact your retailer for advice and assistance.

3. INTENDED USE

The MOPEDIA bed trays are primarily intended for people who have mobility problems, mainly bedridden where it is expected the consumption of meals in bed or chair. However they are adaptable both in hospitals and home.

NOTE:

Check that the product has not been damaged during shipment. In case of damage do not use it and contact your dealer for further instructions.

- Do not use the product for a purpose not indicated in this manual.
- Moretti S.P.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer reserves the right to change the information contained in this document without previous notice.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the groups of BEDS TABLE complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.

4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

4.1 Standards and guidelines reference

Safety tests required by UNI EN 12182

5. GENERAL WARNINGS

- Always pay close attention to the presence of moving parts that could cause limb entrapment and injury.
- For proper use of the device refer to this manual.
- For proper use of the device consult your doctor or therapist.
- Keep the packaged product away from any source of heat because the packaging is made of cardboard.
- The shelf-life of the accessory is determined by the wear of parts that cannot be repaired and/or replaced.
- Be careful when children are nearby.
- Do not exceed the maximum payload.

6. SYMBOLS

- | | |
|---|------------------------------|
|  | Product code |
|  | Unique Device Identification |
|  | CE mark |
|  | Manufacturer |
|  | Batch Lot |
|  | Read the instruction manual |
|  | Medical Device |
|  | Conditions of disposal |

7. GENERAL DESCRIPTION

7.1 View and List of Components

RS988X - Overbed table with folding legs


RS990X - Overbed table with folding legs, reclining in 3 positions


RS992X - Overbed table with wheels, painted steel frame, manual regulation

- A. Big screw - 2pz
- B. Small screw - 2pz
- C. Semicircles on small table - 1pz
- D. Perforated plate - 1pz
- E. Big table - 1pz
- F. Base - 1pz
- G. Castors - 4pz
- H. Column - 1pz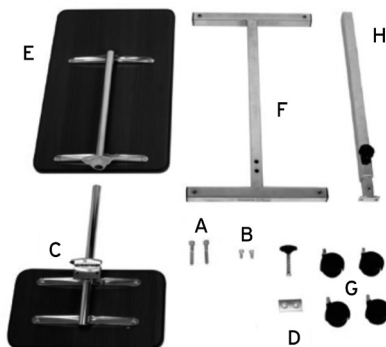

RS997X - Overbed table with wheels, painted steel frame, automatic regulation

- A. Big screw - 2pz
- B. Small screw - 2pz
- C. Column - 1pz
- D. Perforated plate - 1pz
- E. Base - 1pz
- F. Castors - 4pz

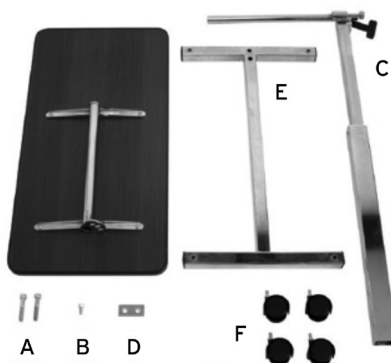8. ASSEMBLING/DISASSEMBLING

8.1 Assembling

RS992X - Overbed table with wheels, painted steel frame, manual regulation

1. Unpack the product and check that the content matches with the description of the parts in the manual. Assemble the four castors.
2. Assemble the column using the perforated plate (fig. D), the two big screws (fig. A) and lock them with the big Allen wrench.
3. Assemble the small tray with the small screws (fig. B) using the Allen wrench.
4. To assemble the bigger tray, insert it in the projecting tube of the tray. Do not put them closer to each other.
5. Insert the spring of the handles (fig. C) between two semicircles.
6. Put the trays closer to each other, insert the locking screw and fix it with the handle.
7. By turning the handle, the big tray can rotate. Tighten the handle again to block it at the required inclination.
8. Now the overbed table is ready for use.

RS997X - Overbed table with wheels, painted steel frame, automatic regulation

Do not remove the safety screw before point "3"

ATTENZIONE!

1. Unpack the product and check that the content matches with the description of the parts in the manual. Assemble the four castors.
2. Assemble the column using the perforated plate (fig. D), the two big screws (fig. A) and lock them with the Allen wrench.
3. Insert the tray in the projecting tube of the column.
4. Assemble the tray with the small screw (fig. B) using the Allen wrench. Remove the black screw (A pag.8) that it is working as a security and replace it with the one inside the bag (B pag.8).
5. Now the overbed table is ready for use.

ENGLISH

8.2 Disassembling

- To disassemble do the exact opposite of the above instructions

9. BEFORE USE

- Always check the status of the mechanical parts to guarantee they are used in total safety.
- Check the device is properly assembled. Pay particular attention to the locking of the knobs.

10. PRECAUTIONS FOR USE

- Do not exceed the maximum load capacity.
- Make sure that the floor surface supporting the overbed table is level.
- Make the adjustment operations without load.
- Check all components are well fixed.
- Always assist the patient during the elevation operation

11. MODALITÀ D'USO HOW TO USE

The assistant must place and adjust the overbed table with the maximum care to avoid sudden movements or dangerous oscillations. An assistant has to check any movement of the patient.

RS988X e RS990X

The overbed table is assembled, just take out the legs

RS992X

Tighten the handle to block at the required height. By turning the handle (E1) the big tray can rotate. Tighten the handle again to block it at the required inclination.


RS997X

By turning the handle (C1) the tray will lift up automatically. Pushing on the tray, it will come down. Tighten the lever to block it at the required height. By turning the handle (C2), the tray can rotate. Tighten the handle again to block it at the required inclination.

Remove the black screw (A) that is working as a security and replace it with the one inside the bag (B).

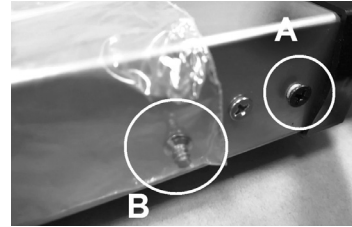**12. MAINTENANCE**

Check the structure periodically: the correct positioning of the tubes, the connections, the hooks, and the complete tightening of screws and / or terminals.

13. CLEANING AND DISINFECTION**13.1 Cleaning**

Use a damp cloth or a solution of lukewarm water and neutral soap.

Do not use abrasive substances, acids, alkali, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of the metal surfaces.

The manufacture company is not responsible for any damage caused by using materials that can damage the product surface or corrosive chemicals during cleaning.

13.2 Disinfection

If you need to disinfect the product, use a common delicate disinfectant neutral detergent.

14. CONDITIONS OF DISPOSAL

When disposing of the device, never use the normal return systems of municipal solid waste. It is recommended instead the disposal through the joint municipal recycling areas for the expected operations of recycling the materials used.

15. SPARE PARTS

For spare parts and accessories, please always refer to the MORETTI master catalogue. ONLY USE ORIGINAL MORETTI ACCESSORIES. MORETTI S.P.A. COMPANY SHALL NOT BE LIABLE FOR ANY INCIDENT CAUSED BY IMPROPER USE.

16. TECHNICAL FEATURES

16.1 Dimension

RS988X

Height 24cm

Width 60.5cm

Depth 39cm

Weight about 2.5 kg

Pay load 5Kg

RS990X

Height 24cm

Width 60.5cm

Depth 39cm

Weight about 3.5kg

Pay load 5Kg

RS992X

Height 80-120cm

Small table Width 20cm

Big table Width 60.5cm

Depth 39cm

Weight about 12kg

Pay load 20Kg

RS997X

Height 78-110cm

Width 77cm

Depth 39cm

Weight about 12 kg

Pay load 20Kg

16.2 Technical specifications

RS988X - RS990X

- Painted steel frame
- Laminate scratch resistant table
- Containment edge made in polypropylene
- Antishock and cut resistant edge made in polyethylene
- Reclining table in three positions (*only for RS990X)
- Available in natural or brown color

RS988X - RS990X

- Painted and chromed steel frame
- Manual adjustable height (*only for RS992X)
- Automatic adjustable height with lever (*only for RS997X)
- Laminate scratch resistant table
- Containment edge made in polypropylene
- Antishock and cut resistant edge made in polyethylene
- Reclining table in more positions
- Small table(*only for RS992X)
- Available in natural or brown color

17. WARRANTY

Every Moretti product is guaranteed for 2 (two) YEARS from the purchase date, except the restrictions specified below. The warranty shall not apply in case of damage caused by improper use, abuse, alterations of the product or any other use not in compliance with the user manual. The proper use of the product is indicated in the user manual. Moretti S.p.A. shall not be liable for any damages or personal injuries due to incorrect assembly/improper use of the device, not compliant with the user manual. Moretti does not guarantee its products against damages or faults in the following cases: natural disasters, non authorized maintenance or reparations, damages caused by electrical supply problems (if provided), use of parts or components not provided by Moretti, failure in accomplishing the guidelines and instructions for use, non authorized alterations, damages due to the transport (different from Moretti original delivery) or failure in maintaining the device as indicated in the manual. The components subject to wear are not covered by this warranty if the damage is due to the normal use of the product.

18. REPAIRING

Warranty repair

If a Moretti product shows material or manufacturing faults during the warranty period, Moretti S.p.A., together with the customer, will consider whether the product fault is covered by warranty. Moretti S.p.A., at its unquestionable discretion, may replace or repair the item under warranty in a Moretti reseller point or at its own qualified offices. Labour costs for repairing the product may be at Moretti's expense if it is determined the reparation is under warranty. A reparation or replacement does not renew nor extend the warranty.

Repair not covered by warranty

A product, not covered by warranty, may be returned for reparation only after receiving prior authorization from Moretti customer service. Labour and delivery costs for an out-of-warranty reparation will be fully at the customer's or reseller's expense. Reparations of out-of-warranty products are guaranteed for 6 (six) months, from the day the customer receives the repaired product.

Non defective device

The customer is informed if, after examining and testing a returning product, Moretti decides the product is not defective. The product will be sent back to the customer and all the return delivery costs will be at his expense.

19. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the day you receive them.

20. EXEMPT CLAUSES

Except what clearly specified in this warranty and in accordance with the law, Moretti S.p.A. does not offer its customers any other declaration, warranty or condition, expressed or implicit, including any declaration, warranty or condition of merchantability, of non violation or non interference, of suitability for a specific aim. Moretti S.p.A. does not guarantee that the use of Moretti products will be continuous and faultless. The duration of possible implicit warranties that may be applied in accordance with the law is limited to the warranty period. Some states or countries do not allow limitations to the duration of an implicit warranty, or the exoneration, or the limitations of accidental or indirect damages related to products for consumers. In such states or countries some exonerations or limitation of this warranty may not be applied to the user. This warranty is subject to modifications without any prior notice.

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