SKEMA[®]FORM

MA MOC330X - MOV330X - MOV333X - MOV340X - MOV350X - MOV355X 04 A_ITA_02 2022

LETTINO DA VISITA MEDICA E FISIOTERAPIA

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INDICE

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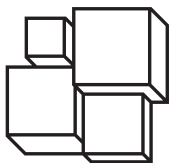
SKEMA[®]FORM

MA MOC330X - MOV330X - MOV333X - MOV340X - MOV350X - MOV355X 04 A_ENG_02 2022

EXAMINATION AND PHYSIOTHERAPY COUCH

INSTRUCTION MANUAL

INDICE

1. CODES	PAG.3
2. INTRODUCTION	PAG.3
3. INTENDED USE	PAG.3
4. DECLARATION OF CONFORMITY	PAG.3
5. GENERAL WARNINGS	PAG.4
6. SYMBOLS	PAG.4
7. GENERAL DESCRIPTIONS	PAG.5
7.1 Drawing and description of the parts of MOC330-MOV330	pag. 5
7.1.1 Drawing and description accessory kit of MOC330-MOV330	pag. 5
7.2 Drawing and description of the parts of MOV333	pag. 6
7.2.1 Drawing and description accessory kit of MOV333	pag. 6
7.3 Drawing and description for the parts of MOV340	pag. 7
7.3.1 Drawing and description accessory kit of MOV340	pag. 7
7.4 Drawing and description for the parts of MOV350	pag. 8
7.5 Drawing and description for the parts of MOV355	pag. 8
7.5.1 Drawing and description accessory kit of MOV350-MOV355	pag. 9
8. ASSEMBLY	PAG.9
8.1 Assembling the couch MOC330-MOV330	pag. 9
8.2 Assembling the couch MOV333	pag. 11
8.3 Assembling the couch MOV340	pag. 12
8.4 Assembling the couch MOV350	pag. 13
8.5 Assembling the couch MOV355	pag. 14
8.6 Assembling the paper holder (OPTIONAL)	pag. 15
9. BEFORE EACH USE	PAG.15
10. PRELIMINARY CHECKS	PAG.15
11. USING THE COUCH	PAG.15
12. MAINTENANCE	PAG.16
13. CLEANING AND DISINFECTION	PAG.17
14. CONDITIONS OF DISPOSAL	PAG.17
15. SPARE PARTS AND ACCESSORIES	PAG.17
16. TECHNICAL SPECIFICATIONS	PAG.18
16.1 Dimension and weight of couches MOC330-MOV330	pag. 18
16.1.1 Technical specifications of couches MOC330-MOV330	pag. 18
16.2 Dimension and weight of couch MOV333	pag. 19
16.2.1 Technical specifications of couch MOV333	pag. 19
16.3 Dimension and weight of couch MOV340	pag. 20
16.3.1 Technical specifications of couch MOV340	pag. 20
16.4 Dimension and weight of couch MOV350	pag. 21
16.4.1 Technical specifications of couch MOV350	pag. 21
16.5 Dimension and weight of couch MOV355	pag. 22
16.5.1 Technical specifications of couch MOV355	pag. 22
17. WARRANTY	PAG.23
18. REPARATIONS	PAG.23
18.1 Reparatons under warranty	pag. 23
18.2 Reparation of an out-of-warranty product	pag. 23
18.3 Non-defective products	pag. 23
19. SPARE PARTS	PAG.23
20. EXONERATION CLAUSES	PAG.24

CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

MOC330X	Examination couch LYBRA - Chromed
MOV330X	Examination couch LYBRA - Painted
MOV333X	Examination couch RYGEL - Painted
MOV340X	Examination and physiotherapy couch RYGEL - Painted
MOV350X	Examination couch RYGEL - 3 sections - Painted
MOV355X	Gynecological Examination couch RYGEL - 3 sections - Painted

X: Indicates the upholstery colour.

2. INTRODUCTION

Thank you for choosing an examination couch from the SKEMA product range by MORETTI S.p.A..

Moretti examination couches have been designed and realized in order to guarantee comfort, safety and reliability. This user manual provides for some suggestions as to how correctly use the product you have chosen and offers a lots of valuable advice for your safety.

Please read carefully through the manual before using the examination couch. Should you have any queries, please contact your retailer for advice and assistance.

3. INTENDED USE

Moretti examination couches are intended for physiotherapy diagnosis and short treatments.

WARNING!

- It is strictly forbidden to use this device for different purposes from those ones specified here in this user manual.
- Moretti S.p.A. declines any responsibility for damages caused by an improper use of the device or by a different purpose from that one specified here in this manual.
- The manufacturer reserves the right to modify the product and this user manual without prior notice, in order to improve the characteristics of the device.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of EXAMINATION COUCHES - SKEMA complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

For all matters concerning a proper use of the product, please refer to this user manual:

- Keep the packing away from any heat source because it is a cardboard box
- The service life of the device is determined by the wear of its non-repairable and/or spare parts and in any case it is no more than 10 years
- Please, pay attention to moving parts that may trap limbs or cause personal injuries
- Always pay attention to children
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

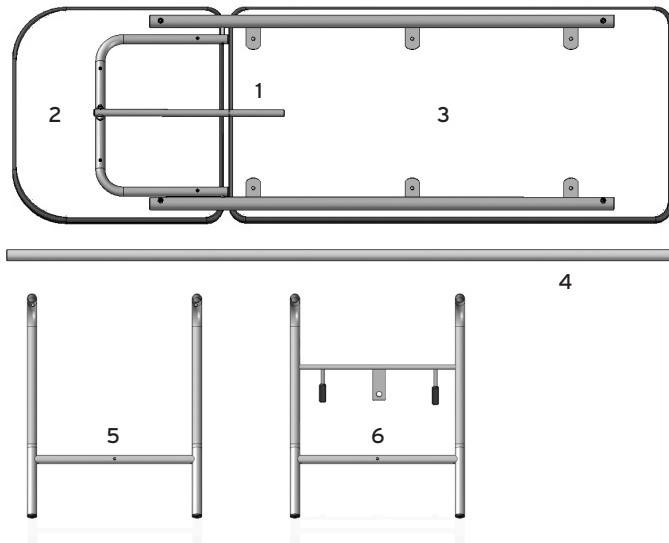
Medical Device

Conditions of disposal

7. GENERAL DESCRIPTIONS


7.1 Drawing and description of the parts of MOC330-MOV330

Parts list

- 1. Frame
- 2. Padded backrest
- 3. Padded couch
- 4. Longitudinal tube for connecting the legs
- 5. Leg (feet)
- 6. Leg (head)

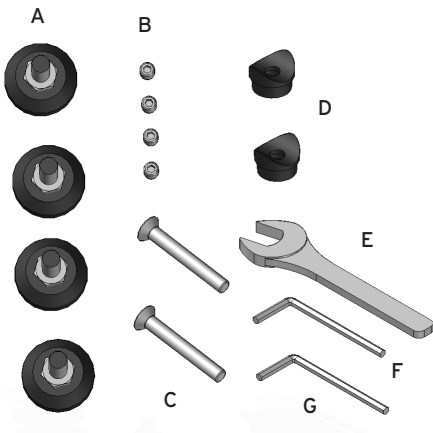
7.1.1 Drawing and description accessory kit of MOC330-MOV330

Parts list

- A. N° 4 feet with adjustment bolt
- B. N° 4 grub screws to secure the legs
- C. N° 2 countersunk screws M 8x60 for the connecting tube for legs
- D. N° 2 countersunk joints for the connecting tube for legs
- E. N° 1 wrench for foot adjustment bolt
- F. N° 1 14mm Hex key to secure the legs
- G. N° 1 15mm Hex key for the connecting tube for legs

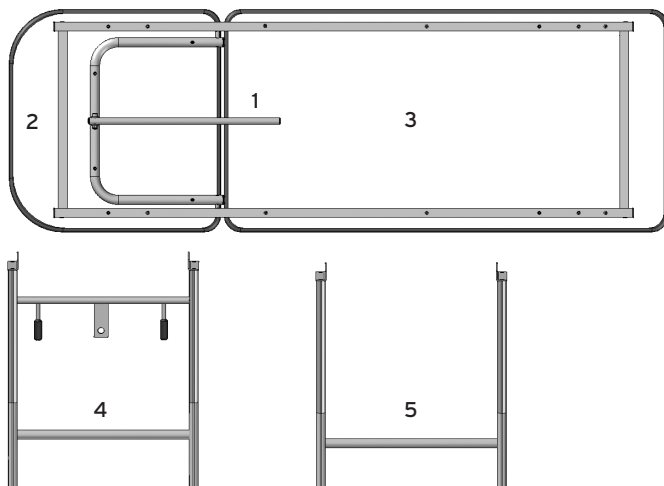7.2 Drawing and description of the parts of MOV333


ENGLISH

Parts list


1. Frame
2. Padded backrest
3. Padded couch
4. Legs (head)
5. Legs (feet)

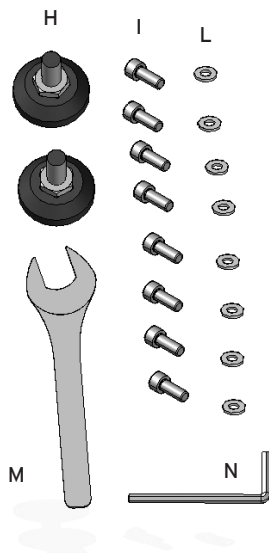7.2.1 Drawing and description accessory kit of MOV333


Parts list

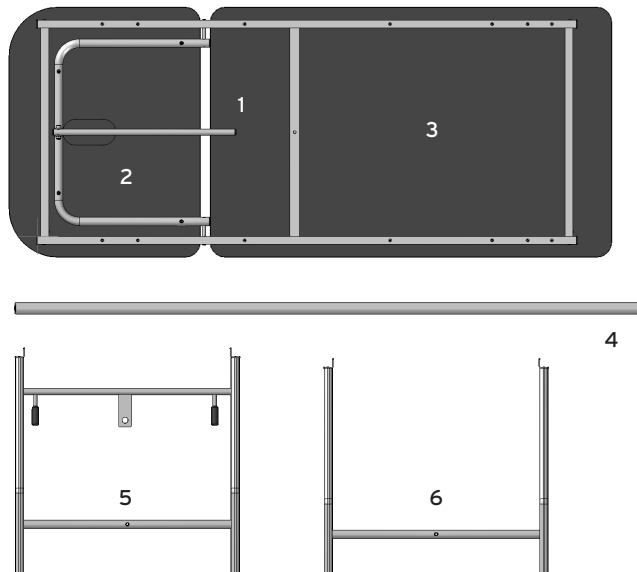- H. N° 2 feet with adjustment bolt
- I. N° 8 round head screws M 6x16 to secure the legs
- L. N° 8 washer, diameter 6 for leg blocking screws
- M. N° 117mm wrench for foot adjustment bolt
- N. N° 1 5mm Hex key to secure the legs



7.3 Drawing and description for the parts of MOV340

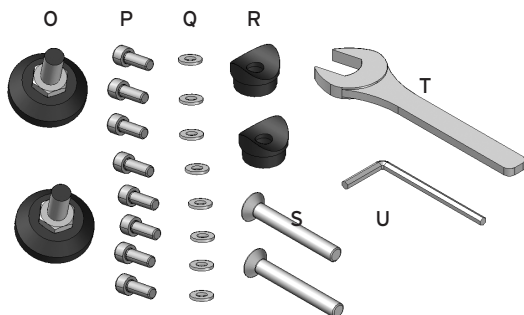Parts list

1. Frame
2. Padded backrest
3. Padded couch
4. Tube for connecting the legs
5. Legs (head)
6. Legs (feet)

7.3.1 Drawing and description accessory kit of MOV340

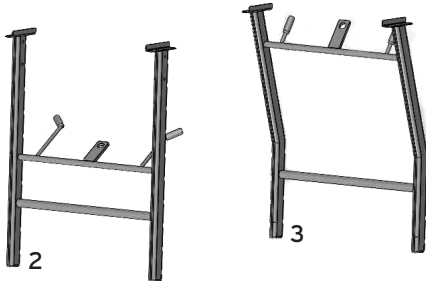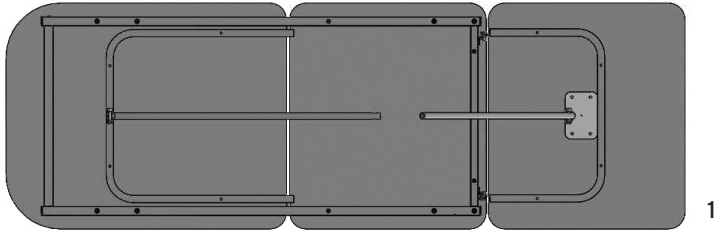Parts list

- O. N° 2 feet with adjustment bolt
- P. N° 8 round head screws M6x16 to secure the legs
- Q. N° 8 washers for leg blocking screws
- R. N° 2 countersunk joints for the connecting tube for legs
- S. N° 2 countersunk screws M 8x60 for the connecting tube for legs
- T. N° 117mm Hex key to adjust feet
- U. N° 15mm Hex key to secure the legs

7.4 Drawing and description for the parts of MOV350

Parts list

1. Frame, comprehensive of:

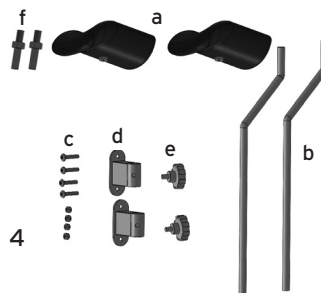
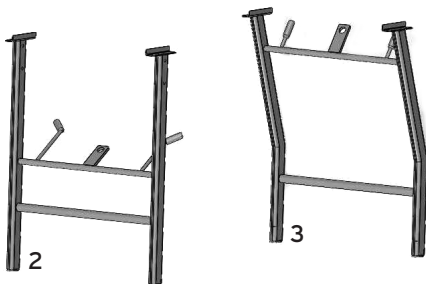
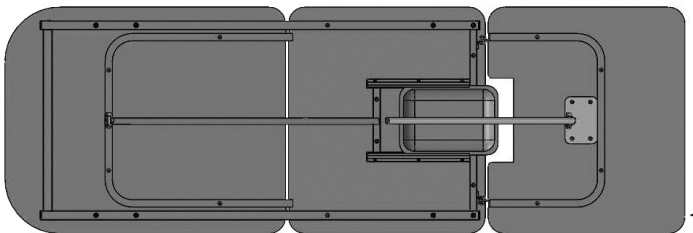
- perimeter frame
- Padded backrest
- Padded seat section
- padded footrest

2. Legs (head)

3. Legs (feet)

For description of accessory kit of MOV350, see 7.5.1.

7.5 Drawing and description for the parts of MOV355


Parts list
1. Frame, comprehensive of:

- perimeter frame
- Padded backrest
- Padded seat section
- padded footrest
- subframe for stainless steel tray
- stainless steel tray for liquid collection (sliding and removable)

2. Legs (head)
3. Legs (feet)
4. Components for leg support:

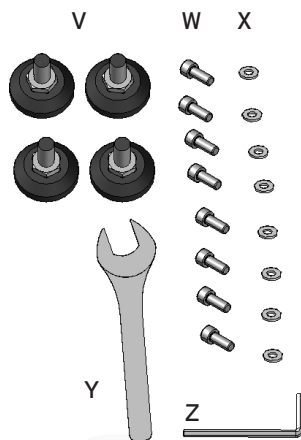- a) anatomic leg supports made of printed black PU
- b) swiveling tubular supports
- c) screws and nuts for fixing the supports
- d) sliding and locking bushings for leg supports
- e) height adjustment and locking handwheels for leg supports
- f) pin with lock nut to be installed on PU leg supports

7.5.1 Drawing and description accessory kit of MOV350-MOV355

Parts list

- V. N° 4 feet with adjustment bolt
- W. N° 8 round head screws M6x16 to secure the legs
- X. N° 8 washers for leg blocking screws M6
- Y. N° 117mm Hex key to adjust feet
- Z. N° 15mm Hex key to secure the legs

8. ASSEMBLY

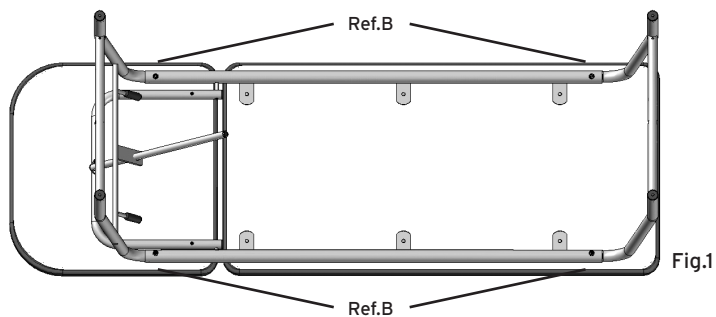
CAUTION!

Perform these operations with the help of another person. Take precautions to prevent injuries and/or damages when moving the couch.


8.1 Assembling the couch MOC330-MOV330

1. Open the packing box and check that the content corresponds to the requested model and the parts have not been damaged by the transportation. Otherwise, please contact immediately your reseller.


2. Assembling the legs

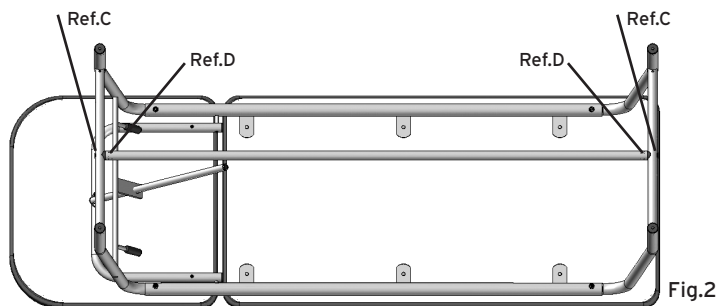3. Assembling the tube that connects the legs

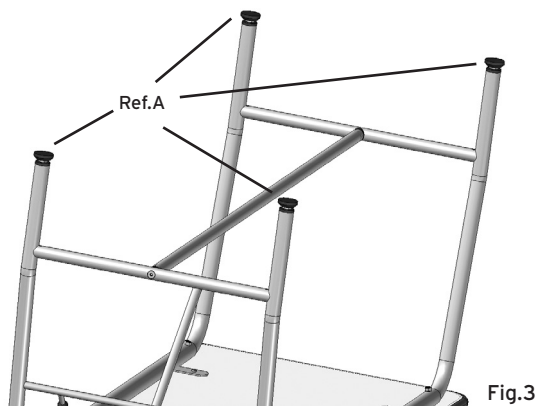4. Foot with adjustment bolt


N.B. Adjust the feet with the provided wrench, so that the couch can be made stable on a flat surface.


YOUR COUCH IS CORRECTLY ASSEMBLED AND READY TO USE

8.2 Assembling the couch MOV333

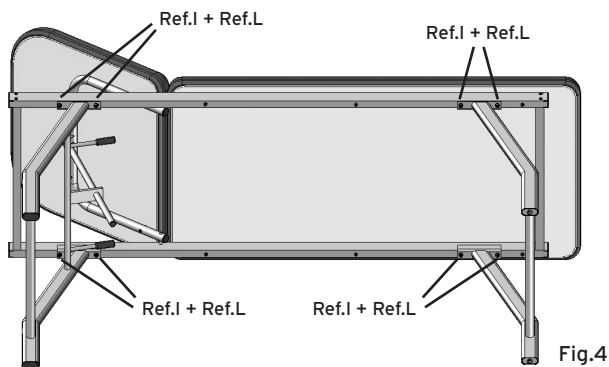1. Open the packing box and check that the content corresponds to the requested model and the parts have not been damaged by the transportation. Otherwise, please contact immediately your reseller.
2. Assembling the legs

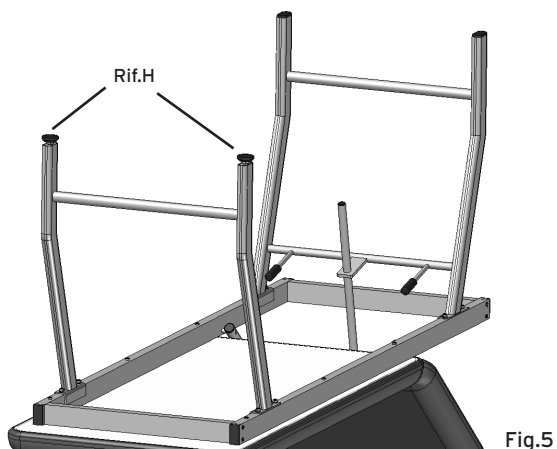
3. Assembling the feet with bolt


YOUR COUCH IS CORRECTLY ASSEMBLED AND READY TO USE

8.3 Assembling the couch MOV340

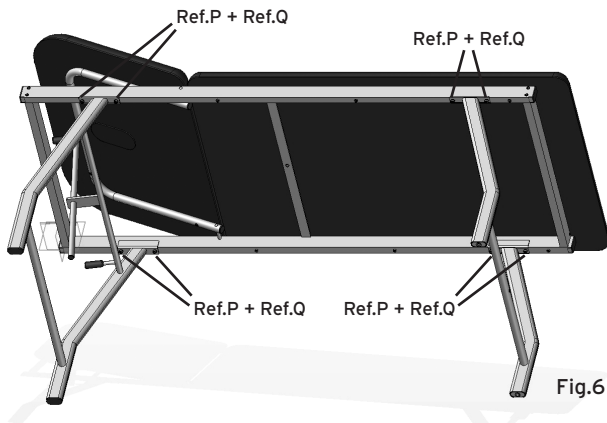
1. Open the packing box and check that the content corresponds to the requested model and the parts have not been damaged by the transportation. Otherwise, please contact immediately your reseller.
2. Assembling the legs

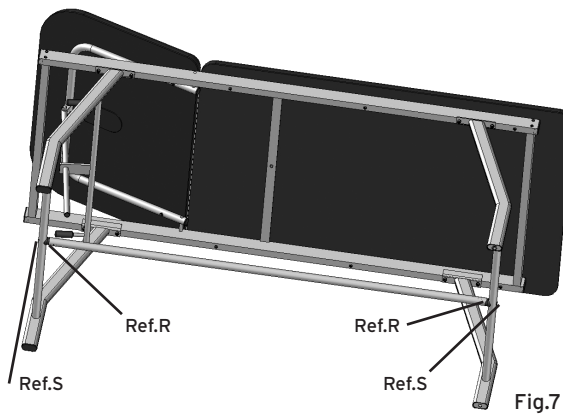3. Assembling the tube that connects the legs

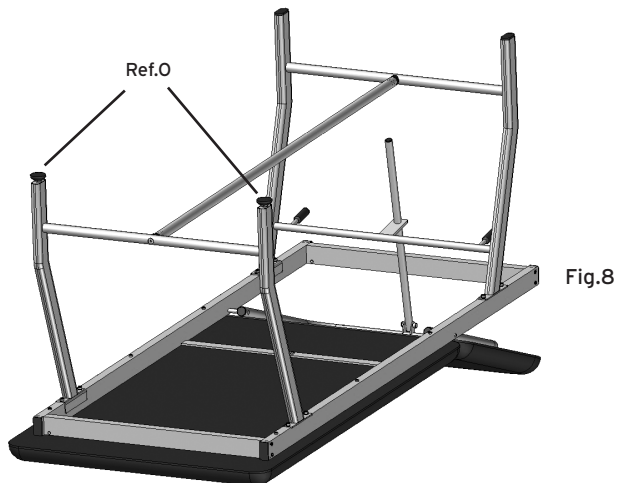4. Foot with bolt


YOUR COUCH IS CORRECTLY ASSEMBLED AND READY TO USE


8.4 Assembling the couch MOV350

1. Open the packing box and check that the content corresponds to the requested model and the parts have not been damaged by the transportation. Otherwise, please contact immediately your reseller.

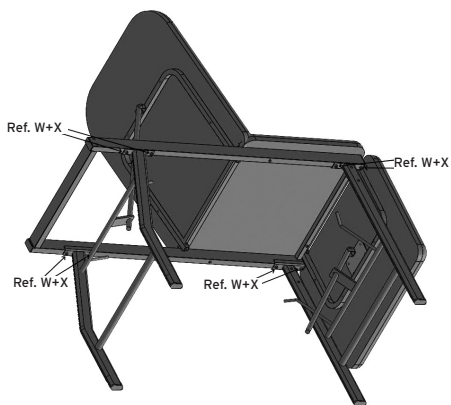2. Assembling the legs

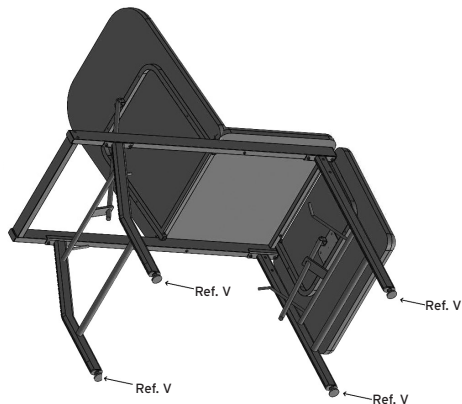3. Assembling the feet with bolt


YOUR COUCH IS CORRECTLY ASSEMBLED AND READY TO USE


8.5 Assembling the couch MOV355

1. Open the packing box and check that the content corresponds to the requested model and the parts have not been damaged by the transportation. Otherwise, please contact immediately your reseller.

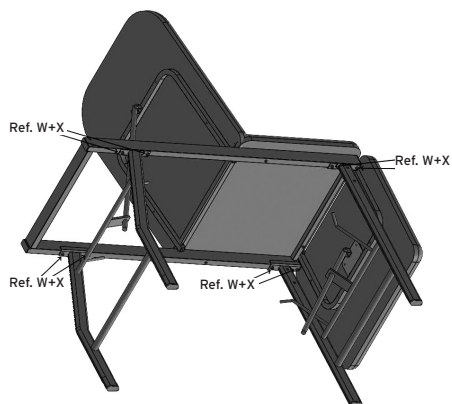
2. Assembling the legs

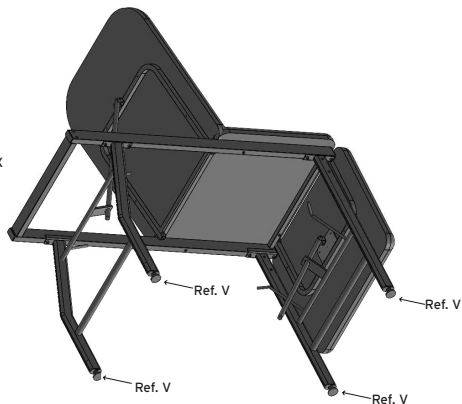
3. Assembling the feet with bolt

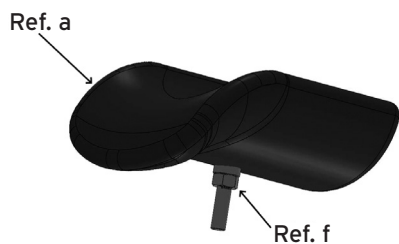4. Installation of pins with lock nut screwed on PU leg supports

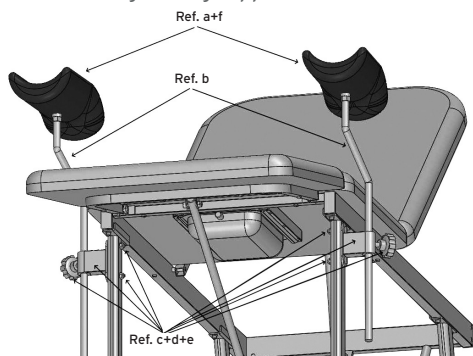5. Assembling the leg supports

YOUR COUCH IS CORRECTLY ASSEMBLED AND READY TO USE

8.6 Assembling the paper holder (OPTIONAL)

MIA370
(only for MOC330-MOV330)

MIA385
(only for MOV333-MOV350-MOV355)

MIA376
(only for MOV340)

N.B. It can be assembled on the head side or on the foot side, except for the MOV350-MOV355 models, where it is possible to install it only on the head side.

YOUR KIT IS CORRECTLY ASSEMBLED AND READY TO USE

9. BEFORE EACH USE

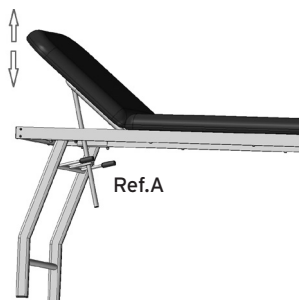
- Always check the mechanical parts for wear so that it can be guaranteed a totally safe use of the device for people and things
- Check the correct assembly of the device. Pay attention to the screws and blocking bolts of the backrest and of the mechanical joints in general.

10. PRELIMINARY CHECKS


- Before any adjustment of the couch, make sure it is in a stable position on its own legs.

11. USING THE COUCH

For all models: You may adjust the backrest inclination, acting on the handles (Ref. A of the picture). In order to secure it, support the backrest with one hand and activate one of the handles with the other hand, lifting it up. Block the backrest at the desired inclination and then release the blocking handle.

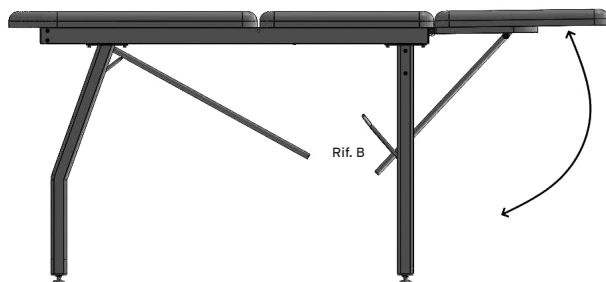
For MOV350 & MOV355 only: The slope of the legrest section can be adjusted of the bed by acting on the handles (ref.B of the image). To unlock it, with one hand support the leg rest section from below and with the other one act on one of the handles by lifting it. Place the leg rest section at the desired angle and then release the locking handle.

For MOV355 only

It is possible to adjust the height of the thigh supports by acting on the tightening handwheel (ref. C. of the image).

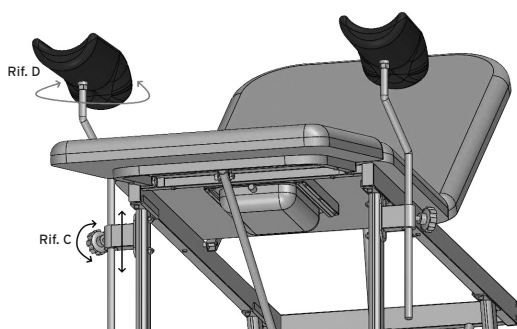It is possible to adjust the orientation of the anatomical leg support cushions by simply rotating the component (ref. D of the image)

CAUTION!

Before placing a patient on the couch, always check the correct safety blocking system of the backrest and of the legrest.

CAUTION!

For MOV350 and MOV355 models, do not sit or place excessive weight in the leg section, to avoid the possibility of overturning the bed.

12. MAINTENANCE

The devices from the SKEMA product range by MORETTI S.p.A. are carefully checked and provided with the EC label before being launched on the market.

As far as patient's and doctor's safety is concerned, you are recommended to have the product suitability checked at least once a year.

For periodic inspection we mean the following parts:

- The couch frame;
- Check all the welding joints. In case of reparation, only use original spare parts and accessories.

13. CLEANING AND DISINFECTION

13.1 Cleaning

Only use a damp cloth and neutral soap. Then dry well before use.

Do not use water warmer than 30° C.

13.2 Disinfection

If necessary, disinfect the device using a common cleaning disinfectant product.

N.B. In no way use acid, alkaline or solvent products as like acetone or thinner.

14. CONDITIONS OF DISPOSAL

As far as the product disposal is concerned, never use the common urban solid waste disposal systems. The product must be disposed of by the municipal sorted waste collection centres, designated for recycling the used materials.

15. SPARE PARTS AND ACCESSORIES

MIA370

Paper holder only for the models MOC330-MOV330

MIA385


Paper holder only for the model MOV333 -MOV350-MOV355

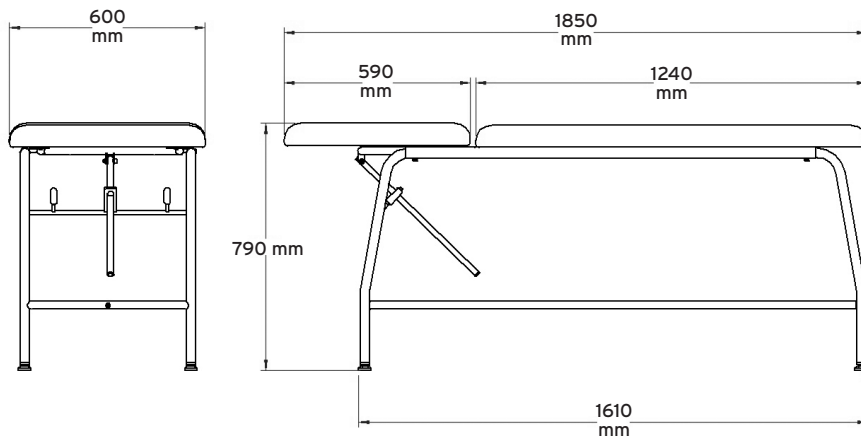MIA376

Paper holder only for the model MOV340

16. TECHNICAL SPECIFICATIONS

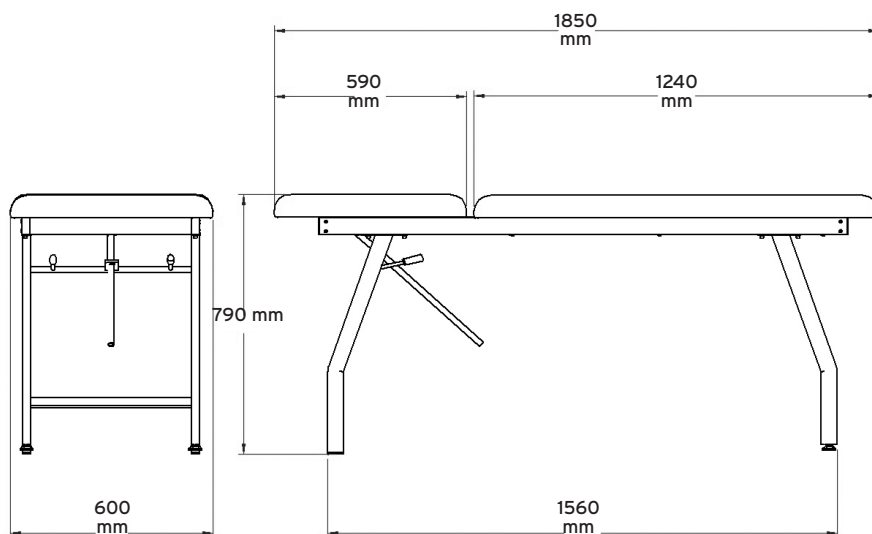
16.1 Dimension and weight of couches MOC330-MOV330

 Packing dimension	 L: 1920 mm	 P: 650 mm	 H: 220 mm
 Packing weight	36 kg		
 Couch weight	33 kg		
 Max. load	180 kg		

16.1.1 Technical specifications of couches MOC330-MOV330

Steel tubular structure diam. 35 mm chromed for MOC codes, painted for MOV codes. Tubular steel legs structure diam. 30 mm. Upholstery in polyurethane density 25, 5 cm high. Upholstery in washable and self-extinguishing imitation leather 11M

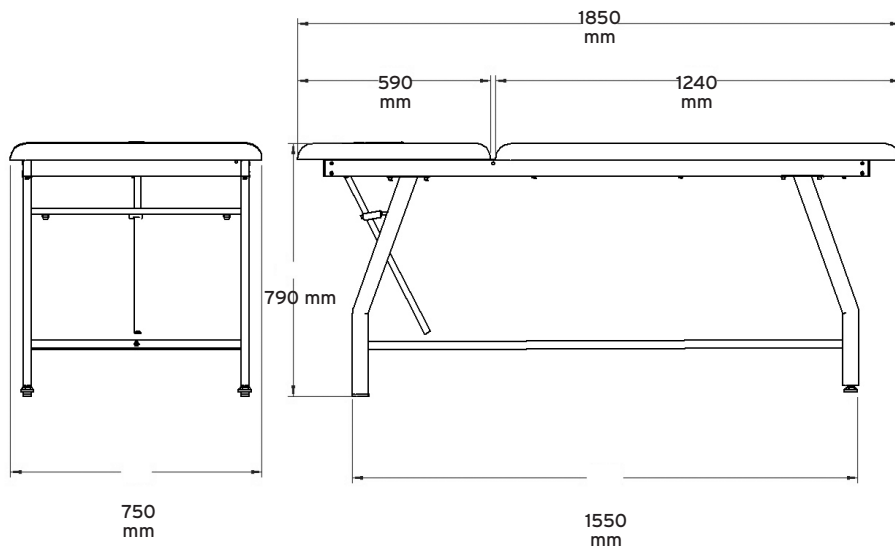
16.2 Dimension and weight of couch MOV333

 Packing dimension	 L: 1920 mm	 P: 650 mm	 H: 220 mm
 Packing weight	39 kg		
 Couch weight	36 kg		
 Max. load	180 kg		

16.2.1 Technical specifications of couch MOV333

Perimeter frame in painted rectangular tubular 40x25 mm. Couch legs frame in oval tubular steel, 50x25 mm. Padded in polyurethane, density 25 high 5 cm. Upholstery in washable and self-extinguishing synthetic leather, 11M.

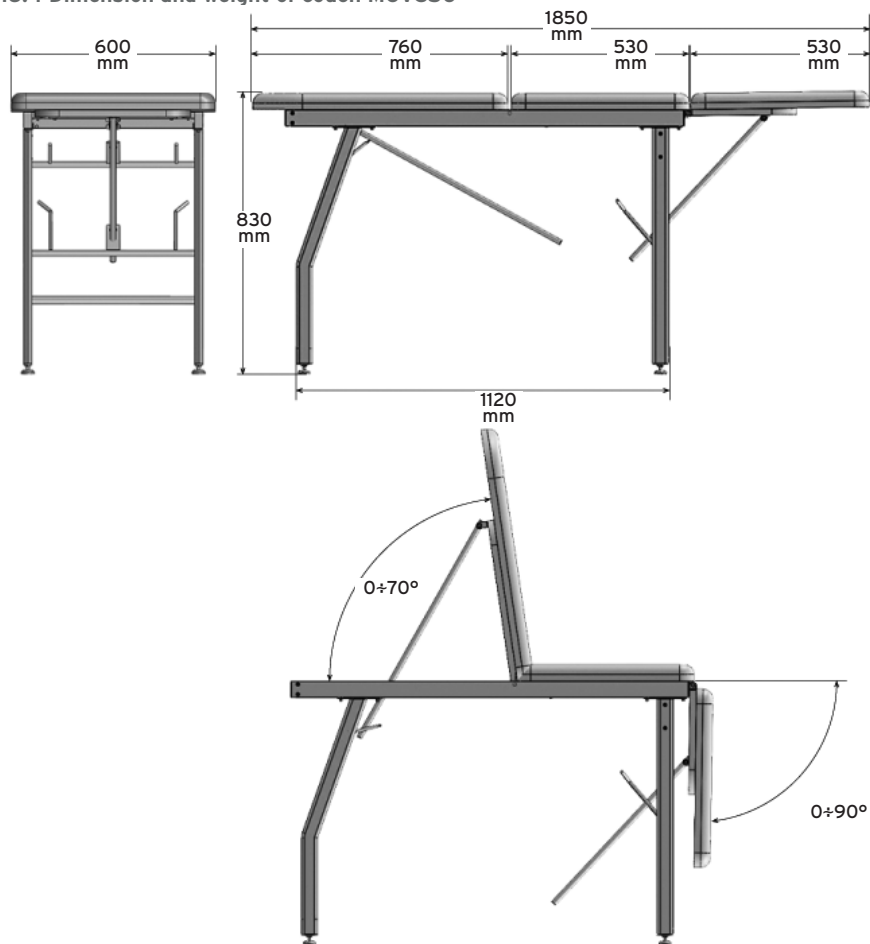
16.3 Dimension and weight of couch MOV340

 Packing dimension	 L: 1920 mm	 P: 800 mm	 H: 220 mm
 Packing weight	43 kg		
 Couch weight	40 kg		
 Max. load	180 kg		

16.3.1 Technical specifications of couch MOV340

Perimeter frame in painted rectangular tubular 40x25 mm. Couch legs frame in oval tubular steel, 50x25 mm. Padded in polyurethane, density 25 high 5 cm. Upholstery in washable and self-extinguishing synthetic leather, 11M.

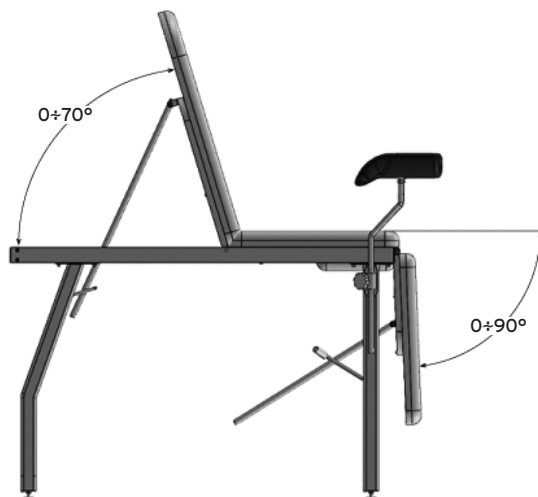
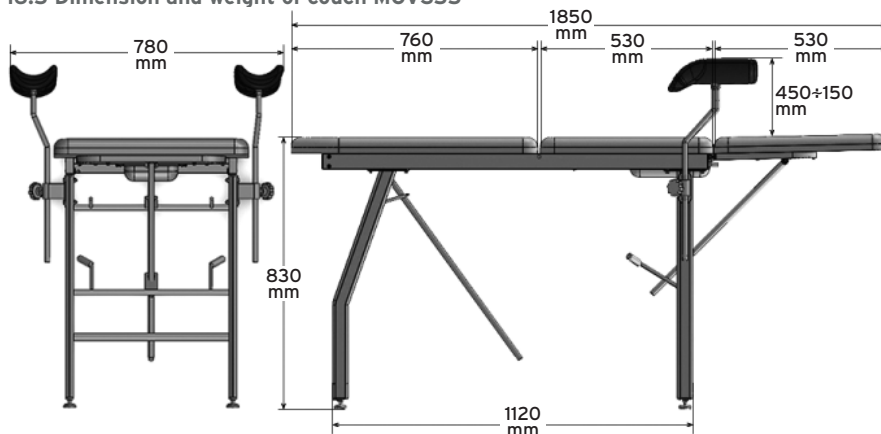
16.4 Dimension and weight of couch MOV350

 Packing dimension	 L: 1920 mm	 P: 650 mm	 H: 220 mm
 Packing weight	35 kg		
 Couch weight	32 kg		
 Max. load	180 kg		

16.4.1 Technical specifications of couch MOV350

Perimeter frame in painted rectangular tubular 50x25 mm. Couch legs frame in oval tubular steel, 50x25 mm. Padding in polyurethane, density 25 high 5 cm. Upholstery in washable and self-extinguishing synthetic leather, 11M class. Three sections, 2 of which (headrest and legrest) with independent movement.

16.5 Dimension and weight of couch MOV355

 Packing dimension	 L: 1920 mm	 P: 650 mm	 H: 220 mm
 Packing weight	39 kg		
 Couch weight	36 kg		
 Max. load	180 kg		

16.5.1 Technical specifications of couch MOV355

Perimeter frame in painted rectangular tubular 50x25 mm. Couch legs frame in oval tubular steel, 50x25 mm. Padding in polyurethane, density 25 high 5 cm. Upholstery in washable and self-extinguishing synthetic leather, 11M class. Three sections, 2 of which (headrest and legrest) with independent movement. Sliding and removable Stainless Steel tray for liquid collection. Adjustable anatomic legs supports in molded PU.

17. WARRANTY

Every Moretti product is guaranteed for 2 (two) years from the purchase date, except the restrictions specified below.

The warranty shall not apply in case of improper use, abuse, alterations of the product or any other use not in compliance with the user manual. The proper use of the product is indicated in the user manual. Moretti S.p.A. shall not be liable for any damages or personal injuries due to incorrect assembly/improper use of the device, not compliant with the user manual.

Moretti does not guarantee its products against damages or faults in the following cases: natural disasters, non authorized maintenance or reparations, damages caused by electrical supply problems (if provided), use of parts or components not provided by Moretti, failure in accomplishing the guidelines and instructions for use, non authorized alterations, damages due to the transport (different from Moretti original delivery) or failure in maintaining the device as indicated in the manual.

The components subject to wear are not covered by this warranty if the damage is due to the normal use of the product.

18. REPARATIONS

18.1 Reparations under warranty

If a Moretti product shows material or manufacturing faults during the warranty period, Moretti S.p.A., together with the client, will consider whether the product fault is covered by warranty. Moretti S.p.A., at its unquestionable discretion, may replace or repair the item under warranty in a Moretti reseller point or at its own qualified offices. Labour costs for repairing the product may be at Moretti's expense if it is determined the repair is under warranty. A repair or replacement does not renew nor extend the warranty.

18.2 Reparation of an out-of-warranty product

A product, not covered by warranty, may be returned for reparation only after receiving prior authorization from Moretti Client Service. Labour and delivery costs for an out-of-warranty reparation will be fully at the client's or reseller's expense. Reparations of out-of-warranty products are guaranteed for 6 (six) months, from the day the client receives the repaired product.

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19. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the day you receive them.

20. EXONERATION CLAUSES

Except what clearly specified in this warranty and in accordance with the law, Moretti S.p.A. does not offer its clients any other declaration, warranty or condition, expressed or implicit, including any declaration, warranty or condition of merchantability, of non violation or non interference, of suitability for a specific aim. Moretti S.p.A. does not guarantee that the use of Moretti products will be continuous and faultless.

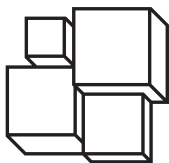
The duration of possible implicit warranties that may be applied in accordance with the law is limited to the warranty period. Some states or countries do not allow limitations to the duration of an implicit warranty, or the exoneration, or the limitations of accidental or indirect damages related to products for consumers. In such states or countries some exonerations or limitations of this warranty may not be applied to the user. This warranty is subject to modifications without any prior notice.

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