**KYARA®**

ITALIANO

MA LTK720\_LTK760\_01 B\_IT\_04-2019

**TERMOFORO A FASCIA  
E GUANCIALE TERMICO  
3 TEMPERATURE**

---

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## INDICE

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## 9. CARATTERISTICHE TECNICHE

 NO BLEACH	 NO IRONING	Model: <span style="border: 1px solid black; padding: 2px 5px;">REF</span>	
 NO DRY CLEAN	 NO WASHER DRY	Voltage: <b>220-240 ~ 50Hz</b> Rated power input: <b>100W</b>	<span style="border: 1px solid black; padding: 2px 5px;">LOT</span>

IMPORTED AND DISTRIBUTED BY MORETTI S.P.A. Cavriglia (AR) Italy. MADE IN P.R.C.

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EN - USED ONLY WITH DETACHABLE CONTROLLER SS03. A TOO LONG APPLICATION CAN RESULT IN SKIN BURNS. DO NOT DRY CLEAN. DO NOT USE WITH INFANTS, A HELPLESS PERSON OR A PERSON INSENSITIVE TO HEAT. HAND WASH ONLY. DO NOT USE IF WET.

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 **MORETTI S.P.A.**

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

**www.morettispa.com** email: **info@morettispa.com****MADE IN P.R.C.**

**KYARA®**

ENGLISH

MA LTK720\_LTK760 01 B\_EN\_04-2019

**WAIST WRAP HEATING PAD -  
HEATING PILLOW  
3 TEMPERATURES**

---

**INSTRUCTION MANUAL**

## INDEX

<b>1. CODES .....</b>	<b>pag.3</b>
<b>2. INTRODUCTION .....</b>	<b>pag.3</b>
<b>3. SAFETY WARNINGS.....</b>	<b>pag.3</b>
<b>4. SYMBOLS.....</b>	<b>pag.4</b>
<b>5. GENERAL DESCRIPTION.....</b>	<b>pag.5</b>
<b>6. INSTRUCTION OF USE.....</b>	<b>pag.5</b>
6.1 Automatic switch-off function.....	pag.5
6.2 Switch-off .....	pag.6
<b>7. CLEANING AND MAINTENANCE.....</b>	<b>pag.6</b>
<b>8. GENERAL CONDITIONS OF DISPOSAL .....</b>	<b>pag.6</b>
<b>9. TECHNICAL FEATURES.....</b>	<b>pag.7</b>
<b>10. TROUBLESHOOTING.....</b>	<b>pag.7</b>
<b>11. WARRANTY .....</b>	<b>pag.8</b>
11.1 Exempt clauses.....	pag.8

## 1. CODES

**LTK720** WAIST WRAP HEATING PAD - 3 TEMPERATURES

**LTK760** HEATING PILLOW - 3 TEMPERATURES

## 2. INTRODUCTION

Thank you for choosing the heating pad of the Kyara line by Moretti. Please read the operating instructions and safety warnings in particular before using the appliance and follow these at all times. Keep this manual and illustrative guide with the appliance for future consultation. Should you pass the device on to another user, make sure to also include this documentation. This manual includes suggestions for a correct use of the appliance and precious advices for your safety. If you have any doubts, please contact the retailer, who will be able to help you and advise you correctly.

**NOTE** Check that all parts of the product have not been damaged during the shipment. In case of damage do not use the product and contact the retailer for further instructions.

## 3. SAFETY WARNINGS

- After having removed the appliance from its packaging, verify its integrity based on the drawing and check for any damage caused during transport. If in doubt, do not use the appliance and contact the local retailer. Do not let children play with the packaging! Keep the plastic wrapping out of the reach of children; risk of suffocation!
- Before connecting the appliance, check that the mains voltage data reported on the identification plate corresponds to that of the available electric mains. The identification plate is attached to the appliance.
- This appliance is intended exclusively as a heating pad for home use. Any other use is improper and potentially dangerous.
- When using for the first time, you may notice a nasty odour due to the synthetic materials.
- Checking the heating pad outside with your hands must not be taken into account as this may give a misleading result of the heat emitted.
- The appliance is not to be used by persons (including children) with reduced physical, sensorial or mental abilities, or with no experience or knowledge, unless supervised by a person responsible for their safety, supervision or after having received instructions regarding its use. Children must be supervised to ensure they do not play with the appliance. Do not use the appliance for ill people, for infants or for persons who are insensitive to heat. This appliance must not be used by children if the control devices have not been set beforehand by a parent or guardian or if the child has not been properly instructed on how to use the device safely.
-  **AVWARNING - NOT** suitable for children under the age of 3. Children under the age of 3 must not use the appliance as they are unable to react in the event of overheating.
- **DO NOT** activate the temperature control with wet hands.
- **DO NOT** pull the power cord or the appliance itself to remove the plug from the socket.
- **DO NOT** immerse the power cord and the remote control.
- Always disconnect the appliance from the mains before cleaning or performing maintenance and when the appliance is not in use.
- In case of a fault or malfunctioning of the appliance, switch it off and do not tamper with it. If the appliance needs to be repaired, contact an authorised technical service centre.
- If the power cord is damaged, it must be replaced by an authorised technical service centre, so as to prevent all risks.
- The appliance must be used solely with the types of control indicated on the label.

- DO NOT place the remote control (1) or the power cable underneath or on top of the heating pad when the appliance is turned on.
-  **DO NOT** use the folded appliance.
- Before storage, let the appliance cool before folding it.
- **DO NOT** leave clothes inside the appliance if folded.
- Inspect the appliance frequently to check for signs of wear and tear. If such signs are found or if the appliance has been used incorrectly, contact an authorised technical assistance centre before continuing to use it.
- Persons with pacemakers must not use the appliance.
- This appliance is not intended for medical use inside of hospitals.
- **DO NOT** use the appliance if it is wet or in locations with an high level of humidity (e.g., bathroom).
- **DO NOT** use the appliance to keep animals warm.
- **DO NOT** leave the appliance running while you sleep.
- If the device is used for an extended period, the selector should be placed on the level 1-2 (minimum heat).
- **DO NOT** use the appliance on swollen or painful parts of the body.
- Immediately switch the device off if you feel pain during the use.
- **DO NOT** use the device if highly flammable gases are present in the air.
- **NEVER** use the device near to a gas source.
- **DO NOT** place on the device flammable materials or materials that easily deteriorate due to the heat (e.g., carton, plastic, paper, candles).
- **NEVER** expose the appliance to other heat sources (such as burners or ovens), sub-zero temperatures, direct sunlight, humidity.
-  **DO NOT INSERT PINS OR NEEDLES.**
- Application of the appliance for a long time at high temperature can burn the skin.

**WARNING!**

In case of improper use, exist the risk of possible injuries.

**4. SYMBOLS**

Product code

Month and year of production

CE mark

Instruction manual

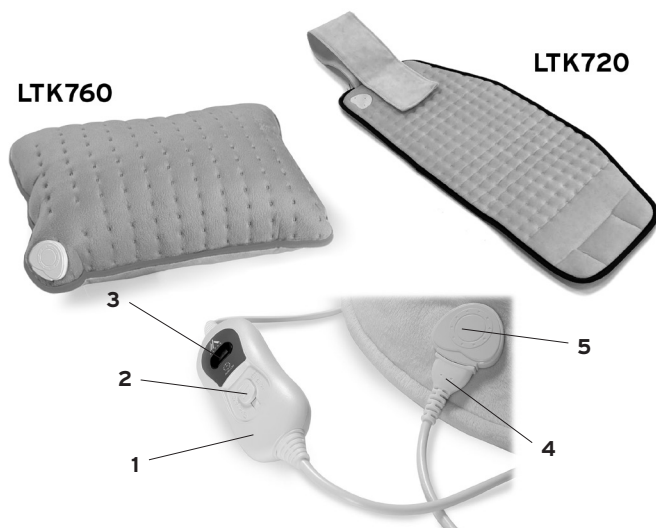
Disposal

Insulation class II

## 5. GENERAL DESCRIPTION

### MAIN COMPONENTS

1. Remote control
2. Temperature selector
3. Display
4. Connector
5. Connector base

### 6. INSTRUCTION OF USE

The heating pad is designed to warm the human body. It allows for targeted application of heat. The heating pad can be placed on parts of the body, such as: neck, shoulders, back, abdomen and joints, depending on the relative model.

#### ATTENTION!

The heating pad can only be used with the control supplied. The relative model is specified on the label sewn on the panel.

- Insert the connector (4) into the connector base (5) to connect the temperature selector to the heating pad.
- Insert the plug in the socket and select the maximum temperature for a short time, in order to warm up the heating pad faster, then select a temperature level from 1 to 3. (Level 1 = minimum temperature; Level 2 = medium temperature; Level 3 = maximum temperature)
- Place the heating pad on the part of the body that is to be heated.

#### 6.1 Automatic switch-off function

- The heating pad is provided with an automatic switch-off system. After 90 minutes of

functioning it automatically turns off.

- To turn the device on once again, place the temperature selector in position "0", wait a few seconds, then select the desired temperature once again.

### 6.2 Switch-off

- Switch the heating pad off by placing the temperature selector in position "0".
- Remove the plug from the socket.

## 7. CLEANING AND MAINTENANCE

Do not use bleach

Hand wash at a temperature not higher than 30°

Do not dry-clean

Do not dry the heating pad in the dryer

Do not iron

- Before cleaning make sure that the heating pad is not connected to the electric main and that it's cold.
- Before cleaning make sure the connection cable is detached.
- The heating pad can be hand-washed (DO NOT USE THE WASHING MACHINE) at a temperature not higher than 30°.
- Use a neutral cleanser.
- DO NOT use bleach.
- DO NOT dry in the dryer, leave the heating pad dry to the air. DO NOT place it on radiators or other heating systems.
- DO NOT iron.
- Make sure that the heating pad is completely dried, before using it again.
- While cleaning do not use chemical or abrasive products.

## 8. GENERAL CONDITIONS OF DISPOSAL

The product is packaged in recyclable materials. Dispose of it in compliance with the environmental protection standards.

The product must be disposed of appropriately, in compliance with European standard 2002/96/EC. The recyclable materials contained in the appliance must be recovered so as to prevent environmental pollution. For further information, contact the local disposal

body or the appliance dealer.

## 9. TECHNICAL FEATURES

**NO BLEACH**   **NO IRONING**   Model:  

**NO DRY CLEAN**   **NO WASHER DRY**   Voltage: **220-240 ~ 50Hz**  

**Rated power input: 100W**

IMPORTED AND DISTRIBUTED BY MORETTI S.P.A. Cavriglia (AR) Italy.      MADE IN P.R.C.

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(Image for illustrative purpose only)

- Measure 29x69 cm (cod. LTK720); measure 40x30 cm (cod. LTK760)
- Power 100W - Voltage 220-240V
- Material 100% polyester

## 10. TROUBLESHOOTING

### ATTENTION!

If the electrical device does not work properly and it is defective, do not attempt to repair it by yourself.

PROBLEM	POSSIBLE CAUSE	SOLUTION
The device does not work.	The device does not receive the power source.	Verify if: <ol style="list-style-type: none"> <li>the power cable is connected to the connecting socket;</li> <li>the plug is insert in the socket;</li> <li>the socket is defective;</li> <li>the fuse of your mains connection has been activated.</li> </ol>
	Any temperature level has been selected. The temperature selector is in position 0.	Move the temperature selector to position 1, 2 or 3.
	The protection against overheating has been activated.	Have the device checked by the manufacturer.

## 11. WARRANTY

Every Moretti product is guaranteed for 2 (two) YEARS from the purchase date, except the restrictions specified below. The warranty shall not apply in case of damage caused by improper use, abuse, alterations of the product or any other use not in compliance with the user manual.

The proper use of the product is indicated in the user manual. Moretti S.p.A. shall not be liable for any damages or personal injuries due to incorrect assembly/improper use of the device, not compliant with the user manual.

Moretti does not guarantee its products against damages or faults in the following cases: natural disasters, non authorized maintenance or reparations, damages caused by electrical supply problems (if provided), use of parts or components not provided by Moretti, failure in accomplishing the guidelines and instructions for use, non authorized alterations, damages due to the transport (different from Moretti original delivery) or failure in maintaining the device as indicated in the manual.

The components subject to wear are not covered by this warranty if the damage is due to the normal use of the product.

### 11.1 Exempt clauses

Except what clearly specified in this warranty and in accordance with the law, Moretti S.p.A. does not offer its customers any other declaration, warranty or condition, expressed or implicit, including any declaration, warranty or condition of merchantability, of non violation or non interference, of suitability for a specific aim. Moretti S.p.A. does not guarantee that the use of Moretti products will be continuous and faultless. The duration of possible implicit warranties that may be applied in accordance with the law is limited to the warranty period.

Some states or countries do not allow limitations to the duration of an implicit warranty, or the exoneration, or the limitations of accidental or indirect damages related to products for consumers. In such states or countries some exonerations or limitation of this warranty may not be applied to the user. This warranty is subject to modifications without any prior notice.

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