ITALIANO

MA CAR900 02 A\_ITA\_11 2020

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**MORETTI S.P.A.**

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

**www.morettispa.com email: info@morettispa.com****MADE IN P.R.C.**

ENGLISH

MA CAR900 02 A\_ENG\_11 2020

**FOLDING HEADREST FOR CA SERIES WHEELCHAIRS**

---

**INSTRUCTION MANUAL**

## INDEX

1. CODE .....	PAG.3
2. INTRODUCTION.....	PAG.3
3. INTENDED USE.....	PAG.3
4. DECLARATION OF CONFORMITY .....	PAG.3
5. GENERAL WARNINGS .....	PAG.4
6. SYMBOLS .....	PAG.4
7. GENERAL DESCRIPTION.....	PAG.4
7.1 Parts view .....	pag. 4
8. ASSEMBLING/DISASSEMBLING .....	PAG.5
8.1 Assembling.....	pag. 5
8.2 Disassembling .....	pag. 5
9. BEFORE USE.....	PAG.5
10. HOW TO USE.....	PAG.5
10.1 How to fold the wheelchair with the headrest assembled .....	pag. 5
11. MAINTENANCE .....	PAG.5
12. CLEANING AND DISINFECTION.....	PAG.5
12.1 Cleaning.....	pag. 5
12.2 Disinfection .....	pag. 5
13. CONDITIONS OF DISPOSAL .....	PAG.6
14. SPARE PARTS/ACCESSORIES.....	PAG.6
15. TECHNICAL FEATURES.....	PAG.6
15.1 Composing material .....	pag. 6
16. WARRANTY .....	PAG.6
17. REPAIRING.....	PAG.6
17.1 Warranty repair.....	pag. 6
17.2 Repair not covered by warranty .....	pag. 6
17.3 No-defective device.....	pag. 6
18. SPARE PARTS.....	PAG.6
19. EXEMPT CLAUSES .....	PAG.7

CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL  
of 5 April 2017 concerning medical devices**1. CODE****CAR900** Folding headrest for CA series wheelchairs**2. INTRODUCTION**

Thank you for purchasing an ARDEA headrest by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advise you properly.

**3. INTENDED USE**

The headrest is used on ARDEA CA series wheelchairs as patient head adjustable and comfortable accessory

**CAUTION!**

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice

**4. DECLARATION OF CONFORMITY**

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of MANUAL WHEELCHAIRS - ARDEA ONE complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017. For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

**Note:** Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

**5. GENERAL WARNINGS**


- DO NOT use this product or any available optional equipment without first completely reading and understanding this instruction manual. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment, otherwise injury or damage may occur
- Advise with physician or therapist to determinate the correct adjustment and the correct use of the headrest
- Keep the packed headrest away from heat sources
- SERVICE LIFE- the headrest use limit is defined by the wear of the parts
- ALWAYS keep hands and fingers clear of moving parts to avoid injury
- DO NOT allow children to play on or operate the headrest
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

**6. SYMBOLS**

	Product code
	Unique Device Identification
	CE mark
	Manufacturer
	Batch Lot
	Read the instruction manual
	Medical Device
	Conditions of disposal

**7. GENERAL DESCRIPTION****7.1 Parts view**

The headrest is composed by the following

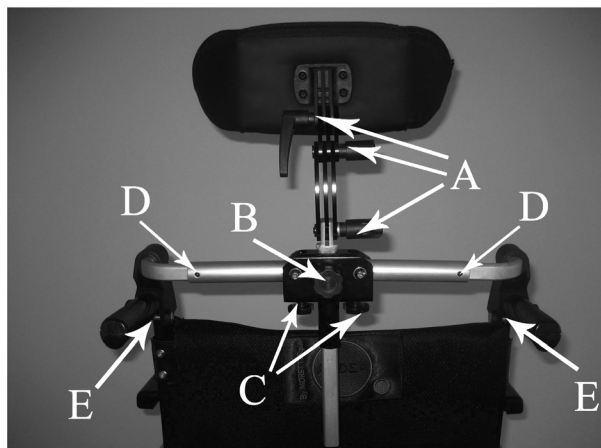(FIG.1)

- A. Knob for depth and angle adjustment
- B. Knob for height adjustment
- C. Locking knob
- D. Allen wrench for width adjustment
- E. Hexagonal screw to lock the headrest on the handles

## **8. ASSEMBLING/DISASSEMBLING**

### **8.1 Assembling**

- Unscrew the Allen wrench to adjust the width (Fig.1 Rif. D)
- Position the headrest on the wheelchair handles (on the wheelchair frame)
- Insert the screws and tight them (Fig.1 Rif.E)
- Once the headrest is correctly assembled, tight the Allen wrenches
- Screw the knobs under the headrest to lock it
- Adjust the height, the depth and the angle of the headrest using the knobs (Fig.1 Rif. A-B)

### **8.2 Disassembling**

- Follow the previous instruction in reversed order

## **9. BEFORE USE**

- Check if the mechanical parts are damaged in order to guarantee a safe use for people and things
- Check the correct assembly, paying particular attention to the assembly on the wheelchair handles

## **10. HOW TO USE**

### **10.1 How to fold the wheelchair with the headrest assembled**

The headrest has been designed to allow an easy and fast closing of the wheelchair, without disassembling. Just unscrew the knobs under the headrest (Fig.1 Rif.C) and close the wheelchair as usual

**NOTE Remember to tight the knobs after the wheelchair is open**

## **11. MAINTENANCE**

The ARDEA accessories by MORETTI are checked carefully and supplied with EC mark, once launched on the market.

For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years.

In case of repair, use only original spare parts and accessories.

## **12. CLEANING AND DISINFECTION**

### **12.1 Cleaning**

Use a damp cloth and mild soap. Then dry thoroughly before use.

### **12.2 Disinfection**

If you need to disinfect the device use a common disinfectant cleaner

**NOTE Never use acids, alkalis or solvents such as acetone**

**13. CONDITIONS OF DISPOSAL** 

When disposing of the device never use the normal return systems of municipal solid waste. It is recommended instead to disposal through the joint municipal ecological islands for the expected operations of recycling the materials used.

**14. SPARE PARTS/ACCESSORIES**

For spare parts and accessories refer to main catalogue

**15. TECHNICAL FEATURES****15.1 Composing material**

Frame -	Anodized aluminum
Pad -	Foam
Cover -	PVC

**16. WARRANTY**

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual. Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to wrong installation or improper use. Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, not authorized alteration, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

**17. REPAIRING****17.1 Warranty repair**

If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

**17.2 Repair not covered by warranty**

A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

**17.3 No-defective device**

The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be paid by the customer.

**18. SPARE PARTS**

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

**19. EXEMPT CLAUSES**

Moretti does not offer any other declaration, explicit or implicit warranty or conditions, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

**WARRANTY CERTIFICATE**

**Product** \_\_\_\_\_

**Purchased on (date)** \_\_\_\_\_

**Retailer** \_\_\_\_\_

**Address** \_\_\_\_\_ **Town/city** \_\_\_\_\_

**Sold to** \_\_\_\_\_

**Address** \_\_\_\_\_ **Town/city** \_\_\_\_\_

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

[www.morettispa.com](http://www.morettispa.com) email: [info@morettispa.com](mailto:info@morettispa.com)

**MADE IN P.R.C.**

**MORETTI S.P.A.**

Via Bruxelles, 3 - Meleto  
52022 Cavriglia (Arezzo)

Tel. +39 055 96 21 11  
Fax. +39 055 96 21 200

[www.morettispa.com](http://www.morettispa.com)  
[info@morettispa.com](mailto:info@morettispa.com)