MOPEDIA®

MA RS844-RS846-RS847-RS848 02 A_ITA_09-2020

SEDIE DA DOCCIA E WC

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ITALIANO

INDICE

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Via _____ Località _____

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com**MADE IN P.R.C.**

NOTE

MOPEDIA®

MA RS844-RS846-RS847-RS848 02 A_ENG_09-2020

SHOWER AND WC CHAIR

INSTRUCTION MANUAL

ENGLISH

INDEX

1. CODES.....	PAG.3
2. INTRODUCTION.....	PAG.3
3. INTENDED USE.....	PAG.3
4. DECLARATION OF CONFORMITY	PAG.3
5. GENERAL WARNINGS	PAG.4
6. SYMBOLS	PAG.4
7. GENERAL DESCRIPTION.....	PAG.4
7.1 View and list of Components	pag. 4
8. ASSEMBLING/DISASSEMBLING	PAG.5
8.1 Assembling RS844.....	pag. 5
8.2 Assembling RS846 - RS847	pag. 6
8.3 Assembling RS848.....	pag. 7
8.4 Disassembling	pag. 7
9. BEFORE TO USE	PAG.7
10. PRECAUTIONS FOR USE	PAG.7
11. HOW TO USE.....	PAG.8
11.1 RS844.....	pag. 8
11.2 RS846 - RS848	pag. 8
11.3 RS847	pag. 9
12. MAINTENANCE	PAG.9
13. CLEANING AND SANITISATION	PAG.9
13.1 Cleaning.....	pag. 9
13.2 Disinfection.....	pag. 10
14. CONDITIONS OF DISPOSAL	PAG.10
15. TECHNICAL FEATURES.....	PAG.10
15.1 Dimensions and weight	pag. 10
15.2 Technical specifications.....	pag. 10
16. WARRANTY	PAG.10
17. REPAIRING.....	PAG.10
18. SPARE PARTS.....	PAG.11
19. EXEMPT CLAUSES	PAG.11

CE I Class Medical Device
REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

RS844 - Tilting shower and WC chair with 4 castors

RS846 - Demountable shower and WC chair with 4 castors

RS847 - Self propelled shower and WC chair

RS848 - Foldable shower and WC chair with 4 castors

2. INTRODUCTION

Thank you for choosing a MOPEDIA shower and WC chair by Moretti S.p.A. MOPEDIA shower and WC chair manufactured by Moretti S.p.A. are designed to satisfy all your requirements for a practical, correct and safe use. This manual provides some useful suggestions to correctly use the device you have selected and offers valuable advices for your safety. We recommend carefully reading the entire manual before using the hospital bed you have purchased. Should you have any queries, please contact your retailer for proper advice or assistance.

NOTE: Check all parts of the product have not been damaged during shipment. In case of damage do not use the product and contact your dealer for further instructions.

CAUTION!

- Do not use the following device for purposes other than those described in this manual.
- Moretti S.p.A. disclaims any liability for damage resulting from improper use of the device or use other than that described in this manual.
- We reserve the right to make changes to the device and this manual without notice in order to improve the characteristics.

3. INTENDED USE

The MOPEDIA shower and WC chairs are created for people with movement difficulty and can be used indoors as substitution of toilet or as aid under the shower.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of SHOWER CHAIRS complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

WARNINGS!

Always pay close attention to the presence of moving parts that could cause limb entrapment and injury.

- For proper use of the device refer to this manual.
- For proper use of the device consult your doctor or therapist.
- Keep the packaged product away from any source of heat because the packaging is made of cardboard.
- The useful life of the accessory is determined by wear of parts cannot be repaired and / or replaced.
- Be careful when children are nearby.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

Product code

Batch Lot

Unique Device Identification

Medical Device

Read the instruction manual

CE mark

Conditions of disposal

Manufacturer

7. GENERAL DESCRIPTION

7.1 View and list of Components

RS844 Tilting WC and shower chair with 4 wheels (Fig.1)

Fig.1

1. Frame - 1 pc
2. Backrest with armrests - 1 pc
3. Pivoting wheels without brake - 2 pcs
4. Pivoting wheels with brake - 2 pcs
5. Headrest - 1 pc
6. Footrests - 2 pcs
7. Seat - 1 pc
8. Seat cover - 1 pc
9. Bucket - 1 pc

RS846 Demountable WC and shower chair with 4 wheels (Fig.2)

Fig.2

1. Left frame - 1 pc
2. Right frame - 1 pc
3. Connecting pipe - 2 pcs
4. Backrest with armrests - 1 pc
5. Footrests - 2 pcs
6. Seat - 1 pc
7. Seat cover - 1 pc
8. Bucket - 1 pc

RS847 Self-propelled WC and shower chair with 4 wheels (Fig.3)

Fig.3

1. Left frame - 1 pc
2. Right frame - 1 pc
3. Connecting pipe - 2 pcs
4. Backrest with armrests - 1 pc
5. Footrests - 2 pcs
6. Seat - 1 pc
7. Seat cover - 1 pc
8. Bucket - 1 pc

RS848 Foldable WC and shower chair with 4 wheels (Fig.4)

Fig.4

1. Foldable frame - 1 pc
2. Backrest with armrests - 1 pc
3. Footrests - 2 pcs
4. Seat - 1 pc
5. Seat cover - 1 pc
6. Bucket - 1 pc

8. ASSEMBLING/DISASSEMBLING

8.1 Assembling RS844

To complete the assembly carry out the following operations:

- Instal the 4 wheels on the frame using screws and washers provided. The wheels without brakes must be installed on the front side of the frame (fig.5);
- Insert the backrest inside the holes on the wheelchair frame and fix the backrest using the two fixing knobs (Fig. 6);
- Insert the headrest in the support already installed on the push handle and tighten the support (Fig.6);
- Place the seat on the wheelchair frame and press down to fix its position, then add the cover (Fig. 7);
- Insert the footrests from the side and rotate them to the front until fastening into the holder (Fig. 8);
- Insert the bucket from the rear side of the shower and WC chair by sliding it into the slots.

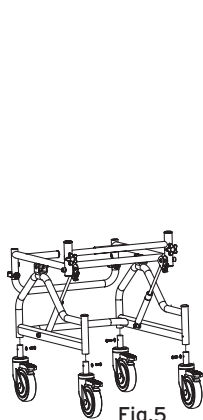
Fig.5

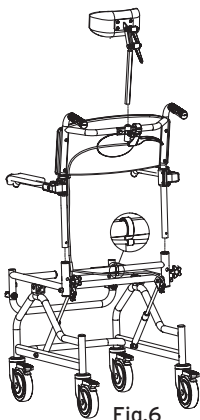
Fig.6

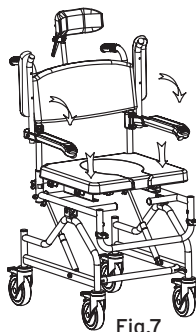
Fig.7

Fig.8

8.2 Assembling RS846 - RS847

To complete the assembly carry out the following operations:

- Insert the fixing tubes on the left or right side frame, align the holes and insert the C-clips as shown in Fig.9;
- Connect the newly assembled part to the other side of the frame by using the same two fixing tubes and two additional C-clips as described in the previous point (Fig.10);
- Insert the backrest inside the holes on the wheelchair frame and fix the backrest using the two fixing knobs (Fig.11);

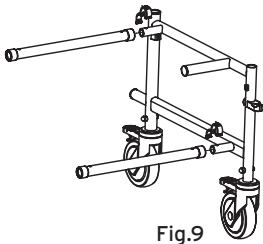
Fig.9

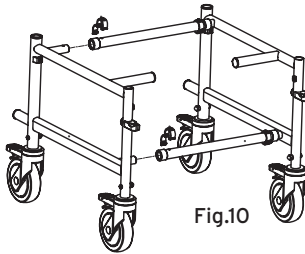
Fig.10

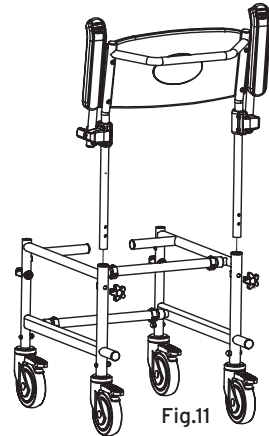
Fig.11

Images are illustrative purposes only

- Place the seat on the wheelchair frame and press down to fix its position, then add the cover (Fig.12);
- Insert the footrests from the side and rotate them to the front until fastening into the holder (Fig.13);
- Insert the bucket from the rear side of the shower and WC chair by sliding it into the slots (Fig.14).

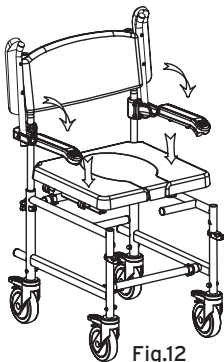
Fig.12

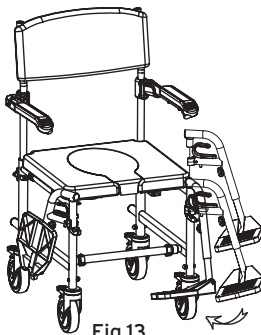
Fig.13

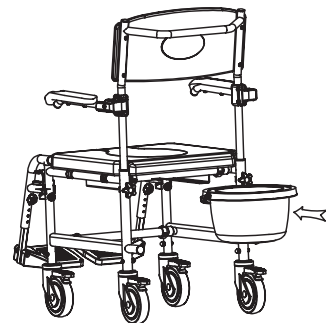
Fig.14

Images are illustrative purposes only

8.3 Assembling RS848

To complete the assembly carry out the following operations:

- Open the frame;
- Insert the backrest inside the holes on the wheelchair frame and fix the backrest using the two fixing knobs (Fig.15);
- Place the seat on the wheelchair frame and press down to fix its position, then add the cover (Fig.16);
- Insert the footrests from the side and rotate them to the front until fastening into the holder (Fig.17);
- Insert the bucket from the rear side of the shower chair by sliding it into the slots (Fig.18).

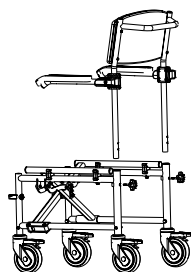
Fig.15

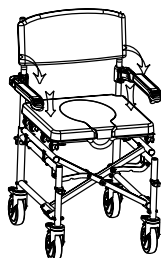
Fig.16

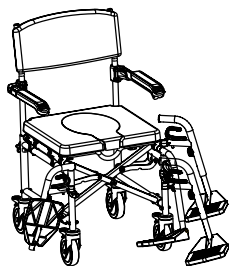
Fig.17

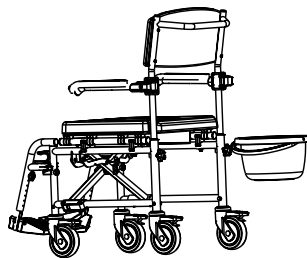
Fig.18

8.4 Disassembling

Follow the procedure described above in reverse.

9. BEFORE TO USE

- Always check the status of the mechanical parts in order to guarantee they are used in total safety for persons and environment.
- Check the proper assembly of the device with attention to the correct insertion and locking screws and nuts of the backrest and footrest.

10. PRECAUTIONS FOR USE

- The manufacturer and distributor are not responsible for any responsibility for the consequences of improper use of the product.
- Always help the patient with walking difficulties to position themselves on the seat.
- Do not use with patients who exceed the maximum weight shown on the label placed on the product frame.
- Avoid sudden movements during positioning.
- Be careful on wet floors, on wet or slippery surfaces, it could make difficult the patient positioning.
- Do not store the product outside. Periodically check the stability of the product and the correct fastening screws and the status of protections rubber.
- In case of doubt for the use, the adjustment or the maintenance of the product, please contact your dealer.
- Do not exceed the maximum load capacity for any reason.
- Assist always the patient during the handling operations.
- Don't skew the chair more than 10% without assistance.
- Block the wheels during patient transfer.
- Wheelchair has to be in a stable surface and wheels locked during user transfer.
- Before leaning or get up, be sure casters are in straight position and rear castors are blocked in order to avoid a turnover.
- Do not lean out of the backrest to avoid a turnover.
- Pay attention to your fingers during assembling, regulations or simply during the use.
- When using the self-propelled wheel, pay attention to your fingers.
- Keep the chair far from fire.
- Use always original accessories and spare parts.

WARNING! Do not exceed the maximum load capacity for any reason!

11. HOW TO USE

Use only the back handle to move the chair, never use other supports. Know your own capabilities and limitations in terms of strength and endurance before attempting to go up a ramp. Assistance is recommended when in slope higher than 10%. If you need to stop in a sloping surface, avoid abrupt movements. During the downhill, do not change suddenly the direction. Lift the chair up if necessary, using the main frame and not armrests or footrests.

WARNING! The patient must not put pressure on footrests when go up and down from the chair.

11.1 RS844

- Setting out to sit with your weight on the seat assembly. Do not sit by pushing on one side only;
- Leaning always on both arms to get up and keep your balance;
- In case of use of the chair by people with movement difficulties, help them always in the sitting or in getting up from the chair.

FOOTREST ADJUSTMENT

The footrests of shower and WC chair can be adjusted according to your needs:

- Unfasten the screw;
- Adjust the footrest height;
- Fasten the screw again. Check that both footrests are adjusted at the same height.

HEADREST ADJUSTMENT

The headrest can be easily adjusted in different positions. To make adjustments, unscrew the knobs on the headrest one at a time and adjust their position. Then tighten the knob firmly. Carry out the same operations also on the other knobs if necessary.

TILT ADJUSTMENT

To tilt the chair back, pull the lever installed on the push handle and push down.

To return the chair to its normal position, simply pull the lever installed on the push handle. The hydraulic pistons are designed to return to the starting position without the user applying any force.

11.2 RS846 - RS848

- Setting out to sit with your weight on the seat assembly. Do not sit by pushing on one side only;
- Leaning always on both arms to get up and keep your balance;
- In case of use of the chair by people with movement difficulties, help them always in the sitting or in getting up from the chair.

HEIGHT ADJUSTMENT

The backrest and the armrests can be adjusted together on two different heights. Unscrew and remove the knobs that lock the backrest to the seat and raise or lower the backrest as necessary. Finally insert and tighten the knobs again.

FOOTREST ADJUSTMENT

The footrests of shower and WC chair can be adjusted according to your needs:

- Unfasten the screw;
- Adjust the footrest height;
- Fasten the screw again. Check that both footrests are adjusted at the same height.

11.3 RS847

Use only the back handle to move the chair, never use other supports. Know your own capabilities and limitations in terms of strength and endurance before attempting to go up a ramp. Assistance is recommended when in slope higher than 10%. If you need to stop in a sloping surface, avoid abrupt movements. During the downhill, do not change suddenly the direction. Lift the chair up if necessary, using the main frame and not armrests or footrests.

**WARNING!**

The patient must not put pressure on footrests when go up and down from the chair.

BRAKES

You can brake the chair pushing the lever on each wheel. If you need to adjust the brake lever, you can unscrew the nut, move the clamp at the required position and then tighten again.

WC

Use the toilet when the wheelchair is completely blocked. To use the toilet take out the removable part of the seat using the handle on the front part. Take out the toilet using the handle.

HEIGHT ADJUSTMENT

The backrest and the armrests can be adjusted together on two different heights. Unscrew and remove the knobs that lock the backrest to the seat and raise or lower the backrest as necessary. Finally insert and tighten the knobs again.

FOOTREST ADJUSTMENT

The footrests of shower and WC chair can be adjusted according to your needs:

- Unfasten the screw;
- Adjust the footrest height;
- Fasten the screw again. Check that both footrests are adjusted at the same height.

12. MAINTENANCE

The MOPEDIA by MORETTI S.p.A. chairs need minimal maintenance and general care, but we suggest you to control regularly:

- Braking system: monthly.
- Cleaning parts (seat, backrest, armpad, wheels and so on) if necessary.
- Cleaning the upholstery of the seat as required.
- General check by qualified personnel every two years.

It is extremely important that the wheelchair is checked at the first use. In particular, please check that all screws and nuts are tight.

13. CLEANING AND SANITISATION

13.1 Cleaning

To clean the aid and the structure use a solution of alcohol and neutral soap (in equal parts) mixed with water.

WARNING! Do not use abrasive substances, acids, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of metal surfaces.

The manufacture company cannot be held responsible for the damage that is caused by using materials that could damage the product surface or corrosive chemicals during cleaning.

13.2 Disinfection

If you need to disinfect the product use a common disinfectant neutral detergent.

WARNING! Sanitising agents are corrosive. Avoid contact with the eyes and skin. Wear protective clothing, gloves, protective glasses and safety shoes. In case of accidental contact, rinse the affected part thoroughly using warm water and consult a doctor immediately.

14. CONDITIONS OF DISPOSAL

When disposing of the device never use the normal return systems of municipal solid waste. It is recommended instead to disposal through the joint municipal ecological islands for the expected operations of recycling the materials used.

15. TECHNICAL FEATURES

15.1 Dimensions and weight

Dimensions* and weight

MEASURES IN CM												
RS844	98	40	102	78	54	40÷55	46	49	49	17 kg	12	12
RS846	85	40	96÷98	73÷75	50	40÷55	46	45÷48	49	14 kg	12	12
RS847	101	40	96÷98	73÷75	50	40÷55	46	45÷48	51	17 kg	60	12
RS848	85	40	96÷98	73÷75	50	40÷55	46	45÷48	49	15 kg	12	12

*The dimension could be change $\pm 0.5\text{cm}$

15.2 Technical specifications

- Powder coated aluminum frame
- PU seat, armrest and back
- Folding armrests
- Removable WC bucket with cover
- Max User weight 136 Kg

16. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual. Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to wrong installation or improper use.

Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, not authorized alteration, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

17. REPAIRING

Warranty repair

If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be cover from warranty. Moretti, at its unquestionable

discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

Repair not cover by warranty

A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

Non defective device

The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be paid by the customer.

18. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

19. EXEMPT CLAUSES

Moretti does not offer any other declaration, explicit or implicit warranty or conditions, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

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