SKEMA[®]FORM

MA BCA114 01 A* _ITA_06-2023

**MATERASSO IGNIFUGO CON FODERO
PER BARELLA CORSIA BC100**

MANUALE D'ISTRUZIONI

INDICE

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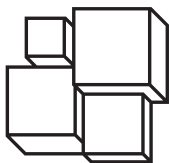
SKEMA[®]FORM

MA BCA114 01 A* _ENG_06-2023

**FIRE-RETARDANT MATTRESS WITH COVER
FOR BC100 WARD STRETCHER**

INSTRUCTION MANUAL

ENGLISH

CONTENTS

1. CODE	PAG.3
2. INTRODUCTION.....	PAG.3
3. INTENDED USE.....	PAG.3
4. EU DECLARATION OF CONFORMITY	PAG.3
5. GENERAL PRECAUTIONS.....	PAG.4
6. SYMBOLS USED.....	PAG.4
7. GENERAL DESCRIPTION.....	PAG.5
8. HOW TO USE	PAG.5
9. CLEANING AND DISINFECTION.....	PAG.5
9.1 Cleaning and disinfecting the covers	pag. 5
10. STORAGE	PAG.5
11. MAINTENANCE	PAG.5
12. DISPOSAL CONDITIONS	PAG.6
13. TECHNICAL SPECIFICATIONS.....	PAG.6
14. WARRANTY	PAG. 6
15. REPAIRS	PAG.6
15.1 Repairs under warranty.....	pag. 6
15.2 Repairing a product not covered by the warranty	pag. 7
15.3 Non-defective products.....	pag. 7
16. SPARE PARTS.....	PAG.7
17. NON-LIABILITY CLAUSE.....	PAG.7

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 on medical devices

1. CODE

BCA114 Fire-retardant mattress with removable cover for BC100 Stretcher

2. INTRODUCTION

Thank you for choosing a mattress for ward stretcher in the SKEMA range by Moretti. Moretti mattresses for ward stretcher have been designed and manufactured to satisfy all your needs for practical, correct and safe use. This manual contains useful suggestions for using your device properly and safely. Please read this entire manual carefully before you use the device purchased. Should you have any queries, please contact your retailer for suitable advice and assistance.

NOTE: indicate several recommendations.

CAUTION: indicate the correct operating or maintenance procedures, in order to avoid damage to the device, the equipment or other items.

WARNING: draw attention to a potential hazard that requires correct procedures or practices to avoid injury.

3. INTENDED USE

The SKEMA mattresses for lane stretcher by Moretti are intended to be used for the lying down of patients on ward stretchers which are in turn intended for the transfer of patients in hospital/healthcare environment.

WARNING!

- Do not use the product for any purpose other than that specified in this manual
- The device must be installed by personnel qualified to perform general nursing procedures and who have received appropriate training
- Moretti S.p.A. declines any and all liability for damages resulting from improper use of the device or any use other than that specified in this manual
- The manufacturer reserves the right to modify the device and its relevant manual without prior notice, in order to improve the product's features

4. EU DECLARATION OF CONFORMITY

MORETTI SpA declares, under its own exclusive responsibility, that the products made and sold by Moretti SpA in the ward stretchers product family conform with the applicable provisions of Regulation (EU) 2017/745 of 5 April 2017 on MEDICAL DEVICES. To this end, MORETTI SpA guarantees and declares as follows, under its own exclusive responsibility:

1. The devices in question satisfy the general safety and performance requirements set out in Annex I to Regulation (EU) 2017/745, as prescribed by Annex IV to said regulation.
2. The devices in question ARE NOT MEASUREMENT TOOLS.
3. The devices in question ARE NOT INTENDED FOR CLINICAL INVESTIGATIONS.
4. The devices in question are sold in NON STERILE packing.
5. The devices in question should be considered as belonging to class I, in accordance with the rules set out in Annex VIII to said regulation.
6. MORETTI SpA maintains and provides to the competent authorities, for at least ten years from the date of manufacture of the last production lot, the technical documentation proving conformity with Regulation (EU) 2017/745.

Note: The complete product codes, the manufacturer's single registration number (SRN), the basic UDI-DI code and any references to standards used are indicated in the EU Declaration of Conformity that MORETTI SPA issues and provides through its own channels.

5. GENERAL PRECAUTIONS

- Please consult this manual carefully for correct use of the device
- Use and store in the horizontal position
- Always sanitise the device before any reuse
- The devices are extremely sensitive to air and light
- Any fading of the colour does not alter the efficacy of the mattress
- The lifetime of the device depends on wear and tear of non-repairable and/or non-replaceable parts
- Store and use the device away from open flames and sources of heat
- Always take care when children are present
- Do not place the patient in direct contact with the mattress. Always use a full cover

6. SYMBOLS USED

	Product code
	Unique device identification
	CE marking
	Manufacturer
	Batch Lot
	Read the instruction manual
	Medical Device
	Disposal conditions
	Maximum washing temperature of 95°C; delicate cycle
	Do not bleach
	Do not iron
	Can be tumble dried; low drying temperature
	Hanging
	Do not dry clean
	Dry on a flat surface

7. GENERAL DESCRIPTION

BCA114 Fire-retardant polyurethane mattress with fire-retardant polyurethane/polyester cover for BC100 Stretcher - 195x65x7 cm

The padding is made in polyurethane foam, density 30 Kg/m³, thickness 7 cm. fire-retardant 11M class.

The polyester and polyurethane mattress covers are completely removable, via a zip around the edge. Breathable, bi-elastic, antibacterial and antifungal fire-retardant 11M class.

8. HOW TO USE

Position the cover on the bed, insert the mattress, then close on three sides using the zip.

9. CLEANING AND DISINFECTION

9.1 Cleaning and disinfecting the covers

Wash on delicate cycle at 95° with mild soap. Rinse thoroughly with plenty of water; do not tumble dry, do not wring, do not use solvents, dry away from sources of heat, do not iron, do not bleach.

Maximum washing temperature of 95°C; delicate cycle

Do not bleach

Do not iron

Can be tumble dried; low drying temperature

Hanging

Do not dry clean

Dry on a flat surface

10. STORAGE

Keep in a cool, dry place, away from damp environments.

11. MAINTENANCE

SKEMA devices manufactured by Moretti are CE marked and carefully checked before being commercialised.

For patient and doctor safety, have the product checked at least every two years by the manufacturer or by an authorised laboratory to verify its suitability for use. Use only original replacement parts and accessories for repairs.

12. DISPOSAL CONDITIONS

Never dispose of the product as normal domestic waste. Dispose of the product at a sorted waste collection centre for recycling.

13. TECHNICAL SPECIFICATIONS

Dimensions and weight

 L:	 D:	 H:	 kg
1950 mm	650 mm	70 mm	3,7 Kg

Padding:

- Polyurethane foam,
- Density 30 Kg/m³
- Fire-retardant 1IM class.

Cover:

- Class 1 fire-retardant
- Zip on three sides
- Breathable
- Antibacterial
- Bi-elastic
- Waterproof
- Washable at 95°
- 42% Polyester - 58% Polyurethane

14. WARRANTY

Moretti products are guaranteed for 2 (two) years from the date of sale against material and manufacturing defects, subject the following limitations. The warranty is voided by improper use, abuse, modifications to the product and failure to follow the instructions. The intended use of the product is given in the user manual. Moretti is not liable for damage, injury or any other consequences resulting from installation or use which are not scrupulously conforming with the instructions given in the installation, assembly and user manual. Moretti does not guarantee its products against damage or defects in the following circumstances: natural disasters, unauthorised repair or maintenance, improper electric power supply (as applicable), use of parts or components not supplied by Moretti, failure to follow the guidelines and instructions for use, tampering, shipping damage (other than the original shipping by Moretti), or failure to perform maintenance as indicated in the manual. Components subject to wear and tear are not covered by this warranty if the damage is caused by normal use of the product.

15. REPAIRS

15.1 Repairs under warranty

If a Moretti product has material or manufacturing defects during the warranty period, Moretti will agree with the client whether the defect is covered by the warranty. Moretti, at its sole discretion, may replace or repair the article at a specified Moretti reseller or its own premises. The costs of labour incurred in repairing the product will be borne by Moretti if it determines that the repair is covered by the warranty. Repair and replacement do not renew the warranty period.

15.2 Repairing a product not covered by the warranty

A product not covered by warranty may be returned for repair only if authorised in advance by Moretti customer service. The costs of labour and shipping incurred by repairs not covered by the warranty are borne by the client or reseller in their entirety. Repairs on products not covered by the warranty are themselves guaranteed for 6 (six) months from the day of reception of the repaired product.

15.3 Non-defective products

The client will be notified if Moretti concludes that the product is not defective after having received and examined it. The product will be returned to the client at his expense.

16. SPARE PARTS

Moretti original replacement parts are guaranteed for 6 (six) months from the day of delivery.

17. NON-LIABILITY CLAUSE

Unless otherwise expressly specified in this warranty and within the limits of the law, Moretti makes no declaration, guarantee or condition, express or implicit, including any future declaration, guarantee or condition of sale, suitability for a given purpose, non violation and non interference. Moretti does not guarantee that the use of its product will be uninterrupted and problem-free. The duration of any implicit guarantee under the law is limited to the warranty period, within the limits of the law. Certain states and countries do not permit limitations on the duration of an implicit guarantee or the exclusion of limitation of accidental or indirect damages in relation to consumer products. In said states and countries, certain exclusions and limitations of this warranty may not apply to the user. This warranty is subject to modification without notice.

WARRANTY CERTIFICATE

Product _____

Date of purchase _____

Reseller _____

Street _____ Town _____

Sold to _____

Street _____ Town _____

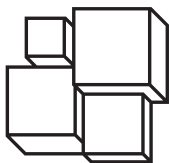
 MORETTI S.P.A.

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) - ITALY - Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN ITALY

** Please consult our website for the latest available version of the user manual*

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