MOPEDIA®

ITALIANO

MA RL100 01 A_ITA_12 2021

SET LAVATESTA GONFIABILE

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INDICE

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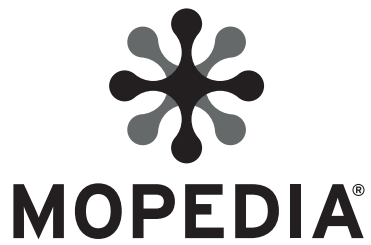
Via _____ Località _____

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com**MADE IN P.R.C.**

ENGLISH

MA RL100 01 A_ENG_12 2021

INFLATABLE HEAD BASIN SET

INSTRUCTION MANUAL

INDICE

1.CODE PAG.3

2. INTRODUCTION..... PAG.3

3. INTENDED USE..... PAG.3

4. GENERAL INFORMATIONS PAG.3

 4.1 WARNING FOR USE..... pag. 3

5. SYMBOLS PAG.4

6. GENERAL DESCRIPTION PAG.4

7. ABEFORE USE PAG.4

8. HOW TO USE..... PAG.4

9. MANTENIANCE..... PAG.5

10. CLEANING AND DISINFECTION PAG.5

11. CONDITION OF DISPOSAL PAG.5

12. TECHNICAL FEATURES..... PAG.5

 12.1 Dimension pag.5

13. WARRANTY PAG.6

14. REPAIRING PAG.6

 14.1 Warranty repair pag.6

 14.2 Repair not cover from warranty pag. 6

 14.3 Non-defective device..... pag. 6

15. SPARE PARTS PAG.6

16. EXEMPT CLAUSES PAG.6

ENGLISH

1. CODE

RL100 Inflatable head basin set

2. INTRODUCTION

Thank you for choosing the inflatable head basin from the MOPEDIA by Moretti range. This product MOPEDIA by Moretti has been designed and manufactured to meet all your requirements for a practical, correct and safe use. This manual contains small tips for the correct use of your chosen device and valuable advice for your safety. We recommend that you read this entire manual carefully before using the device you have purchased.

If you have any doubts, please contact your dealer, who will be able to help and advise you correctly.

Nota: Check that all parts of the product have not been damaged during shipment. In case of damage, do not use the product and contact your dealer for further instructions.

WARNING!

- The use of the following device for purposes other than those defined in this manual is prohibited.
- Moretti S.p.A. accepts no liability for damage resulting from improper use of the device or use other than that indicated in this manual.
- The manufacturer reserves the right to make changes to the device and to the following manual without prior notice in order to improve its characteristics.

3. INTENDED USE

The inflatable MOPEDIA head basin is intended for washing hair for bedridden persons in the home environment.

4. GENERAL INFORMATIONS

- Refer carefully to this manual for the correct use of the device.
- Keep the packed product away from any source of heat as the packaging is made of cardboard.
- The useful life of the device is determined by the wear and tear of non-repairable and/or replaceable parts
- Always pay attention to the presence of children

4.1 WARNING FOR USE

- Do not overinflate the device.
- Do not use if punctured or if it shows signs of thermal deformation.
- Do not use organic solvents and/or alkaline detergents.
- Do not place in contact with sharp or pointed objects.
- Do not dry the headrest using high temperatures.
- Inflate the container using the pump supplied, DO NOT use compressors or other inflating devices.
- Use lukewarm water to avoid burns (maximum 38°C).
- During use, do not leave the patient alone with the headrest full of water.
- Check the integrity of the device regularly.
- Disconnect the hoses by pulling from the end to avoid damaging the welds.
- The head wash set is not a toy: use it only for washing the hair of bedridden persons.
- Keep out of reach of children.

5. SYMBOLS

- | | |
|---|-----------------------------|
|  | Product code |
|  | CE Mark |
|  | Manufacturer |
|  | Batch lot |
|  | Read the instruction manual |
|  | Conditions of disposal |

6. GENERAL DESCRIPTION

- 1 pz PVC inflatable head basin
- 1 pz Water bag
- 1 pz Portable sprinkler
- 1 pz Air pump (non illustrato)
- 1 pz Drain pipe (non illustrato)

7. ABEFORE USE

- Always check the wear condition of the parts to ensure safe use of the device.
- Check the wear condition of the hand shower supplied.
- Ensure that the hand shower hose is correctly connected to the bag and the drain hose.

8. HOW TO USE

Nota: Refer to the general warnings and instructions for use in this instruction manual.

- Inflate the product using the pump supplied and close the cap.
- Place the inflated product at the head of the bed on a previously laid waterproof sheet.
- Connect the hose to the bag and the hand shower.
- Fill the bag with lukewarm water and hang it so that it is higher than the hand shower and make sure it is secure and stable.
- Ensure that a drainage system is in place before putting water into the hand shower.
- Have the user sit in the recess of the headrest.
- When finished washing, raise the user, taking care not to tip the headrest, and remove the drain plug.

9. MANTENIANCE

The products in the MOPEDIA by Moretti range are carefully checked and CE-marked when they are placed on the market.

Only original spare parts and accessories may be used for repairs.

- It is extremely important that the device is checked the first time it is used.
- Rinse the product well after each use with water and dry with a clean cloth.
- Store away from moisture or sources of light and heat.
- Make sure not to leave any residues of skin and hair care products (shampoo, balsam) on the product.

10. CLEANING AND DISINFECTION

10.1 Cleaning

- To clean the headrest, use only mild soap and water and dry with a clean cloth.
- Do not use abrasive or sharp objects or products to clean the various components.
- Do not dry near heat sources or in direct sunlight.

WARNING!

Do not use abrasive substances, acids, alcohol, chlorine-based detergents, disinfectants and acetone, oxides, organic solvents or alkaline detergents, as these may damage the plastic.

The manufacturer shall not be liable for damage caused by the use of materials that may deteriorate the surface of the product or by corrosive chemicals used during cleaning.

10.2 Disinfection

If necessary, disinfect the product using a mild disinfectant detergent or using a commercial, chlorine-free disinfectant.

WARNING!

Sanitising products are corrosive. Avoid contact with the eyes and skin. In case of accidental contact, wash the affected area with plenty of warm water and consult a doctor immediately.

11. CONDITION OF DISPOSAL

When disposing of the device, never use normal solid urban waste disposal systems. It is recommended that the device be disposed of through the municipal recycling centres for the planned recycling of the materials used.

12. TECHNICAL FEATURES

12.1 Dimension

Inflatable head basin	57 cm	47 cm
Water bag	49 cm	30 cm

13. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be apply in the event damages caused by improper use, abuse or alteration, and for the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual. Moretti is not responsible of consequent damages, personal injuries or whatever caused by or in relation with mistaken installation or with device improper use. Moretti does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance or repairs , faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, alteration not authorized, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

14. REPAIRING

14.1 Warranty repair

If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will estimate with customer if the fault can be cover from warranty. Moretti, at its unquestionable discretion, could repair or replace the item, with a Moretti dealer or with own base. Labor cost can be in charge to Moretti if the repair is cover from warranty. A repair or a replacement doesn't extend the warranty.

14.2 Repair not cover from warranty

A product without warranty can be given back after Moretti authorization. The labor and shipping costs for a bath lift out of warranty are to be payed by the customer or by the dealer. The repairs are guaranteed for 6 months as from the good received.

14.3 Non-defective device

The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be payed by the customer.

15. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

16. EXEMPT CLAUSES

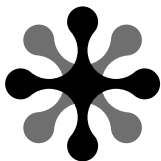
Moretti doesn't offer any other declarations, warranty or conditions, explicit or implicit, including possible declarations, warranties or condition of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti doesn't guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

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