KYARA®

ITALIANO

MA ST187B 01 A_ITA_11-2020

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CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODE

ST187B Hot water bottle in pure rubber - Blue bilaminated

2. INTRODUCTION

Thank you for purchasing the Kyara hot water bottle by Moretti S.p.A. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance.

In case of doubts, we recommend to contact the dealer who can help and advice you properly.

NOTE Check all parts for shipping damages and test before using. In case of damage, do not use the device. Contact your dealer for further instructions.

3. INTENDED USE

Hot water bottles kyara is indicated to relieve body pain. Please note that you have to fill it with drinking water.

CAUTION!

- Do not use the product for a purpose not indicated in this manual.
- Moretti S.p.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of PRODUCTS FOR PERSONAL CARE complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

- For a correct use of the device, refer to this manual
- Make sure that the product is delivered complete and keep it packed away from sources of heat
- The useful life of the device is determined by wear of parts can not be repaired and / or replaced
- Always pay attention to the presence of children.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Conditions of disposal

7. DESCRIPTION AND COMPONENTS

Hot water bottle in pure rubber - Bilaminated. Capacity 2 l.

8. BEFORE USE

Always check the state of wear of the parts in such a way as to ensure it is used in complete safety for persons and property

9. INSTRUCTIONS FOR USE

- Do not use the bottle for purposes not indicated in this manual.
- Moretti SpA is not responsible for purposes not indicated in this manual.
- We suggest you to use protections when you pour the water into the bottle.
- Do not put the bottle if you have skin irritations and always use a cloth between the skin and the bottle.
- To avoid accidental opening pay attention on the bottle if a child use it.

10. HOW TO USE

Verify the surface of the bottle: if you see cracks or damages, don't use it. Clean the internal part of the bottle, if possible. Fill the bottle with hot water (min. 15°C- max. 50°C). Close the bottle with care using the cap, let the air come out. Verify if the bottle loses water. Place the bottle on the body part using a cloth between the skin and the bottle.

WARNING!

Do not put boiling water inside the bag

11. MAINTENANCE AND STORAGE

11.1 Maintenance

The devices of the Kyara line by MORETTI are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. In case of repair, use only original spare parts and accessories.

11.2 Storage

For longer storage of the product we recommend placing the bag completely dry away from light and heat sources within sprinkling a bit talcum powder.

12. CLEANING AND DISINFECTION

12.1 Cleaning

For cleaning both inside and outside of the bag is recommended to use warm water and a neutral detergent. Allow to dry naturally. Do not expose to direct sunlight or sources of artificial heat

12.2 Disinfection

If you need to use to disinfect the products a common disinfectant cleaner

NOTE Never use acids, alkalis or solvents such as acetone or thinner

13. CONDITIONS OF DISPOSAL

When disposing of the device never use the normal systems of provision of municipal solid waste. It is recommended to dispose of the device at the local recycling center.

14. TECHNICAL FEATURES

Material: Rubber
Capacity: 2 l

15. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and for the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual.

Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to mistaken installation or improper use.

Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, alteration not authorized, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

15.1 Exempt clauses

Moretti does not offer any other declarations, warranty or conditions, explicit or implicit, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

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