ITALIANO

MA CS910BL 02 B_ITA_04 2021

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ENGLISH

MA CS910BL 02 B_ENG_04 2021

"ARIES TILTING" ELECTRIC WHEELCHAIR

INSTRUCTION MANUAL

INDEX

1. CODE	PAG.3
2. INTRODUCTION.....	PAG.3
3. INTENDED USE.....	PAG.3
4. EU DECLARATION OF CONFORMITY	PAG.3
6. GENERAL INFORMATION ON ELECTROMAGNETIC DISTURBANCE.....	PAG.4
7. SYMBOLS.....	PAG.5
8. SAFETY	PAG.5
9. WARNINGS ON USE	PAG.6
9.1 General warnings	pag. 6
9.2 Changes	pag. 7
9.3 Before every use.....	pag. 7
9.4 Weight limits.....	pag. 7
9.5 Inflating the tyres	pag. 7
9.6 Temperature	pag. 7
10. GENERAL DESCRIPTION.....	PAG.8
11. COMPONENTS.....	PAG.9
12. ASSEMBLY INSTRUCTIONS.....	PAG.9
12.1 Assembly	pag. 9
12.2 Adjusting comfort	pag. 11
12.3 Motor locking lever	pag. 13
13. USING THE ELECTRIC WHEELCHAIR.....	PAG.13
14. RECOMMENDATIONS FOR SAFE USE	PAG.14
15. GENERAL PRECAUTIONS	PAG.15
15.1 TILTING function.....	pag. 16
16. CHARGING THE WHEELCHAIR.....	PAG.16
17. REPLACEMENT OF THE BATTERY.....	PAG.17
18. REPLACING THE WHEELS	PAG.18
19. CARE AND MAINTENANCE.....	PAG.18
20. TROUBLESHOOTING.....	PAG.19
21. DISPOSAL CONDITIONS	PAG.20
21.1 General conditions of disposal	pag. 20
21.2 Instructions for correct disposal in accordance with European directive 2012/19/EU:.....	pag. 20
21.3 Disposal of exhausted batteries - (Directive 2006/66/EC):	pag. 20
22. TECHNICAL SPECIFICATIONS.....	PAG.20
22.1 Technical specifications.....	pag. 20
22.2 Dimensions.....	pag. 21
23. WARRANTY	PAG.21
24. REPAIRS	PAG.21
24.1 Repairs under warranty.....	pag. 21
24.2 Repairing a product not covered by the warranty	pag. 21
24.3 Non-defective products.....	pag. 22
25. REPLACEMENT PARTS.....	PAG.22
26. NON-LIABILITY CLAUSE	PAG.22

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 on medical devices

1. CODE

CS910BL “Aries Tilting” electric wheelchair

2. INTRODUCTION

Thank you for choosing an **ARDEA MOBILITY** electric wheelchair by Moretti. Moretti electric wheelchairs have been designed and manufactured to satisfy all your needs for practical, correct and safe use. This manual contains useful suggestions for using your device properly and safely. Please read this manual in full before you use the electric wheelchair. Should you have any queries, please contact your retailer for suitable advice and assistance.

Note: Check to ensure that no parts of the wheelchair have been damaged during shipment. Do not use the product if it is damaged and contact the retailer for further instructions.

3. INTENDED USE

The electric wheelchair is intended for people with mobility impairments.

- Do not use the product for any purpose other than that specified in this manual
- The electric wheelchair must always be used by people of sound body and mind and not under the influence of drugs or alcohol.
- Moretti S.p.A. declines any and all liability for damages resulting from improper use of the device or any use other than that specified in this manual
- The manufacturer reserves the right to modify the device and its relevant manual without prior notice, in order to improve the product's features

4. EU DECLARATION OF CONFORMITY

MORETTI SpA declares, under its own exclusive responsibility, that the products manufactured and sold by MORETTI SpA in the ELECTRIC WHEELCHAIR product family conform with the applicable provisions of Regulation (EU) 2017/745 of 5 April 2017 on MEDICAL DEVICES. To this end, MORETTI SpA guarantees and declares as follows, under its own exclusive responsibility:

1. The devices in question satisfy the general safety and performance requirements set out in Annex I to Regulation (EU) 2017/745, as prescribed by Annex IV to said regulation.
2. The devices in question ARE NOT MEASUREMENT TOOLS.
3. The devices in question ARE NOT INTENDED FOR CLINICAL INVESTIGATIONS.
4. The devices in question are sold in NON STERILE packing.
5. The devices in question should be considered as belonging to class I, in accordance with the rules set out in Annex VIII to said regulation.
6. MORETTI SpA maintains and provides to the competent authorities, for at least ten years from the date of manufacture of the last production lot, the technical documentation proving conformity with Regulation (EU) 2017/745.

Note: The complete product codes, the manufacturer's single registration number (SRN), the basic UDI-DI code and any references to standards used are indicated in the EU Declaration of Conformity that MORETTI SPA issues and provides through its own channels.

5. GENERAL PRECAUTIONS

WARNING!

Take extreme care in the presence of moving parts that could trap limbs and cause injury.

- Please consult this manual carefully for correct use of the device.
- Always consult your doctor or therapist for correct use of the device.
- Keep the packaged product away from sources of heat, as the packaging is cardboard.
- The lifetime of the device depends on wear and tear of non-repairable and/or non-replaceable parts.
- Always take care when children are present.
- Maximum user weight: 162 kg.
- Classification: class B (EN 12184).
- The product is not suitable for the visually impaired.
- The device must not be used by children under 12 years of age.
- The electric wheelchair is not designed for use as a seat in a motor vehicle.
- The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

6. GENERAL INFORMATION ON ELECTROMAGNETIC DISTURBANCE

The rapid growth of electronics, especially in communications, has saturated the environment with electromagnetic radiation (EM) emitted by television, radio and communication signals. These EM waves are invisible and are stronger the closer you are to their source. All electrical conductors act as antennas for EM signals and, to varying extents, all electric wheelchairs and scooters may be affected by electromagnetic interference (EMI). EMI may cause unintentional movements and/or irregular control of the vehicle.

The electromagnetic energy emitted by sources such as:

- radio stations;
- TV stations;
- amateur radio stations;
- shop alarm systems;
- mobile telephones and cordless telephones can interfere with electric wheelchairs and electric scooters;

The interference can cause the electromagnetic brake to release, unintentional movement of the device and movement in the wrong direction. It can also permanently damage the device's electronic control unit. In our everyday environment, there are a certain number of intense sources of EMI.

The intensity of the EM energy can be measured in Volts per metre (V/m). Your electric wheelchair is equipped with a protection against EMI which has been tested and certified as compliant with current international regulations, up to a certain intensity. This is called the "level of immunity". The greater the level of immunity, the better the device is protected. At this time, technology can provide a level of immunity of at least 20 V/m; this is sufficient protection against the most common sources of EMI. However, we consider that the following precautions can further minimise the risk.

- A) Do not use portable radio transmitters, CB radios and avoid switching on mobile telephones or other personal communications equipment when the electric wheelchair is switched on.
- B) Do not approach powerful radio and TV transmitters too closely.
- C) If an unintended movement or unintended release of the brake occurs, turn the key to the OFF position as soon as you can do so safely.
- D) Installing additional electrical equipment or accessories and modifying the motor of your electric wheelchair may make it more susceptible to EMI and hence make its use hazardous. Note that doing so also voids the warranty.
- E) In any case, report any unintentional movement of the vehicle to your retailer or to MORETTI SPA, note it down in this manual and note down the fact that there is a source of electromagnetic radiation in your vicinity, if any.

7. SYMBOLS

-  Product code
-  Unique device identification
-  Serial number
-  CE marking
-  Manufacturer
-  Batch Lot
-  Read the instruction manual
-  Medical Device
-  Disposal conditions
-  Warning
-  Production date
-  Disposal of the product according to directive EC/19/2012
-  Type B applied part

8. SAFETY

- Always use the safety belt and keep your feet resting on the footplates at all times.
- Do not use the electric wheelchair under the influence of alcohol.
- Do not use electronic radio transmitters such as walkie-talkies or mobile telephones.
- Make sure there are no obstacles nearby when you are driving the electric wheelchair.
- Do not make a sharp turn or brake suddenly when you are driving the electric wheelchair.
- Do not drive the electric wheelchair in traffic.

- Do not attempt to mount curbs higher than those indicated in the technical specifications.
- Do not put your hands or legs outside the electric wheelchair while driving.
- Do not drive the electric wheelchair on slippery roads or when it is snowing.
- Do not allow unsupervised children to play near the electric wheelchair when the batteries are charging.

Do not use the wheelchair without first reading and understanding this manual completely.

9. WARNINGS ON USE

9.1 General warnings

1. Do not use the electric wheelchair on public highways. Please be aware that it may be difficult for drivers to see you when you are sitting on the electric wheelchair. Always use pedestrian routes. Comply with the rules of pedestrian traffic. Wait for the way to be clear, then proceed with extreme caution.
2. Before getting into or out of your electric wheelchair, always check that it is switched off and that the battery charger is disconnected, to avoid injury to yourself or to others.
3. Always check that the drive wheels are in drive mode before driving. Do not switch off the electric wheelchair when it is still moving, as it would stop very suddenly.
4. Do not use this product or any other optional accessory available without first reading and understanding this manual completely. If you do not understand the warnings, the precautions or the instructions, contact the retailer or a healthcare operator before attempting to use the device, to avoid injury or damage.
5. In certain cases, such as certain medical conditions, the user of the electric wheelchair will need practice in its use in the presence of a qualified assistant, who may be a relative or a professional qualified to assist the user, in use of the electric wheelchair during various everyday activities.
6. Avoid lifting or moving the electric wheelchair by gripping one of its removable parts, including the armrests, seat or fairing. This could cause personal injury and/or damage the wheelchair.
7. Avoid using the electric wheelchair beyond its limits, as described in this manual.
8. Do not sit on the electric wheelchair when it is inside a moving vehicle.
9. Keep your hands away from the wheels (tyres) while driving the electric wheelchair. Please remember that loose clothing could get caught in the wheels.
10. Consult your doctor if you are taking drugs or if you have specific physical limitations. Certain drugs and certain limitations could affect the ability to use the electric wheelchair safely.
11. Always check that the wheel locking/unlocking lever is locked or unlocked.
12. Do not remove the roll-over protection wheels if they are present on the electric wheelchair.
13. Contact with tools could cause an electric shock; do not connect an electrical extension cord to the AC / DC converter or to the battery charger.
14. Although the electric wheelchair is designed for slopes with a gradient up to 8°/14%, try to avoid slopes with a gradient over 6° (10%) and take extreme care when turning on a slope.
15. Do not drive on a slope over the limit of the electric wheelchair.
16. Avoid reversing off steps, pavements or other obstacles. This could cause the electric wheelchair to fall or tip over.
17. Always reduce speed and maintain a stable centre of gravity on sharp bends. Do not use the maximum power on sharp bends.
18. Using the electric wheelchair in the rain, snow or foggy conditions and on icy or slippery surfaces could damage the electrical wiring.

19. Never sit in the electric wheelchair when it is being used with any type of lift or lifting device. The electric wheelchair is not designed for this use. The manufacturer is not liable for any damage or injury caused by improper use.

9.2 Changes

The electric wheelchairs in the ARDEA MOBILITY range by Moretti have been designed and manufactured to satisfy all your needs for practical, correct and safe use. Do not alter, add, remove or disable any part or function of the electric wheelchair under any circumstances. This could cause damage to the electric wheelchair and/or personal injury.

1. The control parameter must only be altered by authorised technicians, for safety reasons.
2. It is recommended to carry out a safety check before every use of the electric wheelchair, in order to ensure that it is functioning safely.

9.3 Before every use

1. Check the tyres are correctly inflated, if present.
2. Check all the electrical connections and make sure they are properly connected and not corroded.
3. Check all the cables and make sure they are connected properly.
4. Check the brakes.

9.4 Weight limits

1. Check the information on maximum capacity in the table of the technical specifications in this manual or on the label on the device. The power of the electric wheelchair is assessed according to the capacity.
2. Comply with the weight limits indicated for your electric wheelchair. Exceeding the maximum capacity invalidates the warranty. The manufacturer cannot be held liable for injury or damage to property caused by failure to comply with the weight limit.
3. Do not carry passengers on the electric wheelchair, as this could alter the centre of gravity and cause tipping over or falls.

9.5 Inflating the tyres

1. If the electric wheelchair has tyres, check the air pressure at least once a week.
2. Correct inflation pressure prolongs the duration of the tyres and guarantees better performance while driving.
3. Do not underinflate or overinflate the tyres. It is extremely important for the tyre pressure to remain between 30-25 psi (2-2.4bar) at all times.
4. Inflating the tyres with an unregulated air source can cause them to be overinflated and burst.

9.6 Temperature

1. Certain parts of the electric wheelchair are affected by changes in temperature. The controller only functions at a temperature ranging between -8 C° and 50 C°.
2. The batteries could freeze at extremely low temperatures and prevent your electric wheelchair from operating. At extremely high temperatures, the electric wheelchair might only function at lower speeds, due to a safety feature of the controller that prevents damage to the motor and other electrical components.
3. Exposing your electric wheelchair to water could cause damage to the electronic or mechanical components. Water can cause an electronic malfunction and/or premature corrosion of the electric components and the frame.

10. GENERAL DESCRIPTION
CS910BL "Aries Tilting" electric wheelchair

ENGLISH

Rear

Joystick

- ON/OFF switch
- "Tilting" button
- Increase speed
- Horn
- Reduce speed
- Faults LED


Control lever

Lights control

11. COMPONENTS

Your Moretti electric wheelchair is usually assembled the first time by the retailer. If this is not the case, start by checking that the cardboard box contains all the parts listed below. (fig.1)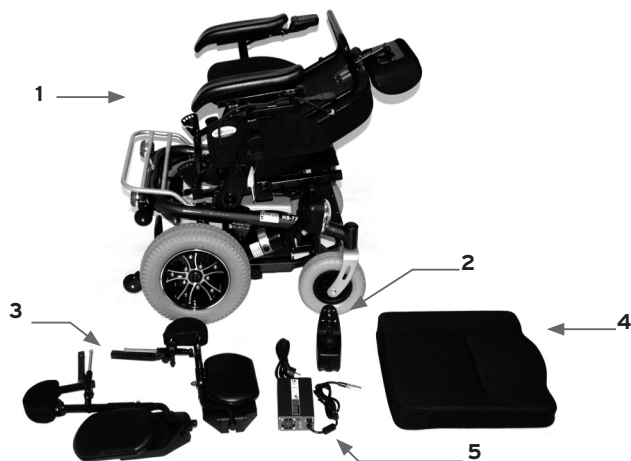

- 1) Main structure
- 2) Joystick
- 3) Footplates
- 4) Battery
- 5) Seat


12. ASSEMBLY INSTRUCTIONS

12.1 Assembly

1) Position the backrest in the erect position, unscrew the two side knobs and choose one of the four possible positions for the desired inclination.

2) Position the seat on the wheelchair, removing the Velcro protections and taking care to position the shaped profile in the front part.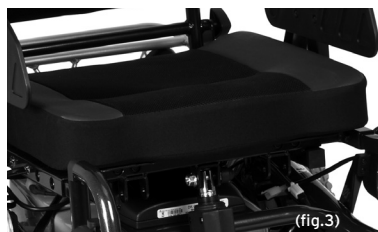

3) Insert the headrest in its housing and block it in place at the desired height by tightening the rear knob.

(fig.4)

4) Connect the joystick to the wheelchair through the cable on the right armrest.

(fig.4)

5) Attach the joystick to the support bar, using the four screws provided. Slide the connection cable as shown in the figure.

N.B. The position of the joystick can be changed from the right armrest to the left armrest. It is recommended to contact the retailer for positioning of the controller.

(fig.5)

5) Mount the two footplates, by inserting the support in the structure, attaching them using the screws present in the structure itself and taking care to match the right side and the left side correctly. The footrests must always be inside the wheelchair. Lower the footrests and adjust the footplates to the desired height

(fig.6)

7) Connect the battery through the connector positioned underneath the seat in the rear part.

12.2 Adjusting comfort

Before starting to use your **ARDEA MOBILITY** electric wheelchair by Moretti, carry out all the adjustments indicated below to obtain a comfortable driving and seating position suited to the user.

1. Adjusting the backrest inclination
2. Adjusting the armrest height
3. Tipping up the armrests
4. Adjusting the footplates
5. Adjusting the headrest
6. Functioning of the lights

1. Adjusting the backrest inclination

Your electric wheelchair has an inclining backrest. To adjust the backrest inclination, unscrew the two side knobs and place it in one of the four available positions, choosing the most appropriate one, and tighten the knobs again until the backrest is locked in place.

2. Adjusting the armrest height (Fig.9)

The height of the armrests on your electric wheelchair is adjustable. To adjust the height, unscrew the knob underneath the armrest and place it in one of the four available positions, choosing the most appropriate one, and tighten the knob again until the armrest is locked in place.

3. Tipping up the armrests (Fig. 10)

Your electric wheelchair has tip-up armrests. To tip up the armrests, pull the knob underneath the armrest adjustment knob outwards. To place the armrest in the position of use again, pull the knob, lower the armrest and release the knob, taking care to ensure that it returns totally into its housing.

4. Adjusting the footplates

Your wheelchair has height-adjustable footplates with a calf rest. Adjust the height of the footplates using the side unlocking lever and raising the footplate at the same time. Position the calf rest in one of the three available positions.


Adjust the height of the footrests by tightening the side safety screws.

(fig.11)

5. Adjusting the headrest

Your electric wheelchair has a height-adjustable headrest. To adjust, loosen the knob and position the headrest in the ideal position.

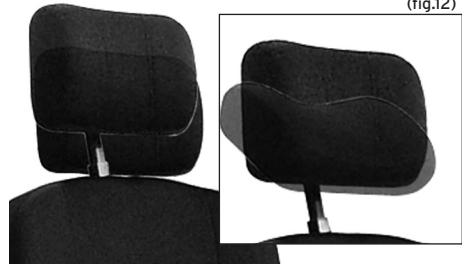(fig.12)

6. Functioning of the lights

The lights on the electric wheelchair are activated and controlled by the light command initially positioned on the left side. Press the ON/OFF button to switch on the light command and activate any type of light.

Use the light-blue button to switch on the front and rear lights and press the same button again to switch them off. Press one of the two green arrow buttons to switch on the direction indicators. Press the same button again to switch off the direction indicator. Press the triangle button to switch on the emergency lights (hazard lights). Press the same button again to switch them off.

(fig.13)

Use of the lights affects the battery and reduces the duration of the batteries, although to a limited extent.

12.3 Motor locking lever

Your Moretti wheelchair has an automatic motor brake on each motor that prevents involuntary movement of the wheelchair. You can disable this system if you need to move the wheelchair manually, using the motor disengage levers. When the levers are in position N (NEUTRAL), the motor is disengaged and the wheelchair may be moved freely. When the lever is in position D (DRIVE), the motor is engaged and the wheelchair can be used with the joystick. If the wheelchair is switched on when the lever is in position N, the alarm lights will indicate an anomaly and the electric wheelchair will be prevented from moving.

(fig.14)

ENGLISH

When the lever is in position N, the motor brake is disabled.

Avoid disabling the motor on sloping ground.

Do not use the wheelchair with the levers in position N (NEUTRAL), unless someone is accompanying you. When the motor disconnection levers are in position N (NEUTRAL), the electromechanical brake is also disabled.

13. USING THE ELECTRIC WHEELCHAIR

Before you get into your electric wheelchair, make sure you are on a flat surface with no obstacles. Although your wheelchair is designed for use on slopes, it is safer to practise using it on a flat surface. Several checks to make before every use are indicated below:

- Check functioning of the horn.
- Check correct functioning of the batteries and charge them if necessary
- Always check correct and stable assembly of all the parts of your electrical wheelchair
- Always check the state of wear and tear and functioning of the parts, in order to guarantee that the device is used in total safety; in particular, check the state of wear and tear, damage to and pressure of the front and rear tyres
- The motor unlocking lever must be in position D (DRIVE).
- It is recommended to select a low speed for the first few times you use it.

Follow these steps to use the electric wheelchair:

1. Sit on your wheelchair and press the ON/OFF button. The LED indicator lights of the battery charger must be lit up. The control and self-diagnosis light must not be flashing.
2. Rest your forearm firmly on the armrest and place your hand on the controller. After switching on the wheelchair, wait two seconds before activating the joystick. This is a safety feature to prevent sudden start-ups.
3. The joystick central lever controls all movements of the wheelchair. If you move the lever forwards, the wheelchair moves forwards. If you move the lever backwards, the wheelchair moves backwards. If you move the lever to the left, the wheelchair moves to the left. If you move the lever to the right, the wheelchair moves to the right. When you release

the joystick lever in the vertical or central position, the automatic brake is activated and the wheelchair stops automatically. The joystick lever also operates as an accelerator. The further you push the lever, the faster your wheelchair will move. The speed also depends on the maximum speed setting, which is set using the joystick.

4. Move the joystick central lever gently, applying stable and controlled pressure.
5. In an emergency, release the joystick central lever and the wheelchair will stop automatically.
6. Never drive at a speed beyond your ability to control the wheelchair.
7. Practise driving in a place where there are no obstacles. Start at the lowest speed and move forwards and backwards. Do this several times. When you feel secure, you can gradually increase the speed by pressing the button depicting a hare, or reduce it by pressing the button depicting a tortoise.
8. The number of green LEDs lit up on the speed control indicator identifies the speed level set.
9. If the battery charge indicator only has two LEDs lit up, the batteries must be charged as soon as possible.
10. If the wheelchair stops and does not function, identify the circuit breakers in the battery containers underneath the seat to reset the circuit. Use the ON/OFF button to switch off the wheelchair, press the reset button and try the electric wheelchair again (Fig. 15).
11. When you have finished travelling or before putting the wheelchair away, always switch off the controller by pressing the ON/OFF button.

(fig.15)

14. RECOMMENDATIONS FOR SAFE USE

- Before changing direction from forwards to backwards or backwards to forwards, bring the control lever into the central position and wait for the wheelchair to come to a complete stop.
- Only use the wheelchair where it would also be safe to walk.
- If you need to go up or down a slope, check that the gradient does not exceed 8% and that the surface has a good grip and is not slippery. Use a slow speed to reverse, go up a slope, down a slope, turn a corner or on uneven surfaces. NEVER use the leg rests to lift the wheelchair. The leg rests are removable and therefore must not be used for lifting.

DRIVING UP A SLOPE

International standards set the maximum safe gradient on which an electric wheelchair can be driven without any difficulty. This value is set as 6°/10%. The Moretti electric wheelchair can travel on slopes with a gradient up to 8°/14%. If you need to drive it on the maximum gradient, you may need to travel at a higher speed. Press the speed button to increase the speed. Be extremely careful when travelling on steep slopes.

DRIVING DOWN A SLOPE

Proceed slowly down steep descents. Press the speed button to reduce the speed. This allows better control of driving. However, your wheelchair will not accelerate on its own down a slope, due to the automatic braking system, which starts functioning if you attempt to drive too fast. In general, you should drive in a linear manner on slopes, without sudden changes in direction, and never stop the wheelchair on a slope that exceeds the safety limit.

GOING OVER OBSTACLES

Go over obstacles slowly and in a forwards direction. Do not attempt to go over obstacles from the side. This will allow you to go over obstacles up to 2 cm. Do not go over obstacles higher than 2 cm.

SELF-DIAGNOSIS

If the LED with a wrench symbol starts flashing immediately after the electric wheelchair is switched on, identify the problem by checking the potential problems on the list in the “Troubleshooting” section of this manual and take the necessary actions.

MOVEMENT

If you need to move the electric wheelchair, switch off the controller by pressing the ON/OFF button, position the motor unlocking levers on N (NEUTRAL), move the electric wheelchair into a secure position, then reposition the motor unlocking levers on D (DRIVE).

15. GENERAL PRECAUTIONS

- The electric wheelchair can only be used by one person at a time
- The maximum total weight carried must not exceed 160KG.
- Before getting into or out of your electric wheelchair, check that it is switched off and that the battery charger is disconnected.
- Always drive carefully and with your feet resting firmly on the footplates, taking care if there are other people present in the area where you are travelling.
- Use pedestrian routes whenever possible. Take extreme care when crossing roads.
- Although the electric wheelchair is designed for slopes with a gradient up to 8°/14%, try to avoid slopes with a gradient over 6° (10%) and take extreme care when turning on a slope
- Do not use the maximum power on sharp bends.
- Only drive the electric wheelchair after checking that all the parts are firmly attached and in the most comfortable driving position for you.
- Do not drive your electric wheelchair if the rear roll-over protection wheels are damaged or have been removed.
- Take extreme care and use a low speed when reversing, when driving down a slope and on uneven ground and when turning on both an ascent and a descent.
- Never place the motors of your electric wheelchair in position “N” (NEUTRAL) on a descent.
- Follow all the rules of the highway code when you are on public roads.
- It is not recommended to use the electric wheelchair on wet or damp ground, as this could be dangerous. Furthermore, the warranty does not cover damage caused by water.

15.1 TILTING function

The wheelchair has a seat adjustment function that is electrically controllable using the joystick, called the Tilting function. To adjust the inclination of the wheelchair seat, press the Tilting button until the number 1 switches on. Pull the joystick control lever backwards to increase the inclination of the wheelchair seat. Push the joystick lever forwards to decrease the inclination of the wheelchair seat. To return to the drive function, press the Tilting button again and hold down until the number 1 switches off.

(fig.16)

Avoid placing your hands in moving parts while the wheelchair is moving to activate the Tilting function .

16. CHARGING THE WHEELCHAIR

Your Moretti ARDEA Mobility electric wheelchair has two 12v/75Ah rechargeable batteries and a 25v/8Ampere battery charger. The batteries must be charged before using the wheelchair for the first time and should be charged after prolonged use. Ensure that the wheelchair is switched off and that the motor unlocking lever is in position D (DRIVE).

The wheelchair should be charged the first time for at least 14 hours. Always check the battery charger indicator on the control panel and check how many LEDs are lit up. It is recommended to charge the batteries when only the red LEDs are lit up.

1. Insert the battery charger connector into the charging socket underneath the controller.
2. Check that the current selector is on 220V (or the voltage available to you). Insert the battery charger plug into a wall socket and switch on the battery charger by pressing the switch to ON.

NOTE: Do not place the battery charger in a place near a potential fire risk.

3. The battery charger has control lights:
 - A light indicating that the battery charger is in operation.
 - A set of lights indicating the percentage charge of the wheelchair battery.
4. Wait for the charge percentage LEDs to indicate 100% before disconnecting the power supply.
5. The battery charger has a limiter that switches it off when the maximum charge is reached.
6. Disconnect the battery charger cable from the electric wheelchair and from the mains power.
7. If the switch-on light starts flashing on connection, the battery charger is functioning correctly.

If not, check that:

- The battery charger cables are properly connected.
- The fuse is working.
- The electric wheelchair is switched off.
- If all these points have been performed correctly, contact your retailer.

NOTE: The time needed for complete charging may vary depending on the initial charge when the charging procedure is started. Charging for longer than necessary does not damage the batteries, as they cannot be overcharged.

Keep these rules in mind:

- Charge the batteries completely at least once every twenty days or more often if you use the wheelchair regularly. Always charge the batteries after a trip of over 3 Km and as regularly as possible, in order to guarantee the best performance and functioning time of the batteries.
- Avoid leaving the batteries inactive for a period exceeding twenty days.
- The batteries and the electric wheelchair will function best after the electric wheelchair has been used and the batteries have been charged more than 10 times. The complete charging time of the batteries may be up to a maximum of ten hours.
- If the battery charger light does not turn green after this time, contact your usual retailer for a check on the battery charger. The battery charger may become slightly warm during use and it has an internal ventilation system. Do not obstruct the ventilation grids.

For safety, follow these guidelines:

1. Do not use the battery charger if the primary power cable is damaged.
2. Do not use extension cables for the primary power supply of the battery charger. There is a risk of fire or electric shock.

17. REPLACEMENT OF THE BATTERY

If you need to replace the batteries, contact your retailer to obtain the correct replacement batteries. Take care when replacing the batteries and do not discard depleted or damaged batteries in the environment.

Note: Dispose of the replaced batteries at authorised recycling centres and not in normal urban waste.

- To replace the batteries, disconnect the cable connecting the motor to the battery, positioned in the rear part.

- Pull the red lever positioned underneath the battery and turn it by 180°

- Remove the side blocks.

- Pull the battery unit using the lever on the rear part of the wheelchair (Fig. 20).

The battery unit is heavy. Take care during transport.

To replace the batteries inside, remove the upper part of the battery pack or replace both batteries inside. Carry out all the operations in reverse to mount the battery.

18. REPLACING THE WHEELS

WARNING! The wheels must be replaced by an authorised retailer or a qualified technician in a workshop

19. CARE AND MAINTENANCE

Looking after your electric wheelchair properly will ensure it is always in the best conditions. Several maintenance guidelines are provided below.

FRAME: If your wheelchair is dirty, use a damp or slightly soapy cloth to clean it. Do not use running water to wash or rinse the wheelchair. This could damage the electrical parts.

SEAT: Use a damp cloth to clean the upholstery. Please note that extensive use of the wheelchair outdoors reduces the lifetime of the upholstery, due to exposure to sunlight. Normal wear and tear from use is not covered by the warranty. The seat cover can also be removed for more thorough washing. It is recommended to use cold water and normal mild detergents for washing.

An authorised retailer will perform all necessary maintenance and repairs on your wheelchair, for both the electrical and electronic parts and for the wheels and batteries. It is also a good idea to store the wheelchair in a dry place after use.

20. TROUBLESHOOTING

Moretti wheelchairs are equipped with a device that constantly controls the electrical and electronic system, to ensure safe and comfortable driving. If a malfunction or error occurs, the LED with a wrench symbol flashes continuously for a specific number of times to indicate that there is a problem when the wheelchair is switched on. The number of flashes indicates the type of malfunction. Several suggestions on resolving the main problems you could have with your wheelchair are provided below. To start self-diagnosis, press the ON/OFF button of the electric wheelchair and count the number of flashes.

PROBLEM	SOLUTION
1 - General error	Position the motor lever on N and then reposition on D. Try switching on.
2 - Battery error	Check the battery connection cables. Try replacing the battery.
3 The left motor is faulty	Check the left motor and the electrical connections.
4 The right motor is faulty	Check the right motor and the electrical connections
5 The left motor is blocked	Check the left motor unlock lever Contact your usual retailer
6 The right motor is blocked	Check the right motor unlock lever Contact your usual retailer
7 - Joystick error	Check the joystick connections. Replace the joystick
8 - Joystick power supply error	Check the joystick connections. Replace the joystick
9 - Joystick connection error	Check the joystick connections. Replace the joystick
10 - Unknown error	Check all the cables and connections. Contact your usual retailer
11 Incompatible control unit	Control unit. The joystick is not recognised by the system. Check the connections Contact your usual retailer

21. DISPOSAL CONDITIONS

21.1 General conditions of disposal

Never dispose of the product as normal domestic waste. Dispose of the product at a sorted waste collection centre for recycling.

21.2 Instructions for correct disposal in accordance with European directive 2012/19/EU:

At the end of its working life, the product must not be disposed of together with normal urban waste. It must be delivered to municipal separated waste collection facilities, or to appropriate dealers that provide this service. Separated waste disposal helps to reduce possible negative effects on the environment and health deriving from improper disposal and allows for recycling the materials comprising the product, which translates into significant energy and resource savings. The product bears the barred bin symbol to underline the obligation of disposing of electro-medical equipment.

21.3 Disposal of exhausted batteries - (Directive 2006/66/EC):

This symbol on the product indicates that the batteries must not be considered as normal domestic waste. Ensuring that the batteries are discarded correctly contributes to preventing potentially negative consequences for health and the environment that would otherwise be caused by their inappropriate disposal. Recycling materials helps to conserve natural resources. Take depleted batteries to the collection points indicated for recycling. You can contact your local council, the local waste disposal service or the shop where you purchased the device for more detailed information on disposal of depleted batteries or the product.

22. TECHNICAL SPECIFICATIONS

22.1 Technical specifications

Seat	45 cm
Wheels	Tyres
Maximum speed	8 km/h
Steering radius	92 cm
Maximum safe slope	8°
Maximum estimated surmountable slope **	8°/14%
Max. step	20 mm
Autonomy*	48 km
Battery specifications	2x75Ah (12V)
Battery charger type	8A
Controller electronics	Dynamic Shark
Motor	2x420W- 4600 revs/min
Brake	Electromechanical
Drive	Two motors on rear wheels

*** The autonomy of the electric wheelchair may vary in relation to:**

- Weight of the patient;
- Slopes along the route;
- Battery wear.

**** The term rated slope is defined in standard EN 12184:2014.**

22.2 Dimensions

Dimensions and weight																
Measurements in cm																
CS910 BL	121	79	40	121	78/84	54	45/61	45	55	67	-	118	36	23	-	162

23. WARRANTY

Moretti products are guaranteed for 2 (two) years from the date of sale against material and manufacturing defects, subject the following limitations. The warranty is voided by improper use, abuse, modifications to the product and failure to follow the instructions. The intended use of the product is given in the user manual. Moretti is not liable for damage, injury or any other consequences resulting from installation or use which are not scrupulously conforming with the instructions given in the installation, assembly and user manual. Moretti does not guarantee its products against damage or defects in the following circumstances: natural disasters, unauthorised repair or maintenance, improper electric power supply (as applicable), use of parts or components not supplied by Moretti, failure to follow the guidelines and instructions for use, tampering, shipping damage (other than the original shipping by Moretti), or failure to perform maintenance as indicated in the manual. Components subject to wear and tear are not covered by this warranty if the damage is caused by normal use of the product.

23.1 Warranty on rechargeable batteries (if provided)

The original and replacement batteries are covered by a ninety-day warranty on performance and a six-month warranty on manufacturing defects or as required by law. If completely charged batteries are left unused for more than three consecutive months, the warranty is automatically invalidated. If completely discharged batteries are left unused for more than three consecutive days, the warranty is automatically invalidated.

24. REPAIRS

24.1 Repairs under warranty

If a Moretti product has material or manufacturing defects during the warranty period, Moretti will agree with the client whether the defect is covered by the warranty. Moretti, at its sole discretion, may replace or repair the article at a specified Moretti reseller or its own premises. The costs of labour incurred in repairing the product will be borne by Moretti if it determines that the repair is covered by the warranty. Repair and replacement do not renew the warranty period.

24.2 Repairing a product not covered by the warranty

A product not covered by warranty may be returned for repair only if authorised in advance by Moretti customer service. The costs of labour and shipping incurred by repairs not covered by the warranty are borne by the client or reseller in their entirety. Repairs on products not covered by the warranty are themselves guaranteed for 6 (six) months from the day of reception of the repaired product.

24.3 Non-defective products

The client will be notified if Moretti concludes that the product is not defective after having received and examined it. The product will be returned to the client at his expense.

25. REPLACEMENT PARTS

Moretti original replacement parts are guaranteed for 6 (six) months from the day of delivery.

26. NON-LIABILITY CLAUSE

Unless otherwise expressly specified in this warranty and within the limits of the law, Moretti makes no declaration, guarantee or condition, express or implicit, including any future declaration, guarantee or condition of sale, suitability for a given purpose, non violation and non interference. Moretti does not guarantee that the use of its product will be uninterrupted and problem-free. The duration of any implicit guarantee under the law is limited to the warranty period, within the limits of the law. Certain states and countries do not permit limitations on the duration of an implicit guarantee or the exclusion of limitation of accidental or indirect damages in relation to consumer products. In said states and countries, certain exclusions and limitations of this warranty may not apply to the user. This warranty is subject to modification without notice.

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