IT

2

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- The devices in question are marketed in NON-STERILE packaging.
- The devices in question are to be considered as belonging to class I in accordance with the provisions of Annex VIII of the aforementioned regulation.
- **Moretti S.p.A.** shall maintain and make available to the Competent Authorities, for at least 10 years from the date of manufacture of the last batch, the technical documentation proving compliance with Regulation 2017/745.

5. Warnings

- Refer to this manual for correct use of the device.
- Be careful around children.
- The useful life of the device is determined by the wear and tear of non-repairable and/or non-replaceable parts, and in any case shall not exceed ten (10) years.
- Check that the product is delivered intact and keep it away from heat sources.
- Never use the bag in any way other than that indicated. Moretti Spa is not responsible for improper use.
- Use only on intact skin. Do not use on skin lesions or lacerations. In any case, always use a protective cloth between the skin and the bag.
- Use only with ice or ice water.
- The user and/or patient must report any serious incident related to the device to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

6. Symbols used

 REF	Product code	 Manufacturer
 UDI	Unique Device Identifier (UDI)	 LOT
 CE	CE marking	 Consult instructions for use
 MD	Medical device	 Disposal conditions

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7. General description

Ice bag made of resistant fabric with screw cap and rubber seal. PVC interior that guarantees long-lasting ice retention.

8. Instructions for use

Always check the condition of the parts to ensure that the device can be used in complete safety for people and property.
If possible, wash the inside of the bag. Fill the bag with ice. Close the bag tightly with the cap provided, taking care to remove any excess air from inside. Check that it is closed properly by checking for any leaks. Place the bag on the chosen part of the body, using a soft cloth between the skin and the bag.

9. Maintenance and storage

Kyara by Moretti devices are carefully checked and CE marked. For the safety of patients and doctors, it is recommended that their suitability for use be checked at least every 2 years by the manufacturer or an authorized laboratory. In case of repairs, use only original spare parts and accessories.
For proper storage, keep the bag dry, away from light and heat sources, sprinkling the inside with a small amount of talcum powder.

10. Cleaning and disinfection

To clean both the inside and outside of the bag, we recommend using warm water and a neutral detergent. Allow to dry naturally. Do not expose to sunlight or artificial heat sources. If it is necessary to disinfect the product, use a common disinfectant detergent.

N.B. Void using acidic, alkaline, or solvent products such as acetone or thinner.

11. General disposal conditions

When disposing of the device, never use normal municipal solid waste collection systems. Instead, it is recommended that the device be disposed of through municipal recycling centers for the recycling of the materials used

Ice Bag INSTRUCTIONS FOR USE

12. Technical specifications

Material: PVC interior / Nylon exterior Size: Ø 28 cm

13. Warranty

All Moretti products are guaranteed against defects in materials or workmanship for a period of 2 (two) years from the date of sale of the product, except for any exclusions and limitations specified below. This warranty is not valid in the event of misuse, abuse, or modification of the product, or failure to follow the instructions for use. The correct intended use of the product is indicated in the user manual. Moretti is not liable for any resulting damage, personal injury, or anything else caused by or related to the installation and/or use of the equipment that does not strictly comply with the instructions in the installation, assembly, and use manuals.

Moretti does not guarantee Moretti products against damage or defects in the following conditions: natural disasters, unauthorized maintenance or repair operations, damage resulting from power supply problems (where applicable), use of parts or components not supplied by Moretti, failure to comply with the guidelines and instructions for use, unauthorized modifications, shipping damage (other than the original shipment from Moretti), or failure to perform maintenance as indicated in the manual. Wearable components are not covered by this warranty if the damage is due to normal use of the product.

14. Disclaimers

Except as expressly stated in this warranty and to the extent permitted by law, Moretti makes no other representations, warranties, or conditions, express or implied, including any representations, warranties, or conditions of merchantability, fitness for a particular purpose, non-infringement, and non-interference. Moretti does not warrant that the use of the Moretti product will be uninterrupted or error-free. The duration of any implied warranties that may be imposed by law is limited to the warranty period, within the limits of the law. Some states or countries do not allow limitations on the duration of an implied warranty or the exclusion or limitation of incidental or consequential damages in connection with consumer products. In such states or countries, certain exclusions or limitations of this warranty may not apply to you. This warranty is subject to change without notice.

CE Class I Medical Device
Regulation (EU) 2017/745 of the European Parliament and of the Council
of 5 April 2017 on medical devices

1. Product Code

ST196 Ice Bag, Ø 28 cm

2. Introduction

Thank you for choosing the **KYARA by Moretti** ice pack. Always consult your doctor before use to ensure correct use of the device. This manual provides useful information on use and safety; please read it carefully before use.

If you have any questions, contact your retailer for assistance. If you have any questions or if the product is damaged, contact your retailer for assistance.

3. Intended use

The **KYARA** ice pack is intended for cooling parts of the human body. Any use other than that indicated in the manual is prohibited. **Moretti S.p.A.** declines all responsibility for damage resulting from improper or non-compliant use.

The manufacturer reserves the right to modify the device and the manual without notice in order to improve its characteristics.

4. EU declaration of conformity

Moretti S.p.A. declares under its sole responsibility that the products manufactured and marketed by **Moretti S.p.A.** itself and belonging to the HEALTHCARE PRODUCTS family comply with the applicable provisions of Regulation 2017/745 on MEDICAL DEVICES of April 5, 2017.

The complete product codes, the Manufacturer Registration Number (SRN), the basic UDI-DI code, and any references to standards used are listed in the EU Declaration of Conformity that **Moretti S.p.A.** issues and makes available through its channels.

To this end, **MORETTI SpA** guarantees and declares under its sole responsibility that:
- The devices in question meet the general safety and performance requirements as required by Annex I of Regulation 2017/745 as prescribed by the annex to the aforementioned regulation.

- The devices in question ARE NOT MEASURING INSTRUMENTS.
- The devices in question ARE NOT INTENDED FOR CLINICAL INVESTIGATION.

WARRANTY CERTIFICATE

Product _____

Purchased on _____

Retailer _____

Address _____ City/Town _____

Sold to _____

Address _____ City/Town _____

Via Bruxelles, 3 - Melegnano 20022 Cavriaglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN P.R.C.