ITALIANO

MA CP810xx 02 A_ITA_09 2020

CARROZZINA COMFY

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INDICE

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www.morettispa.com email: info@morettispa.com

MADE IN P.R.C.

This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

NOTE

ENGLISH

MA CP810xx 02 A_ENG_09 2020

COMFY WHEELCHAIR

INSTRUCTION MANUAL

INDEX

1. CODES.....	PAG.3
2. INTRODUCTION.....	PAG.3
3. INTENDED USE.....	PAG.3
4. DECLARATION OF CONFORMITY	PAG.3
5. GENERAL WARNINGS	PAG.4
6. SYMBOLS	PAG.4
7. GENERAL DESCRIPTION.....	PAG.4
7.1 Parts view	pag. 4
8. ASSEMBLING/DISASSEMBLING	PAG.5
8.1 Opening.....	pag. 5
8.2 Footrest assembling	pag. 5
8.3 Headrest assembling	pag. 6
9. BEFORE USE.....	PAG.6
10. WARNING FOR A CORRECT USE.....	PAG.6
11. HOW TO USE.....	PAG.6
11.1 Get in the wheelchair.....	pag. 6
11.2 Get out of the wheelchair	pag. 6
11.3 Stairs and height difference (how to face them).....	pag. 7
11.4 Use the armrest.....	pag. 7
11.5 Use the footrest.....	pag. 7
11.6 Use the reclining backrest	pag. 8
11.7 Use the 60 cm rear wheels.....	pag. 8
11.8 Use the headrest	pag. 8
11.9 Use the rear handles.....	pag. 8
11.10 Use the anti-tip castors	pag. 8
11.11 Use the brakes	pag. 9
11.12 Transport on vehicles.....	pag. 9
12. MAINTENANCE.....	PAG.10
13. CLEANING AND DISINFECTION	PAG.10
13.1 Cover and padding.....	pag. 10
13.2 Frame and metallic parts	pag. 10
13.3 Brakes and wheels	pag. 10
13.4 Disinfection.....	pag. 10
14. CONDITIONS OF DISPOSAL	PAG.10
14.1 General conditions of disposal	pag. 10
15. SPARE PARTS/ACCESSORIES.....	PAG.10
16. TECHNICAL FEATURES.....	PAG.11
16.1 Dimensions	pag. 11
16.2 Technical Specifics	pag. 11
17. WARRANTY	PAG.12
17.1 Repairing.....	pag. 12
17.2 Spare parts	pag. 12
17.3 Exempt clauses	pag. 12

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

CP810-40	Reclining wheelchair COMFY seat 40 cm
CP810-43	Reclining wheelchair COMFY seat 43 cm
CP810-46	Reclining wheelchair COMFY seat 46 cm

2. INTRODUCTION

Thank you for purchasing a reclining wheelchair **COMFY ARDEA ONE** by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly.

Note: check all parts for shipping damages and test before using. In case of damage, do not use the device. Contact your dealer for further instructions.

3. INTENDED USE

This wheelchair is intended for people with mobility problems and can be used for self-propelling or with the aid of an operator. In all cases however it requires the intervention of an operator for the various adjustments you require.

WARNING!

- Do not use the product for a purpose not indicated in this manual.
- Moretti S.p.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer reserves the right to change the information contained in this document without previous notice.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of MANUAL WHEELCHAIRS COMFY/RELIEF complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

- For correct use of the device, carefully refer to the following manual
- For correct use of the device always consult your doctor or therapist
- Keep the packaged product away from any source of heat as the packaging it is made of cardboard
- The useful life of the device is determined by the wear of non-repairable and / or replaceable parts
- Always pay close attention to the presence of moving parts which could cause limb entrapment and personal injury
- Always pay attention to the presence of children
- Do not lift the wheelchair by removable parts which could accidentally come off during lifting
- The user and / or the patient must report any serious accident occurring in relation to the device to the manufacturer and to the competent authority of the State in which the user and / or the patient is established.


6. SYMBOLS

REF	Product code
UDI	Unique Device Identification
CE	CE mark
	Manufacturer
LOT	Batch Lot
	Read the instruction manual
MD	Medical Device
	Conditions of disposal
SN	Serial Number

7. GENERAL DESCRIPTION

7.1 Parts view

Depending on the COMFY model wheelchair you choose, the wheelchair will be composed by the following parts

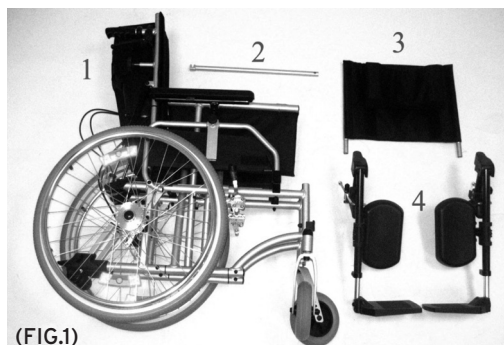(FIG.1)

1. Frame
2. Metal pole for backrest
3. Headrest
4. Elevating footrest

8. ASSEMBLING/DISASSEMBLING

8.1 Opening

(FIG.2)

Note: Refer to the safety warnings in this instruction manual

- Get the handles and pull outwards, then press the seat until the wheelchair is completely open (FIG.2)

(FIG.3)

- Insert the metal pole inside its supports, then lock the knobs. The frame is duly fixed (FIG.3)

8.2 Footrest assembling

(FIG.4)

- Insert the hinges on the frame holes, then rotate inwards until the lever is locked (FIG.4)

8.3 Headrest assembling

(FIG.5)

- Insert the headrest in the backrest support, then lock the knobs once the desired height is found (FIG.5)

9. BEFORE USE

- Check the mechanical parts usage, in order to guarantee a safe use of the device for people and things
- Assure all the parts are correctly assembled. Checking carefully the correct assembling of the rear wheels
- Assure the brake handles are well regulated

10. WARNING FOR A CORRECT USE

- Do not use the wheelchair as seat for vehicle
- Assure the brake handles is duly adjusted and correctly works
- Avoid entrapment of finger into wheels or brakes during the wheelchair use
- The wheelchair temperature increase if exposed to the sun for long period

11. HOW TO USE

WARNING!

- Getting in and out of the wheelchair may be performed independently, be sure you have an adequate mobility and upper body strength
- Do not stand on removable parts

DO NOT STAND ON THE FOOTREST WHEN GETTING IN OR OUT OF THE WHEELCHAIR, OVERTURNING RISK!!!

11.1 Get in the wheelchair

1. Position the wheelchair as close as possible to you
2. Engage wheel locks. We suggest to remove the armrest to ease the patient getting in the wheelchair
3. Lean on the frame with arms and slowly go down towards the seat

11.2 Get out of the wheelchair

1. Position the wheelchair as close as possible alongside the seat to which you are transferring
2. Engage wheel locks. We suggest to remove the armrest to ease the patient getting out of the wheelchair
3. Shift body weight into seat with transfer from the chair to seat you want transfer

11.3 Stairs and height difference (how to face them)

**WARNING!**

The following method must be used only from experienced patients. If you are not experienced, let you help from an assistant.

GET ON: Position the wheelchair at the curb to be crossed. Balance the wheelchair on the rear wheels, in order to lift the front wheels, enough to lean them on the obstacle. Bend your trunk forward and vigorously push the handrail. Continue pushing until the rear wheels are climbing over the obstacle.

GET OFF: To get off the stair follow the reverse procedure, positioning the rear side of the wheelchair at the curb, keep the handrails vigorously and get off the obstacle slowly; then balance the wheelchair on the rear wheels in order to complete the descent.

**WARNING!**

We recommend to use this method only in presence of obstacle lower than 10cm


11.4 Use the armrest

The armrest can be adjusted for height and can be tipped over (Fig.6)

Height adjustment- pull the knob **A**, adjust the height, release the knob to lock the armrest

Tip over- press the spring button **B**. Tip over the armrest. Then bring it back to its position until the button is locked

(FIG.6)

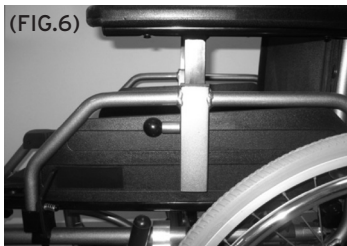11.5 Use the footrest

The footrest can be adjusted in length and inclination (Fig.7)

Length adjustment- loosen the screw **A** using a 5mm Allen wrench. Adjust the footrest length then tighten the screw


Inclination adjustment- push the spring lever **B**. Adjust the inclination, then release the lever to lock into the desired position

(FIG.7)

11.6 Use the reclining backrest


Press the release lever (Fig.8), push slightly the backrest downwards to adjust the inclination of the backrest. Gently push upwards to bring the angle of the backrest back to right-angled

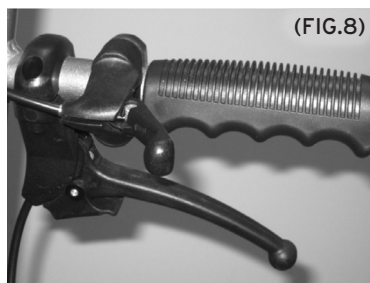**11.7 Use the 60 cm rear wheels**

Use rear wheels for self-propelling. Grasp the handrail and wheel through the desired direction.

11.8 Use the headrest

To adjust the height of the headrest easily detach the padded cushion from Velcro and position to the desired height (Fig.9)

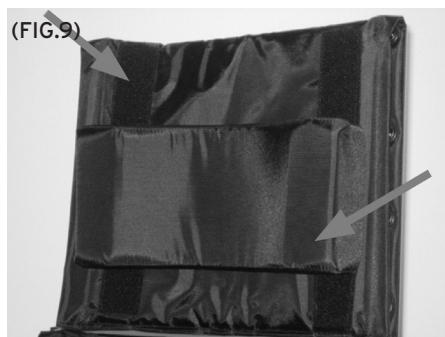**11.9 Use the rear handles**

The rear push handles on the backrest are used by the operator to push the wheelchair or to adjust the backrest angle.

11.10 Use the anti-tip castors

(FIG.10) The anti-tip castors are situated on the wheelchair backside.

To adjust the height pull the ring shown in Figure 10 and release once you have the correct height

11.11 Use the brakes

REAR BRAKES ACTIVATED BY THE OPERATOR: To lock the brakes, press the corresponding ones (right and left) on the rear push handle. Release the levers of the brakes to unlock the wheelchair again. (FIG.11)

Note: the brakes are designed to stay locked automatically once pressed. to activate the brake function, once pressed the brake lever, push the locking lever placed under the teeth graduated sector. To unlock the brake again exert a further pressure on the lever so that the locking element comes out of the toothed sector.

PATIENT OPERATED REAR BRAKES: On the frame at the front of the big rear wheels there is a mechanical brake operated by the patient. To block the wheels push the operating lever forward. To release the brake pull the lever towards the side rear. (FIG.12)

11.12 Transport on vehicles

The wheelchair has been designed to be easily carried. Once the armrest and the footrest are removed, can be closed and easily positioned in the car boot, for example. If this can't be possible, position the wheelchair in front of the car rear seat so that it is stable and cannot slip or slide. Lock the wheelchair with the seatbelt.

12. MAINTENANCE

The **ARDEA ONE** devices by Moretti are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check:

- When using the wheelchair for the first time and during the first month, check the fixing screws of the frame and if necessary tighten carefully
- Every 3 months make a check for the condition of all components and the correct ones locking of the screws and nuts of all mechanical parts, especially those concerning the wheels and the braking system

In case of repair, use only original spare parts and accessories.

Before using follow the "BEFORE USE" chapter

13. CLEANING AND DISINFECTION

13.1 Cover and padding

The cover and padding are in Nylon. Clean with warm water and mild soap.

Note Do not use solvent and abrasive product

13.2 Frame and metallic parts

Clean with warm water and mild soap. Dry with a soft cloth in order not to scratch the painting.

13.3 Brakes and wheels

Clean the wheels with warm water and mild soap. Pay attention in drying the wheels, it's dangerous using the wheelchair with wet wheels.

13.4 Disinfection

If you need to disinfect the device use a common disinfectant cleaner

Note: Never use acids, alkalis or solvents such as acetone

14. CONDITIONS OF DISPOSAL

14.1 General conditions of disposal

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the wheelchair in the appropriate disposal areas for recycling.

15. SPARE PARTS/ACCESSORIES

For spare parts and accessories refer to Moretti main catalogue

16. TECHNICAL FEATURES

16.1 Dimensions

Dimensions and weight

Measurements in cm															
CP810-40								40		62					
CP810-43	116	92	46	138	70	50	50/64	43	89	65	33	20	60	20	5
CP810-46								46		68					100

 Portata
MAX

ENGLISH

16.2 Technical Specifics

- **Frame** Powder coated aluminum
- **Coat** Nylon
- **Armrest and legrest** PU
- **Headrest** Foam
- **Front wheels** PU 20cm diameter
- **Rear wheels** PU 60cm diameter
- **Backrest inclination angle** From 0° to 40°
- **Weight** 20 kg
- **Max weight** 100kg
- **Resistance to ignition** The materials used for cover and padding have been treated with methods specified in Directive EN1021-1/2
- **Reference regulation** EN12183
- **Environmental temperature** -20°C/+ 40°C
- **Storage** Store in a cool place, away from heat sources, such as sunlight (suggested temperature between +10°C and +40°C)

17. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and for the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual. Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to mistaken installation or improper use. Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, alteration not authorized, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

17.1 Repairing

Warranty repair: If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

Repair not covered by warrant: A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

Non defective device: The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be paid by the customer.

17.2 Spare parts

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

17.3 Exempt clauses

Moretti does not offer any other declarations, warranty or conditions, explicit or implicit, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

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MORETTI S.p.A.