ITALIANO

MA CS810 02 B_ITA_04 202

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ENGLISH

MA CS810 02 B_ENG_04 2021

"TAURUS" ADJUSTABLE ELECTRIC WHEELCHAIR

INSTRUCTION MANUAL

INDEX

1. CODE	PAG.3
2. INTRODUCTION	PAG.3
3. INTENDED USE	PAG.3
4. EU DECLARATION OF CONFORMITY	PAG.3
6. GENERAL INFORMATION ON ELECTROMAGNETIC DISTURBANCE	PAG.4
7. SYMBOLS	PAG.5
8. SAFETY	PAG.5
9. WARNINGS ON USE	PAG.6
9.1 General warnings.....	pag. 6
9.2 Changes.....	pag. 7
9.3 Before every use.....	pag. 7
9.4 Weight limits.....	pag. 7
9.5 Inflating the tyres.....	pag. 7
9.6 Temperature.....	pag. 7
10. GENERAL DESCRIPTION	PAG.8
10.1 Front view with description of the parts.....	pag. 8
10.2 Rear view with description of the parts.....	pag. 8
10.3 Main view of the control joystick.....	pag. 8
10.4 Main view of the lighted controller (OPTIONAL).....	pag. 9
11. COMPONENTS	PAG.9
12. ASSEMBLY INSTRUCTIONS	PAG.10
12.1 Assembly.....	pag. 10
12.2 Adjusting comfort.....	pag. 10
12.3 Use of the controls.....	pag. 13
12.4 Disassembly of the wheelchair.....	pag. 14
13. USING THE ELECTRIC WHEELCHAIR	PAG.15
14. RECOMMENDATIONS FOR SAFE USE	PAG.16
15. GENERAL PRECAUTIONS	PAG.17
16. CHARGING THE WHEELCHAIR	PAG.17
17. CARE AND MAINTENANCE	PAG.19
17.1 Annual maintenance.....	pag. 19
17.2 Ordinary maintenance.....	pag. 19
17.3 Cleaning and disinfection.....	pag. 19
18. TROUBLESHOOTING	PAG.19
18.1 Other problems.....	pag. 20
19. DISPOSAL CONDITIONS	PAG.21
19.1 General conditions of disposal.....	pag. 21
19.2 Instructions for correct disposal in accordance with European directive 2012/19/EU:.....	pag. 21
19.3 Disposal of exhausted batteries - (Directive 2006/66/EC):.....	pag. 21
20. TECHNICAL SPECIFICATIONS	PAG.22
20.1 Technical specifications.....	pag. 22
20.2 Dimensions.....	pag. 22
21. WARRANTY	PAG.22
22. REPAIRS	PAG.23
22.1 Repairs under warranty.....	pag. 23
22.2 Repairing a product not covered by the warranty.....	pag. 23
22.3 Non-defective products.....	pag. 23
23. REPLACEMENT PARTS	PAG.23
24. NON-LIABILITY CLAUSE	PAG.23

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL of 5 April 2017 on medical devices

1. CODE

CS810 "Taurus" adjustable electric wheelchair

2. INTRODUCTION

Thank you for choosing an **ARDEA MOBILITY** electric wheelchair by Moretti. Moretti electric wheelchairs have been designed and manufactured to satisfy all your needs for practical, correct and safe use. This manual contains useful suggestions for using your device properly and safely. Please read this manual in full before you use the electric wheelchair. Should you have any queries, please contact your retailer for suitable advice and assistance.

Note: Check to ensure that no parts of the wheelchair have been damaged during shipment. Do not use the product if it is damaged and contact the retailer for further instructions.

3. INTENDED USE

The electric wheelchair is intended for people with mobility impairments.

- Do not use the product for any purpose other than that specified in this manual
- The electric wheelchair must always be used by people of sound body and mind and not under the influence of drugs or alcohol.
- Moretti S.p.A. declines any and all liability for damages resulting from improper use of the device or any use other than that specified in this manual
- The manufacturer reserves the right to modify the device and its relevant manual without prior notice, in order to improve the product's features

4. EU DECLARATION OF CONFORMITY

MORETTI SpA declares, under its own exclusive responsibility, that the products manufactured and sold by MORETTI SpA in the ELECTRIC WHEELCHAIR product family conform with the applicable provisions of Regulation (EU) 2017/745 of 5 April 2017 on MEDICAL DEVICES. To this end, MORETTI SpA guarantees and declares as follows, under its own exclusive responsibility:

1. The devices in question satisfy the general safety and performance requirements set out in Annex I to Regulation (EU) 2017/745, as prescribed by Annex IV to said regulation.
2. The devices in question ARE NOT MEASUREMENT TOOLS.
3. The devices in question ARE NOT INTENDED FOR CLINICAL INVESTIGATIONS.
4. The devices in question are sold in NON STERILE packing.
5. The devices in question should be considered as belonging to class I, in accordance with the rules set out in Annex VIII to said regulation.
6. MORETTI SpA maintains and provides to the competent authorities, for at least ten years from the date of manufacture of the last production lot, the technical documentation proving conformity with Regulation (EU) 2017/745.

Note: The complete product codes, the manufacturer's single registration number (SRN), the basic UDI-DI code and any references to standards used are indicated in the EU Declaration of Conformity that MORETTI SPA issues and provides through its own channels.

ENGLISH

5. GENERAL PRECAUTIONS

WARNING!

Take extreme care in the presence of moving parts that could trap limbs and cause injury.

- Please consult this manual carefully for correct use of the device.
- Always consult your doctor or therapist for correct use of the device.
- Keep the packaged product away from sources of heat, as the packaging is cardboard.
- The lifetime of the device depends on wear and tear of non-repairable and/or non-replaceable parts.
- Always take care when children are present.
- Maximum user weight: 136 kg.
- Classification: class B (EN 12184).
- The product is not suitable for the visually impaired.
- The device must not be used by children under 12 years of age.
- The electric wheelchair is not designed for use as a seat in a motor vehicle.
- The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

6. GENERAL INFORMATION ON ELECTROMAGNETIC DISTURBANCE

The rapid growth of electronics, especially in communications, has saturated the environment with electromagnetic radiation (EM) emitted by television, radio and communication signals. These EM waves are invisible and are stronger the closer you are to their source. All electrical conductors act as antennas for EM signals and, to varying extents, all electric wheelchairs and scooters may be affected by electromagnetic interference (EMI). EMI may cause unintentional movements and/or irregular control of the vehicle.

The electromagnetic energy emitted by sources such as:

- radio stations;
- TV stations;
- amateur radio stations;
- shop alarm systems;
- mobile telephones and cordless telephones can interfere with electric wheelchairs and electric scooters;

The interference can cause the electromagnetic brake to release, unintentional movement of the device and movement in the wrong direction. It can also permanently damage the device's electronic control unit. In our everyday environment, there are a certain number of intense sources of EMI.

The intensity of the EM energy can be measured in Volts per metre (V/m). Your electric wheelchair is equipped with a protection against EMI which has been tested and certified as compliant with current international regulations, up to a certain intensity. This is called the "level of immunity". The greater the level of immunity, the better the device is protected. At this time, technology can provide a level of immunity of at least 20 V/m; this is sufficient protection against the most common sources of EMI. However, we consider that the following precautions can further minimise the risk.

- A) Do not use portable radio transmitters, CB radios and avoid switching on mobile telephones or other personal communications equipment when the electric wheelchair is switched on.
- B) Do not approach powerful radio and TV transmitters too closely.
- C) If an unintended movement or unintended release of the brake occurs, turn the key to the OFF position as soon as you can do so safely.
- D) Installing additional electrical equipment or accessories and modifying the motor of your electric wheelchair may make it more susceptible to EMI and hence make its use hazardous. Note that doing so also voids the warranty.
- E) In any case, report any unintentional movement of the vehicle to your retailer or to MORETTI SPA, note it down in this manual and note down the fact that there is a source of electromagnetic radiation in your vicinity, if any.

7. SYMBOLS

	Product code
	Unique device identification
	Serial number
	CE marking
	Manufacturer
	Batch Lot
	Read the instruction manual
	Medical Device
	Disposal conditions
	Warning
	Production date
	Disposal of the product according to directive EC/19/2012
	Type B applied part

8. SAFETY

- Always use the safety belt and keep your feet resting on the footplates at all times.
- Do not use the electric wheelchair under the influence of alcohol.
- Do not use electronic radio transmitters such as walkie-talkies or mobile telephones.
- Make sure there are no obstacles nearby when you are driving the electric wheelchair.
- Do not make a sharp turn or brake suddenly when you are driving the electric wheelchair.

- Do not drive the electric wheelchair in traffic.
- Do not attempt to mount curbs higher than those indicated in the technical specifications.
- Do not put your hands or legs outside the electric wheelchair while driving.
- Do not drive the electric wheelchair on slippery roads or when it is snowing.
- Do not allow unsupervised children to play near the electric wheelchair when the batteries are charging.

Do not use the wheelchair without first reading and understanding this manual completely.

9. WARNINGS ON USE

9.1 General warnings

1. Do not use the electric wheelchair on public highways. Please be aware that it may be difficult for drivers to see you when you are sitting on the electric wheelchair. Always use pedestrian routes. Comply with the rules of pedestrian traffic. Wait for the way to be clear, then proceed with extreme caution.
2. Before getting into or out of your electric wheelchair, always check that it is switched off and that the battery charger is disconnected, to avoid injury to yourself or to others.
3. Always check that the drive wheels are in drive mode before driving. Do not switch off the electric wheelchair when it is still moving, as it would stop very suddenly.
4. Do not use this product or any other optional accessory available without first reading and understanding this manual completely. If you do not understand the warnings, the precautions or the instructions, contact the retailer or a healthcare operator before attempting to use the device, to avoid injury or damage.
5. In certain cases, such as certain medical conditions, the user of the electric wheelchair will need practice in its use in the presence of a qualified assistant, who may be a relative or a professional qualified to assist the user, in use of the electric wheelchair during various everyday activities.
6. Avoid lifting or moving the electric wheelchair by gripping one of its removable parts, including the armrests, seat or fairing. This could cause personal injury and/or damage the wheelchair.
7. Avoid using the electric wheelchair beyond its limits, as described in this manual.
8. Do not sit on the electric wheelchair when it is inside a moving vehicle.
9. Keep your hands away from the wheels (tyres) while driving the electric wheelchair. Please remember that loose clothing could get caught in the wheels.
10. Consult your doctor if you are taking drugs or if you have specific physical limitations. Certain drugs and certain limitations could affect the ability to use the electric wheelchair safely.
11. Always check that the wheel locking/unlocking lever is locked or unlocked.
12. Do not remove the roll-over protection wheels if they are present on the electric wheelchair.
13. Contact with tools could cause an electric shock; do not connect an electrical extension cord to the AC / DC converter or to the battery charger.
14. Although the electric wheelchair is designed for slopes with a gradient up to 8°/14%, try to avoid slopes with a gradient over 6° (10%) and take extreme care when turning on a slope.
15. Do not drive on a slope over the limit of the electric wheelchair.
16. Avoid reversing off steps, pavements or other obstacles. This could cause the electric wheelchair to fall or tip over.
17. Always reduce speed and maintain a stable centre of gravity on sharp bends. Do not use the maximum power on sharp bends.
18. Using the electric wheelchair in the rain, snow or foggy conditions and on icy or slippery surfaces could damage the electrical wiring.

19. Never sit in the electric wheelchair when it is being used with any type of lift or lifting device. The electric wheelchair is not designed for this use. The manufacturer is not liable for any damage or injury caused by improper use.

9.2 Changes

The electric wheelchairs in the ARDEA MOBILITY range by Moretti have been designed and manufactured to satisfy all your needs for practical, correct and safe use. Do not alter, add, remove or disable any part or function of the electric wheelchair under any circumstances. This could cause damage to the electric wheelchair and/or personal injury.

1. The control parameter must only be altered by authorised technicians, for safety reasons.
2. It is recommended to carry out a safety check before every use of the electric wheelchair, in order to ensure that it is functioning safely.

9.3 Before every use

1. Check the tyres are correctly inflated, if present.
2. Check all the electrical connections and make sure they are properly connected and not corroded.
3. Check all the cables and make sure they are connected properly.
4. Check the brakes.

9.4 Weight limits

1. Check the information on maximum capacity in the table of the technical specifications in this manual or on the label on the device. The power of the electric wheelchair is assessed according to the capacity.
2. Comply with the weight limits indicated for your electric wheelchair. Exceeding the maximum capacity invalidates the warranty. The manufacturer cannot be held liable for injury or damage to property caused by failure to comply with the weight limit.
3. Do not carry passengers on the electric wheelchair, as this could alter the centre of gravity and cause tipping over or falls.

9.5 Inflating the tyres

1. If the electric wheelchair has tyres, check the air pressure at least once a week.
2. Correct inflation pressure prolongs the duration of the tyres and guarantees better performance while driving.
3. Do not underinflate or overinflate the tyres. It is extremely important for the tyre pressure to remain between 30-25 psi (2-2.4bar) at all times.
4. Inflating the tyres with an unregulated air source can cause them to be overinflated and burst.

9.6 Temperature


1. Certain parts of the electric wheelchair are affected by changes in temperature. The controller only functions at a temperature ranging between -8 C° and 50 C°.
2. The batteries could freeze at extremely low temperatures and prevent your electric wheelchair from operating. At extremely high temperatures, the electric wheelchair might only function at lower speeds, due to a safety feature of the controller that prevents damage to the motor and other electrical components.
3. Exposing your electric wheelchair to water could cause damage to the electronic or mechanical components. Water can cause an electronic malfunction and/or premature corrosion of the electric components and the frame.

10. GENERAL DESCRIPTION

CS810 "Taurus" adjustable electric wheelchair

10.1 Front view with description of the parts

- A) Pushing handles
- B) Joystick bracket adjustment knob
- C) Armrest height adjustment knob
- D) Armrest detachment knob
- E) Abdominal belt
- F) Wheelchair control joystick
- G) Footplate locking/unlocking devices
- H) Fibreglass-reinforced nylon footrests
- I) Seat inclination adjustment

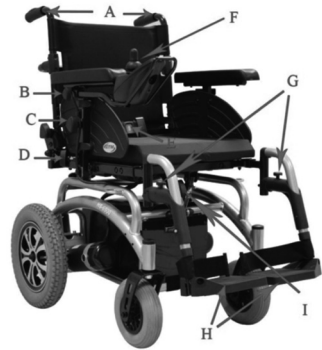
10.2 Rear view with description of the parts

- A) Upper support bar for the backrest
- B) Backrest inclination adjustment knob
- C) Seat width adjustment
- D) Motor unlocking levers (free wheels)
- E) Roll-over protection wheels

10.3 Main view of the control joystick

- A) ON/OFF button
- B) Battery charge indicator
- C) Speed adjustment button (accelerate)
- D) Speed adjustment button (decelerate)
- E) Horn
- F) Direction adjustment lever

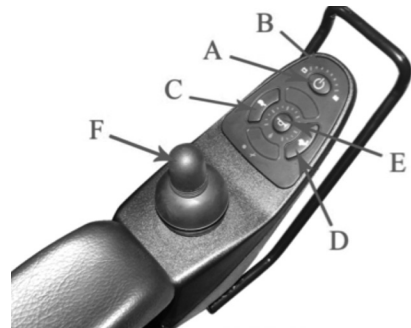10.4 Main view of the lighted controller (OPTIONAL)

- A) Lighting system enable button (ON/OFF)
- B) Emergency lights enable button
- C) Lighted "right turn" indicator button
- C) Lighted "left turn" indicator button
- E) Front lights button

ENGLISH

11. COMPONENTS

Your Moretti electric wheelchair is usually assembled the first time by the retailer. If this is not the case, start by checking that the cardboard box contains all the parts listed below. (fig.1)

(fig.1)

- 1. Electric wheelchair
- 2. Backrest
- 3. Joystick with support
- 4. Nuts, bolts and covers for backrest assembly

12. ASSEMBLY INSTRUCTIONS

12.1 Assembly


1. Assemble the backrest onto the wheelchair using the nuts, bolts and covers provided, as illustrated below

2. Install the joystick with its support on the armrest and tighten the hexagonal screw. After mounting the joystick, also connect the electrical connector

3. Remove the battery cover and connect the two electrical connectors present underneath the seat. Close the battery cover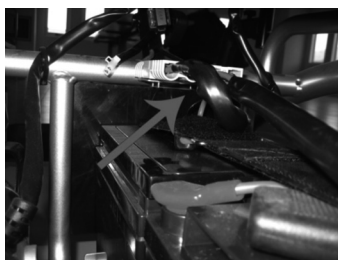

12.2 Adjusting comfort

Before starting to use your **ARDEA MOBILITY** electric wheelchair by Moretti, carry out all the adjustments indicated below to obtain a comfortable driving and seating position suited to the user.

1. Position of the control joystick
2. Armrest height
3. Armrest width
4. Seat depth
5. Seat width
6. Seat inclination
7. Backrest height
8. Backrest width
9. Backrest inclination
10. Footplate depth
11. Footplate height

1. Position of the control joystick

1. Unscrew the knob indicated in the figure
2. Adjust the position of the control joystick support
3. Tighten the knob again

N.B. During adjustment, take care with the electric cable of the joystick, assisting it in sliding into place

2. Armrest height

1. Unscrew the armrest locking knob (ref. C)
2. Adjust the armrest height
3. Tighten the knob firmly

N.B. Take care with the controller cables


3. Armrest width

1. Unscrew the bolt indicated in the figure
2. Adjust the armrest width
3. Tighten the bolt again

4. Seat depth

1. Remove the four screws positioned underneath the seat
2. Choose the positioning hole according to need
3. Tighten the four screws in the positioning hole chosen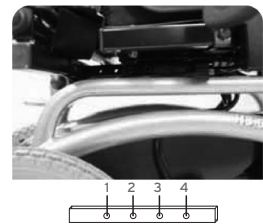

5. Seat width

1. Unscrew and remove the bolts in the front and rear part of the seat
2. Adjust the width of the seat based on the holes present on the support
3. Firmly tighten the screws again

**6. Seat inclination**

1. Unscrew the knob (ref. I) and remove the bolt
2. Adjust the seat inclination according to need
3. Insert the bolt again and firmly tighten the knob

N.B. For safety reasons, do not drive when the seat is inclined!

7. Backrest height

1. Detach the rear Velcro strap of the backrest
2. Unscrew and remove the backrest bolts
3. Adjust the backrest height according to need
4. Insert the bolts again and firmly tighten
5. Reattach the rear Velcro strap

**8. Backrest width**

1. Detach the rear Velcro strap of the backrest
2. Unscrew and remove the screws indicated in the figure
3. Adjust the width of the backrest, choosing the size best suited to the user
4. Insert the screws in the holes again and firmly tighten them
5. Reattach the Velcro strap

9. Backrest inclination

1. Unscrew the two side knobs of the backrest (ref. B)
2. Adjust the backrest inclination according to need
3. Insert the two knobs again and firmly tighten

N.B. For safety reasons, do not drive the wheelchair with the backrest reclined

10. Footplate depth

1. Unscrew and remove the two screws indicated in the figure
2. Adjust the depth of the footplate support
3. Insert the screws and firmly tighten them

N.B. Take care not to interfere with the front wheels while adjusting the footplates

11. Footplate height

1. Unscrew and remove both screws on the footplates indicated in the figure
2. Adjust the height of the footplates according need
3. Insert the screws and firmly tighten them

12.3 Use of the controls

1. Press the ON/OFF button to switch on the control joystick. The battery charge LED indicators will switch on.
2. Use the appropriate buttons to adjust the maximum speed of the wheelchair (Tortoise to reduce speed and Hare to increase it).
3. Press the horn button when deemed necessary.

N.B. Do not use the horn unnecessarily and comply with laws on disturbing the peace.

4. To move the wheelchair **FORWARDS**, push the joystick **FORWARDS**. To turn the wheelchair to the **RIGHT** or to the **LEFT**, push the joystick in the respective directions. To move **BACKWARDS**, pull the joystick **BACKWARDS**

12.4 Disassembly of the wheelchair

1. Press the button on the footplates indicated in the figure, turn the footplate outwards and lift to extract it.

2. Unscrew the knob (ref.2) and extract the armrest upwards.

N.B. Warning!! If the armrest has the joystick mounted on it, first disconnect the electrical connector (ref.1), remove the electric cable from the supports and extract the armrest upwards

3. Rotate the motor unlocking levers to position "D" to prevent any movement of the wheelchair
4. Remove the battery cover

5. Disconnect all the connectors from the electronic control unit positioned underneath the seat

6. Detach the battery Velcro attachment strap.

7. Lift the two batteries, then extract them upwards

8. At this point, your disassembled wheelchair will look as shown in the figure below

13. USING THE ELECTRIC WHEELCHAIR

Before you get into your electric wheelchair, make sure you are on a flat surface with no obstacles. Although your wheelchair is designed for use on slopes, it is safer to practise using it on a flat surface. Several checks to make before every use are indicated below:

- Check correct functioning of the batteries and charge them if necessary
- Always check correct and stable assembly of all the parts of your electrical wheelchair
- Always check the state of wear and tear and functioning of the parts, in order to guarantee that the device is used in total safety; in particular, check the state of wear and tear, damage to and pressure of the front and rear tyres
- Check functioning of the horn.
- The motor unlocking lever must be in position D (DRIVE).
- It is recommended to select a low speed for the first few times you use it.

14. RECOMMENDATIONS FOR SAFE USE

Before sitting on the wheelchair, check that:

- the ignition is switched off (OFF)
- the armrests and footplates are correctly positioned
- the battery charger is disconnected from the wheelchair and the wall socket

Before changing direction from forwards to backwards or backwards to forwards, bring the control lever into the central position and wait for the wheelchair to come to a complete stop.

- Only use the wheelchair where it would also be safe to walk.
- If you need to go up or down a slope, check that the gradient does not exceed 8% and that the surface has a good grip and is not slippery.
- Use a slow speed to reverse, go up a slope, down a slope, turn a corner or on uneven surfaces.

NEVER use the leg rests to lift the wheelchair. The leg rests are removable and therefore must not be used for lifting.

Once seated, check that all adjustments have been made in a manner to guarantee safe and comfortable driving. In particular, the joystick should be adjusted to prevent overtiring the hand during driving

To start moving:

- Set to LOW speed
- Press the ON button
- Comply with the waiting time of around two seconds set by the electronics to prevent accidental movements
- Push the joystick gently FORWARDS to move forwards. The more you push the joystick, the quicker the wheelchair will move. To stop, allow the joystick to return to the central position (vertical)
- To turn left or right, push the joystick gently in the desired direction
- Pull the joystick backwards for reverse

WARNING!
IN AN EMERGENCY, RELEASE THE JOYSTICK. THE WHEELCHAIR WILL STOP AUTOMATICALLY

DRIVING UP A SLOPE

International standards set the maximum safe gradient on which an electric wheelchair can be driven without any difficulty. This value is set as 6°/10%. The Moretti electric wheelchair can travel on slopes with a gradient up to 8°/14%. If you need to drive it on the maximum gradient, you may need to travel at a higher speed. Press the "speed" button to increase the speed. Be extremely careful when travelling on steep slopes.

DRIVING DOWN A SLOPE

Proceed slowly down steep descents. Press the "speed" button to reduce the speed. This allows better control of driving. The wheelchair will not accelerate on its own down a slope in any case, due to the automatic braking system, which starts functioning if you attempt to increase the speed. In general, you should drive in a linear manner on slopes, without sudden changes in direction, and never stop the wheelchair on a slope that exceeds the safety limit.

GOING OVER OBSTACLES

Go over obstacles slowly and in a forwards direction. Do not go over obstacles from the side. This will allow you to go over obstacles up to 3 cm. Do not go over obstacles higher than 3 cm. The wheelchair has a roll-over protection system. These devices may interfere when you are ascending onto or descending from a sidewalk. Take maximum care when going over these obstacles.

SELF-DIAGNOSIS

If the speed indicator LEDs start flashing immediately after switching on the electric wheelchair, identify the problem by checking the potential problems on the list in the “Troubleshooting” section of this manual and take the necessary actions.

MOVEMENT

If you need to move the electric wheelchair, switch off the controller by pressing the ON/OFF button, position the motor unlocking levers on N (NEUTRAL), move the electric wheelchair into a secure position, then reposition the motor unlocking levers on D (DRIVE).

15. GENERAL PRECAUTIONS

- The electric wheelchair can only be used by one person at a time
- The maximum total weight carried must not exceed 136 Kg
- Always use the safety belt on your electric wheelchair
- Before getting into or out of your electric wheelchair, check that it is switched off and that the battery charger is disconnected.
- Always drive carefully and with your feet resting firmly on the footplates, taking care if there are other people present in the area where you are travelling.
- Use pedestrian routes whenever possible. Take extreme care when crossing roads.
- Although the electric wheelchair is designed for slopes with a gradient up to 8°/14%, try to avoid slopes with a gradient over 6° (10%) and take extreme care when turning on a slope
- Do not use the maximum power on sharp bends
- Only drive the electric wheelchair after checking that all the parts are firmly attached, in the most comfortable driving position
- Do not drive the electric wheelchair if the rear roll-over protection wheels are damaged or have been removed
- Lower speed when reversing, driving down a slope, on uneven ground or when turning
- Never place the electric wheelchair motors on “N” (NEUTRAL) on a descent
- Follow all the rules of the highway code
- It is not recommended to use the electric wheelchair on wet or damp ground, as the warranty does not cover damage caused by water

16. CHARGING THE WHEELCHAIR

The electric wheelchair has two 12v/36Ah rechargeable batteries and a 25v/5Ampere battery charger. The batteries must be charged before using the wheelchair for the first time and should be charged after prolonged use. Ensure that the wheelchair is switched off and that the motor unlocking lever is in position D (DRIVE).

WARNING!

The wheelchair should be charged the first time for at least 14 hours. Always check the battery charger indicator on the control panel and check how many LEDs are lit up. It is recommended to charge the batteries when only the red LEDs are lit up.

1. Insert the battery charger connector into the charging socket underneath the controller
2. Check that the current selector is on 220V (or the voltage available to you). Insert the battery charger plug into a wall socket and switch on the battery charger by pressing the switch to ON

NB - Do not place the battery charger near a potential fire risk

3. The battery charger has two control lights:
 - A red light indicating that the battery charger is in operation
 - A yellow light that turns green when the batteries are fully charged
4. Wait for the light to turn green before disconnecting the power supply
5. When the light has turned green, switch off the battery charger, then disconnect the cable from the electric wheelchair and from the mains power
6. If the battery charger light flashes green for over 40 minutes, at any time, this indicates a malfunction

Carry out the following checks

- Check that the battery charger cables are properly connected
- Check that the fuse is working
- Check that the wheelchair is switched off
- If all these points have been performed correctly, contact your retailer

NB - The time needed for complete charging may vary depending on the initial charge when the charging procedure is started. Charging for longer than necessary does not damage the batteries, as they cannot be overcharged.

Keep these rules in mind:

- Charge the batteries completely at least once a month or more frequently if used regularly. Always charge the batteries after a trip of over 3 Km
- If you do not plan to use the wheelchair for a month or longer, make sure that the batteries are completely discharged and charge them completely before using the wheelchair again
- The batteries and the wheelchair will function best after the electric wheelchair has been used and the batteries have been charged more than 10 times
- The complete charging time of the batteries may be up to a maximum of 10 hours. If the battery charger light does not turn green after this time, contact your usual retailer for a check on the battery charger
- The battery charger may become slightly warm during use and it has an internal ventilation system

Do not obstruct the ventilation grids.

For safety, follow these guidelines:

1. Do not use the battery charger if the primary power cable is damaged.
2. Do not use extension cables for the primary power supply of the battery charger. There is a risk of fire or electric shock

If you need to replace the batteries, contact your retailer to obtain the correct replacement batteries. Take care when replacing the batteries and do not discard depleted or damaged batteries in the environment. Dispose of the replaced batteries at authorised recycling centres and not in normal urban waste.

17. CARE AND MAINTENANCE

The devices in the ARDEA range by Moretti are CE marked and carefully checked before being placed on the market. For the safety of the patient, your device should be checked at least once every year by the manufacturer or an authorised laboratory to verify its suitability for use. Use only original replacement parts and accessories for repairs.

17.1 Annual maintenance

It is recommended to have a complete maintenance service performed at an authorised centre at least once a year. This ensures that the wheelchair works properly and avoids future problems.

The annual service should include the following checks:

- Wheels and tyres
- Check on batteries, cables and electrical connections
- The controller's programming in relation to the user's requirements
- General check of the frame and fasteners, such as nuts and bolts
- Check on coverings and upholstery

17.2 Ordinary maintenance

- Clean the controller and the joystick
- Clean the frame and the upholstery to prevent any build up of dirt
- Check that the electrical connections are hooked up properly
- Keep the wheels clean of dirt and residue of any kind
- Lightly lubricate the wheel axles and springs every 3 months if necessary
- Inspect the tyres. The tread should not wear more than 1.5 mm
- Charge the batteries regularly
- Check that the joystick works properly, and that it returns to the centre when released. In case of doubt, contact an authorised service centre
- Check the cable covers for damage and wear. In case of doubt, contact an authorised service centre

17.3 Cleaning and disinfection

- DO NOT immerse the device in water for any reason!
- Use only a damp cloth to clean your wheelchair and wipe gently to avoid damaging the controller

N.B. For more thorough disinfection, use a small amount of neutral detergent and dry it off very carefully, making sure that no fluid enters the device.

- Never use acid, alkali or solvents (acetone or diluent) to clean it

18. TROUBLESHOOTING

Moretti wheelchairs are equipped with a device that constantly controls the electrical and electronic system, to ensure safe and comfortable driving. If a malfunction or error occurs, the LED with a wrench symbol flashes continuously for a specific number of times to indicate that there is a problem when the wheelchair is switched on. The number of flashes indicates the type of malfunction. Several suggestions on resolving the main problems you could have with your wheelchair are provided below. To start self-diagnosis, press the ON/OFF button of the electric wheelchair and count the number of flashes.

N° of LEDs lit	PROBLEM	SOLUTION
1	Faulty battery connection or flat batteries	Check the battery status and the connections. Charge the batteries for at least 8 hours.
2	The left motor is faulty	Check the left motor and the electrical connections.
3	The left motor is short circuiting	Contact your usual retailer
4	The right motor is faulty	Check the right motor and the electrical connections.
5	The right motor is short circuiting	Contact your usual retailer
6	External interference	Check whether the battery charger is connected or whether there is electromagnetic interference
7	The joystick is faulty or is not in the neutral position	Release the joystick and check that it is in the central position. Switch off the wheelchair and switch it back on again.
8	The controller is faulty	Check all controller connections
9	Faulty brake	Check the electrical connections of the motors
10	The controller receives excess voltage	Check the battery status and the connections

18.1 Other problems

1) The electric wheelchair does not move when switched ON:

- Check battery charge level LEDs on the joystick.

All the LEDs should be switched on

- Check the self-diagnosis LEDs. They should be continually on.

If they flash, refer to the above table to solve the problem

- Check all the electrical connections to be sure they are correct

- Check whether the batteries are correctly connected

- If none of these suggestions help solve the problem, contact your usual retailer

2) If the charge lights do not turn green after charging the electric wheelchair for over 14 hours, contact your usual retailer

Remember that the electric wheelchair is equipped with an electronic system that constantly controls the status of the driving system, to ensure safe driving. If an error occurs, the battery charge LED will indicate the type of anomaly by flashing.

19. DISPOSAL CONDITIONS 

19.1 General conditions of disposal

Never dispose of the product as normal domestic waste. Dispose of the product at a sorted waste collection centre for recycling.

19.2 Instructions for correct disposal in accordance with European directive 2012/19/EU:

At the end of its working life, the product must not be disposed of together with normal urban waste. It must be delivered to municipal separated waste collection facilities, or to appropriate dealers that provide this service. Separated waste disposal helps to reduce possible negative effects on the environment and health deriving from improper disposal and allows for recycling the materials comprising the product, which translates into significant energy and resource savings. The product bears the barred bin symbol to underline the obligation of disposing of electro-medical equipment.

19.3 Disposal of exhausted batteries - (Directive 2006/66/EC):

This symbol on the product indicates that the batteries must not be considered as normal domestic waste. Ensuring that the batteries are discarded correctly contributes to preventing potentially negative consequences for health and the environment that would otherwise be caused by their inappropriate disposal. Recycling materials helps to conserve natural resources. Take depleted batteries to the collection points indicated for recycling. You can contact your local council, the local waste disposal service or the shop where you purchased the device for more detailed information on disposal of depleted batteries or the product.

20. TECHNICAL SPECIFICATIONS

20.1 Technical specifications

Seat	37+52 cm
Wheels	Tyres
Maximum speed	8 km/h
Steering radius	92 cm
Maximum safe slope	8°
Maximum estimated surmountable slope **	8°/14%
Max. step	30 mm
Autonomy*	16 km
Battery specifications	2x12V - 36Ah
Battery charger type	5A
Controller electronics	Dynamic
Motor	2x420W - 4600 revs/min
Brake	Electromechanical
Drive	Two motors on rear wheels

* The autonomy of the electric wheelchair may vary in relation to:

- Weight of the patient;
- Slopes along the route;
- Battery wear.

** The term rated slope is defined in standard EN 12184:2014.

20.2 Dimensions

Dimensions and weight																
Measurements in cm																
CS810	106	72	37 42	91 101	73 77.5	49	44 59	37 52	41 51	58 71	-	75	32	20	4	136

21. WARRANTY

Moretti products are guaranteed for 2 (two) years from the date of sale against material and manufacturing defects, subject the following limitations. The warranty is voided by improper use, abuse, modifications to the product and failure to follow the instructions. The intended use of the product is given in the user manual. Moretti is not liable for damage, injury or any other consequences resulting from installation or use which are not scrupulously conforming with the instructions given in the installation, assembly and user manual. Moretti does not guarantee its products against damage or defects in the following circumstances: natural disasters, unauthorised repair or maintenance, improper electric power supply (as applicable), use of parts or components not supplied by Moretti, failure to follow the guidelines and instructions for use, tampering, shipping damage (other than the original shipping by Moretti), or failure to perform maintenance as indicated in the manual. Components subject to wear and tear are not covered by this warranty if the damage is caused by normal use of the product.

21.1 Warranty on rechargeable batteries (if provided)

The original and replacement batteries are covered by a ninety-day warranty on performance and a six-month warranty on manufacturing defects or as required by law. If completely charged batteries are left unused for more than three consecutive months, the warranty is automatically invalidated. If completely discharged batteries are left unused for more than three consecutive days, the warranty is automatically invalidated.

22. REPAIRS**22.1 Repairs under warranty**

If a Moretti product has material or manufacturing defects during the warranty period, Moretti will agree with the client whether the defect is covered by the warranty. Moretti, at its sole discretion, may replace or repair the article at a specified Moretti reseller or its own premises. The costs of labour incurred in repairing the product will be borne by Moretti if it determines that the repair is covered by the warranty. Repair and replacement do not renew the warranty period.

22.2 Repairing a product not covered by the warranty

A product not covered by warranty may be returned for repair only if authorised in advance by Moretti customer service. The costs of labour and shipping incurred by repairs not covered by the warranty are borne by the client or reseller in their entirety. Repairs on products not covered by the warranty are themselves guaranteed for 6 (six) months from the day of reception of the repaired product.

22.3 Non-defective products

The client will be notified if Moretti concludes that the product is not defective after having received and examined it. The product will be returned to the client at his expense.

23. REPLACEMENT PARTS

Moretti original replacement parts are guaranteed for 6 (six) months from the day of delivery.

24. NON-LIABILITY CLAUSE

Unless otherwise expressly specified in this warranty and within the limits of the law, Moretti makes no declaration, guarantee or condition, express or implicit, including any future declaration, guarantee or condition of sale, suitability for a given purpose, non violation and non interference. Moretti does not guarantee that the use of its product will be uninterrupted and problem-free. The duration of any implicit guarantee under the law is limited to the warranty period, within the limits of the law. Certain states and countries do not permit limitations on the duration of an implicit guarantee or the exclusion of limitation of accidental or indirect damages in relation to consumer products. In said states and countries, certain exclusions and limitations of this warranty may not apply to the user. This warranty is subject to modification without notice.

WARRANTY CERTIFICATE

ENGLISH

Product _____

Date of purchase _____

Reseller _____

Street _____ Town _____

Sold to _____

Street _____ Town _____

Via Bruxelles, 3 - Melegnano 20122 Cavriaglia (Arezzo) - ITALY - Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN TAIWAN

NOTES

Series of horizontal lines for taking notes.

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