LOGIKO®

ITALIANO

MA DM130_500_5_30_5_40_5_61 03 A_ITA_10-2020

STETOSCOPI

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INDICE

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
REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

LOGIKO **ECHO** stethoscopes line by MORETTI is a complete line of models that can meet different requirements.

- DM130** Single head stethoscope with flat Head in Anodised Aluminium. It is suitable especially for the pressure measurement. The flat head can be easily inserted under the cuff of the sphygmomanometer.
- DM500** Double head stethoscope, Head in Anodised Aluminium, suitable for high and low frequency sounds.
- DM530** Double head stethoscope, Head in Stainless Steel, suitable for high and low frequency sounds. **Adult model.**
- DM535** Double head stethoscope, Head in Stainless Steel, suitable for high and low frequency sounds, in particular for cardiology auscultation.
- DM540** Double head stethoscope, Head in Stainless Steel, suitable for high and low frequency sounds. **Paediatric model.**
- DM545** Single head stethoscope with flat Head in mat finishing anodised Zync and anatomic head.
- DM561** **Rappaport** stethoscope. It is suitable for a precise auscultation without inconvenience or loss of signal. It offers 5 different auscultation modes thanks to two diaphragm types and three bell types.

2. INTRODUCTION

Congratulations on the purchase of the MORETTI. LOGIKO **ECHO** stethoscope. The line LOGIKO **ECHO** By MORETTI is a synonym for stethoscopes designed and made to assure great performance and acoustical superiority. We suggest you to read this instruction manual carefully before the use of the stethoscope. This manual can give you useful advice for a correct and safe use of the product. In case of any doubts, please contact your retailer for help.

CAUTION!

- Do not use the following device for purposes other than those described in this manual.
- Moretti S.p.A. disclaims any liability for damage resulting from improper use of the device or use other than that described in this manual.
- We reserve the right to make changes to the device and this manual without notice in order to improve the characteristics.

3. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of STETHOSCOPE complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

4. WARNINGS

- Before any single use, control the state of the eartips.
- Once worn-out, the eartips should be replaced to avoid damages to the ear canal.
- Avoid extreme heat and cold.
- Avoid hot sterilization.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

5. SYMBOLS

Product code

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Conditions of disposal

6. USE

For a correct use of the LOGIKO ECHO stethoscope, adjust the spring tension of the headset to your needs. To reduce spring tension in the headset, hold each eartube at the bend near the eartip and gradually pull apart until fully extended. To increase spring tension, grasp the headset with one hand where the eartubes enter the tubing and squeeze until the tubing on the eartube touches the other. Repeat as necessary. To turn the double head, it is sufficient to hold the Y tube in correspondence with the head and turn it by 180° until you hear CLICK.

7. CLEANING

The entire stethoscope can be wiped clean with alcohol or soapy water. If disinfection is required, the stethoscope may be wiped with a 70% isopropyl alcohol solution.

8. CONDITIONS OF DISPOSAL

When disposing of the device never use the normal return systems of municipal solid waste. It is recommended instead to disposal through the joint municipal ecological islands for the expected operations of recycling the materials used.

9. TECHNICAL FEATURES

Product	Material	Diameter of the head	Height of the head
DM130	Aluminium	45 mm	10 mm
DM500	Aluminium	45/31 mm	20 mm
DM530	Stainless STEEL	46/36 mm	23 mm
DM535	Stainless STEEL	47/36 mm	33 mm
DM540	Stainless STEEL	40/26 mm	20 mm
DM545	Anodized Zinc	46 mm	23 mm
DM561	Aluminium	45/31 mm	36 mm

10. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual. Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

11. REPAIRING

11.1 Warranty repair

In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location. The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty.

11.2 Repair not covered by warranty

A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

11.3 Non defective device

The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer.

12. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the receiving date. For spare parts and accessories please refer to Moretti master catalogue.

13. EXEMPT CLAUSES

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
Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

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MORETTI S.p.A.