KYARA®

ITALIANO

MA LTK670 01 A_ITA_02-2022

BILANCE PESANEONATI DA CASA

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INDICE

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 **MORETTI S.P.A.**

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: **info@morettispa.com****MADE IN P.R.C.**

KYARA®

ENGLISH

MA LTK670 01 A_ENG_02-2022

BABY SCALES

INSTRUCTIONS MANUAL

INDICE

1. CODE	PAG.3
2. INTRODUCTION.....	PAG.3
3. INTENDED USE.....	PAG.3
4. GENERAL WARNINGS	PAG.3
5. SYMBOLS	PAG.3
6. DESCRIPTION	PAG.4
7. ASSEMBLING	PAG.4
8. BEFORE USE	PAG.4
9. HOW TO USE	PAG.4
10. MAINTENANCE	PAG.5
11. CLEANING AND DISINFECTION	PAG.5
11.1 Cleaning.....	pag. 5
11.2 Disinfection	pag. 5
12. CONDITIONS OF DISPOSAL	PAG.5
12.1 General conditions of disposal	pag. 5
13. DISPOSAL OF BATTERIES AND ELECTRONIC COMPONENTS	PAG.5
DISPOSAL INSTRUCTIONS, AS REQUIRED BY DIRECTIVE 2012/19/UE:	PAG.5
14. DISPOSAL OF WASTE BATTERIES - (DIRETTIVA 2006/66/CE)	PAG.5
15. TECHNICAL FEATURES	PAG.6
16. WARRANTY	PAG.6
17. REPAIRING.....	PAG.6
17.1 Warranty repair.....	pag. 6
17.2 Repair not cover from warranty	pag. 6
17.3 Non-defective device	pag. 6
18. SPARE PARTS.....	PAG.6
19. EXEMPT CLAUSES	PAG.7

1. CODE

LTK670 Baby scale

2. INTRODUCTION

Thank you for choosing KYARA by Moretti Baby scale. The product has been designed and manufactured to meet all your requirements for practical, correct and safe use. This user manual contains small suggestions for the correct use of your chosen device and valuable advice for your safety. We recommend that you read this entire manual carefully before using the scale in question. If you have any doubts, please contact your dealer, who will be able to help and advise you correctly.

NOTE Check that all parts of the product have not been damaged during shipment. In case of damage, do not use the product and contact your dealer for further instructions.

ATTENTION!

- It is forbidden to use the following device for purposes other than those defined in this manual
- Moretti S.p.A. declines any responsibility for damage resulting from improper use of the device or use other than that indicated in this manual.
- The manufacturer reserves the right to make changes to the device and to the following manual without prior notice in order to improve their characteristics.

3. INTENDED USE

This scale is intended to detect babies weight.

4. GENERAL WARNINGS

- DO NOT use this product or any available optional equipment without first completely reading and understanding this instruction manual. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment otherwise injury or damage may occur
- Advise with physician or therapist to determinate the correct adjustment and the correct use of the scale
- Keep the packed scale away from heat sources
- SERVICE LIFE- the scale use limit is defined by the wear of the parts
- Protect the scale from impact with hard objects, moisture, dust, chemicals, liquid cosmetics and great temperature fluctuation
- Do not place any objects on the scale when not in use
- Strong electromagnetic fields may impair the accuracy of the scale
- Remove the battery if the scale is not going to be used for a long period of time

5. SYMBOLS

Product code

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Waste Electrical & Electronic Equipment (WEEE)

Conditions of disposal

ENGLISH

6. DESCRIPTION

1. DISPLAY
2. ON/TARE BUTTON
3. MODE BUTTON
4. MEM BUTTON
5. BATTERY COMPARTEMENT

7. ASSEMBLING

Battery assembling

These kind of scale do not require special assembly operations, excluding the installation of the battery that must be completed in the type and model stated in this manual and the polarity marked on the battery compartment.

8. BEFORE USE

Place the scale on a level and solid surface in order to ensure the scale weighs correctly.

9. HOW TO USE

NOTE: Refer to the general and operating instructions in this instruction manual.

1. Install four AA lithium batteries in the battery compartment, observing polarity.
N.B. THIS OPERATION MUST BE CARRIED OUT ONLY ONCE WHEN USING THE PRODUCT FOR THE FIRST TIME.
2. Press the ON/TARE button to turn the device on

3. Using the scale's MODE button, select the unit of measure (lb=pound/ kg=Kilogram)
4. The scale is ready for measurement as soon as the "0.000kg" screen appears on the display.
5. After one minute of inactivity the scale will automatically switch off.
6. Place the infant correctly on the scale
7. If you wish to place a towel between the infant and the scale, place it before switching on the device
8. To zero the scale that has already detected a weight, press the TARE button
9. The scale automatically registers the weight once it has been stabilised, press MEM to display it again
10. When the "Lo" screen appears replace the batteries

10. MAINTENANCE

The KYARA devices by Moretti S.p.A. are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years.

11. CLEANING AND DISINFECTION

11.1 Cleaning

Use only a damp, non-abrasive cloth and neutral detergents to clean the balance.

ATTENTION!

Do not use abrasive substances, acids, alcohol, chlorine-based detergents, disinfectants and acetone, oxides, organic solvents or alkaline detergents, as these substances may damage the plastic.

The manufacturer will not be responsible for damage caused by the use of materials that may deteriorate the surface of the product or by chemical corrosives used during cleaning.

11.2 Disinfection

If necessary, disinfect the product using a mild disinfectant detergent or using a commercial, chlorine-free disinfectant.

ATTENTION!

Sanitising products are corrosive. Avoid contact with the eyes and skin. In case of accidental contact, wash the affected area thoroughly with warm water and consult a doctor immediately.

12. CONDITIONS OF DISPOSAL

12.1 General conditions of disposal

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the scale in the appropriate disposal areas for recycling.

13. DISPOSAL OF BATTERIES AND ELECTRONIC COMPONENTS DISPOSAL INSTRUCTIONS, AS REQUIRED BY DIRECTIVE 2012/19/UE:

At the end of its service life, the product may not be disposed together with normal municipal waste. It must be delivered to the municipal collection disposal centre or to the reseller, who provides this service. Sorted waste disposal reduces the environmental and health consequences of improper disposal and enables its materials to be recycled, this resulting in energy and resource savings. To highlight the requirement for sorted disposal of electromedical equipment, the product bears the barred bin symbol.

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This symbol on the battery or on the packaging indicates that the battery provided with this product shall not be treated as household waste. By ensuring these batteries are disposed of correctly, you will help prevent potentially negative consequences for the environment and human health which could otherwise be caused by inappropriate waste handling of the battery. The recycling of the

materials will help to conserve natural resources. At the end of their life hand the batteries over to the applicable collection points for the recycling of waste batteries. For more detailed information about recycling of this product or battery, please contact your local Civic Office, your household waste disposal service or the shop where you purchased the product.

15. TECHNICAL FEATURES

Dimensions	 L:28,5 cm  P: 42,5 cm  H: 8,5 cm
Max weight	20kg
Division	5gr
LCD Display	Blue backlight
Power supply	4 AA lithium batteries
Function	Tare/Memory
Auto-off	1'

16. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be apply in the event damages caused by improper use, abuse or alteration, and for the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual. Moretti is not responsible of consequent damages, personal injuries or whatever caused by or in relation with mistaken installation or with device improper use. Moretti does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance or repairs , faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, alteration not authorized, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

17. REPAIRING

17.1 Warranty repair

If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will estimate with customer if the fault can be cover from warranty. Moretti, at its unquestionable discretion, could repair or replace the item, with a Moretti dealer or with own base. Labor cost can be in charge to Moretti if the repair is cover from warranty. A repair or a replacement doesn't extend the warranty.

17.2 Repair not cover from warranty

A product without warranty can be given back after Moretti authorization. The labor and shipping costs for a bath lift out of warranty are to be payed by the customer or by the dealer. The repairs are guaranteed for 6 months as from the good received.

17.3 Non-defective device

The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be payed by the customer.

18. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

19. EXEMPT CLAUSES

Moretti doesn't offer any other declarations, warranty or conditions, explicit or implicit, including possible declarations, warranties or condition of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti doesn't guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

ENGLISH**WARRANTY CERTIFICATE****Product** _____**Purchased on (date)** _____**Retailer** _____**Address** _____ **Town/city** _____**Sold to** _____**Address** _____ **Town/city** _____ **MORETTI S.P.A.**

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: **info@morettispa.com****MADE IN P.R.C.**

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