MOPEDIA®

MA RP691 02 A_ITA_09-2020

ROLLATORS

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ITALIANO

INDICE

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CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices**1. CODES**

RP691 4-wheel steel rollator with antibrachium rest ERA

2. INTRODUCTION

Thank you for purchasing the rollator, MOPEDIA by MORETTI S.p.A., realized to help people with movement difficulty inward and outward, during voyages or in everyday life. The production has been made according to the highest quality and safety standards in accordance with European Directive on Medical devices (2017/745).

This instruction manual gives you suggestion for a correct use and for your safety. We recommend you to read carefully this instruction manual before using. In case of doubts, please contact your dealer.

Note: Check all parts to verify it won't be any shipping damage. In case of shipping damages, do not use. Contact the dealer for further instructions.

WARNING!

- Do not use the product for a purpose not indicated in this manual.
- Moretti Spa declines all responsibilities for any consequences resulting from an incorrect use of this product and from a different use compared to what indicated in this manual.
- The manufacturer can modify the device or the instruction manual without prior notification

3. INTENDED USE

This rollator fits for people with movement problems. Can be used as assistance to facilitate ambulation or even servo-assisted ambulatory. The rollator provides stability, support and security during the deambulation. Anyway, it can't be used by people with evident malfunction of the hand or the arm, even by people with balance problems. The rollator can be used inward and outward. We recommend using on flat surface where the wheels can reach the land. It can't be used on rock ground or on slanted area as the rollator can be unstable. The device is not intended to carry people or things.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of ROLLATORS - MOPEDIA complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017. For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation

6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. WARNINGS

5.1 General warnings

- DO NOT install or use the device before reading and understanding this manual; If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional dealer or technical personnel before attempting to install this equipment to avoid injure or damage;
- The rollator should only be used by patients with a suitable motor capacity which ensures their use without risks;
- Advise with physician or therapist to determinate the correct adjustment and the correct use;
- Physician or therapist should contribute to height adjustment for the maximum support, for a correct brake activation and for a general assembly;
- The rollators are not intended to be used as self-propelled devices.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

5.2 Warnings for use

1.
 - Insure the brake cable to be correctly set;
 - Verify the correct operation of the brake after every height adjustment to avoid a free flexion of the cable;
 - Breaking this simple rules can cause personal damages;
 - The rollator can be used only on a hard, flat surface. Do not use the rollator to climb the stairs
 - Leaning always on both antibrachium supports to maintain balance;
 - Pay particular attention to wet, icy or snowy surfaces;
 - Do not overload the rollator;
 - Do not use the rollator seat to carry persons or property, the seat is for the user's session with the parking brake engaged;
 - All wheels must be in contact with the floor at any time during use to ensure the proper balancing of the rollator and maximum safety;
 - The brake must be engaged before using the seat.

Note When the rollator is used in steady position, the handbrake must be locked.

5.3 Stability warnings

- The rollator can provide ambulatory assistance to an individual weighing up to 136 kg INCLUDING the loading bag;
- The basket has a weight limitation of 5 kg. To avoid displacement and risk of overturning, do not overload it.

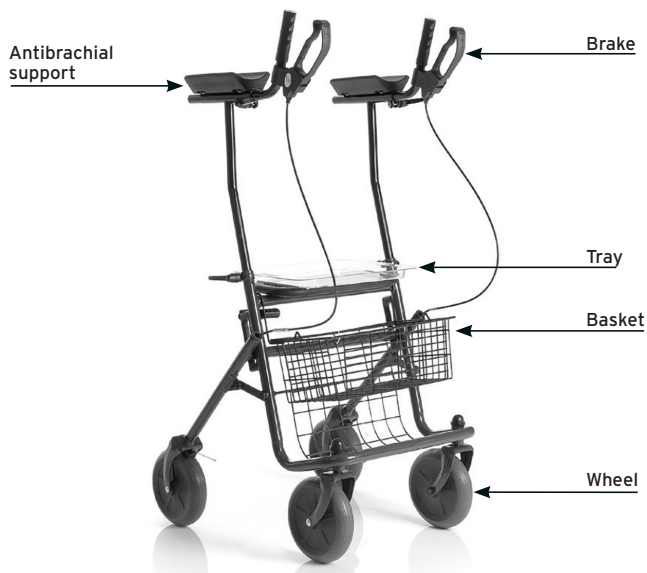
5.4 Assembling warnings

- Once the rollator has been opened and assembled, verify that the seat support is blocked on both sides and verify that the safety system is fasten in the correct position;
- Verify that both antibrachium supports tubes are correctly inserted in the frame and secured with locking knobs;
- Do not hanging anything on the rollator frame to avoid distortions of the tubes when the rollator is open and damage the stability. Injury may also be caused by distortions;
- Take care to entrapment of fingers and / or clothing during the opening and closing.

6. SYMBOLS

-  Product code
-  Unique Device Identification
-  CE mark
-  Manufacturer
-  Batch Lot
-  Read the instruction manual
-  Medical Device
-  Conditions of disposal

7. GENERAL DESCRIPTION

8. BEFORE USE

Before using the device, always check:

- That all parts of the rollator are locked safely;
- That all screws are tightened;
- That the seat is properly locked;
- That the locking of the seat is correctly inserted;
- The correct operation of the brake and the regular brake.
- The state of wear of the mechanical parts involved so as to ensure the use of the device in total safety for people and things

9. ASSEMBLING INSTRUCTION

9.1 Before assembling the product

Unpack the product and check that no damage has occurred during shipping. In the event when the rollator is damaged, please contact your nearest dealer.

9.2 Open the rollator

Open the rollator pulling the front part of the frame by leveraging on the rear legs.

9.3 Pushing handles assembly

Place the antibrachium supports' tubes in the frame. Turn the locking knobs until they are locked.

9.4 Adjustment of the antibrachium supports

Adjust the height of the antibrachium supports using the side knobs. Unscrew the knob, place the antibrachium support to the desired height, turn the knob clockwise to tighten and fasten it again. Repeat for the other antibrachium support and make sure that both antibrachium supports are positioned at the same height.

WARNING!

Make sure that the antibrachium supports are in line with the direction of movement of the rollator. If the handle does not stay straight, there is wobbling or tipping over risk for the patient, especially if you push it harder.

9.5 Folding the rollator

Make lever on the lateral bars and bring back the front of the rollator.

9.6 Tray

The tray has to be used only to support small objects.

WARNING!

Do not place hot objects, sharp or heavy on the tray. To fix the tray to the frame, insert the support that are on the back of the tray in the corresponding slots on the seat.

9.7 Basket

The shopping basket is hung to the rollator frame.

WARNING! Maximum capacity for basket 5 kg. Use the basket for small items only. Do not overload the basket to prevent imbalances and the risk of overturning.

9.8 Bag

The bag is attached to the frame of the rollator through the clips.

WARNING! Maximum capacity for bag 5 kg. Use the bag for small items only. Do not overload the bag to prevent imbalances and the risk of overturning.

9.9 Seat

Before sitting, place the rollator on a flat and solid surface and then lock the brakes. Make sure the brakes are securely locked and that the rollator is in a stable position.

WARNING! Sit down distributing your weight on the seat. Do not sit by pushing only on one side..

10. HOW TO USE

- Start pushing when you are in an upright position with both hands on the handle;
- Make sure your hands are dry before using the rollator to avoid the risk of slipping;
- Walking maintaining an upright position with his eyes facing forward and not down;
- Do not push the rollator too far away from the body, this may cause falls and loads inadequate;
- Do not use the rollator to overcome steps higher than 20mm;
- When using outdoors, prefer the sidewalks properly prepared in accordance regulations for the removal of architectural barriers.

11. BRAKES

The correct functioning of the brakes depends on the wear and the filth remaining on the wheels. Verify the brakes and wheels conditions before every use.

11.1 To brake

To brake the rollator pull the brake lever with your fingers towards themselves until the rollator slows down or stops. To continue to move the rollator, release the brake handles.

11.2 Lock the brake

Press towards the opposite direction on the brake lever with the palm of your hand, when you hear a "click" wheels are locked.

WARNING! When the user is stationary, the brakes must be locked

11.3 To unlock the wheels

Pulling on both brake levers until they are unlocked.

12. CARE AND MAINTANANCE

The devices MOPEDIA by Moretti SpA on the market are carefully checked and carry the CE mark. For the safety of the patient and the doctor is recommended to be checked by the manufacturer or by an approved laboratory, at least every 2 years, the suitability for use of your rollator. If repairs are to be used only original spare parts and accessories. The useful life of the product depends on the use, maintenance, storage and cleaning.

13. CLEANING AND DISINFECTION

13.1 Cleaning

To clean the product, use only a wet cloth and dry it with a clean and dry cloth, or with warm water and neutral soap. The manufacture company cannot be held responsible for the damage that is caused by using materials that could damage the product surface or corrosive chemicals during cleaning.

13.2 Disinfection

If you need to disinfect the product use a common disinfectant neutral detergent.

WARNING! Do not use abrasive substances, acids, alcohols, detergents based on chlorine, disinfections and acetone, because the se substances cause abrasion of the plastic parts and bustingof metal surfaces.

14. WHEEL CARE

1. Clean the wheels with warm water and mild detergent. Wipe dry with a clean cloth.
2. Periodically lubricate the wheels.

15. CONDITIONS OF DISPOSAL

When disposing of the device never use the normal return systems of municipal solid waste. It is recommended instead to disposal through the joint municipal ecological islands for the expected operations of recycling the materials used.

16. TECHNICAL FEATURES

Measures in cm*

ERA										
Product code	RP691									
Seat	38x16									
Frame	Steel									
Dimensioni e pesi										
										
RP691	64	98-120	61	60	44	31	10 Kg	20	20	136 Kg

17. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be apply in the event damages caused by improper use, abuse or alteration, and for the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual. Moretti is not responsible of consequent damages, personal injuries or whatever caused by or in relation with mistaken installation or with device improper use. Moretti does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance or repairs , faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, alteration not authorized, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

18. REPAIRING

18.1 Warranty repair

If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will estimate with customer if the fault can be cover from warranty. Moretti, at its unquestionable discretion, could repair or replace the item, with a Moretti dealer or with own base. Labor cost can be in charge to Moretti if the repair is cover from warranty. A repair or a replacement doesn't extend the warranty.

18.2 Repair not cover from warranty

A product without warranty can be given back after Moretti authorization. The labor and shipping costs for a bath lift out of warranty are to be payed by the customer or by the dealer. The repairs are guaranteed for 6 months as from the good received.

18.3 Non-defective device

The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be payed by the customer.

19. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

20. EXEMPT CLAUSES

Moretti doesn't offer any other declarations, warranty or conditions, explicit or implicit, including possible declarations, warranties or condition of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti doesn't guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

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