MA DMV520 02 A\_05-2020

**OTOSCOPIO**

---

**MANUALE D'ISTRUZIONE**

ITALIANO

## INDICE

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**CE** I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT  
AND OF THE COUNCIL of 5 April 2017 concerning medical devices

## 1. CODE

**DMV520** Optical fiber otoscope

## 2. INTRODUCTION

Thank you for purchasing an otoscope **Dimed by Moretti S.p.A.** Their design and qualities guarantee comfort, safety and reliability to the user. This instruction manual contains advices and for your safety and for a correct use of the aid. We suggest you to read this instruction manual before the use of the product. In case of doubts, contact your retailer.

## 3. INTENDED USE

### OTOSCOPE

A medical device that allows the analysis of the auditory canal.

These devices should be used by professional and trained personnel only.

## 4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of OTOSCOPE - OPHTALMOSCOPE complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

## 5. SYMBOLS

 Product code

 Medical Device

 Unique Device Identification

 CE mark

 Read the instruction manual

 Manufacturer

 Conditions of disposal

 Batch Lot

## 6. ASSEMBLING OTOSCOPE

Unscrew the base and insert inside two alkaline batteries (AA) 1,5 V Place them with the positive pole orientated to the head of the device. Fasten the base again.

## 7. WARNING

Read carefully this instruction manual for a correct use. Pay attention to the presence of children.

## 8. HOW TO USE

These devices can be used by professional and trained personnel only.

### OTOSCOPE

Adjust the adaptor on the base, considering the auditory canal of the patient (usually the 4 mm model is for adult, while 2,4 mm is for children). Switch on the optical fiber light with the switch on the handle. Insert the speculum in the patient ear to do the physical examination. If necessary, you can also use the magnifying glass in the head of the device (possible magnification 3x).

## 9. MAINTENANCE

Dimed by Moretti S.p.A. devices are CE marked and checked before the launch on the market. We suggest you to remove batteries if you don't use the device. Use only original accessories and spare parts

## 10. REPLACEMENT

### HEAD

Switch off the light, unscrew the head anticlockwise and replace it with a new head. Screw clockwise until the end.

## HANDLE

Unscrew the handle and remove the batteries. Screw the handle again clockwise until the end

## BATTERIES

Unfasten the final part of the handle, place there 2 AA 1,5V batteries with positive pole turned to the head of the ENGLISH device, then fasten again.

ENGLISH

### WARNING!

Wait at least 5 minutes after turning off the light of the device because it could heat.

## 11. CLEANING

Use only a damp cloth.

N.B. the otoscope speculum can be sterilized for 20 minutes at 134° C.

## 12. GENERAL WARNINGS

Do not use the products for purposes not indicated in this manual. Moretti S.p.A. declines all responsibility for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product. The manufacturer can modify the device or the instruction manual without prior notification.

## 13. TREATMENT OF FLAT BATTERIES

ADVICE FOR THE CORRECT DISPOSAL OF PRODUCT - EUROPEAN DIRECTIVE 2002/96/EC: Batteries are hazardous waste. Do not dispose them together with household garbage. At the end of their life hand the batteries over to the applicable collection points for the recycling of waste

batteries. For more detailed information about recycling of this product or battery, please contact your local Civic Office, or the shop where you purchased the product.

## 14. TECHNICAL FEATURES

### OTOSCOPE

Lighting system	Optical fiber
Brightness	18,5 Lumen
Magnification	3X
Lighting bulb	2,5V 0,7° diameter 3,8 mm
Batteries	2 AA 1,5V alkaline
Working conditions	Temperature: 10°C/40°C Humidity: 30% - 75% Pressure: 700hPa - 1060hPa
Storage conditions	Temperature -40°C/55°C Humidity 10% - 90% Pressure 500hPa - 1060hPa

## 15. SPARE PARTS/ACCESSORIES

Check the Moretti S.p.A. catalogue to see the spare parts and the accessories.

## 16. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual. Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or

improper use. Moretti Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

## 17. REPAIRING

### Warranty repair

In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location. The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty.

### Repair not covered by warranty

A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

**Non defective device**

The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer.

**17. SPARE PARTS**

The original Moretti spare parts are guaranteed for 6 (six) months from the receiving date. For spare parts and accessories please refer to Moretti master catalogue.

**18. EXEMPT CLAUSES**

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

---

**NOTES****ENGLISH**

---

---

---

---

---

---

---

---

---

---

**WARRANTY CERTIFICATE****ENGLISH**

Product \_\_\_\_\_

Purchased on (date) \_\_\_\_\_

Retailer \_\_\_\_\_

Address \_\_\_\_\_ Town/city \_\_\_\_\_

Sold to \_\_\_\_\_

Address \_\_\_\_\_ Town/city \_\_\_\_\_

 MORETTI S.P.A.

Via Bruxelles, 3 - Melegnano 20122 Cavriaglia (Arezzo) Tel. +39 055 96 21 11

[www.morettispa.com](http://www.morettispa.com) email: [info@morettispa.com](mailto:info@morettispa.com)

MADE IN P.R.C

**MORETTI S.p.A.**  
Via Bruxelles, 3 - Meleto  
52022 Cavriglia (Arezzo)

Tel. +39 055 96 21 11  
Fax. +39 055 96 21 200

[www.morettispa.com](http://www.morettispa.com)  
[info@morettispa.com](mailto:info@morettispa.com)