MA DMV540x 02 A_09-2020

**OTOSCOPIO MINI LUCE
CONVENZIONALE**

MANUALE D'ISTRUZIONE

ITALIANO

INDICE

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
CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT
AND OF THE COUNCIL of 5 April 2017 concerning medical devices

1. CODES

DMV520B Mini otoscope conventional light - Blue

DMV520N Mini otoscope conventional light - Black

2. INTRODUCTION

Thank you for purchasing an otoscope **DIMED by Moretti S.p.A.** Its design and quality guarantee comfort, safety and reliability to the user. This instruction manual contains advices for your safety and for a correct use of the device. We suggest you to read this instruction manual before the use of the product. In case of doubts, contact your retailer.

3. INTENDED USE

The Otoscope is a medical device that allows the analysis of the auditory canal. This device can be used by professional and trained personnel only.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of OTOSCOPE - OPHTALMOSCOPE complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. CAUTION

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without prior notification
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to

6. SYMBOLS

 Product code

 Medical Device

 Unique Device Identification

 CE mark

 Read the instruction manual

 Manufacturer

 Conditions of disposal

 Batch Lot

7. GENERAL DESCRIPTION

The DMV540 otoscopes by Moretti S.p.A. are conventional light medical devices, with halogen xenon bulb. They include 8 reusable tips, 2 for each dimension (2,4mm-3mm-4mm-5mm), and they are used to view the tympanic membrane and external ear canal.

8. WARNING FOR A CORRECT USE





For a correct use take reference to the manual.

- Keep the device away from heat sources
- Protect the device from water, moisture, impact, dropping, direct sunlight, heat and fire
- Do not touch the lens directly by hand or something hard
- Keep the device out of the reach of infants and children
- Do not disassemble or attempt to repair the device or components

9. ASSEMBLING

Unscrew the handle and insert inside two alkaline batteries (AA) 1,5 V Place them with the positive pole orientated to the head of the device. Fasten the handle again.

10. HOW TO USE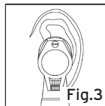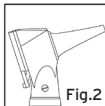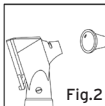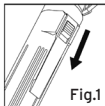

- Turn on (or off) the device by pushing downwards (upwards) as Fig.1;
- Select a reasonable tip for patient's ear canal and push it on to the hand and screw the tip clockwise, refer to Fig.2. generally the tip size of 4mm is for adult and 2.4 mm is for children;
- Put the otoscope with a tip onto the ear canal of patient (Fig.3), and examine through the viewing window which is 3X magnification;

11. MAINTENANCE

DIMED by Moretti S.p.A. devices are CE marked and checked before the launch on the market. We suggest you to remove batteries if you don't use the device for long time.


12. PERIODIC SAFETY CHECK

The following safety check should be performed at least every 12 months by a qualified person who has adequate training, knowledge and practical experience to perform these tests.

- Inspect the equipment and accessories for mechanical and functional damage.

- Verify that the device functions properly as described in the instructions for use.

13. CHANGING SPARE PARTS

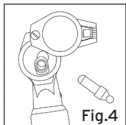Bulb (Fig.4)

- Turn off the power and screw the viewing window off the slot on bottom of the head;
- Pull out the old bulb in the metal support and replace with new one;
- Screw the viewing window back on the slot.

WARNING!


Wait at least 5 minutes after turning off the light of the device because it could heat.

Head

Switch off the light, unscrew the head anticlockwise and replace it with a new head. Screw clockwise until the end.

Handle

Unscrew the handle and remove the batteries. Screw the handle again clockwise until the end.

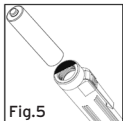Batteries (Fig.5)

- Screw the head off the handle;
- Remove the old AA batteries and replace with new ones. Ensure the anode of the batteries points to the head of the device;
- Screw the head back on handle.

**WARNING!**

- AA alkaline batteries are recommended;
- Replace all batteries simultaneously and do not use alkali manganese batteries with lithium as bulb life will be reduced;
- Do not leave worn batteries in the handle;
- Please take out batteries from the device if not in use for a prolonged period;

14. CLEANING

Clean the surface of the device with 75% alcohol solution at least once a week, avoiding the magnifying glass in the head. Corrosive and polishing detergent is prohibited. Otoscope tips can be sterilized with saturated steam for 20 minutes at 134° C.

**WARNING!**

Do not soak, autoclave or let water dip into the device.

15. DISPOSAL CONDITION

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the otoscope in the appropriate disposal areas for recycling.

DISPOSAL OF BATTERIES AND ELECTRONIC COMPONENTS DISPOSAL INSTRUCTIONS, AS REQUIRED BY DIRECTIVE 2012/19/UE:

At the end of its service life, the product may not be disposed together with normal municipal waste. It must be delivered

to the municipal collection disposal centre or to the reseller, who provides this service. Sorted waste disposal reduces the environmental and health consequences of improper disposal and enables its materials to be recycled, this resulting in energy and resource savings. To highlight the requirement for sorted disposal of electromedical equipment, the product bears the barred bin symbol.

TREATMENT OF EXHAUSTED BATTERIES (Directive 2006/66/EC):

This symbol indicates that the batteries may not be treated like common household waste. Make sure the batteries are disposed of properly; this prevents the possible environmental and health consequences of incorrect disposal. Recycling helps to conserve natural resources. Deliver exhausted batteries to a proper recycling centre. For more detailed information regarding the disposal of exhausted batteries or the product itself, contact the municipality, local waste disposal service or the reseller from whom you purchased the device.

16. SPARE PARTS AND ACCESSORIES

Check the Moretti S.p.A. catalogue to see the spare parts and the accessories.

- DMV510** Set 4 reusable tips
- DMV511** Disposable tips 2,75mm - 100 pcs
- DMV512** Disposable tips 4,75mm - 100 pcs
- DRV238** Spare bulb

17. TECHNICAL FEATURES

DIMENSION	24 x 37 x 165 mm
WEIGHT	53.20 g (w/o batteries)
ILLUMINATION	Direct illumination
LIGHTING SYSTEM	Conventional light with halogen xenon bulb
LUMINANCE	24 Lumen
MAGNIFICATION	3X
TIP SIZES	2.4mm; 3mm; 4mm; 5mm
BULB	2,5 V 0,75°, 3060K, 25 ore
BATTERIES	2xAA 1,5V alkaline

Technical specification

- Operating conditions
 - Temperature: 10°C / 35°C
 - Humidity: 30% - 75%
 - Air Pressure: 700hPa - 1060hPa
- Storage & transport conditions
 - Temperature: -20°C/70°C
 - Humidity: 10% - 90%
 - Pressure: 500hPa - 1060hPa

18. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual. Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or

improper use. Moretti Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

Rechargeable batteries warranty

The original batteries and the spare batteries are covered by a 90 days warranty during a period of 6 month, with regards to manufacturing defeat or to what concern the law regulation. If charge batteries are unused for more than 3 consecutive months, the warranty is void. If exhausted batteries are unused for more than 3 consecutive days, the warranty is void.

19. REPAIRING

Warranty repair

In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location. The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty.

Repair not covered by warranty

A product not covered by warranty can be returned for repairing only after receiving prior authorization from

Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

Non defective device

The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer.

20. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the receiving date. For spare parts and accessories please refer to Moretti master catalogue.

21. EXEMPT CLAUSES

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
Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

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MORETTI S.p.A.