MOPEDIA®

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INDICE

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REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL of 5 April 2017 concerning medical devices

1. CODE

RS963 FOLDING GRAB BAR

2. INTRODUCTION

Thank you for purchasing a MOPEDIA bath handle by Moretti S.p.A. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly.

NOTE Check if any part has shipping damages and test before using. In case of damage, do not use the device. Contact your dealer for further instructions.

WARNING!

- Do not use the product for a purpose not indicated in this manual.
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer reserves the right to change the information contained in this document without previous notice.

3. INTENDED USE

The Mopedia by Moretti folding grab bars is an aid for people with problems to sit down and stand up from toilet.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of SAFETY BATH HANDLES complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

5.1 General warnings

- DO NOT use this product or any available optional equipment without

first completely reading and understanding this instruction manual.

- Keep the packed product away from heat sources
- The bath handle is not designed to hold or support your full body weight.
- This handle bar is designed to assist your natural motion, as when stepping into or out of a shower.
- Do not attach to moving surfaces.
- DO NOT allow children to play on or operate the bath chair.
- The user and/or the patient will have to report any serious accident that have occurred related to the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

5.2 Warnings for use

For a correct use please make reference to the following manual. Pay attention to the presence of children.

WARNING!

Be sure that the safety block is correctly placed in order to stop the grab bar in the horizontal and in the vertical position.

6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Max loading weight

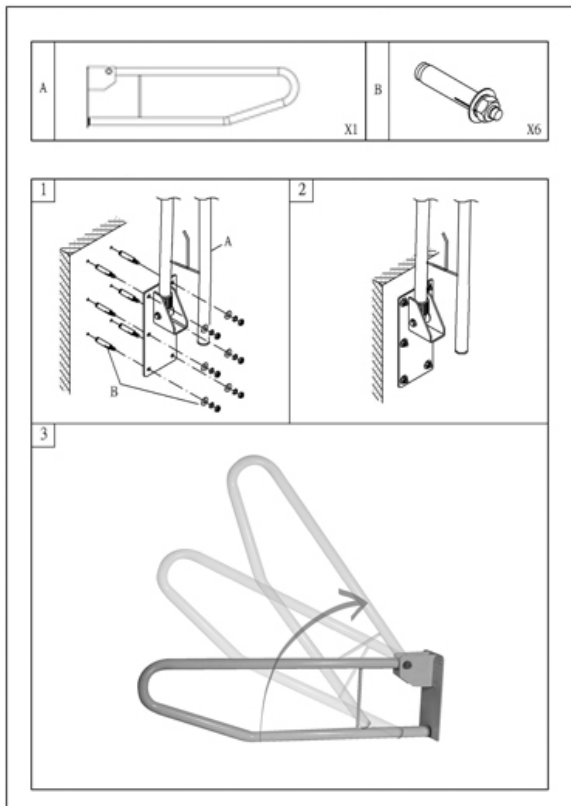
Conditions of disposal

7. BEFORE USE

- Check if the frame is damaged in order to guarantee a safe use for people and things.
- Check the correct assembly of the device.

8. ASSEMBLING

- Choose a solid and safe wall to install the handle.
- Make sure that it is possible to drill the wall safely for people and things.
- Make sure to position the handle bar at the most suitable height for users.
- The grab bar has to be assembled as in the drawing: it can be moved horizontally and vertically
- Make 6 bores with the right diameter for screws.
- Fix the grab bar to the wall with the screws and fasten them carefully.

Note: Stainless steel screws are recommended to reduce the risk of rust.

9. HOW TO USE

The grab bar can be moved into two positions, according to your needs:

1. Vertical position, when the aid is not used.
2. Horizontal position, when the patient needs to use it.

Lift the grab bar and unlock the safety block. Take-down the grab bar and move it to the wall to block it again. If you need to change from horizontal to vertical position, follow the inverse up mentioned procedure. The grab bar in horizontal position is an aid to seat and stand up from the toilet.

WARNING!

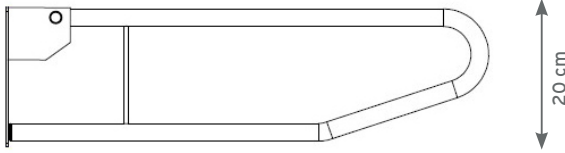
Do not overload the grab bar but use it to maintain the right balance and to reduce the effort sitting and standing up from toilet.

10. MAINTENANCE

Mopedia by Moretti devices are CE marked and checked before the launch on the market. Before each use, be sure screws are correctly placed into the wall. Check also that screw on the grab bar turning point is fastened.

11. TECHNICAL FEATURES

MATERIAL: the grab is composed by steel.

LENGTH:	70 cm
TUBE DIAMETER:	2,5 cm
HEIGHT:	20 cm
WIDTH:	10 cm
GRAB BAR WEIGHT:	2 Kg
MAX LOADING WEIGHT:	80 Kg

12. CLEANING AND DISINFECTION

12.1 Cleaning

Use a damp cloth and mild soap. Then dry thoroughly before use.

12.2 Disinfection

If you need to disinfect the device use a common disinfectant cleaner.

Note: Never use acids, alkalis or solvents such as acetone.

13. CONDITIONS OF DISPOSAL

In case of disposal don't use the inserting container for municipal waste. It is recommended to dispose of the device through the municipal sorted waste collection centre for the planned recycling operations of the materials used.

14. SPARE PARTS AND ACCESSORIES

For spare parts only refer to MORETTI master catalogue.

15. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual. Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

16. REPAIR

Warranty repair

In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at

Moretti location. The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty.

Repair not covered by warranty

A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

Non defective devices

The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer.

17. SPARE PARTS

For spare parts only refer to MORETTI master catalogue.

18. EXEMPT CLAUSES

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

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